
Cleaners and Disinfectants for SonoSite Products

SonoSite, Inc.

21919 30th Drive SE

Bothell, WA 98021

USA

T: 1-888-482-9449 or 1-425-951-1200

F: 1-425-951-1201

SonoSite Ltd

Alexander House

40A Wilbury Way

Hitchin

Herts SG4 0AP

UK

T: +44-1462-444800

F: +44-1462-444801

Caution: | Federal (United States) law restricts this device to sale by or on the order of a physician.

"MicroMaxx," "TITAN," "SonoSite TITAN," "SonoCalc," and "SonoRES" are trademarks of SonoSite, Inc.

Non-SonoSite product names may be trademarks or registered trademarks of their respective owners.

SonoSite products may be covered by one or more of the following U.S. patents: 4454884, 4462408, 4469106, 4474184, 4475376, 4515017, 4534357, 4542653, 4543960, 4552607, 4561807, 4566035, 4567895, 4581636, 4591355, 4603702, 4607642, 4644795, 4670339, 4773140, 4817618, 4883059, 4887306, 5016641, 5050610, 5095910, 5099847, 5123415, 5158088, 5197477, 5207225, 5215094, 5226420, 5226422, 5233994, 5255682, 5275167, 5287753, 5305756, 5353354, 5365929, 5381795, 5386830, 5390674, 5402793, 5,423,220, 5438994, 5450851, 5456257, 5471989, 5471990, 5474073, 5476097, 5479930, 5482045, 5482047, 5485842, 5492134, 5517994, 5529070, 5546946, 5555887, 5603323, 5606972, 5617863, 5634465, 5634466, 5636631, 5645066, 5648942, 5669385, 5706819, 5715823, 5718229, 5720291, 5722412, 5752517, 5762067, 5782769, 5800356, 5817024, 5833613, 5846200, 5860924, 5893363, 5916168, 5951478, 6036643, 6102863, 6104126, 6113547, 6117085, 6142946, 6203498 B1, 6371918, 6135961, 6364839, 6383139, 6416475, 6447451, 6471651, 6569101, 6575908, 6604630, 6648826, 6835177, D0280762, D0285484, D0286325, D0300241, D0306343, D0328095, D0369307, D0379231, D456509, D461895, 6817982, D509900, 10/062179. Other patents pending.

P06703-01 07/2006

Copyright 2006 by SonoSite, Inc.

All rights reserved.

Introduction

This document provides information about the compatibility of cleaners and disinfectants used on SonoSite ultrasound systems and transducers in the cleaning and disinfecting process. All cleaners and disinfectants listed in [Table 1](#) have been tested by SonoSite for use with its products. See the ultrasound system user guide for cleaning and disinfecting information and additional warnings and cautions.

WARNING: Disinfectants and cleaning methods listed are recommended by SonoSite for compatibility with product materials, not for biological effectiveness. Refer to the disinfectant label instructions for guidance on disinfection efficacy and appropriate clinical uses.

Systems

The exterior surface of the ultrasound system and the accessories can be cleaned and disinfected using a recommended cleaner or disinfectant.

Table 1: Cleaners and Disinfectants Compatible with Sonosite Systems

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	180/180 PLUS	TITAN	Micro-Maxx	iLook
AbcoCide (4)	USA	Liquid	Glutaraldehyde	N	A	A	A
Accel Wipes	CAN	Wipe	Hydrogen Peroxide	N	A	A	N
Accel Plus	CAN	Wipe	Hydrogen Peroxide	N	N	N	N
Accel TB	CAN	Wipe	Hydrogen Peroxide	N	N	N	N
Aidal Plus	AUS	Liquid	Glutaraldehyde	N	N	N	N
Alkacide	FRA	Liquid	Glutaraldehyde	A	A	A	N
Alkazyme	FRA	Liquid	Quat. Ammonia	A	A	A	N
Anioxyde 1000	FRA	Liquid	Peracetic Acid	N	N	N	A
Aquatabs (1000 PPM)	IRL	Tablet	Sodium Dichloroisocyanurate	A	A	A	N
Aquatabs (2000 PPM)	IRL	Tablet	Sodium Dichloroisocyanurate	N	N	N	N

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with Sonosite Systems (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	180/180 PLUS	TITAN	Micro-Maxx	iLook
Aquatabs (5000 PPM)	IRL	Tablet	Sodium Dichloroisocyanurate	N	N	N	N
Ascend (4)	USA	Liquid	Quat. Ammonia	A	A	A	A
Asepti-HB	USA	Liquid	Quat. Ammonia	N	N	N	N
Asepti-Steryl	USA	Spray	Ethanol	N	A	N	N
Asepti-Wipes	USA	Wipe	Propanol (Isopropyl Alcohol)	N	A	A	N
Bacillocid rasant	DEU	Liquid	Glut./Quat. Ammonia	A	N	N	A
Banicide (4)	USA	Liquid	Glutaraldehyde	N	N	N	A
Bleach (4)	USA	Liquid	NaCl Hypochlorite	N	N	N	N
Cavicide (4)	USA	Liquid	Isopropyl	N	A	A	N
Caviwipes	USA	Wipes	Isopropanol	N	A	A	N
Chlor-Clean	GBR	Liquid	Sodium Dichloroisocyanurate	N	N	N	N
Cidex 14 (2) (4) (5)	USA	Liquid	Glutaraldehyde	A	A	A	A
Cidex OPA (2) (3) (4) (5)	USA	Liquid	Ortho-phthaldehyde	N	A	A	A
Cidex Plus (2) (4) (5)	USA	Liquid	Glutaraldehyde	A	A	A	A
Clorox Wipes	USA	Wipes	Isopropanol	N	N	N	N
Control III (4)	USA	Liquid	Quat. Ammonia	N	A	A	N
Coverage Spray (4)	USA	Spray	Quat. Ammonia	N	A	N	N
DentaSept	FRA	Liquid	Quat. Ammonia	A	N	N	N
Dentured Alcohol	USA	Liquid	Ethanol	N	N	N	N
DisCide	USA	Wipes	Isopropyl Alcohol	N	N	N	N
DisOPA	JPN	Liquid	Ortho-phthaldehyde	N	N	N	A

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with Sonosite Systems (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	180/180 PLUS	TITAN	Micro-Maxx	iLook
Dispatch (4)	USA	Spray	NaCl Hypochlorite	N	N	N	N
End-Bac II	USA	Liquid	Quat. Ammonia	N	A	A	N
Endozime AW Plus	FRA	Liquid	Propanol	A	A	A	A
Envirocide (4)	USA	Liquid	Isopropyl	N	A	A	N
Enzol	USA	Cleaner	Ethylene Glycol	A	A	N	A
Expose	USA	Liquid	Isopropyl	A	A	A	N
Gigasept AF (3)	DEU	Liquid	Quat. Ammonia	N	N	N	N
Gigasept FF	DEU	Liquid	Bersteinsaure	N	N	N	N
Glutaraldehyde SDS	USA	Liquid	Glutaraldehyde	N	A	A	N
Hexanios	FRA	Liquid	Polyhexanide/Quat. Ammonia	A	A	A	A
Hi Tor Plus	USA	Liquid	Chloride	A	N	N	N
Hibiclens	USA	Cleaner	Chlorhexidine	A	A	A	A
Hydrogen Peroxide	USA	Liquid	Hydrogen Peroxide	N	N	N	N
Isopropanol Alcohol	ALL	Liquid	Alcohol	N	N	N	N
Kodan Tücher	DEU	Liquid	Propanol	N	N	N	N
Kohrsolin ff	DEU	Liquid	Glutaraldehyde	N	N	N	N
Korsolex basic (3)	DEU	Liquid	Glutaraldehyde	N	A	A	N
Korsolex FF	DEU	Liquid	Glutaraldehyde	U	U	A	U
LpHse (4)	USA	Liquid	O-phenylphenol	N	A	A	N
Lysol	USA	Spray	Ethanol	N	N	N	N
Lysol IC Spray	USA	Spray	Ethanol	N	A	A	N
Madacide 1 (4)	USA	Liquid	Isopropanol	N	N	N	N

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with Sonosite Systems (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	180/180 PLUS	TITAN	Micro-Maxx	iLook
Matar (4)	USA	Liquid	O-phenylphenol	N	N	N	N
MetriCide 14 (2) (4) (5)	USA	Liquid	Glutaraldehyde	A	A	A	A
MetriCide 28 (2) (4) (5)	USA	Liquid	Glutaraldehyde	A	A	A	A
MetriZyme	USA	Cleaner	Propylene Glycol	A	A	A	A
Mikrobak forte	DEU	Liquid	Ammonium Chloride	A	A	A	A
Mikrozid Tissues (3)	DEU	Wipe	Ethanol/Propanol	N	A	A	N
Nuclear	FRA	Spray	Alcohol/Biguanide	A	N	N	A
Precise (4)	USA	Spray	O-phenylphenol	N	N	N	N
Ruthless	USA	Spray	Quat. Ammonia	N	A	A	N
Sagrosept Wipes	DEU	Wipe	Propanol	N	N	N	N
Salvanios pH 7	FRA	Liquid	Quat. Ammonia	A	A	A	A
Sani-Cloth HB	USA	Wipe	Quat. Ammonia	N	N	N	N
Sani-Cloth Plus	USA	Wipe	Quat. Ammonia	N	N	N	A
Sklar (4)	USA	Liquid	Isopropanol	N	N	N	N
Sporicidin (2) (4)	USA	Liquid	Phenol	A	N	N	N
Sporicidin Wipes (2) (4)	USA	Wipe	Phenol	N	N	N	N
Staphene (4)	USA	Spray	Ethanol	N	A	N	N
Steranios	FRA	Liquid	Glutaraldehyde	N	A	A	A
Super Sani-Cloth	USA	Wipe	Isopropyl Alcohol	N	N	N	N
T-Spray	USA	Spray	Quat. Ammonia	N	A	N	N
T-Spray II	USA	Spray	Alkyl/Chloride	N	A	A	A
TASK 105	USA	Spray	Quat. Ammonia	A	A	A	A

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with Sonosite Systems (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	180/180 PLUS	TITAN	Micro-Maxx	iLook
TBQ (4)	USA	Liquid	Alkyl	N	N	N	N
Thericide Plus	USA	Liquid	Quat. Ammonia	N	A	N	N
Thericide Plus Wipes	USA	Wipe	Quat. Ammonia	N	A	A	N
Tor (4)	USA	Spray	Quat. Ammonia	N	N	N	N
Transeptic	USA	Cleaner	Alcohol	N	A	N	N
Tristel	GBR	Liquid	Chlorine Dioxide	N	A	A	N
Tristel Wipes				N	A	A	N
Vesphene II (4)	USA	Liquid	Sodium/ o-Phenylphenate	N	A	A	N
Virex II 256	USA	Liquid	Ammonium Chloride	A	N	N	N
Virex TB (4)	USA	Liquid	Quat. Ammonia	N	N	N	N
Wavicide -01 (2) (4) (5)	USA	Liquid	Glutaraldehyde	N	A	N	A
Wavicide -06 (4)	USA	Liquid	Glutaraldehyde	A	N	N	N
Wex-Cide (4)	USA	Liquid	O-phenylphenol	A	A	A	A

(1) Compatible but no EPA Registration

(2) Has FDA 510(k)

(3) Has CE Mark

(4) EPA Registered

(5) FDA 510(k) cleared liquid sterilant or high level disinfectant

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Transducers

To disinfect the transducer, use the immersion or wipe method. Immersible transducers can be disinfected only if the product labeling of the compatible disinfectant indicates it can be used with an immersion method. See the ultrasound system user guide for cleaning and disinfecting information and additional warnings and cautions.

Table 1: Cleaners and Disinfectants Compatible with SonoSite Transducers

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	C8	C11	C11 Cast Urethane	C15 C15e	C15 Cast Urethane	C60 C60e	HFL38	HST	ICT	L25 L25e	L38 L38e	P10	P17	SLA	SLT
				P03363 P03741	P02645-04 P02645-05 P03361-11 P04103	P01960 P02645-02 P02645-03 P03361-10	P00798-04 P01677 P02594 P02461 P04102	P00798-03	P00005 P02098 P02115 P04100 P04969 P05483	P05697	P02954 P04099	P00006 P01343 P01368 P02097 P02450 P04105 P04538	P02976 P04034 P04791 P06519	P01088 P01910 P04101 P05482	P05182	P05661 P06073	P05174	P05180
AbcoCide 14 (4)	USA	Liquid	Glutaraldehyde	A	A	A	A	A	A	U	U	A	A	A	A	A	A	A
Accel Wipes	CAN	Wipe	Hydrogen Peroxide	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Accel Plus	CAN	Liquid	Hydrogen Peroxide	N	N	N	N	N	N	N	U	N	N	N	N	N	N	N
Accel TB	CAN	Wipe	Hydrogen Peroxide	N	N	N	N	N	N	N	U	N	N	N	N	N	N	N
Aidal Plus	AUS	Liquid	Glutaraldehyde	A	A	A	A	A	A	A	A	A	A	A	A	A	A	A
Alkacide	FRA	Liquid	Glutaraldehyde	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Alkazyme	FRA	Liquid	Quat. Ammonia	A	A	A	A	A	A	A	N	A	A	A	A	A	A	A
Aquatabs (1000)	IRL	Tablet	Sodium Dichloroisocyanurate	A	A	A	A	A	A	N	U	A	A	A	A	A	A	A
Aquatabs (2000)	IRL	Tablet	Sodium Dichloroisocyanurate	A	A	A	A	A	A	N	U	A	A	A	A	A	A	A
Aquatabs (5000)	IRL	Tablet	Sodium Dichloroisocyanurate	N	N	N	N	N	N	N	U	N	N	N	N	N	N	N
Anioxyde 1000	FRA	Liquid	Peracetic Acid	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Ascend (4)	USA	Liquid	Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with SonoSite Transducers (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	C8	C11	C11 Cast Urethane	C15 C15e	C15 Cast Urethane	C60 C60e	HFL38	HST	ICT	L25 L25e	L38 L38e	P10	P17	SLA	SLT
				P03363 P03741	P02645-04 P02645-05 P03361-11 P04103	P01960 P02645-02 P02645-03 P03361-10	P00798-04 P01677 P02594 P02461 P04102	P00798-03	P00005 P02098 P02115 P04100 P04969 P05483	P05697	P02954 P04099	P00006 P01343 P01368 P02097 P02450 P04105 P04538	P02976 P04034 P04791 P06519	P01088 P01910 P04101 P05482	P05182	P05661 P06073	P05174	P05180
Asepti-HB	USA	Liquid	Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Asepti-Steryl	USA	Spray	Ethanol	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A
Asepti-Wipes	USA	Wipe	Propanol (Isopropyl Alcohol)	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Bacillocid rasant	DEU	Liquid	Glut./Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Banicide (4)	USA	Liquid	Glutaraldehyde	A	A	A	A	A	A	U	A	A	A	A	A	A	A	A
Bleach (4)	USA	Liquid	NaCl Hypochlorite	A	A	A	A	A	A	A	A	A	A	A	A	A	A	A
Cavicide (4)	USA	Liquid	Isopropyl	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A
Caviwipes	USA	Wipes	Isopropanol	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A
Chlor-Clean	GBR	Liquid	Sodium Dichloroisocyanurate	A	A	A	A	A	A	N	U	A	A	A	A	A	A	A
Cidex 14 (2) (4) (5)	USA	Liquid	Glutaraldehyde	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Cidex OPA (2) (3) (4) (5)	USA	Liquid	Ortho-phthaldehyde	A	A	A	A	A	A	A	A	A	A	A	A	A	A	A
Cidex Plus (2) (4) (5)	USA	Liquid	Glutaraldehyde	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Clorox Wipes	USA	Wipes	Isopropanol	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Control III (4)	USA	Liquid	Quat. Ammonia	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A
Coverage Spray (4)	USA	Spray	Quat. Ammonia	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A
DentaSept	FRA	Liquid	Quat. Ammonia	N	N	N	N	N	N	N	U	N	N	N	N	N	N	A
Denatured Alcohol	USA	Liquid	Ethanol	N	N	N	N	N	N	N	U	N	N	N	N	N	N	N

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with SonoSite Transducers (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	C8	C11	C11 Cast Urethane	C15 C15e	C15 Cast Urethane	C60 C60e	HFL38	HST	ICT	L25 L25e	L38 L38e	P10	P17	SLA	SLT
				P03363 P03741	P02645-04 P02645-05 P03361-11 P04103	P01960 P02645-02 P02645-03 P03361-10	P00798-04 P01677 P02594 P02461 P04102	P00798-03	P00005 P02098 P02115 P04100 P04969 P05483	P05697	P02954 P04099	P00006 P01343 P01368 P02097 P02450 P04105 P04538	P02976 P04034 P04791 P06519	P01088 P01910 P04101 P05482	P05182	P05661 P06073	P05174	P05180
DisCide Wipes	USA	Wipes	Isopropyl Alcohol	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A
DisOPA	JPN	Liquid	Ortho-phthaldehyde	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Dispatch (4)	USA	Spray	NaCl Hypochlorite	A	A	A	A	A	A	A	A	A	A	A	A	A	A	A
End-Bac II	USA	Liquid	Quat. Ammonia	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A
Endozime AW Plus	FRA	Liquid	Propanol	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Envirocide (4)	USA	Liquid	Isopropyl	A	N	A	A	A	A	U	U	A	N	A	A	A	A	A
Enzol	USA	Cleaner	Ethylene Glycol	A	A	A	A	A	A	A	A	A	A	A	A	A	A	A
Expose	USA	Liquid	Isopropyl	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Gigasept AF (3)	DEU	Liquid	Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Gigasept FF	DEU	Liquid	Bersteinsäure	N	N	N	N	N	N	N	A	N	N	N	N	N	N	A
Glutaraldehyde SDS	USA	Liquid	Glutaraldehyde	A	A	A	A	A	A	U	U	A	A	A	A	A	A	A
Hexanios	FRA	Liquid	Polyhexanide/Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Hi Tor Plus	USA	Liquid	Chloride	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A
Hibiclens	USA	Cleaner	Chlorhexidine	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Hydrogen Peroxide	USA	Liquid	Hydrogen Peroxide	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Isopropanol Alcohol	Available in all countries	Liquid	Alcohol	N	N	N	N	N	N	N	U	N	N	N	N	N	N	N
Kodan Tücher	DEU	Liquid	Propanol	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with SonoSite Transducers (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	C8	C11	C11 Cast Urethane	C15 C15e	C15 Cast Urethane	C60 C60e	HFL38	HST	ICT	L25 L25e	L38 L38e	P10	P17	SLA	SLT
				P03363 P03741	P02645-04 P02645-05 P03361-11 P04103	P01960 P02645-02 P02645-03 P03361-10	P00798-04 P01677 P02594 P02461 P04102	P00798-03	P00005 P02098 P02115 P04100 P04969 P05483	P05697	P02954 P04099	P00006 P01343 P01368 P02097 P02450 P04105 P04538	P02976 P04034 P04791 P06519	P01088 P01910 P04101 P05482	P05182	P05661 P06073	P05174	P05180
Kohrsolin ff	DEU	Liquid	Glutaraldehyde	A	A	A	A	A	A	U	U	A	A	A	A	A	A	A
Korsolex basic (3)	DEU	Liquid	Glutaraldehyde	N	N	N	N	N	N	N	U	N	N	N	N	N	N	A
LpHse (4)	USA	Liquid	O-phenylphenol	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Lysol	USA	Spray	Ethanol	N	N	N	N	N	N	N	U	N	N	N	N	N	N	N
Lysol IC Spray (4)	USA	Spray	O-phenylphenol	A	A	N	A	N	A	N	U	A	A	A	A	A	A	A
Madacide 1 (4)	USA	Liquid	Isopropanol	A	N	N	A	N	A	A	U	A	N	A	A	A	A	A
Matar (4)	USA	Liquid	O-phenylphenol	A	A	A	A	A	A	U	U	A	A	A	A	A	A	A
MetriCide 14 (2) (4) (5)	USA	Liquid	Glutaraldehyde	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
MetriCide 28 (2) (4) (5)	USA	Liquid	Glutaraldehyde	A	A	A	A	A	A	A	A	A	A	A	A	A	A	A
MetriZyme	USA	Cleaner	Propylene Glycol	A	A	A	A	A	A	A	A	A	A	A	A	A	A	A
Mikrobak forte	DEU	Liquid	Ammonium Chloride	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Mikrozid Wipes (3)	DEU	Wipe	Ethanol/Propanol	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A
Nuclear	FRA	Spray	Alcohol/Biguanide	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Precise (4)	USA	Spray	O-phenylphenol	N	N	N	N	N	N	N	U	N	N	N	N	N	N	N
Ruthless	USA	Spray	Quat. Ammonia	A	N	N	A	N	A	A	N	A	N	A	A	A	A	A
Sagrosept Wipe	DEU	Wipe	Propanol	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A
Salvanios pH 7	FRA	Liquid	Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Sani-Cloth HB	USA	Wipe	Quat. Ammonia	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with SonoSite Transducers (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	C8	C11	C11 Cast Urethane	C15 C15e	C15 Cast Urethane	C60 C60e	HFL38	HST	ICT	L25 L25e	L38 L38e	P10	P17	SLA	SLT
				P03363 P03741	P02645-04 P02645-05 P03361-11 P04103	P01960 P02645-02 P02645-03 P03361-10	P00798-04 P01677 P02594 P02461 P04102	P00798-03	P00005 P02098 P02115 P04100 P04969 P05483	P05697	P02954 P04099	P00006 P01343 P01368 P02097 P02450 P04105 P04538	P02976 P04034 P04791 P06519	P01088 P01910 P04101 P05482	P05182	P05661 P06073	P05174	P05180
Sani-Cloth Plus	USA	Wipe	Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Sklar (4)	USA	Liquid	Isopropanol	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A
Sporicidin (2) (4)	USA	Liquid	Phenol	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Sporicidin Wipes (2) (4)	USA	Wipe	Phenol	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Staphene (4)	USA	Spray	Ethanol	A	A	N	A	N	A	N	U	A	A	A	A	A	A	A
Steranios	FRA	Liquid	Glutaraldehyde	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Super Sani-Cloth	USA	Wipe	Isopropyl Alcohol	N	N	N	N	N	N	N	N	N	N	N	N	N	N	A
T-Spray	USA	Spray	Quat. Ammonia	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A
T-Spray II	USA	Spray	Alkyl/Chloride	A	A	A	A	A	A	A	A	A	A	A	A	A	A	A
TASK 105	USA	Spray	Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
TBQ (4)	USA	Liquid	Alkyl	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Thericide Plus	USA	Liquid	Quat. Ammonia	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Thericide Plus Wipes	USA	Wipe	Quat. Ammonia	A	A	A	A	A	A	A	N	A	A	A	A	A	A	N
Tor (4)	USA	Spray	Quat. Ammonia	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A
Transeptic	USA	Cleaner	Alcohol	N	N	N	N	N	N	N	U	N	N	N	N	N	N	N
Tristel	GBR	Liquid	Chlorine Dioxide	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A
Tristel Wipes	GBR	Wipe	Chlorine Dioxide	N	N	N	N	N	N	N	U	N	N	N	N	N	N	A

A = Acceptable for use with system or transducer/cable.

N = Not Acceptable (do not use)

U = Untested (do not use)

Table 1: Cleaners and Disinfectants Compatible with SonoSite Transducers (Continued)

Disinfection and Cleaning Solutions	Country of Origin	Type	Active Ingredient	C8	C11	C11 Cast Urethane	C15 C15e	C15 Cast Urethane	C60 C60e	HFL38	HST	ICT	L25 L25e	L38 L38e	P10	P17	SLA	SLT
				P03363	P02645-04	P01960	P00798-04	P00798-03	P00005	P05697	P02954	P00006	P02976	P01088	P05182	P05661	P05174	P05180
				P03741	P02645-05	P02645-02	P01677		P02098		P04099	P01343	P04034	P01910		P06073		
					P03361-11	P02645-03	P02594		P02115			P01368	P04791	P04101				
					P04103	P03361-10	P02461		P04100			P02097	P06519	P05482				
							P04102		P04969			P02450						
									P05483			P04105						
												P04538						
Vesphene IIse(4)	USA	Liquid	Sodium/ o-Phenylphenate	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Virex II 256	USA	Liquid	Ammonium Chloride	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A
Virex TB (4)	USA	Liquid	Quat. Ammonia	A	N	A	A	A	A	A	U	A	N	A	A	A	A	A
Wavicide -01 (2) (4) (5)	USA	Liquid	Glutaraldehyde	N	N	N	N	N	N	N	A	N	N	N	N	N	N	N
Wavicide -06 (4)	USA	Liquid	Glutaraldehyde	A	A	N	A	N	A	A	U	A	A	A	A	A	A	A
Wex-Cide (4)	USA	Liquid	O-phenylphenol	A	A	A	A	A	A	A	U	A	A	A	A	A	A	A

- (1) Compatible but no EPA Registration
- (2) Has FDA 510(k)
- (3) Has CE Mark
- (4) EPA Registered
- (5) FDA 510(k) cleared liquid sterilant or high level disinfectant

A = Acceptable for use with system or transducer/cable.
 N = Not Acceptable (do not use)
 U = Untested (do not use)

Caution:

To avoid damaging the transducer, follow the disinfectant manufacturer's instructions. Soaking times exceeding values recommended by the manufacturer and/or inadequate rinsing may reduce transducer life time or cause damages. See the *TEE Transducer User Guide* for important transducer care and cleaning instructions.

Table 2: Disinfectants Compatible with TEE Transducers

Disinfectant	Active Ingredient	TEE
Cidex	Glutaraldehyde	A
Cidex OPA	Ortho-phthalaldehyde	A
Cidex Plus	Glutaraldehyde	A
Cidezyme/Enzol	Enzymatic detergent	A
Klenzyme	Enzymatic detergent	A
Metricide	Glutaraldehyde	A
PeraSafe	Sodium Perborate	A
STERIS System 1	Peracetic acid	N
STERRAD	Hydrogen peroxide gas plasma	N
Wavicide-01	Glutaraldehyde	A

