

SIMPLE "POINT-
AND-SHOOT"
ACCESS, PRECISELY

Ultrasound combines with advanced magnetic technology to achieve a breakthrough in needle guidance.

Make your vascular access procedures faster and more precise.

- **Efficiency** is increased by achieving higher first-pass success rates through real-time updates of needle location and trajectory.*
- Improve **patient satisfaction** with shorter overall procedure time.
- AxoTrack simultaneously displays anatomy and needle location, **reducing the risk** of damaging critical underlying structures.
- The small footprint of the **uniquely designed transducer** provides easier access to alternative central line insertion sites.

First-pass success rates after 15 minutes of training

| USGIV | Overall* | Subclavian* | Internal jugular* |
|---------------|----------|-------------|-------------------|
| With AxoTrack | 99% | 98% | 100% |
| Conventional | 37% | 22% | 53% |

* In an *in vitro* study P<0.001. Ferre, R.M. et al. The Use of a Novel Device Improves Ultrasound Guided IV Access. Supplement to Annals of Emergency Medicine, September 2010; 56(3):S74.



Advanced technology for simple needle guidance.


The AxoTrack System is composed of two primary components: a specialized transducer and a disposable, sterile procedure kit.

Transducer

P11x / 10-5 MHz Phased Array
Applications: Venous and vascular access
Maximum Depth: 12 cm

AxoTrack I Sterile Procedure Kit

Top Shield with Needle Clamp
Bottom Shield with Needle Guide and Protective Cap
Needle Assembly with Integral Ring Magnet
Sterile Cable Sleeve (80cm length)
Elastic Bands (2)

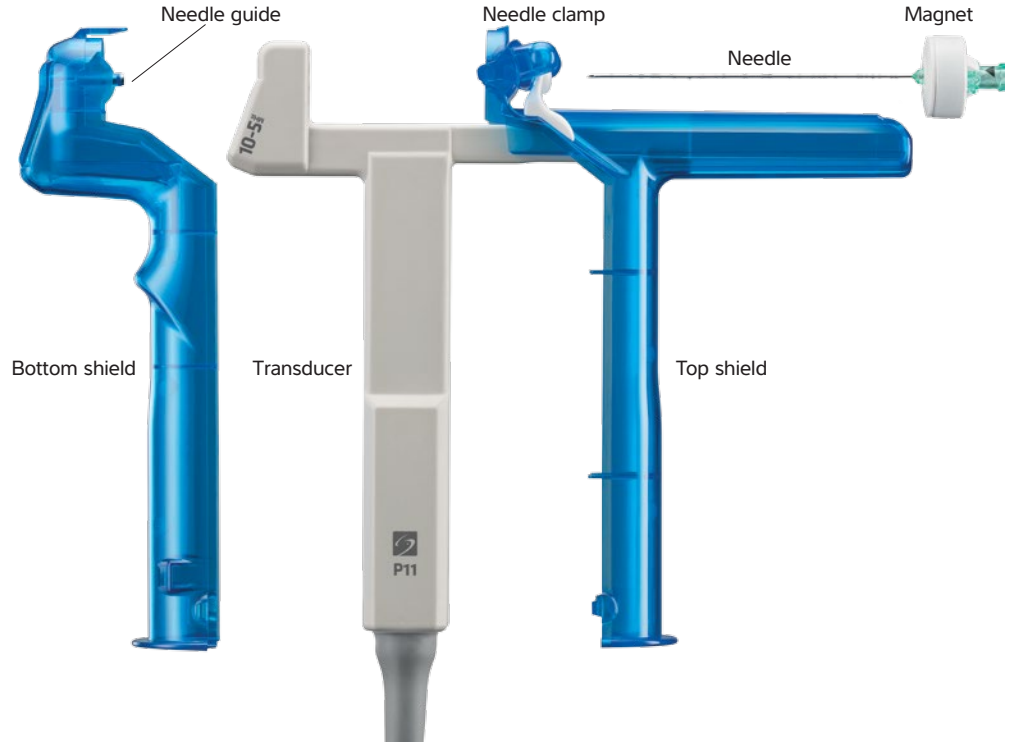

AxoTrack® Needle Guidance Technology

Confidence and control in targeting. Precisely.

Contact 1-877-299-1783 or www.sonosite.com/sonosite-axotrack to learn more.



FUJIFILM SonoSite, Inc.
Worldwide Headquarters
21919 30th Drive SE, Bothell, WA 98021-3904
Tel: +1 (425) 951-1200 or +1 (877) 657-8050
Fax: +1 (425) 951-6800
www.sonosite.com/product/sonosite-axotrack

SONOSITE, the SONOSITE logo and AXOTRACK are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a trademark and registered trademark of FUJIFILM Corporation in various jurisdictions. AxoTrack and AxoTrack Needle Guidance Technology are trademarks of Soma Access Systems LLC. All other trademarks are the property of their respective owners.
Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved. Subject to change.