Barcode Scanner (USB) Installation Instructions

The Sonosite PX ultrasound system is designed to work specifically with the JADAK JDK-10175 USB barcode scanner. Do not use other JADAK Flexpoint HS-1M models. Before using the scanner for the first time, follow these instructions to configure the scanner to work with a Sonosite PX system. See the *JADAK Flexpoint HS-1M Quick Start Guide* or system user guide from Fujifilm Sonosite for information on safety and cleaning.

Configuring the JADAK barcode scanner

To connect and configure the JADAK barcode scanner for use with Sonosite PX

Note Perform this procedure only once for each new barcode scanner you are configuring to work with a Sonosite PX ultrasound system.

- 1 Connect the barcode scanner to a USB port on the ultrasound system.

See *JADAK Flexpoint HS-1M Quick Start Guide* included with the barcode scanner.

For location of USB ports, see the Sonosite PX system user guide.

- 2 Turn on the ultrasound system.

- 3 Use the barcode scanner to scan the following barcodes in the order listed:

- 1 USB COM Port Emulation

- 2 Data Format Configuration

Your barcode scanner is now configured to work with Sonosite PX.

- 4 Once a scanner has been configured, it will appear on a Sonosite PX system's USB setup page when connected to a USB port. You can confirm the scanner is operating properly by opening a patient information form on your system, scanning a patient barcode, and verifying that the information populates into the appropriate fields.

English

Deutsch

Español

Français

Italiano

Nederlands

Manufacturer	EC Authorized Representative	Australia Sponsor	
FUJIFILM Sonosite, Inc. 21919 30th Drive SE Bothell, WA 98021 USA T: +1-888-482-9449 or +1-425-951-1200 F: +1-425-951-1201	FUJIFILM Sonosite B.V. Joop Geesinkweg 140 1114 AB Amsterdam, The Netherlands	FUJIFILM Sonosite Australasia Pty Ltd 114 Old Pittwater Road BROOKVALE, NSW, 2100 Australia	6/2020

SONOSITE is a registered trademark of FUJIFILM Sonosite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value From Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Copyright © 2020 FUJIFILM Sonosite, Inc. All rights reserved.

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Copyright © 2020 FUJIFILM SONOSITE, Inc. Alle Rechte vorbehalten.
Copyright © 2020 FUJIFILM Sonosite, Inc. Reservados todos los derechos.
Copyright © 2020 FUJIFILM Sonosite, Inc. Tous droits réservés.
Copyright © 2020 FUJIFILM Sonosite, Inc. Tutti i diritti riservati.
Copyright © 2020 FUJIFILM Sonosite, Inc. Alle rechten voorbehouden.
Copyright © 2020 FUJIFILM Sonosite, Inc. Alle rettigheder forbeholdes.