

# Bar Code Scanner (USB)

## Installation Instructions

Follow these instructions to connect and program any of these USB bar code scanners:

- ▶ **JADAK flexpoint HS-1M**
- ▶ **Symbol LS2208**

When connected, the bar code scanner is listed on the system's USB or USB Devices setup page.

See the *Bar Code Scanner User Guide* from FUJIFILM SonoSite for information on safety and cleaning.

## Configuring the JADAK bar code scanner

### To connect and program the JadaK bar code scanner

- 1 Attach the scanner cord to the bar code scanner and to a USB port on the ultrasound system.

See *JADAK flexpoint HS-1<sup>M</sup> Quick Start Guide* included with the bar code scanner.

For location of USB ports, see the ultrasound system user guide.

- 2 Turn on the ultrasound system.
- 3 Scan the following two bar codes in the order listed:

USB Kybrd  
STX Prefix  
ETX Suffix



DOS CTRL  
ASCII MODE



- 4 To ensure the scanner has been configured to work with your ultrasound system, FUJIFILM SonoSite recommends opening a patient information form on your system and scanning a bar code. Your system is successfully configured when you scan a bar code and the patient ID (and associated patient information, if any) populates the patient information form.

## Configuring the Symbol bar code scanner

### To connect and program the Symbol bar code scanner

- 1 Attach the scanner cord to the bar code scanner and to a USB port on the ultrasound system.

See *Symbol LS2208 Quick Start Guide* included with the bar code scanner.

For location of USB ports, see the ultrasound system user guide.

2 Turn on the ultrasound system.

3 Scan the following 9 bar codes in the order listed:

For instructions to scan, see *Symbol LS2208 Quick Start Guide*.

**Note**

If you have trouble scanning a bar code, try holding the scanner farther away from the bar code or at a different angle.

Set All Defaults



Code 39 - Any Length



Erase All Rules



Begin New Rule



Send Control B



Send All Data  
That Remains



Send Control C



Beep Once



Save Rule



- 4 Scan additional programming bar codes as desired.
- 5 To ensure the scanner has been configured to work with your ultrasound system, FUJIFILM SonoSite recommends opening a patient information form on your system and scanning a bar code. Your system is successfully configured when you scan a bar code and the patient ID (and associated patient information, if any) populates the patient information form.

**Note**

You can program the scanner with various user preferences, such as beeper volume. Refer to *Symbol LS2208 Quick Start Guide* or, for more comprehensive information, to the *Symbol LS 2208 Product Reference Guide*, included with the scanner and available on the Motorola website at [www.motorola.com](http://www.motorola.com).

SonoSite and the SonoSite logo are registered or unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Copyright© 2015 FUJIFILM SonoSite, Inc. All rights reserved.

<b><u>Manufacturer</u></b>	<b><u>EC Authorized Representative</u></b>	<b><u>Australia Sponsor</u></b>	9/2015
FUJIFILM SonoSite, Inc. 21919 30th Drive SE Bothell, WA 98021 USA T: 1-888-482-9449 or 1-425-951-1200 F: 1-425-951-1201	Emergo Europe Molenstraat 15 2513 BH, The Hague The Netherlands	FUJIFILM SonoSite Australasia Pty Ltd 114 Old Pittwater Road BROOKVALE, NSW, 2100 Australia	



P12173-07

