

Basket and Tray

Installation Instructions


Overview

These instructions describe assembly and installation of the following:

Product	Stand installed on
Basket	<ul style="list-style-type: none"> Edge™ Stand H-Universal™ Stand V-Universal™ Stand
Tray	<ul style="list-style-type: none"> V-Universal Stand

To install additional items, refer to their separate installation instructions. Install items in the order listed on the enclosed insert *Installation sequence of products on the stand*.


Each product is packaged separately. If installing multiple products on the stand, you may receive duplicate tools, cords, and instructions across kits. Please disregard any items not needed for your individual setup.

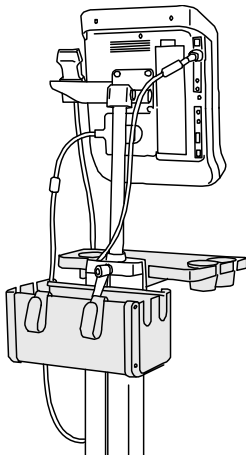




Tray and basket, shown installed on V-Universal Stand with S Series™ ultrasound system



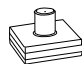
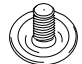
Contents

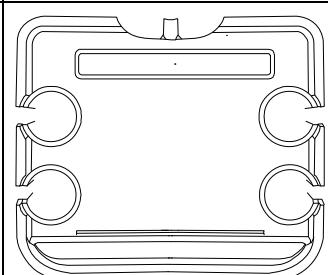Tray (V-Universal)


Description (Quantity)	Illustration
Tray (1)	

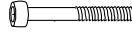Hook (2)	
Cup (4)	
40mm Allen-head screws (2)	

Basket

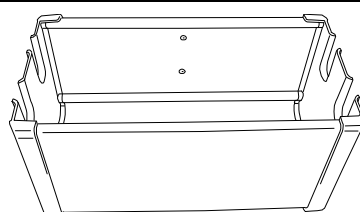Description (Quantity)	Illustration
Basket (1)	
Hook (2)	
Nut plate (2)	
Accessory screw (2)	

Tool included

Description	Illustration
4mm Allen wrench	

Assembly


1 Lock wheels.

- ❖ Press down the lever on two or more wheels.


2 Attach tray (V-Universal).

- 1 Insert the four cups: Align the grooves and press into place.
- 2 Position the tray against the front of the column, aligning the two holes.

- 3 Insert the two 40mm Allen-head screws, and tighten them with the Allen wrench.

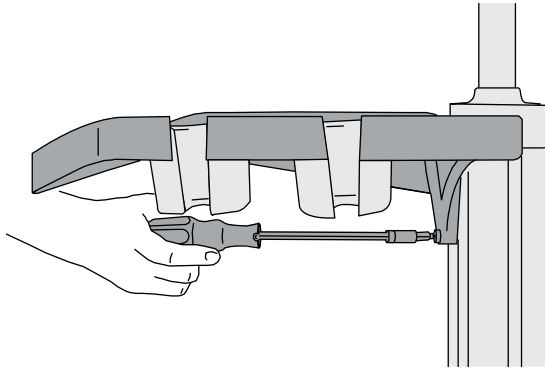


- 4 Attach hooks to the front of the tray. The smaller bend attaches to the tray. The large bend hangs outward.

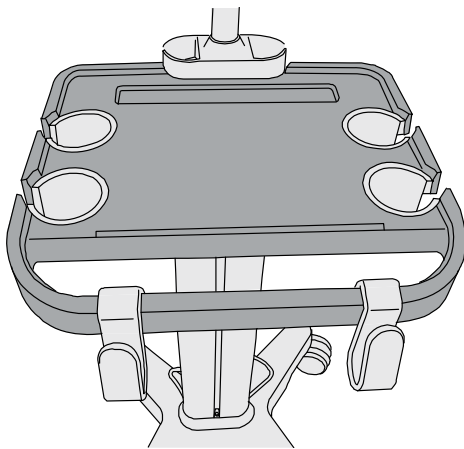

3 Attach basket.

- 1 Insert each nut plate into the groove at the lower back of the column, and slide it up the column. The flat, rubber side of the plate faces toward the column.

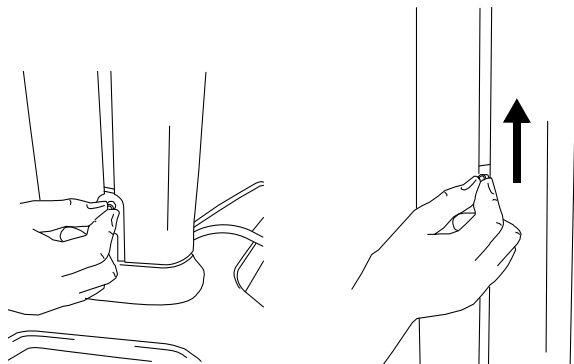


- 2 Position the two nut plates where desired. Make sure that their spacing accommodates the two holes on the basket.
- 3 Place the side of the basket that has two holes onto the column. The two nuts pass through the two holes on the basket.

Note: You can reposition the height-adjustment lever if necessary: Pull the lever out, turn it to the desired position, and release.

- 4 Using the Allen wrench, tighten an accessory screw in each of the two holes on the basket.
- 5 Attach hooks to the basket.

The smaller bend attaches to the basket. The large bend hangs outward.

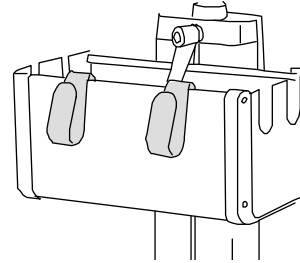

Edge, H-Universal, S Series, SonoSite, V-Universal, and the SonoSite logo are registered (in some jurisdictions) and unregistered trademarks owned by SonoSite, Inc

Non-SonoSite product names may be trademarks or registered trademarks of their respective owners.

The SonoSite ultrasound system referenced in this document may be covered by one or more of the following U.S. patents: 5722412, 5817024, 5893363, 6135961, 6203498, 6364839, 6371918, 6383139, 6416475, 6447451, 6471651, 6569101, 6648826, 6575908, 6604630, 6817982, 6835177, 6962566, 7169108, 7449640, 7534211, 7549961, 7588541, 7591786, 7604596, 7643040, 7686766, 7694814, 7727153, 7740586, 7804970, 7809400, 7819807, 7841575, 7849250, 7867168, 7883276, D456509, D461895, D509900, D538432, D544962, D558351, D559390, D591423, D592750, D592760, D625014, D625015, and by the following counterpart foreign patents: AU727381, AU730822, CA2372152, CA2372158, CA2373065, CN ZL 97113678.5, CN ZL 98106133.8, CN ZL 98108973.9, CN ZL 200830007734.8, DE60021552.0, DE60029777.2, DE60034670.6, DE69730563.5, DE6980539.6, DE69831698.3, DE60200423816.3-08, FR0815793, FR0875203, FR0881492, FR1175713, FR1180970, FR1589878, GB0875203, GB0881492, GB1175713, GB1180970, GB1180971, GB1589878, IT0815793, IT0881492, IT1175713, IT1589878, KR528102, KR532359, NO326202, NO326814, NZ542968, RCD000897368-0001, SP0815793, SP0881492, SP1589878. Patents pending.

Caution: Federal (United States) law restricts this device to sale by or on the order of a physician.

<p>SonoSite, Inc. 21919 30th Drive SE Bothell, WA 98021 USA T: 1-888-482-9449 or 1-425-951-1200 F: 1-425-951-1201</p>	<p>SonoSite Ltd Alexander House 40A Wilbury Way Hitchin, Herts SG4 OAP UK T: +44-1462-444800 F: +44-1462-444801</p>
<p>09/2011 Copyright 2011 by SonoSite, Inc. All rights reserved.</p>	<p>CE</p>

P13941-01

