SonoSite Edge II

SonoSite SII

User Guide Errata

Manufacturer

FUJIFILM SonoSite, Inc.

21919 30th Drive SE

Bothell, WA 98021 USA

T: 1-888-482-9449 or 1-425-951-1200

F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.

Joop Geesinkweg 140

1114 AB Amsterdam,

The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd

114 Old Pittwater Road

BROOKVALE, NSW, 2100

Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

FUJIFILM is a registered trademark of FUJIFILM Corporation. SonoSite, the SonoSite logo, SonoSite Edge, and SonoSite SII are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P21659-03

Publication date: April 2018

Copyright © 2018 FUJIFILM SonoSite, Inc. All rights reserved.

SonoSite Edge II and SonoSite SII User Guide Errata

| | |
|---|----------|
| Introduction | 1 |
| Document conventions | 1 |
| Getting Help | 2 |
| Getting Help (SonoSite Edge II) | 2 |
| Getting Started | 2 |
| Intended uses | 3 |
| System Setup | 3 |
| Connectivity settings (SonoSite SII) | 3 |
| Connectivity setup (SonoSite Edge II) | 3 |
| Network Status settings | 3 |
| Imaging (SonoSite SII) | 3 |
| Measurements and Calculations (SonoSite SII) | 4 |
| General calculations | 4 |
| Measurement References (SonoSite SII) | 5 |
| Measurement accuracy | 5 |
| Measurement publications and terminology | 5 |
| Cleaning and Disinfecting | 6 |
| Safety | 6 |
| Clinical safety | 6 |
| Compatible accessories and peripherals | 7 |
| Supported transducers (SonoSite SII) | 7 |
| Glossary (SonoSite Edge II) | 8 |

Introduction

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the user guide.

Getting Help

For technical support, please contact FUJIFILM SonoSite as follows:

| | |
|---|--|
| Phone (U.S. or Canada) | 877-657-8118 |
| Phone (outside U.S. or Canada) | 425-951-1330, or call your local representative |
| Fax | 425-951-6700 |
| Email | ffss-service@fujifilm.com |
| Web | www.sonosite.com |
| Europe Service Center | Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451 |
| Asia Service Center | +65 6380-5581 |

Printed in the U.S.

Getting Help (SonoSite Edge II)

The following has been corrected in the SonoSite Edge II system user guide; the revision will be made in the next update.

| | |
|--------------|---------------------------|
| Email | ffss-service@fujifilm.com |
|--------------|---------------------------|

Getting Started

The following content was either not included or incorrect in the SonoSite Edge II and SonoSite SII system user guides; the revisions will be made in the next update.

Intended uses

Prostate imaging applications

You can assess the prostate and surrounding anatomical structures for the presence or absence of pathology.

Superficial imaging applications

You can assess the breast, thyroid, testicle, lymph nodes, hernias, musculoskeletal structures, soft tissue structures, spine, ophthalmic structures, and surrounding anatomical structures for the presence or absence of pathology. You can use the system to provide ultrasound guidance for biopsy and drainage procedures, vascular line placement, and peripheral nerve blocks.

System Setup

Connectivity settings (SonoSite SII)

All references to PDAS should be changed to SiteLink in the SonoSite SII system user guide; the revision will be made in the next update.

Connectivity setup (SonoSite Edge II)

The following reference has been updated in the SonoSite Edge II system user guide; the revision will be made in the next update.

To enable wireless connection

❖ See *Setting Up a Network Connection*.

Network Status settings

If your Network Status screen displays a failed wireless device message, your network password may be expired. Make sure that you have an updated network password before connecting your wireless device.

Imaging (SonoSite SII)

The following footnotes were missing from **Table 4-5. Imaging modes and exams available by transducer** in the SonoSite SII system user guide; the revision will be made in the next update.

^aExam type abbreviations are as follows: Abd = Abdomen, Art = Arterial, Bre = Breast, Crd = Cardiac, Gyn = Gynecology, Msk = Musculoskeletal, Neo = Neonatal, Nrv = Nerve, OB = Obstetrical, Oph = Ophthalmic, Pro = Prostate, SmP = Small Parts, Spn = Spine, Sup = Superficial, Ven = Venous.

^bThe optimization settings for 2D are Res, Gen, and Pen.

^cThe optimization settings for CPD and Color are low, medium, and high (flow sensitivity) with a range of PRF settings for Color depending on the setting selected.

^dFor more information refer to the P11x Transducer User Guide, included with the P11x transducer. The P11x transducer is not licensed for use in Canada.

Measurements and Calculations (SonoSite SII)

General calculations

Volume calculation

- WARNING**
- ▶ To avoid incorrect calculations, verify that the patient information, date, and time settings are accurate.
 - ▶ To avoid misdiagnosis or harming the patient outcome, start a new patient form before starting a new patient exam and performing calculations. Starting a new patient form clears the previous patient's data. The previous patient's data will be combined with the current patient if the form is not first cleared.

The volume calculation involves three 2D distance measurements: D1, D2, and D3. After all measurements are saved, the result appears on-screen and in the patient report.

The volume calculation is available in the following exam types: Abdomen, Arterial, Breast, Gynecological, Musculoskeletal, Nerve, Small Parts, Vascular, and Superficial.

To calculate volume

Do the following for each image you need to measure:

- 1 On a frozen 2D image, tap **Calcs**.
- 2 Do the following for each measurement you need to take:
 - a From the calculations menu, under **Volume**, select the measurement name.
If **Volume** is not available in a Gyn exam, select **Gyn** and then select **Volume**.
 - b Using the touchpad or the touchscreen, position the calipers
 - c Tap **Save Calc** to save the calculation.
A check mark appears next to the saved measurement.
- 3 To save a picture of the finished calculation, tap .
- 4 Tap **Back** to exit the calculation.

Measurement References (SonoSite SII)

The following information was not included in the SonoSite SII system user guide; the revision will be made in the next update.

Measurement accuracy

Table 1: M Mode Measurement and Calculation Accuracy and Range

| M Mode Measurement Accuracy and Range | System Tolerance | Accuracy By | Test Method | Range |
|---------------------------------------|---|-------------|----------------------|-------------|
| Distance | < +/- 2% plus 1% of full scale ^a | Acquisition | Phantom ^b | 0–26 cm |
| Time | < +/- 2% plus 1% of full scale ^c | Acquisition | Phantom ^d | 0.01–10 sec |
| Heart Rate | < +/- 2% plus (Full Scale ^c * Heart Rate/100)% | Acquisition | Phantom ^d | 5–923 bpm |

^aFull scale for distance implies the maximum depth of the image.

^bAn RMI 413a model phantom with 0.7 dB/cm MHz attenuation was used.

^cFull scale for time implies the total time displayed on the scrolling graphic image.

^dFUJIFILM SonoSite special test equipment was used.

Measurement publications and terminology

General references

Hip Angle/d:D Ratio

Morin, C., Harcke, H., MacEwen, G. "The Infant Hip: Real-Time US Assessment of Acetabular Development." *Radiology* 177: p.673-677, December 1985.

Percent Area Reduction

Zwiebel W.J., J.A. Zagzebski, A.B. Crummy, et al. "Correlation of peak Doppler frequency with lumen narrowing in carotid stenosis." *Stroke*, 3: (1982), p.386-391.

$$\% \text{ Area Reduction} = [1 - A2(\text{cm}^2)/A1(\text{cm}^2)] * 100$$

where: A1 = original area of the vessel in square cm

A2 = reduced area of the vessel in square cm

Percent Diameter Reduction

Handa, Nobuo et al., "Echo-Doppler Velocimeter in the Diagnosis of Hypertensive Patients: The Renal Artery Doppler Technique," *Ultrasound in Medicine and Biology*, 12:12 (1986), p.945-952.

$$\% \text{ Diameter Reduction} = [1 - D2(\text{cm})/D1(\text{cm})] * 100$$

where: D1 = original diameter of the vessel in cm

D2 = reduced diameter of the vessel in cm

Cleaning and Disinfecting

The following website was corrected in the SonoSite Edge II and SonoSite SII user guides; the revision will be made in the next update.

www.sonosite.com/products/transducers

The following footnote was updated in the cleaning and disinfecting tables.

Refer to the cleaners and disinfection tool available at www.sonosite.com/support/cleaners-disinfectants for a more complete list of approved cleaners and disinfectants.

Safety

Clinical safety

The following warning has been updated in the SonoSite Edge II and SonoSite SII user guides; the revision will be made in the next update.

WARNING

FUJIFILM SonoSite does not recommend the use of high-frequency electromedical devices in proximity to its systems. FUJIFILM SonoSite equipment has not been validated for use with high-frequency electrosurgical devices or procedures. Use of high-frequency electrosurgical devices in proximity to its systems may lead to abnormal system behavior or shutdown of the system.

To avoid the risk of a burn hazard, do not use the transducer with high-frequency surgical equipment. Such a hazard may occur in the event of a defect in the high-frequency surgical neutral electrode connection.

Compatible accessories and peripherals

The following warning was added to the SonoSite Edge II and SonoSite SII user guides; the revision will be made in the next update.

WARNING

If peripherals are connected to the system, ensure that the system and peripherals are connected to the same AC Mains branch circuit.

Supported transducers (SonoSite SII)

The following redundant section was removed from the SonoSite SII user guide. The same information is present in Table 9-2 of the user guide; the revision will be made in the next update.

Table 2: Supported transducers

| Description | Maximum Cable Length |
|-------------------------------------|----------------------|
| C11x transducer | 6.0 ft/1.8 m |
| rC60xi transducer standard/armored | 5.5 ft/1.7 m |
| HFL38xi transducer standard/armored | 5.5 ft/1.7 m |
| HFL50x transducer | 5.5 ft/1.7 m |
| HSL25x transducer | 8.0 ft/2.4 m |
| ICTx transducer | 5.5 ft/1.7 m |
| L25x transducer | 7.5 ft/2.3 m |
| L38xi transducer standard/armored | 5.5 ft/1.7 m |
| L52x transducer | 7.9 ft/2.4 m |
| P10x transducer | 6.0 ft/1.8 m |
| P11x transducer | 6.5 ft/2.0 m |
| rP19x transducer standard/armored | 6.0 ft/1.8 m |

For transducers, the maximum cable length is measured between the strain reliefs. The stated length does not include the lengths of cable in the following locations: underneath the strain reliefs, inside the transducer enclosure, and inside the transducer connector.

Glossary (SonoSite Edge II)

The IMT term has been removed from the SonoSite Edge II user guide abbreviation list; the revision will be made in the next update.

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