


SonoSite Edge II

SonoSite SII


Footswitch Assembly Instructions

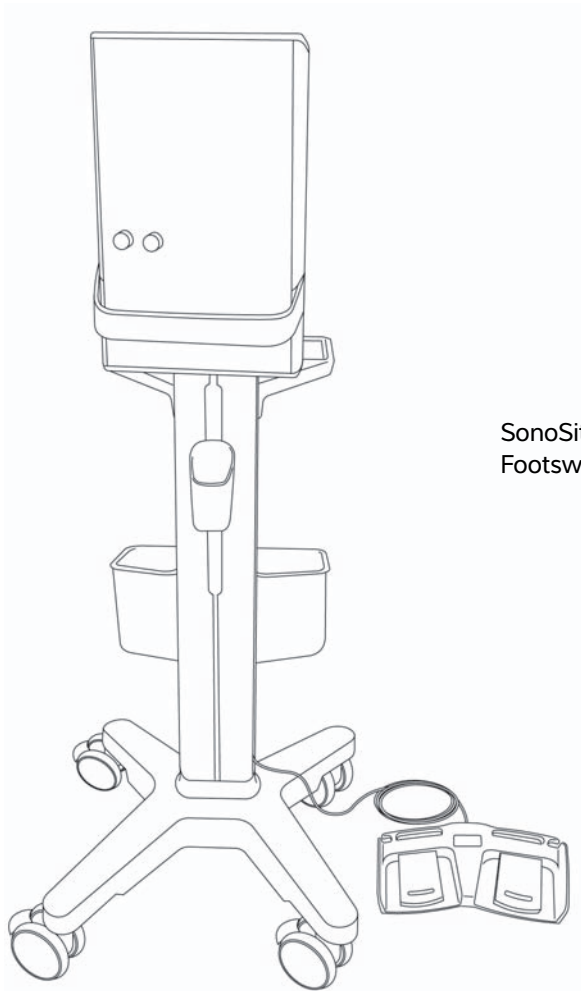


SonoSite SII system with
Footswitch installed.

Unpacking the box

Footswitch (1)

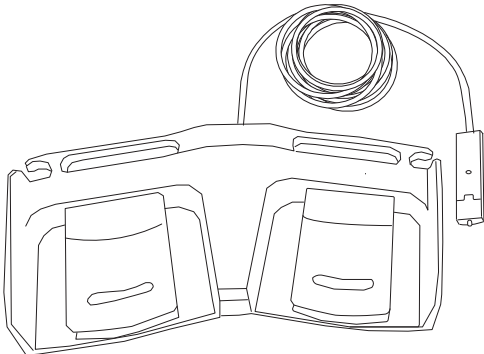

6 ft USB extension cord (1)*

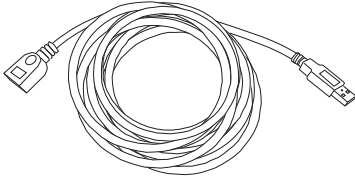


*Available for wall-mounted SonoSite SII systems

Installing the footswitch on the SonoSite Edge II

STEP 1


Plug the footswitch into the USB port located on the side of the system or the back of the mini-dock.

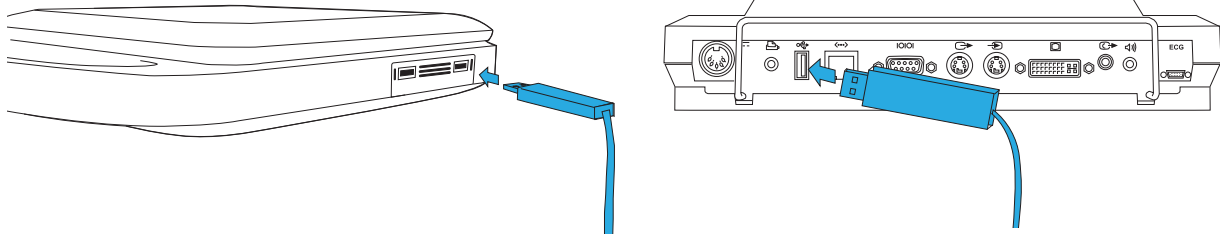

Installing the footswitch on the SonoSite SII


STEP 1

Plug the footswitch or USB extension cord (if connecting to a wall-mounted system) into one of the USB ports located on the back of the system.

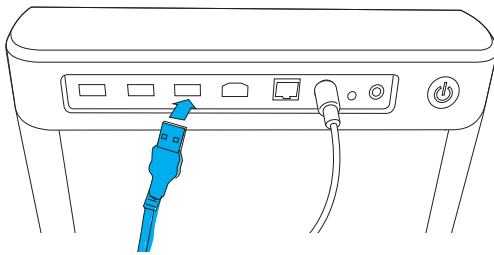

STEP 2

For wall-mounted systems, run the USB extension cord through any arm cable management system. Then attach the cord to the footswitch cable.

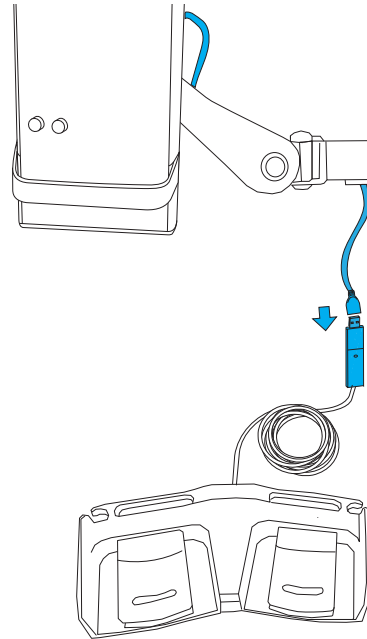


WARNING

Do not place the connection between the footswitch and the USB extension cord on the floor or in areas where liquids are present.

STEP 3

Secure the cable to prevent unplugging the footswitch.

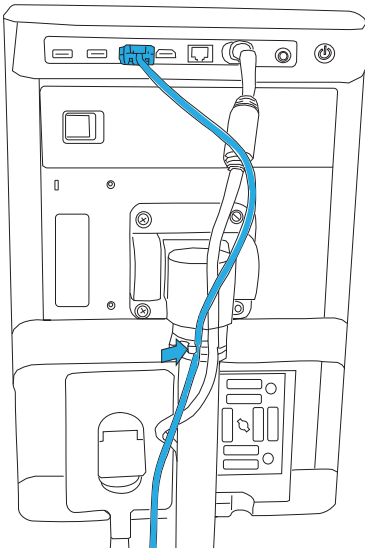

Programming the footswitch

- ❖ To program the footswitch, select a function for the left and right footswitches on the **Footswitch** setup page. See the system user guide for more details.

Cleaning and disinfecting

For cleaning and disinfecting instructions, see the system user guide.

Caution | Federal (United States) law restricts this device to sale by or on the order of a physician.

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite
Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia



SonoSite Edge, SonoSite SII, SonoSite, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Published May 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

P05997-02

