



# Edge Stand: Latch Upgrade Installation Instructions

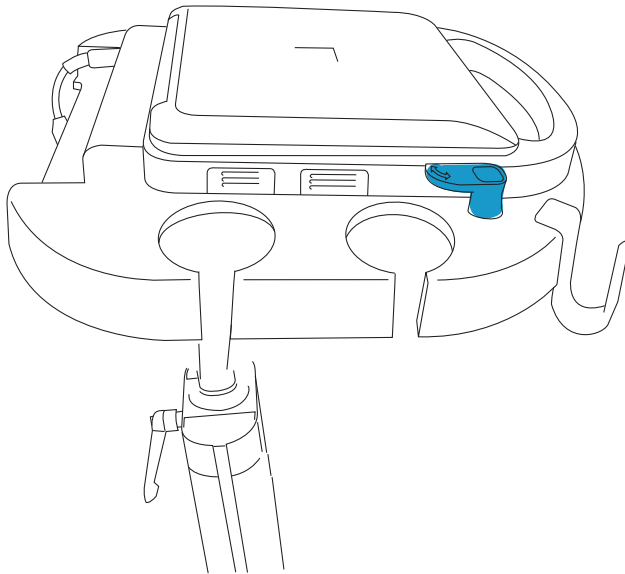


# Unpacking the box


Stand latch and cap (left)

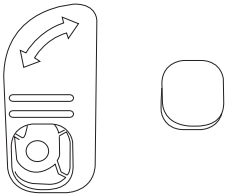

Stand latch and cap (right)

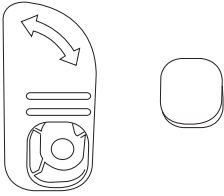

3/8" Allen-head screw, extras (2)

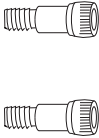

1/8" Allen wrench (1)




Curved washer, extras (2)



# Preparing the stand

## STEP 1


Press down the lever on two or more wheels.

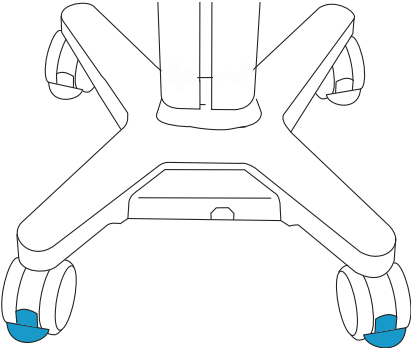

# Replacing the latches


## STEP 2

Use a sharp-ended tool to remove the caps hiding the screws.

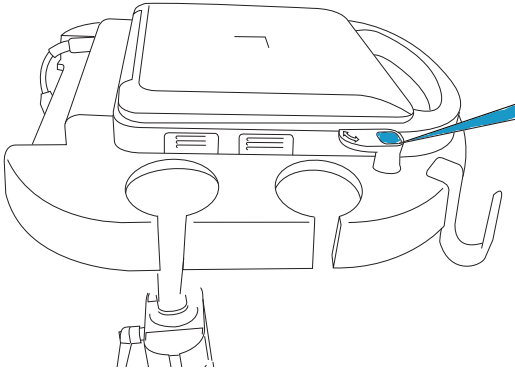


## STEP 3

Use the Allen wrench to loosen the screws, and remove the old latches.

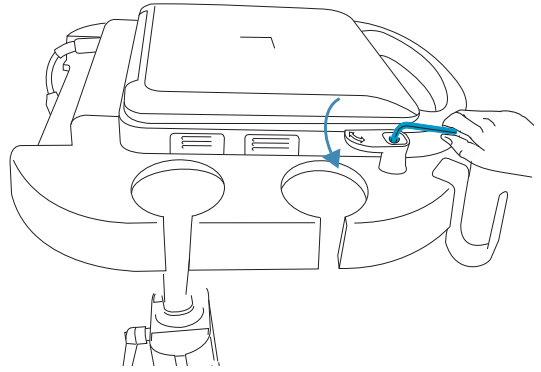


## STEP 4

Place each screw through a curved washer and into the new latches.

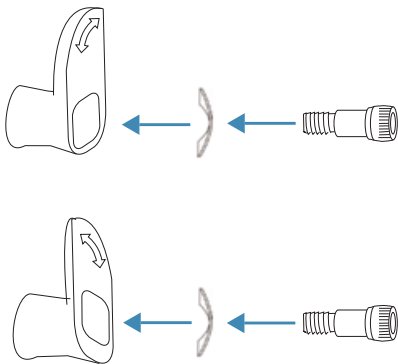


## STEP 5

Install the new latches onto the system platform as shown.

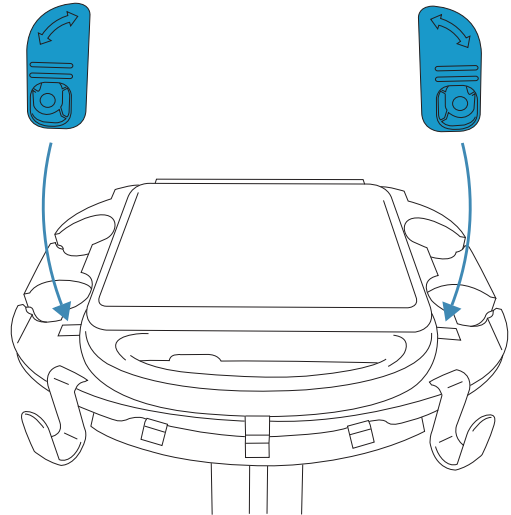


## STEP 6

Use the Allen wrench to tighten the screws

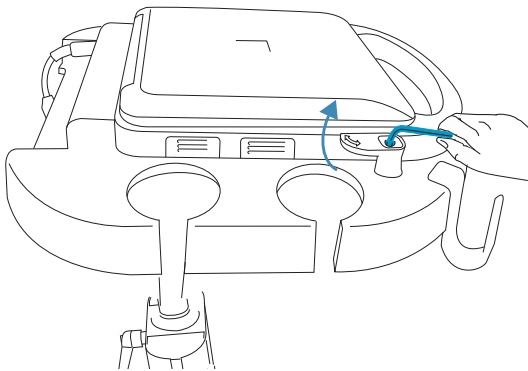

## STEP 7

Place the caps over the screws, and press down.

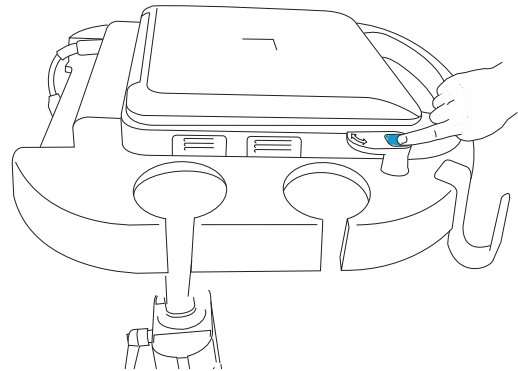

### Caution:

Federal (United States) law restricts this device to sale by or on the order of a physician.

Edge, SonoSite, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Patents: US 8,088,071; US 7,591,786; US 7,534,211; US 6,447,451; CA 2372158; and EP 1589878.

Copyright © 2015 FUJIFILM SonoSite, Inc. All rights reserved.

#### **Manufacturer**

FUJIFILM SonoSite, Inc.  
21919 30th Drive SE  
Bothell, WA 98021 USA  
T: 1-888-482-9449 or 1-425-951-1200  
F: 1-425-951-1201

#### **EC Authorized Representative**

Emergo Europe  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

#### **Australia Sponsor**

FUJIFILM SonoSite Australasia Pty Ltd  
114 Old Pittwater Road  
BROOKVALE, NSW, 2100  
Australia

11/2015



P21954-01

