


Edge Stand H-Universal Stand Assembly Instructions

FUJIFILM
Value from Innovation


SonoSite

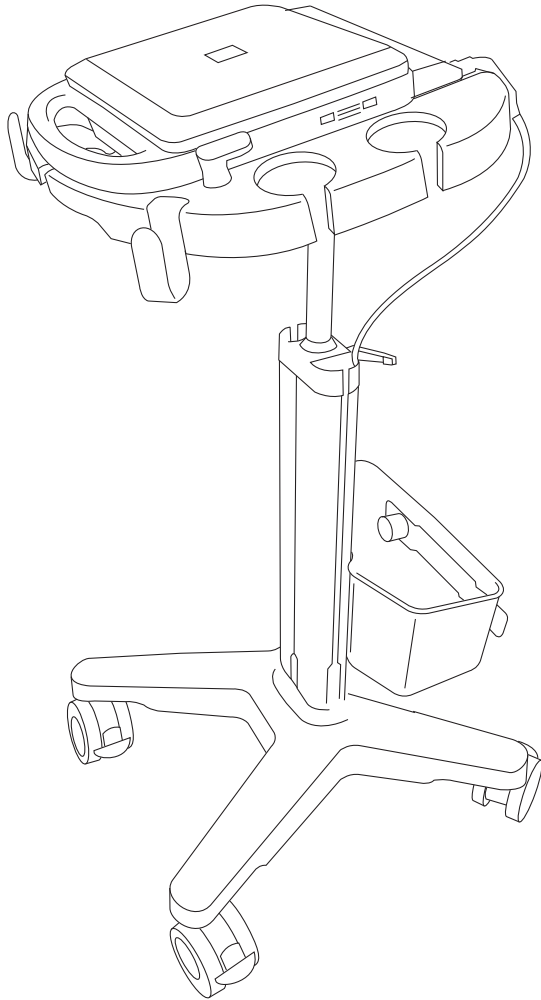


*Assembled Edge Stand
with storage bin.*

Unpacking the box

Column (1)

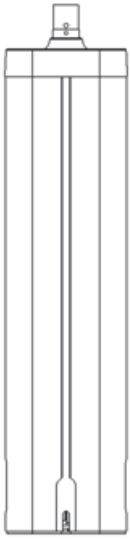


Base (1)

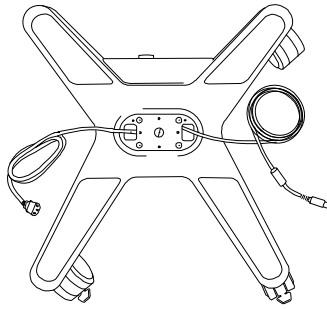


Magnetic bit driver (1)



5/32" tamper-resistant Allen wrench (1)

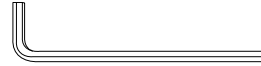


System platform (1)

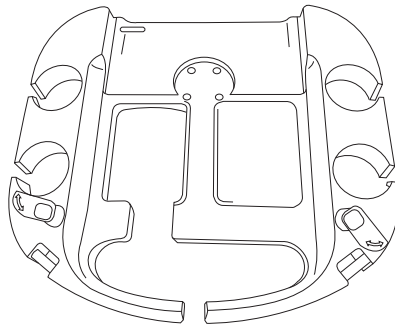


Allen bit (1)



Cord catch
preinstalled (4)

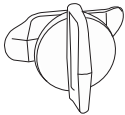


Storage bin holder and bracket
preinstalled (1)

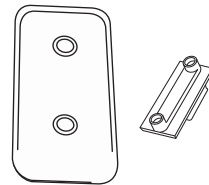

Storage cup (4)



Storage bin (1)*

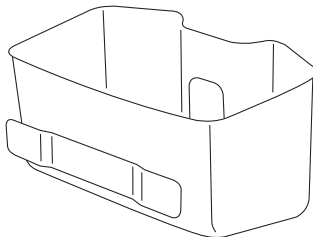

Stand hook (2)



1/2" tamper-resistant screw (1)



12mm Allen-head screw (4)




* These items packaged separately from the stand. An optional second storage bin is available for purchase.

Preparing the stand

STEP 1


Press down the lever on two or more wheels.

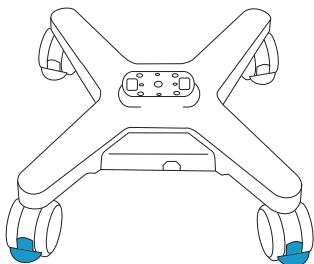

Attaching the column to the base


STEP 2

Place the column in the two pegs, and slide the cords in the grooves of the column.

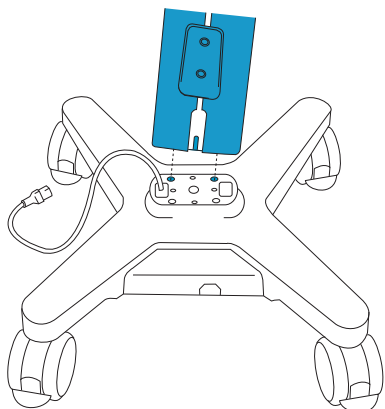


STEP 3

Insert the Allen bit into the bit driver.

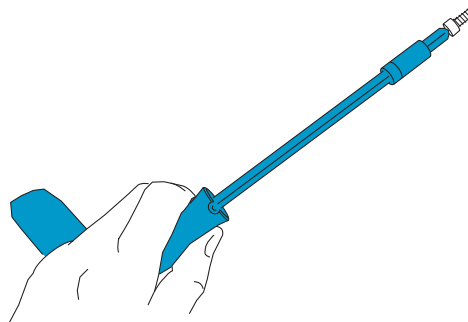


STEP 4

Place an Allen-head screw on the magnetic bit driver, and tighten the screw on the back of the column.

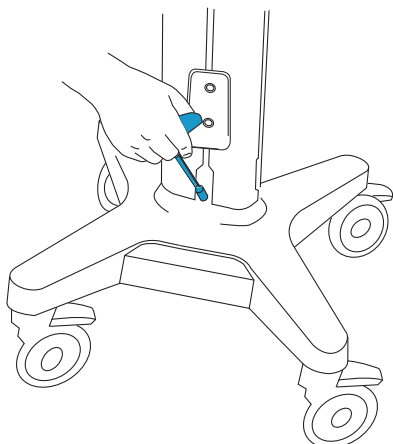

STEP 5


Repeat Step 4 with the other three Allen-head screws on the front and sides of the column.

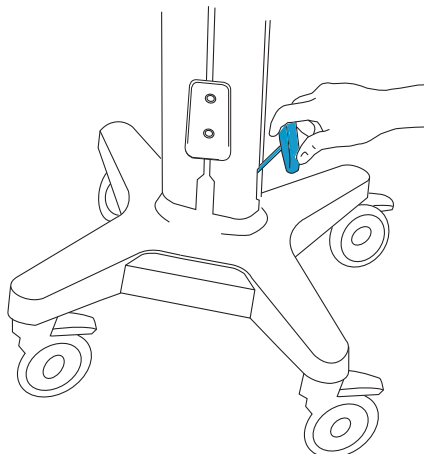

Securing the cables


STEP 6

Turn cord catches counter-clockwise, and pull them away from column sides.

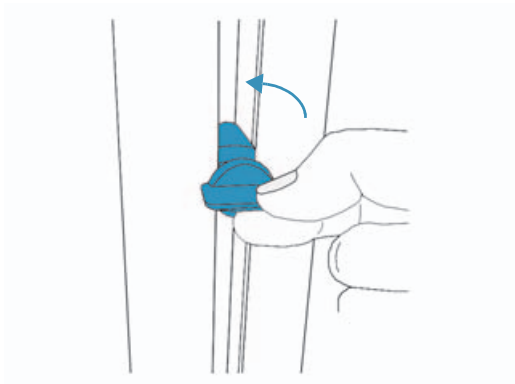


STEP 7

Place cables into the groove on either side of the column.

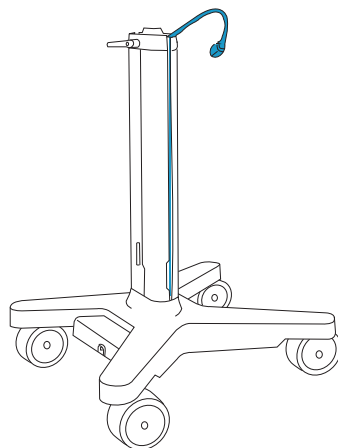

STEP 8


Attach the cord catches, and turn clockwise to tighten.

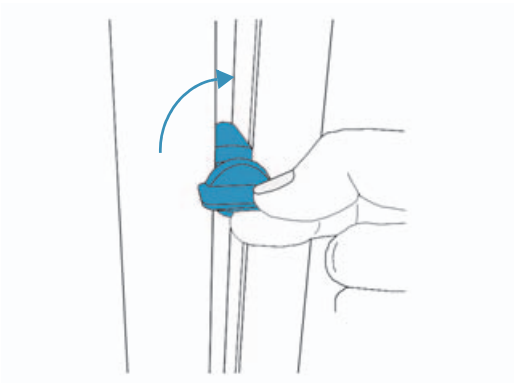

Attaching the system platform


STEP 9

Align the peg on the column pole with the notch on the cylinder of the system platform.

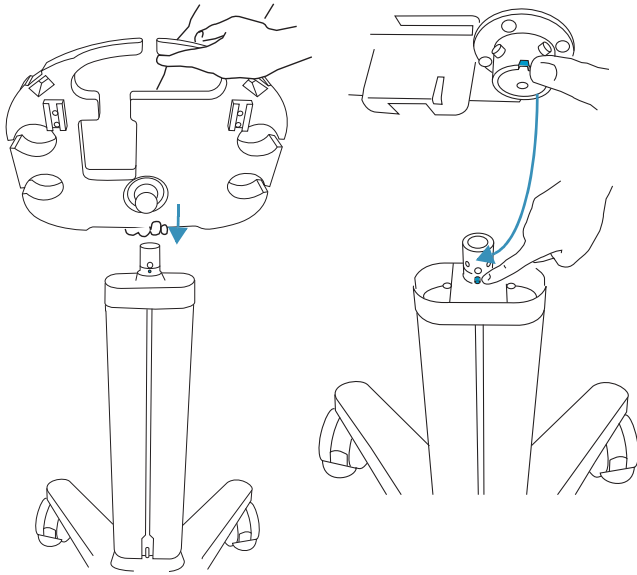


STEP 10

Use the Allen wrench to tighten the 1/2" screw into the back of the cylinder.

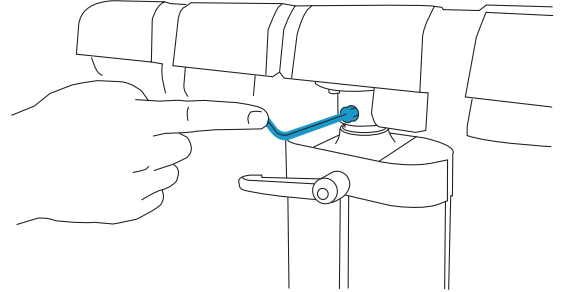


STEP 11

Use the Allen wrench to tighten the two preinstalled set screws.

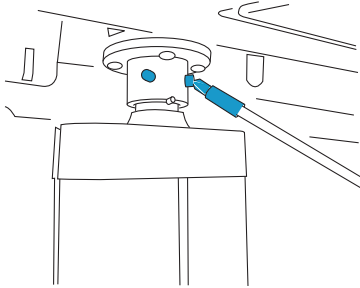


STEP 12

Insert the four cups and press into place.

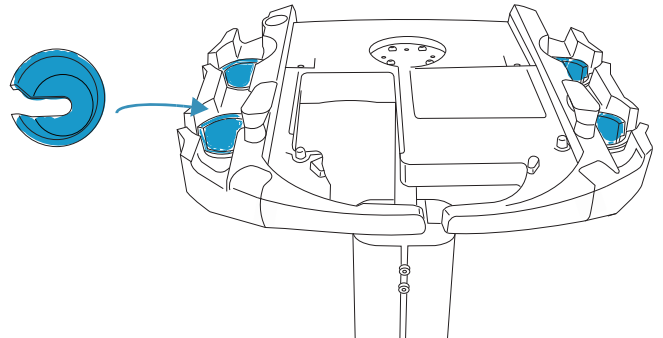

STEP 13


Insert the small end of a hook into the slot at the front of the system platform, and press into place. Repeat for other side.

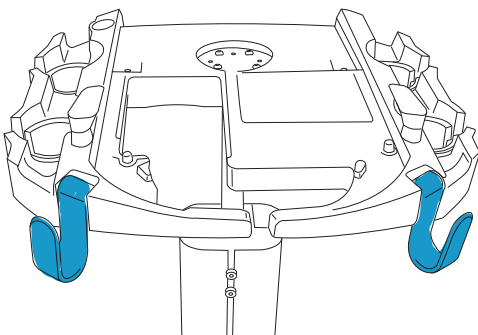

Inserting the system and attaching the mini-dock and storage bin


STEP 14

Lower the system into the system platform, and make sure the transducer cable is positioned between the center of the stand.

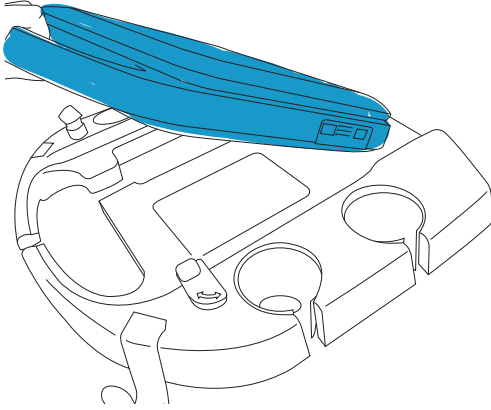


STEP 15

Attach mini-dock.

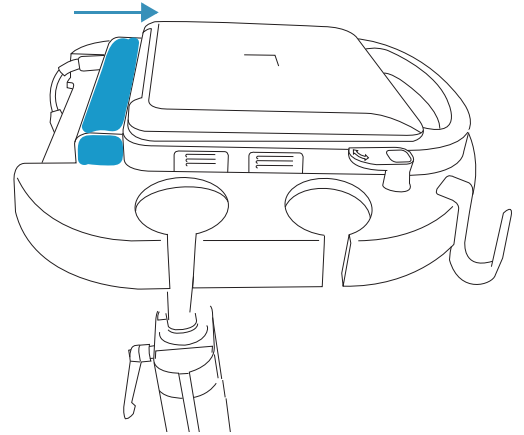


STEP 16

Rotate the latches to secure the system.

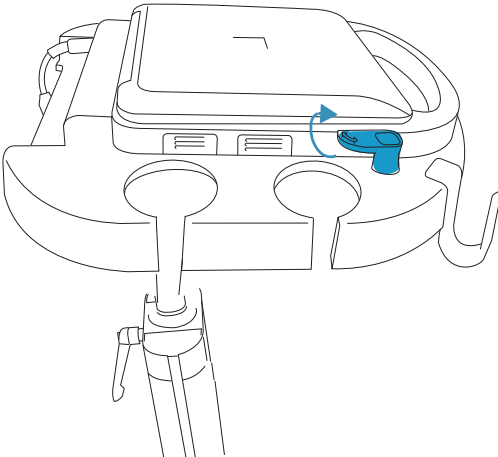

STEP 17


Attach the storage bin to the pre-installed mounting bracket.

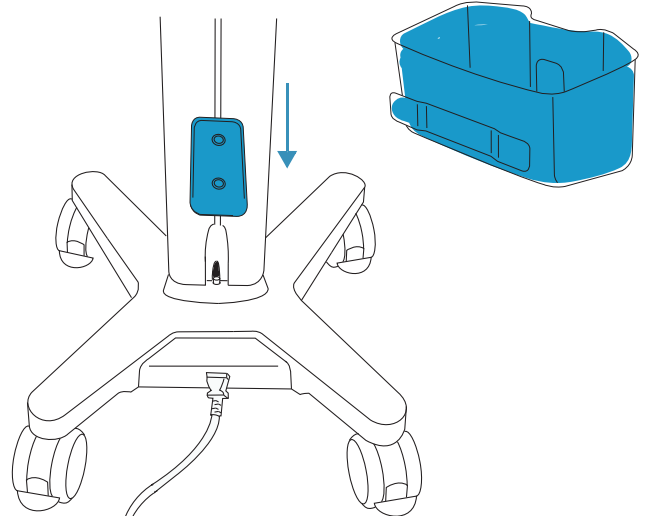

Connect the system


STEP 18

Connect the DC power supply cord.

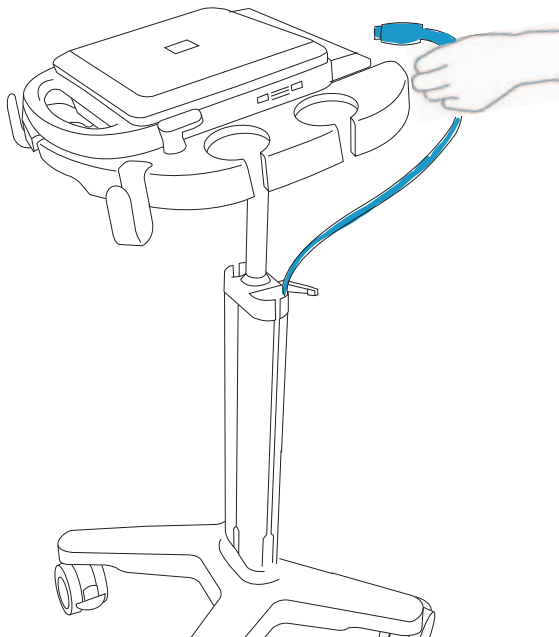

STEP 19

Connect the system AC power cord to a hospital-grade electrical outlet.

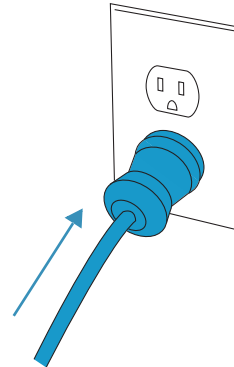

Connectivity

WARNING: To avoid the risk of a shock hazard, do not plug the power cord of the Edge Stand or H-Universal Stand into a multiple portable socket outlet (MPSO) or into an extension cord.

Caution: To avoid damaging the system, use only peripherals recommended by FUJIFILM SonoSite with the ultrasound system.

Cables and connectors

Table 1: Connectivity symbols on mini-docks












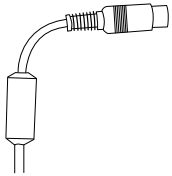
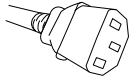

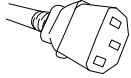
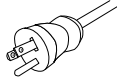
Symbol	Definition	Symbol	Definition
	DC input		S-video in
	Print control		DVI video out
	USB		Composite video out
	Ethernet		Audio out
	RS-232 (DVD recorder, VCR, or bar code scanner)		ECG/Footswitch
	S-video out		

Table 2: Cables and connectors

Cable [Letter] [Name] [Length] [Illustration]	Connector 1 [Illustration] [Connects to]	Connector 2 [Illustration] [Connects to]
A DC power supply cord 6.8 ft./2 m n/a	 System or mini-dock	Pre-installed
J AC power cord 39 in./1 m n/a	 Printer	Pre-installed
K System AC power cord 10 ft./3 m 	 AC adapter 1 on stand base	 Wall outlet

Caution: Federal (United States) law restricts this device to sale by or on the order of a physician.

Edge, SonoSite, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Patents: US 8,088,071; US 7,591,786; US 7,534,211; US 6,447,451; CA 2372158; and EP 1589878.

Copyright © 2015 FUJIFILM SonoSite, Inc. All rights reserved.

Manufacturer

FUJIFILM SonoSite, Inc.
 21919 30th Drive SE
 Bothell, WA 98021 USA
 T: 1-888-482-9449 or 1-425-951-1200
 F: 1-425-951-1201

EC Authorized Representative

Emergo Europe
 Molenstraat 15
 2513 BH, The Hague
 The Netherlands

Australia Sponsor

FUJIFILM SonoSite
 Australasia Pty Ltd
 114 Old Pittwater Road
 BROOKVALE, NSW, 2100
 Australia

11/2015



P22131-01

