SonoSite Mini-Dock

User Guide

FUJIFILM
Value from Innovation

SonoSite

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

Sonosite, the Sonosite logo, M-Turbo, Edge, MicroMaxx, and TITAN are registered and unregistered trademarks of FUJIFILM Sonosite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Patents: 8,088,071; 7,849,250; 7,591,786; 7,534,211; DE 602004023816; FR 1589878 and GB 1589878.

Part Number: P05403-05

Publication Date: September 2015

Copyright © 2015 FUJIFILM SonoSite, Inc. All Rights reserved.

SonoSite Mini-Dock

User Guide

Introduction	1
Document conventions	1
Getting help	1
Attaching and removing the mini-dock	2
Mini-dock connectors	3
Cleaning and Disinfecting	4

Introduction

The SonoSite mini-dock provides connections for accessories and peripherals on any of the following ultrasound systems:

- ▶ SonoSite Edge II ultrasound system
- ▶ Edge ultrasound system
- ▶ M-Turbo ultrasound system
- ▶ MicroMaxx ultrasound system
- ▶ *TITAN* ultrasound system

Note

Each ultrasound system uses a different model mini-dock.

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the user guide.

Getting help

In addition to this document, contact FUJIFILM SonoSite Technical Support:

Phone
(U.S. or Canada)

877-657-8118

Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	service@sonosite.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

Attaching and removing the mini-dock

WARNING

To avoid the risk of electrical shock to the patient, do not touch the ungrounded connectors on the back of the mini-dock while simultaneously touching a patient

Cautions

- ▶ To avoid damaging the mini-dock, do not perform high current grounding impedance testing involving this part. Accessible metal of the mini-dock is not protectively earthed.
- ▶ To avoid damaging equipment, remove mini-dock from system prior to transporting.

To attach mini-dock

- 1 Ensure that the ultrasound system is turned off.

Connecting the system and mini-dock while the power is on may cause the mini-dock to function improperly.


- Slide the mini-dock feet under the ultrasound system, and firmly press the mini-dock into the system connector. The mini-dock is correctly attached when no gap is between the system and mini-dock.

Note

If at any time the ultrasound system loses connectivity, turn off the system and reattach the mini-dock and system. Always verify that the mini-dock and system are correctly attached after changing a transducer. When you change a transducer, the mini-dock may loosen.

To remove mini-dock

- Grasp the handle on the ultrasound system and push back on the release lever.
- Pull the ultrasound system forward.

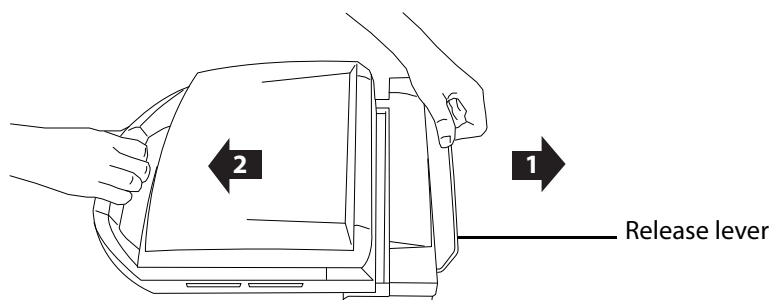Mini-dock connectors

Each connector on the mini-dock has a symbol that describes its use.

Table 1-1: Connectivity symbols on mini-dock

Symbol	Definition	Symbol	Definition
	DC input		S-video in
	Print control		DVI video out
	USB		Composite video out
	Ethernet		Audio out
	RS-232 (DVD recorder, VCR, or bar code scanner)		ECG/Footswitch
	S-video out		

Cleaning and Disinfecting

Caution

Use only recommended cleaners or disinfectants on surfaces. Immersion-type disinfectants are not approved for use on surfaces.

Clean and disinfect

The exterior surface of the mini-dock can be cleaned and disinfected with a recommended cleaner or disinfectant. For recommended disinfectants, see [Table 1-2](#).

- 1 Remove the system, unplug the power supply, and detach any cables.
- 2 Clean the surfaces using a soft cloth lightly dampened in a mild soap or detergent cleaning solution.
Apply the solution to the cloth rather than the surface.
- 3 Wipe the surfaces with the disinfecting solution.
- 4 Air dry or towel dry with a clean cloth.

Compatible cleaners and disinfectants

The following cleaners and disinfectants are recommended for use on the mini-dock.

Table 1-2: Cleaners and disinfectants compatible with the SonoSite mini-dock

Disinfectant	Compatibility	Disinfectant	Compatibility
Sanicloth AF3	✓	Aniosyme DD1- 5%	✓
Accel TB Wipes	✓	Anioxide 1000	✓
Accel Intervention Wipes	✓	Airkem A-33	✓
Alcohol - Isopropyl	✓	Asepti HB	✓
Alcohol-Ethanol	✓	Asepti Steryl	✓
Anios Wipes	✓	Asepti Wipe II	✓

Table 1-2: Cleaners and disinfectants compatible with the SonoSite mini-dock (continued)

Disinfectant	Compatibility	Disinfectant	Compatibility
Baxedin Pre-op	✓	Echo Clean Lingettes	✓
Caviwipes	✓	End Bac II	✓
Cidalkan	✓	Endosporine	✓
Cidalkan Lingettes	✓	Envirocide	✓
Cidex	✓	Hydrogen Peroxide 3%	✓
Cidex OPA	✓	Incidin Plus 1%	✓
Cidex Plus	✓	Incidin Plus 3%	✓
Clinell Universal Sanitising Wipes	✓	MadaCide-1	✓
Clorox Hydrogen Peroxide Wipes	✓	Medzyme	✓
Control III	✓	Metricide OPA Plus	✓
Coverage Spray	✓	Mikrobac Forte 2%	✓
Coverage Germicidal Surface Wipes	✓	Mikrobac Tissues	✓
Coverage Plus Germicidal Surface Wipes	✓	NU - Cidex	✓
Coverage Spray TB	✓	Rely+on PeraSafe	✓
DisCide Ultra Disinfecting Towlettes	✓	Percept RTU wipes	✓
Dynacide PA	✓	Protex Spray	✓

Table 1-2: Cleaners and disinfectants compatible with the SonoSite mini-dock (continued)

Disinfectant	Compatibility	Disinfectant	Compatibility
Protex Wipes	✓	Trigene Advance Wipes	✓
Revital-Ox Resert HLD	✓	Tristel Wipes	✓
Sanicloth AF	✓	Tristel Solo	✓
Sanicloth HB	✓	T-Spray II	✓
Sanicloth Plus	✓	Virox 5 RTU Wipes	✓
Simple Chlor	✓	Virex II 256	✓
Sporicidin	✓	Virex TB	✓
Staphene	✓	Wet Task Wipes	✓
Steranios 2%	✓	Wex-Cide	✓
Steranios 20%	✓	Wood Wyant Ultra Wipes	✓
Super Sanicloth	✓	Wipes Plus Disinfecting Wipes	✓

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Désinfectant	Compatibilité	Désinfectant	Compatibilité
Clorox Hydrogen Peroxide Wipes	✓	Medzyme	✓
Control III	✓	Metricide OPA Plus	✓
Coverage Spray	✓	Mikrobac Forte 2 %	✓
Coverage Germicidal Surface Wipes	✓	Mikrobac Tissues	✓
Coverage Plus Germicidal Surface Wipes	✓	Nu - Cidex	✓
Coverage Spray TB	✓	Rely+on PeraSafe	✓
DisCide Ultra Disinfecting Towlettes	✓	Percept RTU wipes	✓
Dynacide PA	✓	Protex Spray	✓
Protex Wipes	✓	Trigene Advance Wipes	✓
Revital-Ox Resert HLD	✓	Tristel Wipes	✓
Sanicloth AF	✓	Tristel Solo	✓
Sanicloth HB	✓	T-Spray II	✓
Sanicloth Plus	✓	Virox 5 RTU Wipes	✓
Simple Chlor	✓	Virex II 256	✓
Sporicidin	✓	Virex TB	✓
Staphene	✓	Wet Task Wipes	✓

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