---

# PowerPack

*User Guide*

CE

**PowerPack User Guide**

---

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---

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T: +44-1462-444800

F: +44-1462-444801

**Caution:** | United States federal law restricts this device to sale by or on the order of a physician.

H-Universal, M-Turbo, NanoMaxx, S Series, SonoSite, V-Universal, and the SonoSite logo are trademarks or registered trademarks of SonoSite, Inc.

Non-SonoSite product names may be trademarks or registered trademarks of their respective owners.

The SonoSite ultrasound system(s) referenced in this document may be covered by one or more of the following U.S. patents: 5722412, 5817024, 5893363, 6135961, 6203498, 6364839, 6371918, 6383139, 6416475, 6447451, 6471651, 6569101, 6648826, 6575908, 6604630, 6817982, 6835177, 6962566, 7169108, 7449640, 7534211, 7549961, 7588541, 7591786, 7604596, 7643040, 7686766, D456509, D461895, D509900, D538432, D544962, D558351, D559390, D591423, D592750, D592760, and by the following counterpart foreign patents: AU727381, AU730822, CA2372158, CA2373065, CN ZL 97113678.5, CN ZL 98106133.8, CN ZL 98108973.9, CN ZL 200830007734.8, DE60021552.0, DE60029777.2, DE60034670.6, DE69730563.5, DE6980539.6, DE69831698.3, DE60 2004 23 816.3-08, FR0815793, FR0875203, FR0881492, FR1175713, FR1180970, FR1589878, GB0875203, GB0881492, GB1175713, GB1180970, GB1180971, GB1589878, IT0815793, IT0881492, IT1175713, IT1589878, KR528102, KR532359, NO326202, NO326814, NZ542968, RCD000897368-0001, SP0815793, SP0881492, SP1589878. Patents pending.

P13536-02 08/2010

Copyright 2010 by SonoSite, Inc.

All rights reserved.

# PowerPack User Guide

## Contents

Introduction .....	1
About the PowerPack .....	2
Charging the PowerPack battery .....	2
Troubleshooting .....	3
Cleaning and disinfecting .....	4
Safety .....	4
Specifications .....	6

## Introduction

The *PowerPack User Guide* contains operating, regulatory, and safety information. To attach or remove the PowerPack or its battery pack, refer to the assembly instructions.

For information on the stand or ultrasound system, see its user guide.

**WARNING:** To avoid the risk of patient injury, misdiagnosis, or user injury, read all the warnings in the ultrasound system user guide and supplemental guides.

## Conventions

The user guide follows these conventions:

- A **WARNING** describes precautions necessary to prevent injury or loss of life.
- A **Caution** describes precautions necessary to protect the products.
- Numbered steps in procedures must be performed in order.
- Items in bulleted lists do not require performance in sequence.
- Single-step procedures begin with ❖.

Symbols and terms used on the system are defined in the ultrasound system user guide.

## Customer assistance and comments

Questions and comments are encouraged. SonoSite is interested in your feedback regarding the system and the user guide. Please call SonoSite at 1-888-482-9449 in the US. Outside the US, call the nearest SonoSite representative. You can also e-mail SonoSite at [comments@sonosite.com](mailto:comments@sonosite.com).

For technical support, please contact SonoSite as follows:

### SonoSite Technical Support

Phone (US or Canada):	1-877-657-8118
Phone (Outside US and Canada):	1-425-951-1330 Or call your local representative.
Fax:	1-425-951-6700
E-mail:	<a href="mailto:service@sonosite.com">service@sonosite.com</a>
Web site:	<a href="http://www.sonosite.com">www.sonosite.com</a>

### Europe Service Center

Phone:	+44-(0)1462-444-800
E-mail:	<a href="mailto:uk.service@sonosite.com">uk.service@sonosite.com</a>

## About the PowerPack

**Caution:** To ensure proper operation of the PowerPack and ultrasound system, always have the system battery in the ultrasound system.

The PowerPack contains a battery pack secured within a housing that mounts on the V-Universal™ stand or H-Universal™ stand. The PowerPack extends the system battery running time up to 140% for any of the following ultrasound systems:

- M-Turbo® ultrasound system
- S-Series™ ultrasound system
- NanoMaxx™ ultrasound system

The PowerPack powers the system only. If you need to run AC-powered peripherals, connect the stand to AC power.

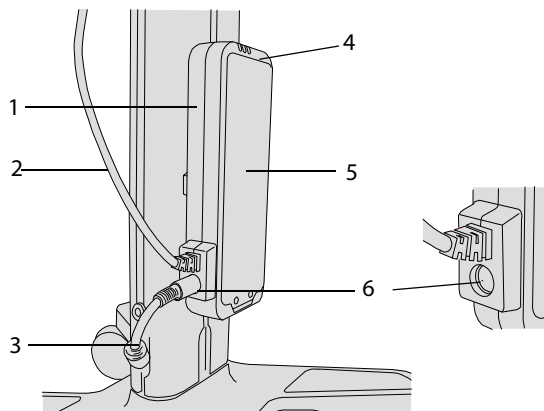
Figure 1 PowerPack mounted on stand:

- 1 Housing
- 2 PowerPack DC power cord
- 3 Power-supply DC power cord (from stand)
- 4 Indicators. See "Indicators" on page 3.
- 5 Battery pack
- 6 DC input port, connected (left) and disconnected (right)

## Charging the PowerPack battery

A fully discharged PowerPack battery charges in 12 hours if the ultrasound system is turned off. The system battery charges before the PowerPack battery charges.

The PowerPack powers the system as well as charges the system battery (if discharged). If the PowerPack battery becomes depleted, the system runs on the system battery.

If the stand is connected to AC power, the system runs on AC power instead of battery power.

### To charge the PowerPack battery

1 Make sure that the following are connected:

- Power supply DC power cord from the stand (A) to the PowerPack. (See "PowerPack cables and connectors" on page 3.)
- PowerPack DC power cord to the system directly, to the NanoMaxx dock, or to the mini-dock (depending on your system and configuration).

2 Connect to AC power: Connect the system AC power cord (K) to the stand and a wall outlet.

*Note: If using the PowerPark, you connect to AC power by docking the stand.*

The PowerPack battery (and system battery if discharged) charge regardless of whether the system is running.

## Indicators

The PowerPack indicators show available battery power. While the battery charges, an indicator flashes.

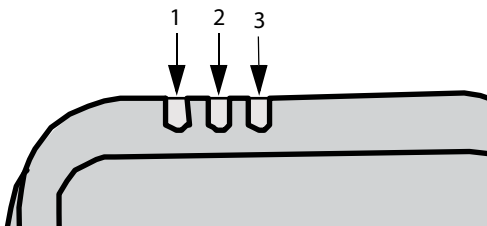
Figure 2 Indicators show available battery power:

- 1 Up to 1/3
- 2 1/3 to 2/3
- 3 At least 2/3

## Troubleshooting

### Symptoms and solutions

**PowerPack doesn't charge** If the PowerPack ceases to charge and its indicators are off, it may be in a frozen state. Disconnect and then reconnect the following cords:

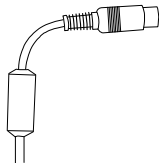
- PowerPack DC power cord (**Z**) from the system
- System AC power cord (**K**), if present, from the wall outlet

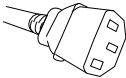
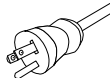
If the problem persists, contact SonoSite Technical Support. (See "[SonoSite Technical Support](#)" on page 1.)

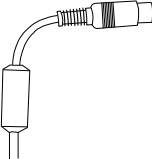
**Charging takes longer than expected** If charging takes a long time (for example, the charge indicator is always on), try disconnecting the PowerPack DC power cord (**Z**) from the system and running the system on the system battery only. If the fully charged system battery discharges quickly (for example, in a few minutes), it needs replacing. Contact SonoSite Technical Support.

## PowerPack cables and connectors

Cable [Letter] [Name] [Illustration]	Connector 1 [Illustration] [Connects to]	Connector 2 [Illustration] [Connects to]
---	--	--

<b>A</b> Power-supply DC power cord 6.8 ft./2 m 	 PowerPack DC input port	<i>Pre-installed on stand</i>
--	---	-------------------------------

<b>K</b> System AC power cord 10 ft./3 m 	 Stand	 Wall outlet
---	---	--

<b>Z</b> PowerPack DC power cord 	<i>Pre-installed on PowerPack</i>	 DC input on mini-dock, NanoMaxx dock, or system
--	-----------------------------------	---

## Cleaning and disinfecting

**WARNING:** To avoid electrical shock, before cleaning always disconnect the system from the power supply and remove it from the stand.

To avoid injury, always use protective eyewear and gloves when performing cleaning and disinfecting procedures.

To avoid infection, ensure that the solution expiration date has not passed.

To avoid infection, the level of disinfection required for a product is dictated by the type of tissue it contacts during use. Ensure that the solution strength and duration of contact are appropriate for the equipment. For information, see the disinfectant label instructions and the recommendations of the Association for Professionals in Infection Control and Epidemiology (APIC) and FDA.

**Caution:** Use only recommended cleaners or disinfectants on surfaces. Immersion-type disinfectants are not tested for use on surfaces.

To avoid damaging the battery, do not allow cleaning solution or disinfectant to come in contact with the battery terminals.

### To clean and disinfect the PowerPack

You can clean and disinfect the exterior surface of the PowerPack using a recommended cleaner or disinfectant. For the most recent list of approved products, see [www.sonosite.com](http://www.sonosite.com).

- 1 Unplug the power supply, and detach any cables.

- 2 Clean the surfaces using a soft cloth lightly dampened in a mild soap or detergent cleaning solution.

Apply the solution to the cloth rather than the surface.

- 3 Wipe the surfaces with the disinfecting solution. Follow the manufacturer's instructions for the disinfecting solution, including exposure time and temperature for effective disinfection.

- 4 Air dry or towel dry with a clean cloth.

## Safety

Observe the following warnings and cautions before using accessories and peripherals with the ultrasound system and transducers.

See also the battery safety information in the ultrasound system user guide.

### Electrical safety classification

Class II equipment	The PowerPack is classified as Class II equipment when connected to the AC power supply.
Internally powered equipment	The PowerPack is classified as internally powered equipment when not connected to the power supply (battery only).
Non AP/APG	The PowerPack is not suitable for use in the presence of flammable anaesthetics.

## Electrical safety

### WARNING:

To avoid the risk of electrical shock:

- Use only properly grounded equipment. Shock hazards exist if the power supply is not properly grounded. Grounding reliability can only be achieved when equipment is connected to a receptacle marked “Hospital Only” or “Hospital Grade” or the equivalent. The grounding wire must not be removed or defeated.
- Use only accessories and peripherals recommended by SonoSite, including the power supply. Use the PowerPack only with SonoSite power supply P09823.
- Do not disassemble or alter the battery.
- Use only with specified approved power supplies.
- Do not plug the stand’s power cord into a multiple portable socket outlet (MPSO) or use an extension cord.

To avoid the risk of electrical shock or injury, do not open the battery enclosure on the PowerPack battery and housing.

To avoid the risk of electrical shock and fire hazard, inspect the power supply, plugs, cables, and power cords on a regular basis for damage.

### WARNING:

To avoid possible electrical shock or electromagnetic interference, verify proper operation and compliance with relevant safety standards for all equipment before clinical use. Connecting additional equipment to the ultrasound system constitutes configuring a medical system. SonoSite recommends verifying that the system, all combinations of equipment, and accessories connected to the ultrasound system comply with JACHO installation requirements and/or safety standards AAMI-ES1, NFPA 99, IEC 60601-1-1, and IEC 60601-1-2, and are certified according to IEC 60950 (Information Technology Equipment (ITE)).

### Caution:

Electrostatic discharge (ESD), or static shock, is a naturally occurring phenomenon. ESD is common in conditions of low humidity, which can be caused by heating or air conditioning. Static shock is a discharge of the electrical energy from a charged body to a lesser or non-charged body. The degree of discharge can be significant enough to cause damage to the PowerPack. The following precautions can help reduce ESD: anti-static spray on carpets, anti-static spray on linoleum, and anti-static mats.

To prevent the battery from draining, do not connect the PowerPack DC power cord to its own—or another PowerPack’s—DC input port.

## Equipment safety

**Caution:** Excessive bending or twisting of cables can cause a failure or intermittent operation.

Improper cleaning or disinfecting of any part of the accessories and peripherals can cause permanent damage. For cleaning and disinfecting instructions, see “Cleaning and disinfecting” on page 4 and the manufacturer’s operating instructions.

## Electromagnetic compatibility (EMC)

See the safety section of the ultrasound system user guide and supplemental guides for information on electromagnetic compatibility with IEC 60601-1-2, including the Manufacturer’s Declaration.

## Symbols

The following symbols and phrases appear on the product. For additional symbols, see the ultrasound system user guide.

### Symbols and phrases

TUV Rhineland of North America. The “C” and “US” indicators signify that the product has been evaluated to the applicable CSA and ANSI/UL standards for use in Canada and the US, respectively.

YYYY-MM Universal date representation where YYYY is the year in the Gregorian calendar and MM is the month of the year between 01 (January) and 12 (December). May also be represented as YYYY-MM-DD where DD is the day of the month between 01 and 31.

## Specifications

### Dimensions (approximate)

**Width:** 3.75 in. (9.5 cm)

**Depth:** 1.5 in. (3.8 cm)

**Length:** 10.25 in. (26 cm)

**Weight:** 2.9 lbs. (1.3 kg)

### Battery

The battery contains lithium-ion cells plus electronics, a temperature sensor, and battery contacts.

**Rating:** 10.8 VDC, 9 Ah, 97.2 Wh

### Environmental limits

**Operating:** 10 to 35°C; 15 to 95% R.H.; 700 hPa to 1060 hPa

**Shipping/Storage:** -20 to 60°C; 15 to 95% R.H.; 500 hPa to 1060 hPa

### Electrical

**Input #1:** 15 VDC, 5.0 A

**Input #2:** 9.0 – 12.6 VDC, 2.3 A

**Output #1:** 15 VDC, 5.0 A

**Output #2:** 9.0 – 12.6 VDC, 1.8 A

Combined output not exceeding 75 watts

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