

# S Series™ Transducer Holder


## Assembly Instructions

### Introduction


Attach the transducer holder to the S Series™ ultrasound system before attaching the system to the S Series Stand or other supporting device.

See the *S Series Stand User Guide* or *VHM™ Wall Mount Installation Instructions for S Series™ Ultrasound System* for information on cleaning and disinfecting and on setting up the ultrasound system to use accessories and peripherals.

### Contents

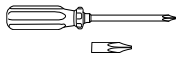

Magnetic  
screwdriver &  
Phillips bit

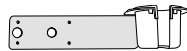

Transducer  
holder



8mm  
Phillips-head  
screws (4)

---

### Attach transducer holder to system

- 1 Place the transducer holder on the back of the system, with the cup facing toward the front.  
The two flat pegs above the transducer connector pass through the large holes on the holder.
- 2 Insert the Phillips bit into the magnetic screwdriver.
- 3 Using the magnetic screwdriver, tighten the four 8mm Phillips-head screws. See [Figure 1](#).

To remove the transducer holder, loosen the four screws on the back of the transducer holder by using a Phillips screwdriver. See [Figure 1](#).

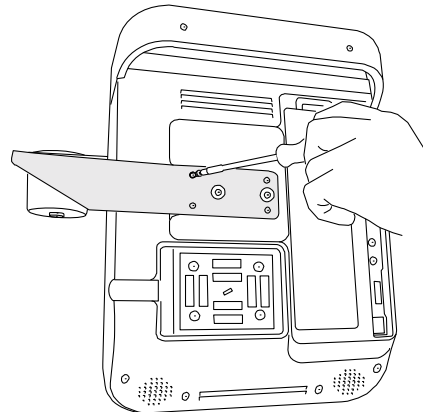


Figure 1 Tighten transducer holder.

S Series, SonoSite, and the SonoSite logo are registered trademarks or trademarks of SonoSite, Inc.

Non-SonoSite product names may be trademarks or registered trademarks of their respective owners.

This SonoSite product(s) referenced in this document may be covered by one or more of the following U.S. patents: 5722412, 5817024, 5893363, 6135961, 6203498, 6364839, 6371918, 6383139, 6416475, 6447451, 6471651, 6569101, 6648826, 6575908, 6604630, 6817982, 6835177, 6962566, 7169108, 7449640, D456509, D461895, D509900, D538432, D544962, D558351, D559390, and by the following counterpart foreign patents: AU727381, AU730822, CA2373065, CN98106133.8, CN98108973.9, DE60021552.0, DE60029777.2, DE60034670.6, DE69730563.5, DE6980539.6, DE69831698.3, FR0815793, FR0875203, FR0881492, FR1175713, FR1180970, GB0815793, GB0875203, GB0881492, GB1175713, GB1180970, GB1180971, IT0815793, IT0881492, IT1175713, KR528102, KR532359, NO326202, NZ542968, RCD000897368-0001, SP0815793, SP0881492. Patents pending.

**Caution:** Federal (United States) law restricts this device to sale by or on the order of a physician.

**SonoSite, Inc.**  
21919 30th Drive SE  
Bothell, WA 98021 USA  
T: 1-888-482-9449 or  
1-425-951-1200  
F: 1-425-951-1201

**SonoSite Ltd**  
Alexander House  
40A Wilbury Way  
Hitchin, Herts  
SG4 OAP UK  
T: +44-1462-444800  
F: +44-1462-444801

03/2009  
Copyright 2009 by  
SonoSite, Inc.  
All rights reserved.



P12237-01

