SonoRemote[™] Control

User Guide

CE

SonoRemote™ Control User Guide

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Caution:

Federal (United States) law restricts this device to sale by or on the order of a physician.

M-Turbo, S Series, SonoRemote, SonoSite, and the SonoSite logo are either a registered trademark or trademark of SonoSite, Inc.

Non-SonoSite product names may be trademarks or registered trademarks of their respective owners.

This SonoSite product(s) referenced in this document may be covered by one or more of the following U.S. patents: 5722412, 5817024, 5893363, 6135961, 6203498, 6364839, 6371918, 6383139, 6416475, 6447451, 6471651, 6569101, 6648826, 6575908, 6604630, 6817982, 6835177, 6962566, 7169108, 7449640, D456509, D461895, D509900, D538432, D544962, D558351, D559390, and by the following counterpart foreign patents: AU727381, AU730822, CA2373065, CN98106133.8, CN98108973.9, DE60021552.0, DE60029777.2, DE60034670.6, DE69730563.5, DE6980539.6, DE69831698.3, FR0815793, FR0875203, FR0881492, FR1175713, FR1180970, GB0815793, GB0875203, GB0881492, GB1175713, GB1180970, GB1180971, IT0815793, IT0881492, IT1175713, KR528102, KR532359, NO326202, NZ542968, RCD000897368-0001, SP0815793, SP0881492. Patents pending.

P11771-02 04/2009 Copyright 2009 by SonoSite, Inc. All rights reserved.

English

Contents

Kit contents 1 About the remote control 1 Connecting the remote control 4 Controlling the ultrasound system 5 Using multiple ultrasound systems or remote controls 7 Working in a sterile environment 8 Maintenance 8 Specifications 10 Safety 10	Introduction	1
About the remote control	Kit contents	1
Connecting the remote control	About the remote control	1
Controlling the ultrasound system	Connecting the remote control	4
Using multiple ultrasound systems or remote controls	Controlling the ultrasound system	5
Working in a sterile environment	Using multiple ultrasound systems or remote controls	7
Maintenance	Working in a sterile environment	8
Specifications	Maintenance	8
Safety	Specifications	.10
	Safety	.10

Introduction

The SonoRemote Control User Guide explains how to use the SonoRemote[™] control with the M-Turbo[®] ultrasound system or S Series[™] ultrasound system.

For more information on the ultrasound system, see the ultrasound system user guide.

Conventions

The user guide follows these conventions:

- A **WARNING** describes precautions necessary to prevent injury or loss of life.
- A **Caution** describes precautions necessary to protect the products.
- Numbered steps must be performed in order.
- Single-step procedures begin with *.
- Items in bulleted lists do not require performance in sequence.

For a definition of labeling symbols on the product and packaging, see the ultrasound system user guide.

Customer comments

Questions and comments are encouraged. Please call SonoSite at 1-888-482-9449 in the US. Outside the US, call the nearest SonoSite representative. You can also e-mail SonoSite at comments@sonosite.com. For technical support, see "Technical support" on page 9.

Kit contents

The kit for the SonoRemote control contains the following items:

- Remote control
- USB cable
- Bluetooth[®] USB adapter
- Microphone
- SonoRemote Control User Guide
- Sterile sheaths for remote control (5)
- MAC address labels (14)

See "To find the MAC address and apply the label" on page 3.

About the remote control

The remote control lets you control the ultrasound system from up to 30 feet away. Using six buttons or voice commands, you can perform the following tasks:

- Freeze (or unfreeze) an image
- Save an image
- Adjust the gain
- Adjust the depth
- Review the cine buffer

Figure 1 Remote control

1	Power switch	7	Microphone port
	(on top)		(on bottom)

- 2 Power indicator
- 3 Depth adjustment buttons
- 4 Freeze button
- 8 USB port (on bottom)
- 9 USB indicator
 - 10 Gain adjustment buttons (live image) Cine review buttons (frozen image)
- 11 Bluetooth indicator
- 6 Microphone indicator

Save button

5

Figure 2 Power switch on upper edge of remote control

Indicators

The indicators have different states for different activity. Except where noted, the indicators are active only if the power is on.

Power indicator

- Green, solid Remote control is ready to use.
- **Green, blinking** Battery is charging.

The indicator blinks whether the power is on or off. If the power is off, the indicator turns off when the battery finishes charging. (See "To charge the battery" on page 3.)

A faster blink indicates that the temperature is too high or low. (See "Temperature and humidity limits" on page 10.) An unusually fast blink indicates that the battery may need replacing. (See "Technical support" on page 9.)

- **Green and yellow, blinking alternately** Battery is charging in power save mode.
- Yellow, blinking Battery is low.
- **Yellow, solid** Remote control is in power save mode.

Microphone indicator

- **Solid** Microphone is connected.
- **Blinking** Remote control is in training. (See "To train the remote control" on page 6.)

Bluetooth indicator

• **Solid** Remote control is connected by Bluetooth technology.

- Blinking fast Remote control is trying to establish a connection by Bluetooth technology.
- **Blinking slowly** Remote control is searching for Bluetooth devices.

USB indicator

• **Solid** Remote control is connected by USB cable.

Turning on power and charging the battery

Before using the remote control, make sure that the battery is charged.

You can charge one remote control while using another that is connected by Bluetooth technology.

To turn the remote control on or off

Press the power switch U.
See Figure 1 and Figure 2 on page 2.

To charge the battery

- 1 Connect the ultrasound system to AC power.
- **2** Turn on the ultrasound system and remote control.
- **3** Connect the remote control by USB cable. (See "To connect by USB cable" on page 4.) Use a USB port that is on the system, not on a dock.

Note: If you connect to a dock instead of to the system and if the system enters power save mode, the remote control does not charge.

The battery fully charges in less than three and a half hours. If the remote control power is on, the ultrasound system ignores the sleep and power delay settings for three hours, allowing the battery to charge.

If the system enters power save mode, the battery continues to charge. If the system turns off, the battery does not charge.

Power save mode

The remote control beeps three warning beeps and enters power save mode after the following period of inactivity:

- (Microphone connected) 15 minutes without a voice command
- (Microphone disconnected) 5 minutes without a button press

All indicators but the power indicator turn off. After 30 minutes in power save mode, the remote control turns off.

To wake from power save mode

Press a button, or say a voice command.

About the MAC address

The remote control's Media Access Control (MAC) address appears in the ultrasound system's Bluetooth Devices list and helps you confirm that the correct Bluetooth device is connected.

Find the MAC address and apply the label to the remote control to ensure that this unique identification information is readily available.

To find the MAC address and apply the label

- 1 Remove a MAC address label from its protective backing.
- 2 Adhere the label to the back of the remote control, in the rectangle at the bottom.
- Find the MAC address: Connect by Bluetooth technology. (See "To connect by Bluetooth technology" on page 4.) The MAC address, BlueHID ##:## (where ##:## are four unique numbers or letters), appears in the Bluetooth Devices list.
- **4** Using a permanent marker, write the MAC address on the label.

Connecting the remote control

You can connect the remote control either by USB cable or wirelessly by Bluetooth technology.

When the system is connected, the USB indicator or Bluetooth indicator is on, and the remote-control icon **••••** appears on the system screen.

Connecting by USB cable

- WARNING: To avoid electrical shock, inspect the USB cable for damage. Do not use the USB cable if it is damaged.
- Caution: To avoid exceeding emissions limits, use only the USB cable provided by SonoSite. Other USB cables may exceed emissions limits.

To avoid damaging the USB cable, do not excessively bend or twist it. Bending or twisting the USB cable can cause a failure or intermittent operation.

To avoid disconnecting the USB cable, maintain slack in the cable, and do not pull on the remote control.

To connect by USB cable

- **1** Turn on the ultrasound system and remote control.
- 2 Connect the smaller end of the USB cable to the USB port on the remote control. (See Figure 1 on page 2 for the port location.)
- **3** Allowing slack in the USB cable, connect the larger end of the USB cable to a USB port *c* on the ultrasound system.

Connecting by Bluetooth technology

WARNING:

To avoid connecting to and controlling the wrong ultrasound system, always do the following:

- Before the ultrasound procedure, confirm connection to the correct system. See "Testing the connection" on page 5. The remote control may maintain a connection from a distance greater than 30 feet.
- Disconnect the remote control after use. See "To disconnect the remote control" on page 5.

With Bluetooth technology, the remote control can control the ultrasound system wirelessly, from up to 30 feet away.

You connect by Bluetooth technology through the ultrasound system's USB Devices setup page. Only one system can connect to the remote control at a time. Similarly, only one remote control can connect to an ultrasound system at a time. If you disconnect or otherwise lose the connection, the Bluetooth indicator blinks quickly.

To connect by Bluetooth technology

- **1** Turn on the system.
- Insert the Bluetooth USB adapter into a USB port or on the system.
- 3 On the USB Devices setup page, select Bluetooth. (See "To display the USB Devices setup page" on page 5.)

Bluetooth is available about a minute after you insert the Bluetooth USB adapter.

4 Turn on the remote control.

The Bluetooth indicator blinks fast.

5 Simultaneously press **▶|** and **□** on the remote control until the beep, and then release.

The Bluetooth indicator blinks slowly.

6 Select **Search** in the lower lefthand corner of the ultrasound system screen.

The ultrasound system searches for Bluetooth devices. The process can take several minutes.

7 Select the remote control (BlueHID ##:##) in the Bluetooth Devices list, and then select Connect at the bottom of the screen.

In the Bluetooth Devices list, the remote control's status changes to Connected.

Note: The USB Devices setup page lists both the remote control (KBD) and the Bluetooth USB adapter (BTD).

To disconnect the remote control

- 1 On the ultrasound system's USB Devices setup page, select **Bluetooth**. (See "To display the USB Devices setup page.")
- 2 Select the remote control (BlueHID ##:##) in the Bluetooth Devices list, and then select Disconnect at the bottom of the screen.

To display the USB Devices setup page

- Do one of the following:
 - (M-Turbo) Press the SETUP key, and then select **USB Devices**.
 - (S Series) In 2D imaging, click **Setup** on the vertical row of control keys, and then click **USB Devices**.

If **Setup** is on another page, press **Page x/x** until it appears.

Testing the connection

Before the ultrasound procedure, test the connection to confirm that the correct system is connected.

To test the connection

Press a button on the remote control.
The ultrasound system is connected if it responds as expected.

Controlling the ultrasound system

WARNING:

If the ultrasound system fails to respond to the remote control during an ultrasound procedure, use the controls on the ultrasound system instead.

If a message that requires a user response appears on-screen, use the controls on the ultrasound system to respond.

After connecting the remote control, you can control the ultrasound system by pressing the six buttons or using six voice commands.

Buttons

The buttons on the remote control perform the following commands.

Ŧ	Decreases the depth
	Increases the depth
7	Increases the gain (live image) Reviews the cine buffer (frozen image)
7	Decreases the gain (live image) Reviews the cine buffer (frozen image)
M	Freezes (or unfreezes) the image
	Saves the image

See also "Reviewing the cine buffer" on page 7.

You can press the buttons to control the ultrasound system even if the microphone is connected.

Voice commands

WARNING: To avoid unexpected system responses to voice commands, test them before the ultrasound procedure. See "To test the voice commands" on page 7.

If the remote control fails to respond to voice commands during an ultrasound procedure, use the buttons instead.

You can control the ultrasound system by speaking commands into a microphone connected to the remote control. Before using voice commands, train the remote control to recognize your voice.

Whenever you connect the microphone, the remote control is automatically ready for voice commands.

See also "Reviewing the cine buffer" on page 7.

To connect the microphone

Insert the microphone connector into the microphone port. See Figure 1 on page 2.

The microphone indicator turns on. The remote control beeps three warning beeps if not previously trained.

To train the remote control

You can train the remote control with your own phrases instead of the default phrases in this procedure. See "Using custom phrases" on page 7.

Read the entire procedure before beginning. Allow enough time for training: about 15 to 30 minutes. If you have to stop for more than a few seconds during training, disconnect the microphone and begin again.

- **1** Set up the equipment and environment:
 - **a** In a quiet area, turn on the remote control.
 - **b** Connect the microphone.
 - c Put on any garments, such as a mask or gown, that will be between your mouth and the microphone during ultrasound procedures.

- **d** Attach the microphone to your clothing, near your mouth and where positioned during ultrasound procedures (for example, under the gown).
- 2 Start training mode: Press (i) until the beep, and then release. If the remote control beeps three times, clear the memory (within

10 seconds) by pressing ► until the long beep.

The microphone indicator starts blinking. If these responses do not occur, disconnect the microphone and restart the procedure.

3 Say and then repeat each phrase in the following table, in order, and wait for the response.

Saying each phrase in order associates it with the correct response from the ultrasound system.

Use the same volume and clarity that you use during ultrasound procedures.

	e ć		ŧ	
	Say:	Wait for:	Repeat:	Wait for:
1	Depth up	beep	Depth up	2 beeps
2	Gain down	beep	Gain down	2 beeps
3	Gain up	beep	Gain up	2 beeps
4	Depth down	beep	Depth down	2 beeps
5	Freeze	beep	Freeze	2 beeps
6	Save	beep	Save	long beep

The long beep indicates that the training is complete.

4 Test the voice commands.

To test the voice commands

- **1** Set up the equipment and environment:
 - **a** Turn on the ultrasound system and the remote control.
 - **b** Connect the following:
 - The microphone to the remote control
 - The remote control to the ultrasound system (See "Connecting the remote control" on page 4.)
 - Put on any garments, such as a mask or gown, that will be between your mouth and the microphone during ultrasound procedures.
 - **d** Attach the microphone to your clothing, near your mouth and where positioned during ultrasound procedures (for example, under the gown).
- **2** Say each phrase that you used to train the remote control, and make sure that the ultrasound system responds as expected.

If the ultrasound system does not respond as expected, retrain the remote control, or use the buttons. See also "Troubleshooting" on page 9.

Using custom phrases

You can train the remote control with your own phrases instead of the suggested phrases. In some environments, the remote control responds best to custom phrases. If you train the remote control with custom phrases, make sure to say them in order. Phrases do the following:

Phrase 1 Decreases depth

Phrase 2 Decreases gain

Phrase 3 Increases gain (live image), reviews the cine buffer (frozen image)

Phrase 4 Increases depth (live image), reviews the cine buffer (frozen image)

Phrase 5 Freezes (or unfreezes) the image

Phrase 6 Saves the image

Choose phrases that are easy to remember, sound different from each other, and sound different from other phrases used during ultrasound procedures. Longer phrases typically work better than shorter phrases because they contain more sound information. Also, try to vary the length of phrases.

Reviewing the cine buffer

When the image is frozen, you can review the cine buffer by using the buttons and voice commands that adjust the gain on a live image.

To review the cine buffer

- **1** Freeze the image.
- **2** Do any of the following:
 - (Buttons) Press \ to move forward or \ to move backward.
 - (Voice commands) Say the increase-gain command to move forward. Say the decrease-gain command to move backward.

To return to the live image, unfreeze the image.

Using multiple ultrasound systems or remote controls

Using only one remote control per ultrasound system is the best way to ensure that you do not inadvertently connect to another system or remote control.

If you use multiple ultrasound systems with one remote control—or multiple remote controls with one ultrasound system—take precautions to ensure that you do not connect to the wrong device:

• When connecting by Bluetooth technology, always disconnect the remote control from the ultrasound system after use. See "To disconnect the remote control" on page 5.

- Before the ultrasound procedure, test the connection. See "Testing the connection" on page 5.
- If you work in emergency settings, make sure to set up your remote control before entering the workplace. Plan enough time to train and connect it.
- If using multiple remote controls with one ultrasound system, make sure that the previously used remote control is disconnected. Check the Bluetooth Devices list to verify that the MAC address for the connected remote control matches the remote control you use.

Working in a sterile environment

WARNING:

To avoid contamination after encasing the remote control in a sterile sheath, inspect the sheath for holes or tears.

Some transducer sheaths contain natural rubber latex and talc, which can cause allergic reactions in some individuals. Refer to 21 CFR 801.437, User labeling for devices that contain natural rubber.

If the remote control is outside the sterile field and fails to respond to voice commands during the ultrasound procedure, use the controls on the ultrasound system. Take proper precautions to maintain the sterile environment. If you work in a sterile environment, consider including the following tasks when setting up:

- Protect the sterile environment from the remote control by doing one of the following:
 - Encasing the remote control in the recommended sterile sheath.
 - If using voice commands, putting the remote control under the gown in a pocket or in a holster designed for mobile devices. However, take precautions to ensure that you can use the remote control buttons or the ultrasound system controls if the remote control fails to respond to voice commands.
- If using voice commands, put on the microphone before putting on the gown.
- If connecting by USB cable, encase the USB cable in a sterile sheath. See "Sterile sheaths" on page 10.
- If using voice commands and if the remote control will be outside the sterile field, turn on the remote control before setting up the sterile field, or turn it on using a sterile technique.

Be aware that the remote control enters power save mode after a period of inactivity. See "Power save mode" on page 3.

Maintenance

If the remote control needs repair or service, return it to SonoSite. Contact SonoSite Technical Support for instructions. See "Technical support" on page 9.

Troubleshooting

Unexpected beeps during training If the remote control does not beep as expected during training, it may not detect your voice. Try any of the following:

- Speak more loudly or softly.
- Place the microphone nearer to your mouth.
- Make sure that the microphone is oriented toward your mouth.
- Use different phrases. See "Using custom phrases" on page 7.
- Follow the training instructions exactly, including clearing the memory.

Ultrasound system does not respond If the

ultrasound system appears connected but stops responding, turn the remote control off and then on again.

If connecting by Bluetooth technology, make sure that the remote control is connected to the ultrasound system that you are using.

If connecting by USB cable, check cable connections.

Ultrasound system behaves unexpectedly If the system behaves unexpectedly (for example, the image unexpectedly freezes or changes), make sure that it is not connected to another remote control. The ultrasound system may respond to the connected remote control from a distance greater than 30 feet.

Power save starts before Bluetooth connection

If the remote control enters power save mode before the ultrasound system finishes connecting by Bluetooth technology, simultaneously

press M and n on the remote control until the beep, and then release.

Technical support

For technical support, contact SonoSite as follows.

SonoSite Technical Support

Phone (US or Canada):	1-877-657-8118
Phone (Outside US and Canada):	1-425-951-1330 Or call your representative.
Fax:	1-425-951-6700
E-mail:	service@sonosite.com
Web site:	www.sonosite.com Click Resources > Support & Service.

Europe Service Center

Phone:	+44-(0)1462-444-800
E-mail:	uk.service@sonosite.com

Cleaning and disinfecting

For the most recent cleaning and disinfectant information, see www.sonosite.com. Click **Resources**, and then click **Documentation**.

Caution: To avoid damaging the remote control, do not spray liquids on it. Use only a damp cloth or paper tissues to clean it.

To clean and disinfect the remote control, USB cable, or microphone

1 Clean the surface using a soft cloth lightly dampened in a mild soap or detergent cleaning solution.

Apply the solution to the cloth rather than the surface.

- 2 Wipe the surfaces with a recommended disinfectant. Follow the manufacturer's instructions, including contact duration.
- **3** Air dry or towel dry with a clean cloth.

Recommended disinfectants

Disinfectant	Country of Origin	Туре	Active Ingredient
Hydrogen Peroxide*	n/a	Liquid	Hydrogen Peroxide
Sani-Cloth HB*	USA	Wipe	Quat. Ammonia
Super Sani-Cloth*	USA	Wipe	lsopropyl Alcohol

* Not for use on the microphone clip. (The microphone clip is detachable from the microphone.)

Specifications

Sterile sheaths

The recommended sheath is as follows:

• Remote control only:

Sterile 15.2 x 25.4 cm (6" x 10") polyethylene cover, CIVCO brand

- Package of 5: CIVCO 610-1049
- Package of 24: CIVCO 610-1050
- Remote control with USB cable:

Sterile 10.2 x 147 cm (4" x 58") CIV-Flex cover, CIVCO brand

• Package of 24: CIVCO 610-1000

One package of five sheaths for the remote control is included. For additional sheaths, contact CIVCO. See www.civco.com.

Dimensions

Width: 2.4 in. (6.1 cm) Height: 4.8 in. (11.1 cm) Depth: 0.8 in. (2 cm) Weight: 4.4 oz (124 grams)

Temperature and humidity limits

Operating: 10 to 40° C, 15-95% RH

Storage: -20 to 60° C, maximum 95% RH

Safety

Clinical safety

WARNING: To avoid injury to the patient or user, inspect the remote control for damage before use. If the remote control is damaged, do not use it.

Electrical safety

WARNING: To avoid electrical shock, use only accessories that are recommended by SonoSite with the remote control.

Radio device regulatory notices

For more information on electromagnetic compatibility, including electromagnetic emission and electromagnetic immunity limits, see the ultrasound system user guide.

US: FCC notice

The SonoRemote control has been tested and found to comply with the limits for a Class A digital device pursuant to Part 15 of the Federal Communications Commission (FCC) rules. These limits are designed to provide reasonable protection against harmful interference in an industrial installation. This device generates, uses, and can radiate radio frequency energy and,

if not installed and used in accordance with the instructions, may cause harmful interference to radio communications and can affect the performance of other electrical equipment in the vicinity.

Operation conditions

Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

FCC RF safety requirement

The radiated output power of the wireless LAN and Bluetooth adapter is far below the FCC radio frequency exposure limits. Nevertheless, the remote control and these adapters shall be used in such a manner that the potential for human contact during normal operation is minimized as follows:

• Users are requested to follow the RF safety instructions on wireless option devices that are included in the user manual of each RF option device.

Caution: To comply with FCC RF exposure compliance requirements, a separation distance of at least 8 inches (20 cm) must be maintained between wireless adapters and all persons.

- High power radars are allocated as primary users of the 5.25 to 5.35 GHz and 5.65 to 5.85 GHz bands. These radar stations can cause interference with and/or damage this device.
- An improper installation or unauthorized use may cause harmful inference to radio communications.
- Changes or modifications not expressly approved by the manufacturer could void the user's authority, which is granted by the FCC, to operate this device.

Canada: Low-power license-exempt radio communication devices (RSS-210)

This Class A digital apparatus complies with Canadian ICES-003.

Common information

Operation is subject to the following two conditions:

- This device may not cause interference.
- This device must accept any interference, including interference that may cause undesired operation of the device.

European Union (EU): Declaration of Conformity for EU countries

SonoSite declares that this remote control device is in compliance with the essential requirements and other relevant provisions of R&TTE Directive 1999/5/EC as attested by conformity with the following harmonized standard:

- Article 3.1(b) EMC
 - EN301 489-1 V1.8.1
 - EN301 489-17 V1.3.2
- Article 3.2 Spectrum Usages
 - EN300 328 V1.7.1

Applicable countries

As of March 2009, EU member states are Austria, Belgium, Bulgaria, Cyprus, Czech Republic, Denmark, Estonia, Finland, France, Germany, Greece, Hungary, Ireland, Italy, Latvia, Lithuania, Luxembourg, Malta, Netherlands, Poland, Portugal, Romania, Slovakia, Slovenia, Spain, Sweden, and United Kingdom. This device must be used in strict accordance with the regulations and constraints in the country of use. For more information, contact the local office in the country of use.

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