


Wireless Adapter Installation Instructions

Installing the wireless adapter on SonoSite X-Porte	1
To install on a system in kiosk mode	1
To install on a system in desktop mode	2
Installing the wireless adapter on SonoSite Edge	2
Installing the wireless adapter on SonoSite Edge II	2
Installing the wireless adapter on SonoSite S Series	2
Installing the wireless adapter on SonoSite SII	3
Installing the wireless adapter on SonoSite M-Turbo	3

Contents

Wireless adapter (1)

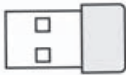

Caution

To avoid damaging the system:


- ▶ Use only peripherals recommended by FUJIFILM SonoSite with the ultrasound system.
- ▶ Turn off the system power before inserting the wireless adapter.
- ▶ When using the SonoSite X-Porte in desktop mode, remove the wireless adapter before returning the system to kiosk mode.

Installing the wireless adapter on SonoSite X-Porte


To install on a system in kiosk mode

Plug the wireless adapter into the USB port on the back of the SonoSite X-Porte system beneath the Power button. Choose the diagram below that best fits your SonoSite X-Porte configuration:

CONFIGURATION 1

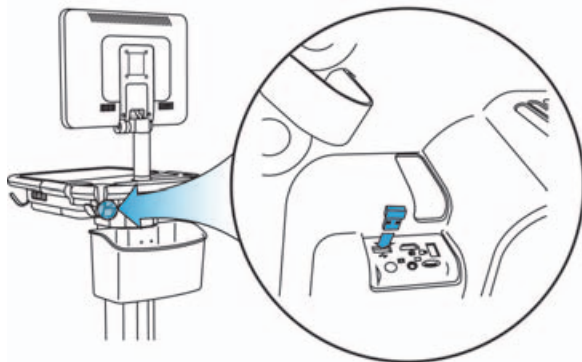


CONFIGURATION 2

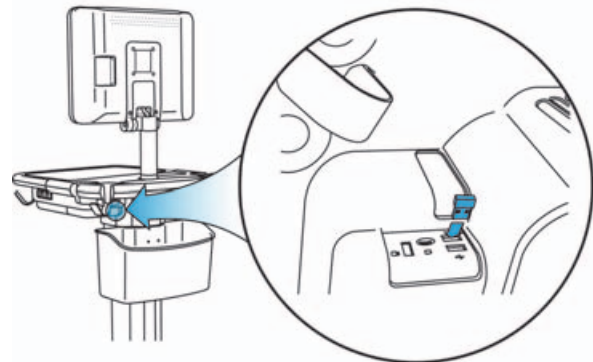

To install on a system in desktop mode

On the right side of the system, plug the wireless adapter into the rearmost of the three USB ports.

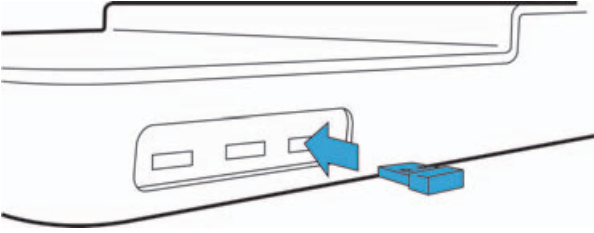

Installing the wireless adapter on SonoSite Edge

Install the wireless adapter in a USB port on the right side of the system.




Installing the wireless adapter on SonoSite Edge II

Install the wireless adapter in a USB port on the right side of the system.




Installing the wireless adapter on SonoSite S Series

Install the wireless adapter in a USB port on the upper right side of the system.

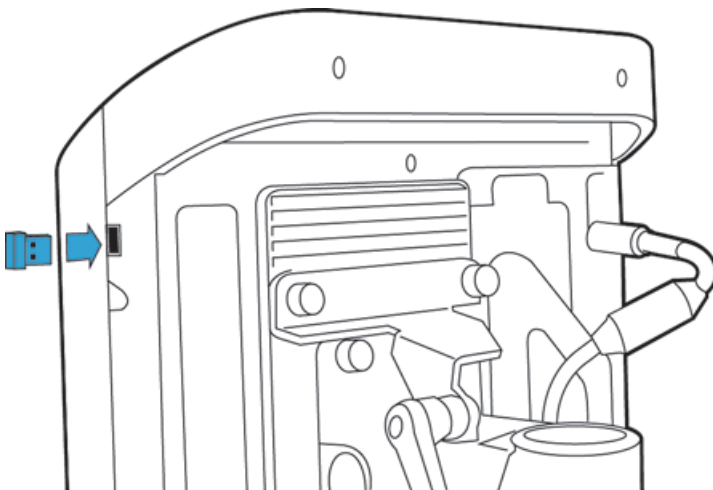


Installing the wireless adapter on SonoSite SII

Install the wireless adapter in a USB port on the upper back of the system.

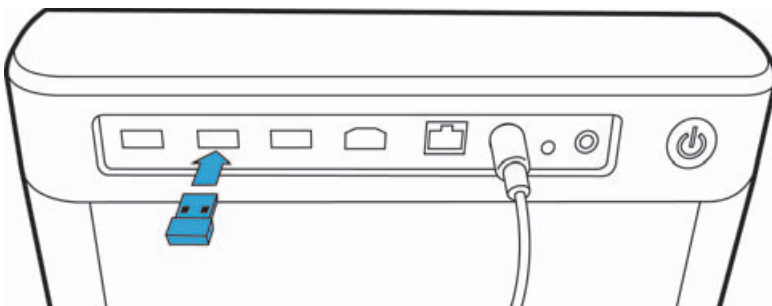

Installing the wireless adapter on SonoSite M-Turbo

Install the wireless adapter in a USB port on the right side of the system.

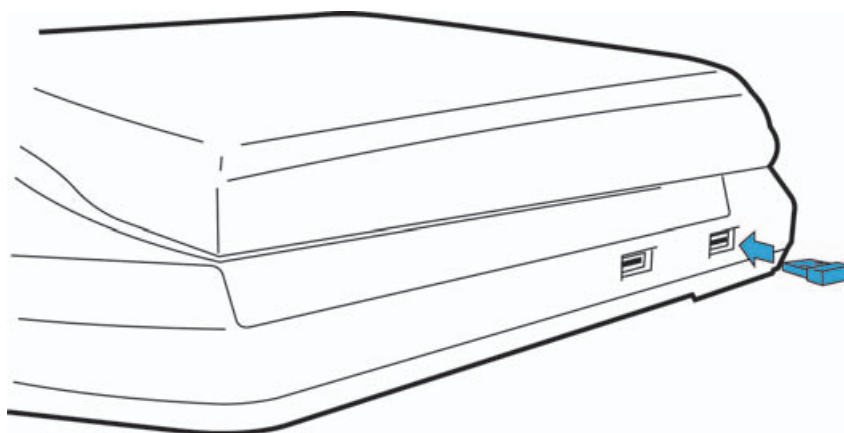

Manufacturer

FUJIFILM SonoSite, Inc.

21919 30th Drive SE
Bothell, WA 98021
USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

Emergo Europe

Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd

114 Old Pittwater Road
BROOKVALE, NSW 2100
Australia

X-Porte, SonoSite Edge, SonoSite S Series, SonoSite SII, SonoSite M-Turbo, SonoSite, and the SonoSite logo are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Publication Date: November 2016

Copyright © 2016 FUJIFILM SonoSite, Inc. All Rights reserved.



P22661-03

