Setting Up a Wireless Network

User Guide

FUJIFILM
Value from Innovation

SonoSite

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: +1-888-482-9449 or
+1-425-951-1200
F: +1-425-951-1201

EC Authorized Representative

Emergo Europe
Molenstraat 15
2513 BH, The Hague
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Sonosite, the Sonosite logo, SonoSite SII, SonoSite Edge II, SonoSite M-Turbo, SonoSite S Series, SonoSite Edge, and SonoSite NanoMaxx are registered and unregistered trademarks of FUJIFILM Sonosite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

DICOM is a registered trademark of the National Electrical Manufacturers Association.

All other trademarks are the property of their respective owners.

Part Number: P12062-06

Publication Date: March 2016

Copyright © 2016 FUJIFILM SonoSite, Inc. All Rights reserved.

Contents

Introduction	1
Document conventions	1
Changes in version P12062-06	2
System and hardware requirements	3
Connecting the ultrasound system	3
Client certificates	3
Connecting the wireless USB dongle	4
Troubleshooting	5

Introduction

To archive wirelessly from the SonoSite Edge II, SonoSite SII, SonoSite X-Porte, SonoSite M-Turbo, SonoSite S Series, SonoSite Edge, or SonoSite NanoMaxx ultrasound system, FUJIFILM SonoSite supports the network setup described in this document. Use the setup instructions from the router manufacturer for general information only.

After connecting the ultrasound system, you need to configure a location through the Connectivity setup page on the ultrasound system and to connect the wireless USB dongle. For instructions to configure a location, see *Sending and Receiving DICOM Data on SonoSite Systems*.

For more information about the ultrasound system, see the ultrasound system user guide.

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the user guide.

Changes in version P12062-06

Feature	Description
System and hardware requirements	Updated the wireless description.

Getting help

In addition to this document, contact FUJIFILM SonoSite Technical Support:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	service@sonosite.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

System and hardware requirements

- ▶ Ultrasound system: SonoSite Edge II, SonoSite SII, SonoSite X-Porte, SonoSite M-Turbo, SonoSite S Series, Edge, or SonoSite NanoMaxx.
- ▶ 802.11 wireless USB dongle

WARNING The wireless USB dongle is a radio transmitter designed for use as a mobile device as defined by the Federal Communications Commission (FCC). Do not place the transmitter on a patient's or user's body. Ensure that the transmitter is located at least 20 cm (8 in) from a patient's or user's body. Locating the transmitter closer than 20 cm (8 in) to a person's body voids the user's FCC authority to operate the equipment.

- Caution**
- ▶ Operating the wireless USB dongle without a regulatory label or the correct country stamp is illegal. See the documentation from the manufacturer of the wireless USB dongle.
 - ▶ To avoid damage to the wireless USB dongle when using the MDSm or MDS Lite II, remove or insert the USB dongle only when the system is in the stand.
 - ▶ For environmental operating and storage specifications, see the operating instructions from the router manufacturer.

Connecting the ultrasound system

To archive wirelessly, you can connect the ultrasound system to an existing wireless network. DICOM setup requires a wireless access point on an existing wireless network with a DICOM server. See your network administrator.

Client certificates

Although you can connect without client certificates, they help to ensure network security.

You can obtain certificates from your network administrator. After importing a certificate, you select it while configuring the location.

To import or delete wireless certificates

The system supports Base64 and DER encoded format and the following file types: CER, PFX, and PVK.

- 1 On the Connectivity setup page on the ultrasound system, select **Wireless Certificates**.

A list of certificates on the system appears.

2 Do one of the following:

- ▶ Import certificates: Insert the USB storage device that contains the certificates. Verify that the certificates on the system plus those on the USB storage device do not exceed 20 certificates. Delete certificates if necessary. Then select **Import** on-screen.
- ▶ Delete certificates: Select the certificates to delete, and select **Delete**.

3 Select **Back** on-screen.

4 Select **Done** on-screen.

5 Restart the system.

Connecting the wireless USB dongle

After you configure a location on the ultrasound system, connect the wireless USB dongle, which transmits the signal from the ultrasound system to the network.

To connect the wireless USB dongle

- ❖ Insert the wireless USB dongle as specified in the instructions included in the dongle supply mounting kit. You may also insert the dongle in:
 - ▶ (SonoSite S Series and SonoSite X-Porte) A USB port on the side of the ultrasound system.
 - ▶ (SonoSite Edge II, SonoSite SII, SonoSite M-Turbo, SonoSite Edge, or SonoSite NanoMaxx) Any USB port on the ultrasound system.

The system automatically detects the device. Drivers are already installed.

Troubleshooting

If you have trouble connecting the ultrasound system to the wireless network, see [Table 1, “Troubleshooting”](#) on page 5 for suggestions. If the trouble continues, contact FUJIFILM SonoSite.

Table 1: Troubleshooting

Symptom	Solution
The ultrasound system does not connect.	<ul style="list-style-type: none"> ❖ Check the following for information about the network: <ul style="list-style-type: none"> ▶ Make sure that the following icons appear in the system status area on the ultrasound system screen: <ul style="list-style-type: none"> ▶ Wireless  ▶ Ultrasound-connected  ▶ On the ultrasound system, display the Network Status setup page. The page displays three lines of general information about the connection and five additional lines about the wireless connection (if present).
The ultrasound system does not authenticate, and the wireless icon does not appear on-screen.	<ul style="list-style-type: none"> ❖ Check the following <ul style="list-style-type: none"> ▶ Make sure that the system is communicating with your access point: Make sure that the network name on the system is the same as the network name (SSID) of the access point. The default network name is SONOSITE. The network name is case sensitive. ▶ If you use WEP or WPA-PSK, make sure that your WEP keys or WPA/WPA2 Pre-Shared key match between the access point and ultrasound system. These keys are case sensitive. ▶ The Network Status setup page may show that you associated successfully, but you are unable to get a DHCP address because the WEP key is set incorrectly. ▶ If you select the 802.1x RADIUS security option, make sure that the access point is turned on and properly configured with 802.1x enabled and that the Shared Secret key between the access point and authentication server match. ▶ Use the PING utility from the server to make sure that the authentication server is active and can communicate with the access point.

Table 1: Troubleshooting (continued)

Symptom	Solution
The system does not display the wireless icon after you select a valid location.	<ol style="list-style-type: none">1 Turn off the ultrasound system.2 Remove the wireless USB dongle.3 Verify the following:<ul style="list-style-type: none">▶ All wireless settings for the selected location are valid.▶ The selected wireless certificate is in Base64 format.▶ The system date and time are correct.4 Insert the wireless USB dongle.5 Turn on the system.
The system displays the wireless icon but does not communicate with DICOM.	<ul style="list-style-type: none">❖ Make sure that the DICOM setup is correct. See <i>Sending and Receiving DICOM Data on SonoSite Systems</i>.
You cannot import wireless certificates.	<ul style="list-style-type: none">❖ Make sure that the USB storage device contains wireless certificates by viewing its contents on a PC.

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