---

# SonoSite X-Porte

## ***User Guide Errata***

**Manufacturer**

FUJIFILM SonoSite, Inc.  
21919 30th Drive SE  
Bothell, WA 98021 USA  
T: 1-888-482-9449 or  
1-425-951-1200  
F: 1-425-951-1201

**EC Authorized Representative**

Emergo Europe  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

**Australia Sponsor**

FUJIFILM SonoSite Australasia Pty Ltd  
114 Old Pittwater Road  
BROOKVALE, NSW, 2100  
Australia

**Caution**

United States federal law restricts this device to sale by or on the order of a physician.

SONOSITE, X-Porte, and the SONOSITE logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P21471-01

Publication date: January 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

0086

# SonoSite X-Porte User Guide Errata

|                                    |          |
|------------------------------------|----------|
| <b>Introduction</b> .....          | <b>1</b> |
| Document conventions .....         | 1        |
| Getting help .....                 | 1        |
| <b>Setting up the system</b> ..... | <b>2</b> |
| USB Settings .....                 | 2        |
| eFilm Lite image-Viewer .....      | 3        |

## Introduction

This user guide errata has been issued to remove references to the eFilm Lite image-viewer product from the *SonoSite X-Porte User Guide*.

## Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the system user guide.

## Getting help

For technical support, please contact FUJIFILM SonoSite as follows:

|   |   |
|---|---|
| <b>Phone<br/>(U.S. or Canada)</b>             | 877-657-8118                                    |
| <b>Phone<br/>(outside U.S. or<br/>Canada)</b> | 425-951-1330, or call your local representative |
| <b>Fax</b>                                    | 425-951-6700                                    |
| <b>Email</b>                                  | ffss-service@fujifilm.com                       |

|                              |  |
|------------------------------|--|
| <b>Web</b>                   | www.sonosite.com   |
| <b>Europe Service Center</b> | Main: +31 20 751 2020<br>English support: +44 14 6234 1151<br>French support: +33 1 8288 0702<br>German support: +49 69 8088 4030<br>Italian support: +39 02 9475 3655<br>Spanish support: +34 91 123 8451 |
| <b>Asia Service Center</b>   | +65 6380-5581  |

Printed in the U.S.

The following content was either not included or incorrect in the system user guide; the revisions will be made in the next update.

## Setting up the system

### USB Settings

Information about the eFilm Lite image-viewer has been removed.

#### To specify export options

- 1 On the **USB** settings page, select an **Export Type**:
  - ▶ **DICOM Export** creates DICOMDIR files readable by a DICOM reader. DICOM is an optional feature.
  - ▶ **Multimedia Export** organizes files in a standard folder structure. Video clips export in MJPEG video saved as AVI. To view them, FUJIFILM SonoSite recommends Windows Media Player or QuickTime 7.0 or later.
- 2 Select an **Image Format** for your export type. For JPEG image format, also select a JPEG compression. A high compression has a smaller file size but less detail (see "Limitations of JPEG format"). The image format affects both still images and video clips.

For optimal DICOM image quality, select RGB image format and low compression.

- 3 (**Multimedia Export** only) Select a sort order from the **Sort By** list.
- 4 Select **Auto Export** to enable automatic export of ended exams. See also "To export patient exams automatically to a USB memory stick."
- 5 (**DICOM Export** only) Select any of the following:

- ▶ **Include CDA** to export the CDA report. See “CDA Report settings.”
- ▶ **Include SR** to export the Structured Report.

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