SonoSite iViz

User Guide Supplement

Manufacturer

FUJIFILM SonoSite, Inc.

21919 30th Drive SE

Bothell, WA 98021 USA

T: 1-888-482-9449 or 1-425-951-1200

F: 1-425-951-1201

EC Authorized Representative

Emergo Europe

Molenstraat 15

2513 BH, The Hague

The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd

114 Old Pittwater Road

BROOKVALE, NSW, 2100

Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite, the SonoSite logo, and SonoSite iViz are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P22312-03

Publication date: February 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

SonoSite iViz

User Guide Supplement

Configuring SonoSite iViz	1
Configuring SonoSite iViz settings	1
Configuring preferences	1

The following content was either not included or incorrect in the system user guide; the revisions will be made in the next update.

Configuring SonoSite iViz

Configuring SonoSite iViz settings

Configuring preferences


Two new preferences have been added:

- ▶ Anonymize on USB Share
- ▶ USB Structured Reports

To configure preferences

- 1 From the SonoSite iViz **Settings** screen, tap **Preferences**.
- 2 From the **Preferences** screen, the following settings are available:
 - ▶ **Units** - Select the units of measurement used by SonoSite iViz.
 - ▶ **Generic Volume Calcs** - Can be measured two different ways, depending on the type of structures you are measuring. Select either of the following:
 - ▶ **Ellipsoid** - (default) Length x width x height x $\pi/6$.
 - ▶ **Cuboid** - Length x width x height.
 - ▶ **Thermal Index** - Select any of the following:
 - ▶ **TIB** - Thermal index in bone
 - ▶ **TIC** - Thermal index in the cranium

- ▶ **TIS** - Thermal index in soft tissue
- ▶ **Auto Save Patient Form** - When turned on, this feature automatically saves the patient form periodically during editing to help prevent the loss of patient information.
- ▶ **Clip Recording** - Select either of the following:
 - ▶ **Prospective** - During a scan, this option records a standard length clip of the scan after tapping **Save**.
 - ▶ **Retrospective** - During a scan, this option records a standard length clip of the scan before tapping **Save**.
- ▶ **Clip Length** - From the drop-down menu, choose a standard recording length for clips.
- ▶ **Anonymize on USB Share** - When turned on, this feature removes identifying information from reports saved to USB. The default setting is OFF.
- ▶ **USB Structured Reports** - When turned on, this feature saves information to USB as a structured report. The default setting is OFF.
- ▶ **Scan Power Management** - Choose the settings that will provide the best balance between performance and battery life while scanning:

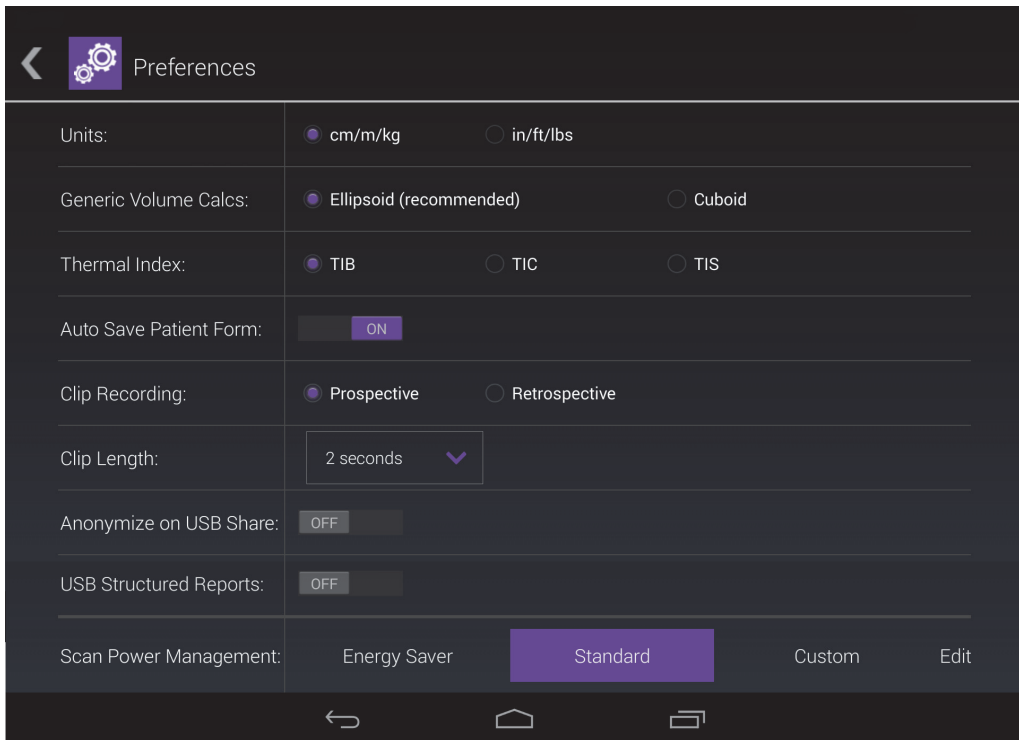- ▶ **Energy Saver** - Engages low-power modes quickly during periods of inactivity to get the most scanning time from your battery charge.
- ▶ **Standard** - Allows longer periods of inactivity before engaging low-power modes to provide a reasonable balance between scanning availability and battery life.
- ▶ **Custom** - Choose your own power management settings:
 - ▶ **Slow frame rate** - Activates in scanning mode after a period of inactivity. Automatically returns to normal operation when scanning resumes or you touch the controls. When active, a flashing white ❄️ appears on the screen. To turn this feature off, select **Never**.
 - ▶ **Freeze** - Activates after a continued period of inactivity after switching to Slow frame rate mode. You must unfreeze manually to resume scanning. When active, a solid blue ❄️ appears on the screen. To turn this feature off, select **Never**.
 - ▶ **Sleep** - Activates after a continued period of inactivity after switching to Freeze mode. To wake the system, quickly press and release the power button. To turn this feature off, select **Never**.
- ▶ **Power off** - Turns off the iViz system after a period of time in sleep mode, regardless of whether scanning or not. The system must be powered on manually.

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