---

# Edge Stand H-Universal Stand

*User Guide*

**FUJIFILM**  
Value from Innovation

**SonoSite**

---

**Manufacturer**

FUJIFILM SonoSite, Inc.  
21919 30th Drive SE  
Bothell, WA 98021 USA  
T: 1-888-482-9449 or 1-425-951-1200  
F: 1-425-951-1201

**EC Authorized Representative**

FUJIFILM SonoSite B.V.  
Joop Geesinkweg 140  
1114 AB Amsterdam,  
The Netherlands

**Australia Sponsor**

FUJIFILM SonoSite Australasia Pty Ltd  
114 Old Pittwater Road  
BROOKVALE, NSW, 2100  
Australia

**Caution** | United States federal law restricts this device to sale by or on the order of a physician.

Edge, SonoSite, M-Turbo, MicroMaxx, SonoSite, Titan, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners

Patents: US 8,088,071; US 7,591,786; US 7,534,211; US 6,447,451; CA 2372158; and EP 1589878.

Part Number: P12223-04

Publication Date: November 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All Rights reserved.

# Edge Stand H-Universal Stand

## User Guide

<b>Introduction</b> .....	<b>1</b>
Document conventions .....	2
Getting Help .....	2
<b>Stand features</b> .....	<b>3</b>
<b>Using the stand</b> .....	<b>4</b>
<b>Inserting and removing the ultrasound system</b> .....	<b>4</b>
<b>Connecting the ultrasound system</b> .....	<b>5</b>
<b>Using peripherals and accessories</b> .....	<b>6</b>
Printer or bar code scanner .....	6
Triple Transducer Connect .....	7
<b>Troubleshooting</b> .....	<b>8</b>
Connectivity diagram .....	8
Cables and connectors .....	9
<b>Cleaning and disinfecting</b> .....	<b>11</b>
<b>Safety</b> .....	<b>12</b>
Electrical safety .....	12
Equipment safety .....	13
Clinical safety .....	14
Electromagnetic compatibility (EMC) .....	14
<b>Specifications</b> .....	<b>14</b>
Stand dimensions (approximate) .....	14
Temperature, humidity, and atmospheric pressure limits .....	14

## Introduction

This user guide explains how to use the Edge Stand (used with the SonoSite Edge II and Edge systems) or the H-Universal Stand and peripherals or accessories. For information on a specific peripheral, see the manufacturer's instructions included with that product. For information on the transducers or ultrasound systems, see the ultrasound system user guide.

If you need to disassemble the stand, refer to its assembly instructions<sup>7</sup> available with the stand at [www.sonosite.com](http://www.sonosite.com).

### WARNINGS

- ▶ To avoid the risk of patient injury, misdiagnosis, or user injury, read all the warnings in the ultrasound system user guide and supplemental guides.
- ▶ To avoid injury or equipment damage that could result from the stand toppling over, do not apply undue force to the stand. Positioning the stand platform at a low level may help to alleviate this potential hazard.

## Document conventions

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

Symbols and terms used on the system are explained in the ultrasound system user guide.

## Getting Help

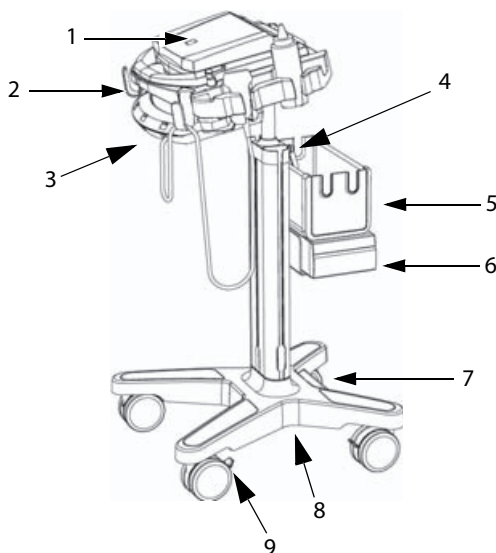
For technical support, please contact FUJIFILM SonoSite as follows:

<b>Phone (U.S. or Canada)</b>	877-657-8118
<b>Phone (outside U.S. or Canada)</b>	425-951-1330, or call your local representative
<b>Fax</b>	425-951-6700
<b>Email</b>	service@sonosite.com
<b>Web</b>	www.sonosite.com
<b>Europe Service Center</b>	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
<b>Asia Service Center</b>	+65 6380-5589

Printed in the U.S.

## Stand features

The stand provides a mobile work platform and a storage area for transducers and other supplies. It also provides connections for accessories and peripherals.

**Figure 1** Stand with accessories. H-Universal Stand shown with M-Turbo ultrasound system, and optional basket, black-&-white printer, and Triple Transducer Connect.

1	Ultrasound system	6	Black-&-white printer
2	Hook (1 of 4) for cables and other items	7	AC adapter 1: Connector for system power cord
3	Triple Transducer Connect	8	AC adapter 2: Connector for AC power cord (under stand base)
4	Height-adjustment lever	9	Locking lever on wheel
5	Basket or storage bin*		

\*Storage bin is positioned under the printer. Refer to the *Storage Bin Installation Instructions*.

## Using the stand

### To raise or lower the system platform

- 1 Make sure that the area above the platform is clear.
- 2 Facing the back of the stand, turn the height-adjustment lever counter-clockwise.

The pole may rise automatically from the column.

- 3 Raise or lower the pole to the desired height, and then tighten the lever.

To tighten the lever, turn it clockwise.

If lowering the pole, avoid grasping its base. Skin could get caught and pinched in the gap.

- 4 If you want to reposition the lever (for example, parallel to the pole), pull the lever out, turn it to the desired position, and release.

### To lock or unlock a wheel

- ❖ Push the locking lever on the wheel:
  - ▶ Push down to lock.
  - ▶ Push up to unlock.

## Inserting and removing the ultrasound system

### To insert the ultrasound system

- 1 Rotate system latches outward.
- 2 Place the ultrasound system on the system platform, ensuring that the transducer cable is positioned between the handles of the stand.
- 3 Secure by rotating the system latches inward.

### To remove the ultrasound system

- 1 Disconnect any cords, cables, or TTC attached to the system.
- 2 Rotate latches outward.
- 3 Lift the ultrasound system.

## Connecting the ultrasound system

Attach the mini-dock if you use the system with AC-powered peripherals.

### To attach the mini-dock

**Note** | Make sure that the battery is installed before you attach the mini-dock.

- 1 Turn off the ultrasound system.

The mini-dock may not function properly if you connect the system while the power is on.

- 2 Place the mini-dock on the system platform.

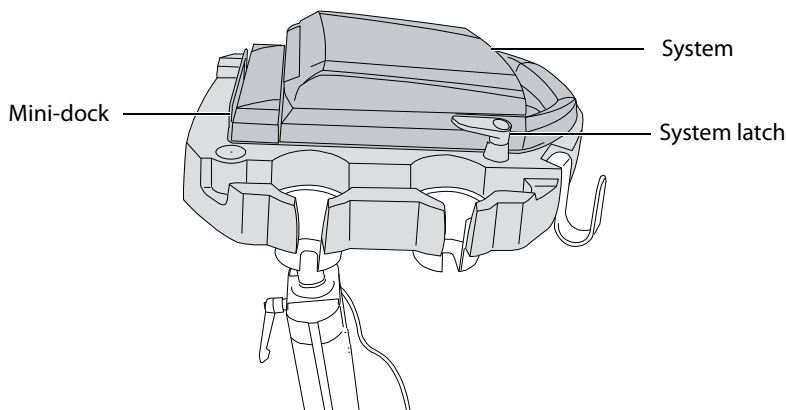
- 3 Slide the mini-dock feet under the ultrasound system and firmly press the mini-dock into the I/O connector on the back of the system.

The mini-dock is correctly attached when no gap is between the system and mini-dock.

See [Figure 2](#).

**Note** | If at any time the ultrasound system should lose connectivity, turn off the system and reattach the mini-dock and system.

Always verify that the mini-dock and system are correctly attached after changing a transducer. When you change a transducer, the mini-dock may loosen.

**Figure 2** System platform with system and mini-dock attached (H-Universal stand with M-Turbo system shown).

## To connect AC power

- 1 If using a mini-dock, ensure that it is connected to the system.
- 2 Connect the cables. See [“Cables and connectors”](#) on page 9.
  - ▶ DC power cord (A)
  - ▶ System AC power cord (K)

## Using peripherals and accessories

**Caution** | To avoid damaging the system, use only accessories and peripherals recommended by SonoSite with the ultrasound system.

For specific information on using a peripheral, including warnings and cautions, see the manufacturer’s operating instructions. For the bar code scanner, see also Bar Code Scanner User Guide.

To connect a peripheral or accessory, see its assembly instructions.

### Optional Peripherals and Accessories for Stand

#### AC-powered peripherals

- ▶ Black-and-white printer
- ▶ Color printer
- ▶ (MicroMaxx, or *TITAN*) Serial bar code scanner

#### USB peripherals

- ▶ (SonoSite Edge II, Edge, M-Turbo) USB bar code scanner
- ▶ (SonoSite Edge II, Edge, M-Turbo) USB footswitch

#### Accessories

- ▶ Mini-dock
- ▶ Triple Transducer Connect

## Printer or bar code scanner

You configure the system for the printer or serial bar code scanner on the system’s Connectivity setup page. For instructions, see the ultrasound system user guide.

For additional information on the bar code scanner, see the *Bar Code Scanner User Guide*.

### To print the screen

- 1 Turn on the printer.
- 2 (Color Printer Only) Change the input setting from S-video to video.
- 3 Do one of the following:
  - ▶ Use the controls on the printer for printing images. Follow the manufacturer’s instructions.

- ▶ If the large A or B key (SonoSite Edge II, Edge, M-Turbo) or Delta key (MicroMaxx or TITAN) is programmed to print, press it. For instructions to program the key, see the ultrasound system user guide.

You can also save images and print them from the patient list. For instructions, see the ultrasound system user guide.

## Triple Transducer Connect

The Triple Transducer Connect (TTC) is an optional three-transducer module that lets you simultaneously connect up to three transducers while the ultrasound system is docked.

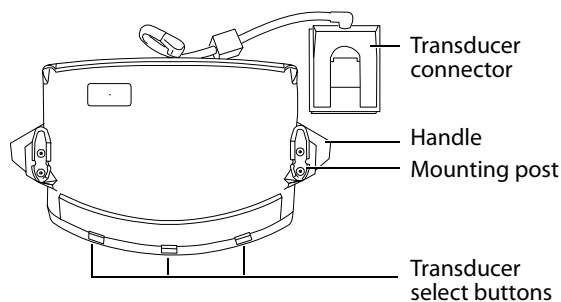
Figure 3 Triple Transducer Connect

### To connect a transducer to the TTC

- 1 Pull the transducer latch up and rotate 90°.
- 2 Align and insert the transducer connector to the connector on the bottom of the TTC.
- 3 Turn the latch and press down, securing the transducer connector to the TTC.

### To select a transducer

#### Note

When you use the TTC, there may be a minor reduction of transducer penetration.

- ❖ Push a transducer select button on the TTC.

When selecting another transducer, the system restarts. The light on the button flashes momentarily and then turns solid green.

Only one transducer is accessible at a time.

## To remove the TTC

- 1 Remove any attached transducers.
- 2 Disconnect the transducer connector from the system.
- 3 Pull out the knob on each of the two mounting brackets, and slide the TTC out.

## Troubleshooting

Symptom	Solution
Printer does not print.	<ul style="list-style-type: none"><li>▶ Check the printer selection on the Connectivity setup page. See the ultrasound system user guide.</li><li>▶ Check the cable connections. See the printer installation instructions.</li><li>▶ Ensure that the printer is turned on and set up properly. See the printer manufacturer's instructions, if necessary.</li></ul>
Stand tips over more easily than expected.	<ul style="list-style-type: none"><li>▶ Lower the platform on the stand.</li><li>▶ Avoid placing heavy objects on the platform. Too much weight on the platform can cause the stand to become less stable.</li></ul>

## Connectivity diagram

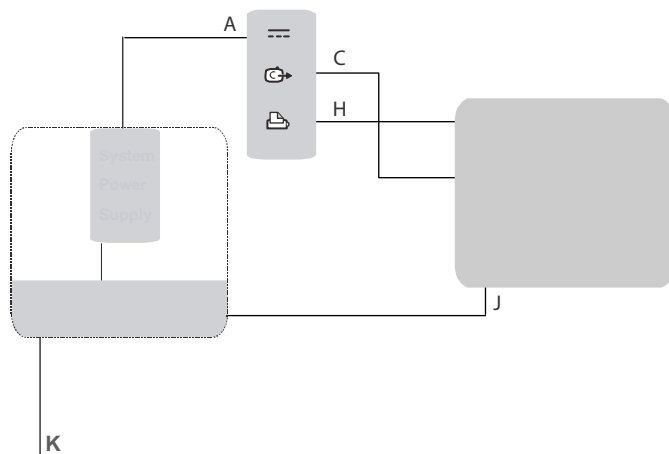
Use the following diagram for assistance disconnecting and reconnecting peripherals. For more detailed information, refer to the assembly instructions. Additional copies are available at [www.sonosite.com](http://www.sonosite.com).

See also **"Cables and connectors"** on page 9.

### Caution

To avoid damaging the system, use only accessories and peripherals recommended by SonoSite with the ultrasound system.

## Connectivity of printer and system

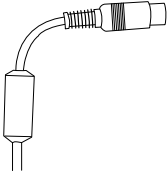
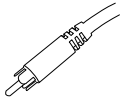
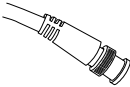
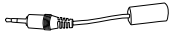
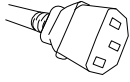
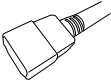
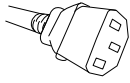
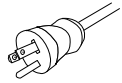
## Cables and connectors

**Table 1: Connectivity symbols on mini-docks**

Symbol	Definition	Symbol	Definition
	DC input		S-video in
	Print control		DVI video out
	USB		Composite video out
	Ethernet		Audio out
	RS-232 (DVD recorder, VCR, or bar code scanner)		ECG/Footswitch
	S-video out		

**Table 2: Cables and connectors**

<b>Letter</b>	<b>Name Length</b>	<b>Illustration</b>	<b>Connector 1</b> [[Illustration] [Connects to]	<b>Connector 2</b> [[Illustration] [Connects to]	<b>Peripheral</b>
<b>A</b>	DC power supply cord 6.8 ft./2 m	See <i>“Stand features”</i> on page 3.	 System or mini-dock	Pre-installed	n/a
<b>C</b>	Composite video cable 6 ft./1.8 m		 Mini-dock	 Printer	Printer
<b>H</b>	Print control cable 6 ft./1.8 m		 Mini-dock	 Printer	Printer
<b>J</b>	AC power cord 39 in./1 m		 Printer	 AC adapter 2 on stand base	Printer
<b>K</b>	System AC power cord 10 ft./3 m		 AC adapter 1 on stand base	 Wall outlet	n/a

## Cleaning and disinfecting

### WARNINGS

- ▶ To avoid electrical shock, before cleaning always disconnect the system from the power supply and remove it from the stand.
- ▶ To avoid injury, always use protective eyewear and gloves when performing cleaning and disinfecting procedures.
- ▶ To avoid infection:
  - ▶ Ensure that the solution expiration date has not passed.
  - ▶ The level of disinfection required for a product is dictated by the type of tissue it contacts during use. Ensure that the solution strength and duration of contact are appropriate for the equipment. For information, see the disinfectant label instructions and the recommendations of the Association for Professionals in Infection Control and Epidemiology (APIC) and FDA.

### Caution

Use only recommended cleaners or disinfectants on surfaces. Immersion-type disinfectants are not tested for use on surfaces.

To clean and disinfect a peripheral, refer to its manufacturer's instructions. For the bar code scanner, see also the *Bar Code Scanner User Guide*.

### To clean and disinfect the stand or TTC

You can clean and disinfect the exterior surface of the stand or TTC using a recommended cleaner or disinfectant. For the most recent list of approved products, see [www.sonosite.com](http://www.sonosite.com).

- 1 Turn off the ultrasound system and any accessory equipment. Disconnect the system and accessory equipment from AC power.
- 2 Clean the surfaces of the stand to remove any debris or bodily fluids. Use the following procedure:

- a** Use either a pre-moistened wipe or a soft cloth dampened with cleaner or disinfectant.

### Approved cleaners and disinfectants for the stand

Cleaner/disinfectant	Minimum wet contact time <sup>1</sup>
SaniCloth AF3 (gray top) <sup>2</sup>	3 minutes
SaniCloth Plus (red top)	3 minutes
PI-Spray II	10 minutes

<sup>1</sup> For maximum effectiveness, the component being cleaned must remain wet with disinfectant for a minimum period of time.

<sup>2</sup> Qualified for use as an intermediate-level disinfectant for mycobacteria. For a more complete list, refer to the cleaners and disinfection document available at [www.sonosite.com](http://www.sonosite.com)

- b** Remove all gel, debris, and bodily fluids from the stand.
- c** With a new wipe, clean all parts of the stand by wiping from the clean areas to the soiled areas. This method helps to avoid cross-contamination.
- d** Observe the minimum wet contact time. Monitor the stand parts for wet appearance. Re-apply with a new wipe if no longer wet.
- 3** Allow the stand to air dry in a clean, well-ventilated space.

For information about cleaning the peripherals, see the manufacturer's instructions.

## Safety

Observe the following warnings and cautions before using accessories and peripherals with the ultrasound system and transducers.

### Electrical safety

The stand is not an electrical device and does not require ground bond testing. The stand is for facilitating the mobility of peripherals and their connection to the ultrasound system.

**WARNINGS**

- ▶ To avoid the risk of electrical shock:
  - ▶ Do not plug the power cord of the Edge Stand or H-Universal Stand into a multiple portable socket outlet (MPSO) or use an extension cord.
  - ▶ Use only properly grounded equipment. Shock hazards exist if the power supply is not properly grounded. Grounding reliability can only be achieved when equipment is connected to a receptacle marked “Hospital Only” or “Hospital Grade” or the equivalent. The grounding wire must not be removed or defeated.
  - ▶ Use only accessories and peripherals recommended by SonoSite, including the power supply. Connection of accessories and peripherals not recommended by SonoSite could result in electrical shock. Contact SonoSite or your local representative for a list of accessories and peripherals available from or recommended by SonoSite.
  - ▶ To avoid the risk of electrical shock or injury, the equipment shall be connected to a center-tapped single phase supply circuit when users in the United States connect the equipment to a 240V supply system.
- ▶ To avoid the risk of electrical shock and fire hazard, inspect the power supply, plugs, cables, and power cords on a regular basis for damage.

## Equipment safety

**Cautions**

- ▶ Excessive bending or twisting of cables can cause a failure or intermittent operation.
- ▶ Improper cleaning or disinfecting of any part of the accessories and peripherals can cause permanent damage. For cleaning and disinfecting instructions, see **“Cleaning and disinfecting”** on page 11 and the manufacturer’s operating instructions.
- ▶ Accessible metal on the signal inputs and outputs on the back and side of the ultrasound system is not protectively earthed. Do not perform a high current ground impedance test involving this part.

## Clinical safety

**WARNING:**

To avoid misdiagnosis, do not use non-medical (commercial) grade peripheral monitors. These monitors have not been verified or validated by SonoSite as being suitable for diagnosis.

## Electromagnetic compatibility (EMC)

See the safety section of the ultrasound system user guide and supplemental guides for information on electromagnetic compatibility with IEC 60601-1-2, including the Manufacturer's Declaration.

## Specifications

For information on the peripherals, see the manufacturer's specifications.

### Stand dimensions (approximate)

Width: 20.25 in. (51.4 cm)

Depth: 22 in. (55.9 cm)

Height: 45 in. (114.3 cm) maximum, 33 in. (83.8 cm) minimum

Height range: 12 in. (30.5 cm)

Weight: 40.0 lbs. (18.1 kg)

Safe working load per storage bin: 5 kg

Total stand weight with system and peripherals: 50 kg maximum

### Temperature, humidity, and atmospheric pressure limits

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