Intracavity Transducer (ICT) Transducer Care

Step 1 of 2: Clean the transducer

WARNINGS

- ▶ Be sure to use the appropriate Personal Protective Equipment (PPE) whenever necessary.
- ▶ **DO NOT** skip any steps or abbreviate the cleaning process in any way.


Caution

- ▶ **DO NOT** allow bodily fluids to dry on the transducer. Always clean the transducer immediately after use.

Table 1: Compatible cleaners and disinfectants for cleaning the system and transducer

Cleaner	Wet Contact Times
SaniCloth Plus (red top)	3 minutes
SaniCloth AF3 (grey top)	3 minutes
PI-Spray II	10 minutes

Immediately after extracting the device from the patient, wipe the transducer from the cable to the scanhead, with a pre-moistened wipe from [Table 1, "Compatible cleaners and disinfectants for cleaning the system and transducer"](#) or a cloth dampened with cleaner. Be sure to remove all visible biological material.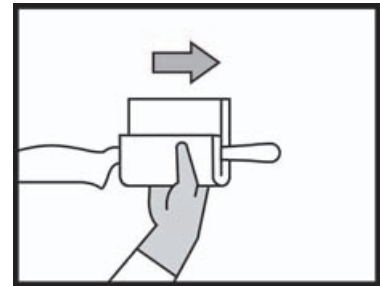

Caution

- ▶ **DO NOT** use unapproved cleaning agents.
- ▶ **DO NOT** spray or immerse the connector with any fluid.

Note

- ▶ After thoroughly cleaning, proceed to disinfection in step 2 of 2.

Step 2 of 2: Disinfect and store the transducer

WARNINGS

- ▶ Be sure to use the appropriate Personal Protective Equipment (PPE) whenever necessary.
- ▶ **DO NOT** skip any steps or abbreviate the disinfection process in any way.
- ▶ **DO NOT** use unapproved disinfecting agents.

Cautions

- ▶ **DO NOT** immerse the connector in any fluid.
- ▶ **DO NOT** soak the transducer longer than recommended by the chemical manufacturer.

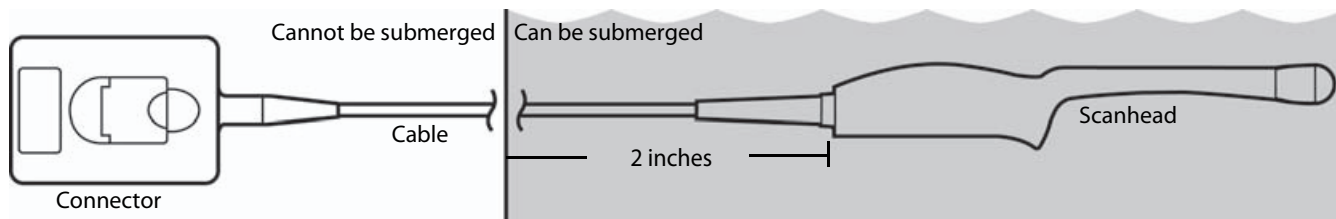
Table 2: Approved high-level disinfectants and soak times

Disinfectant	Temperature	Disinfectant Soak Duration	Clean Water Soak Time
Cidex	23° C (73° F)	45 minutes	3 x 1 minute

If using an automated disinfection process, follow manufacturer procedures. For manual disinfection, do the following:


At the disinfection station:

A

Check the expiration of the disinfectant.

Check the temperature and concentration of the solution.

Disinfect the transducer and cable by soaking in the disinfection fluid.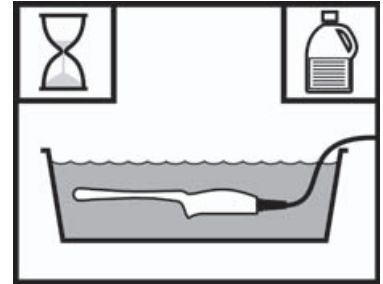


Rinse the transducer by soaking it for at least one minute, three separate times, in a large volume of sterile or filtered water (at least two liters).

B

Discard the rinse water after each rinse.

WARNING


Chemical disinfectants can cause harm to the patient if not completely removed.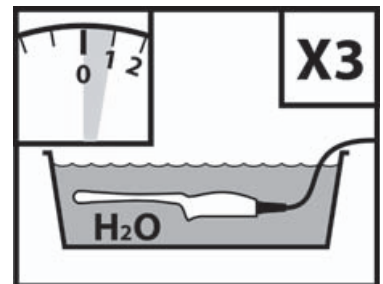

Dry the transducer and cable with a sterile, non-linting towel. Store the transducer in accordance with your organization's policies. For more information on cleaning, disinfecting, transporting, and storing the ICT transducer, refer to the ultrasound system user guide.

C

Inspect the transducer for damage, such as cracks or splitting that might allow fluid to enter.

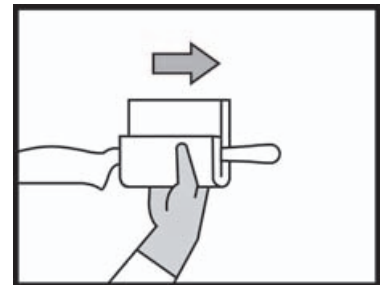Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

Federal (United States) law restricts this device to sale by or on the order of a physician.

SonoSite and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Patents: US 6,371,918, CA 2,373,065, DE 60021552.0, FR 1175173 and GB 1175173.

Copyright © 2017 FUJIFILM SonoSite, Inc. All Rights reserved.

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