GETTING STARTED GUIDE

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite X-Porte, PowerPark, SonoSite, and the SonoSite logo are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Patents: US 9,162,255; US 9,151,832; US 9,113,825; US 8,956,296; US 8,876,719; US 8,861,822; US 8,858,436; US 8,834,372; US 8,805,047; US 8,527,033; US 8,500,647; US 8,439,840; US 8,355,554; US 8,216,146; US 8,213,467; US 8,147,408; US 8,137,278; US 8,088,071; US 8,066,642; US 8,052,606; US 7,819,807; US 7,804,970; US 7,740,586; US 7,686,766; US 7,604,596; US 7,591,786; US 7,588,541; US 7,534,211; US 7,449,640; US 7,169,108; US 6,962,566; US 6,648,826; US 6,575,908; US 6,569,101; US 6,471,651; US 6,416,475; US 6,383,139; US 6,364,839; US 6,203,498; US 6,135,961; US 5,893,363; US 5,817,024; US 5,782,769; US 5,722,412; US D712539; US D712038; US D712037; AU: 730822; AU: 727381; CA: 2,372,152; CA: 2,371,711; CN: 103237499; CN: 102753104; CN: 101231457; CN 98108973.9; CN: 98106133.8; CN: 97113678.5; DE: 69831698.3; DE: 69830539.6; DE: 69730563.5; DE: 602004027882.3; DE: 602004023816.3; DE: 60034670.6; DE: 60029777.2; EP: 1589878; EP: 1552792; EP: 1180971; EP: 0875203; EP: 0815793; EP 1180970; EP 0881492; ES: 2229318; ES: 159878; ES: 1552792; ES: 0881492; FR: 158978; FR: 1552792; FR: 1180970; FR: 0881492; FR: 0875203; FR: 0815793; GB: 158978; GB: 1552792; GB: 1180971; GB: 1180970; GB: 0881492; GB: 0875203; GB: 0815793; IT: 1589878; IT: 1552792; IT: 0881492; IT: 0815793; JP: 5782428; JP: 4696150; KR: 532359; KR: 528102; NO: 326814; NO: 326202.

Part Number: P14584-06

Publication date: November 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

SonoSite X-Porte Getting Started Guide

Overview	1
Document conventions	1
Getting Help	2
Unpacking the system	2
Setting up the system	7
Charging the batteries and connecting the system to power	9
Connecting a transducer	11
Turning the system on and off	13
Basic operating instructions	14

Overview

The *SonoSite X-Porte Getting Started Guide* explains how to unpack, set up, and begin using the SonoSite X-Porte ultrasound system.

For more in-depth instructions on how to use the system, see either:

- ▶ SonoSite X-Porte on-board help, available on the system. To access Help on the system, tap **MORE**, and then tap **Help**.
- ▶ *SonoSite X-Porte User Guide*, available on the CD included with the system and on the FUJIFILM SonoSite website (www.sonosite.com).

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.

For a description of labeling symbols that appear on the product, see “Labeling Symbols” in the *SonoSite X-Porte User Guide*.

Getting Help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.


Unpacking the system

Open the box


The packing box includes an accessory box, a PowerPark docking station (if ordered), and a crate that holds the system in place.

- 1 Remove the straps from the packing box.
- 2 Raise the lid on the packing box. Avoid damaging the lid, as it will be used later as a ramp.
- 3 Remove the tape from the accessory box.

- 4** Lift the accessory box out of the packing box, and set it aside.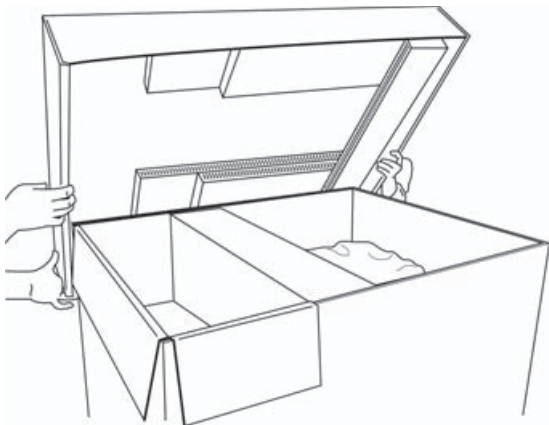

- 5** Remove the sides of the packing box.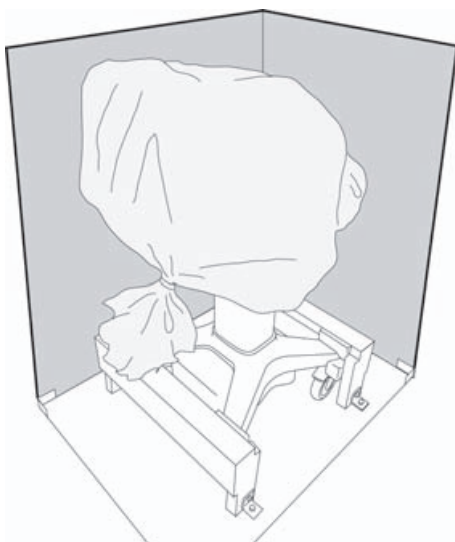


- 6** If a PowerPark docking station is included with the system, remove the PowerPark box:

- a** Remove the strap from the PowerPark box on the back of the crate.

- b** Remove the PowerPark box, and carry it to the PowerPark installation location.

Notes

See the PowerPark Assembly Instructions, inside the PowerPark box, for instructions on assembling the PowerPark docking station.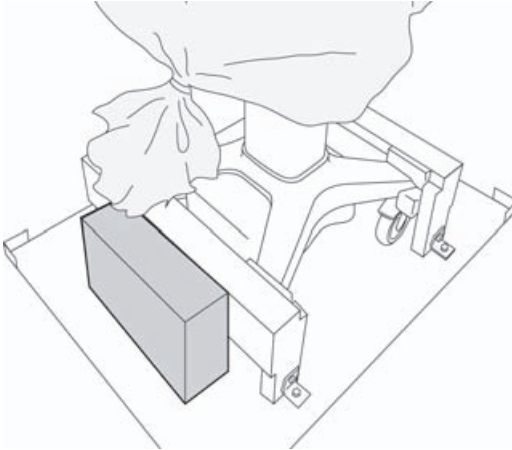


Set up the ramp


The crate lid can be used as a ramp that helps you remove the system from the crate.

- 1** With a box cutter, cut all four corners of the lid so that it lays flat.
- 2** With the corrugated ramp supports on the bottom, place the highest end of the ramp next to the base of the crate.

- 3 Insert the end flaps of the lid into the metal brackets.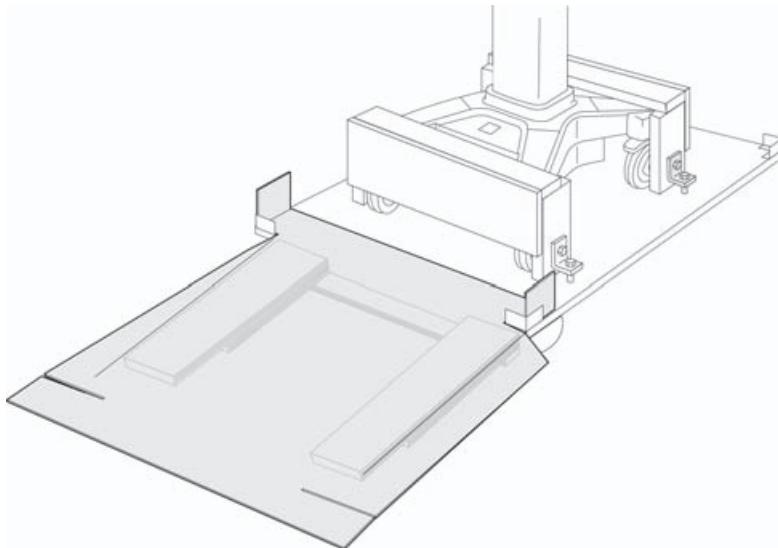


- 4 Using a 3/8-inch wrench, loosen and remove the wheel clamps.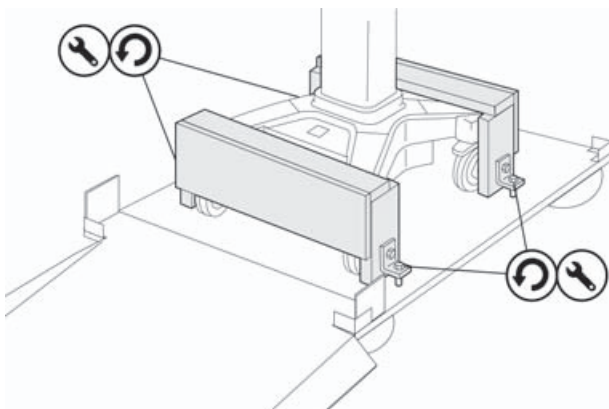

Remove the system

- 1 To unlock the wheels, press up on the rear of the locking lever on the wheels.


- 2 Roll the system down the ramp.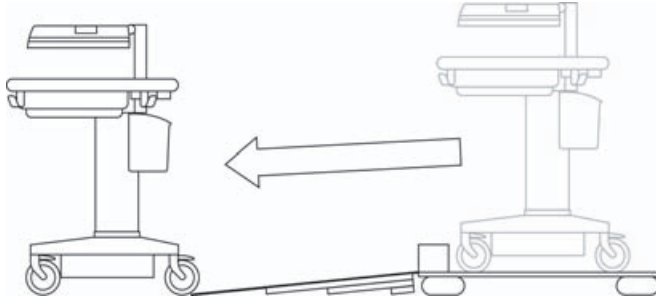

- 3 Remove the plastic bag and packaging materials from the system, and inspect the system for damage.
- 4 Wheel the system to its intended destination. Push the bar on the front of the system to move it.


Caution

To avoid tipping, when transporting the system, always push the system using the bar on the front. Do not move the system by pushing or pulling on any other components, such as the clinical monitor.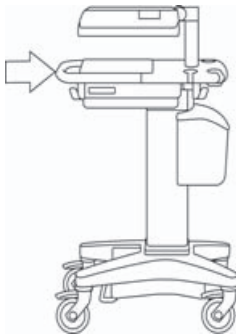


- 5 When the system is at its desired location, lock the wheels by pressing down on the lever.

Setting up the system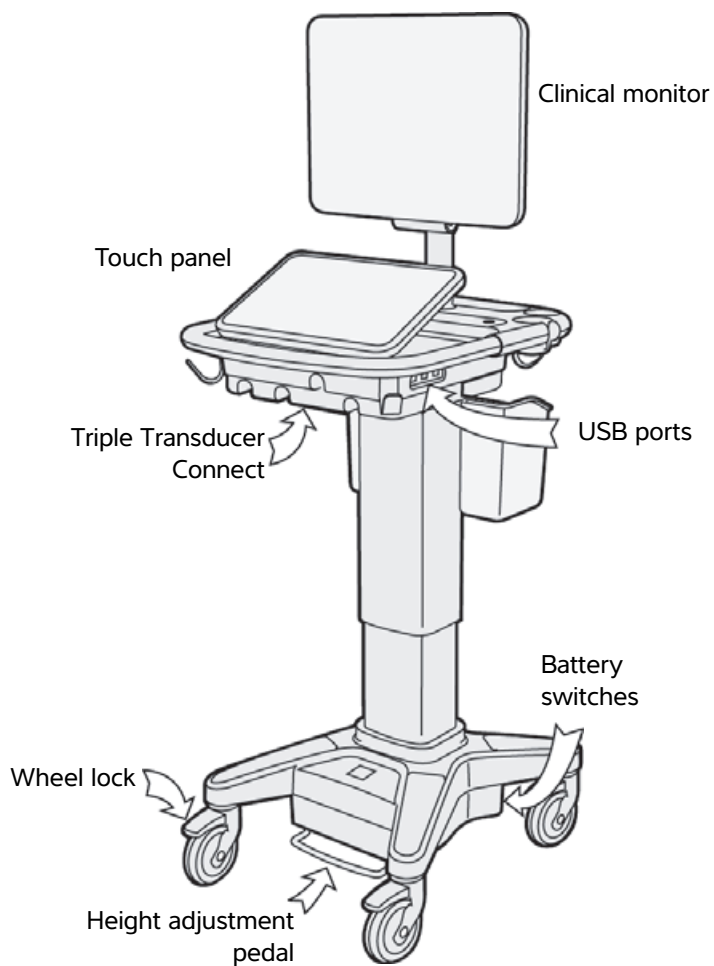

- 1 Press the height-adjustment pedal to raise the stand.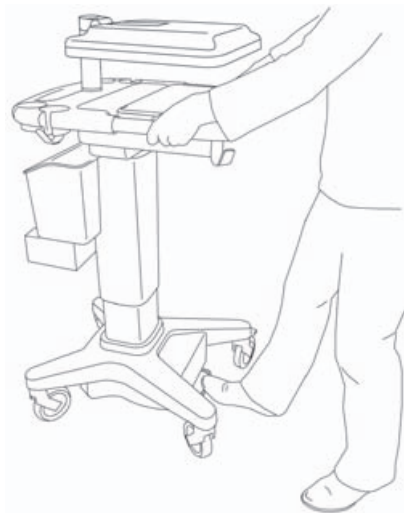

- 2 Grasp the clinical monitor on both sides and tilt it upward.


- 3 Grasp both sides of the touch panel and pull it forward or push it backward to the desired angle.


Charging the batteries and connecting the system to power

- 1 Remove the yellow paper tab in front of the battery compartment door.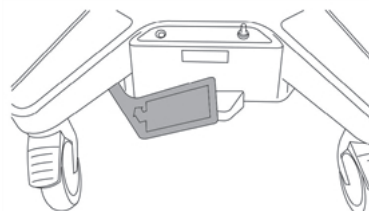

- 2 Open the battery compartment door located at the base of the stand.

The following figure shows all three switches depressed to the  symbol, which is the OFF position.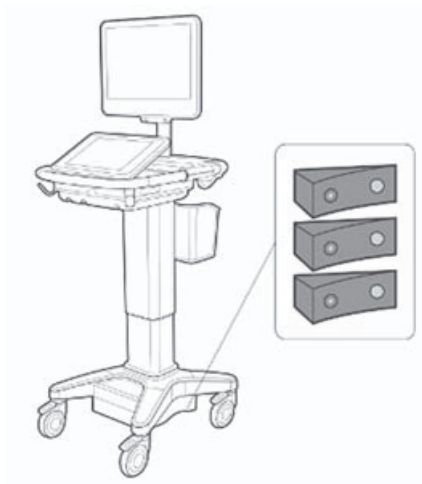


- 3 Depress all three battery switches to the  symbol, which is the ON position.
- 4 Close the battery compartment door. Press the door to ensure that it is locked.

WARNINGS


- ▶ Verify that the hospital supply voltage corresponds to the power supply voltage range.
- ▶ Plug the system only into a grounded hospital-grade outlet.
- ▶ Use only power cords provided by FUJIFILM SonoSite with the system.

- 5 Connect the system to AC power. Use one of the following methods, depending on your configuration:

- a** Connect the AC power cord, located in the accessory box, to the stand, and then connect the AC power cord to a hospital-grade outlet. The battery will charge whenever the system is connected to AC power. The location of the AC power cord socket may depend on the accessories that are installed on the system.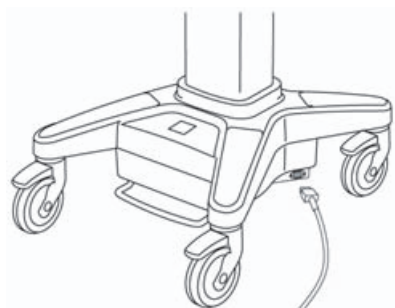


PowerPark installed

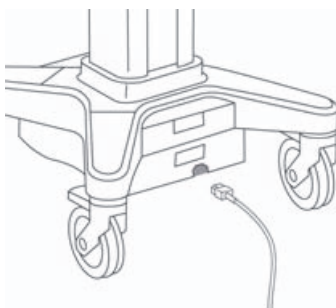No PowerPark

- b** Wheel the SonoSite X-Porte system backward into the PowerPark dock.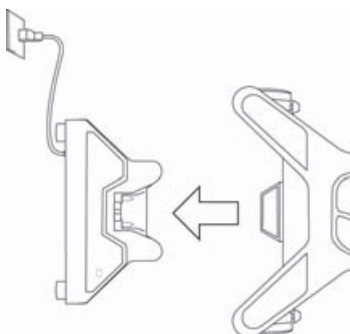

- 6** While connected to AC power, you can turn on the system and use it immediately. Before using the system on battery power for the first time, be sure to charge the battery for a minimum of four hours.

Connecting a transducer

The Triple Transducer Connect (TTC) is a three-transducer module that lets you simultaneously connect up to three transducers to the ultrasound system. The TTC is standard with the system.


Notes

If the TTC is not installed on your system, you can connect one transducer directly to the SonoSite X-Porte system.


To connect a transducer to the TTC

- 1** Pull the transducer latch up, and rotate it clockwise.


- 2** Insert the transducer connector into one of the three TTC connectors on the bottom of the TTC.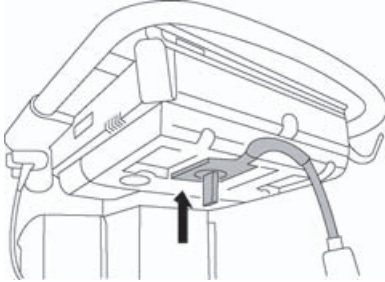

- 3** Make sure that the connector is firmly seated, and then turn the latch handle counterclockwise.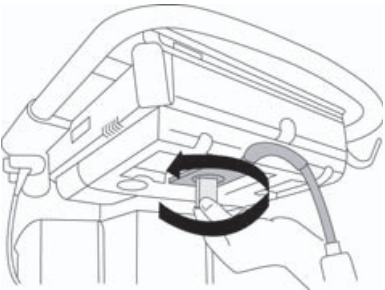

- 4** Press the latch handle up, securing the transducer connector to the TTC.

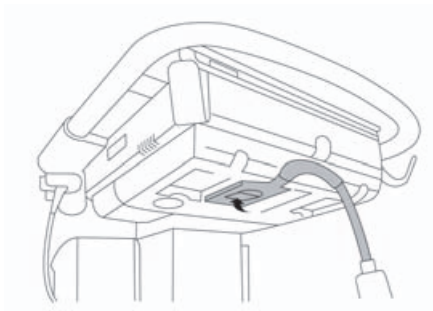Turning the system on and off

WARNINGS

- ▶ Verify that the hospital supply voltage corresponds to the power supply voltage range.
- ▶ Plug the system only into a grounded hospital-grade outlet.
- ▶ Use only power cords provided by FUJIFILM SonoSite with the system.

The system can be powered by the internal battery or by AC power.

To turn the system on

- 1 If you are operating the system using AC power, connect the AC power cord to the stand, and then connect the AC power cord to a hospital-grade outlet.
- 2 Press the power button.

The power button turns green when the system is ready for scanning.

Cautions

- ▶ Do not use the system if an error message appears on the clinical monitor. Note the error code and turn off the system. Call FUJIFILM SonoSite or your local representative.
- ▶ When using AC power, position the system to allow easy access to disconnect it.

Connect the system to AC power to maintain battery charge, especially if the system will not be used for several days.

To turn the system off

- ❖ Press the power button.

Note

Restarting the system while it is performing data-intensive background activities, such as transferring patient files, can result in loss of patient data. FUJIFILM SonoSite recommends waiting at least three minutes before restarting an unresponsive system. To power down an unresponsive system, press and hold the Power button until the system shuts down. This procedure may take five seconds or longer.

Basic operating instructions

Notes

For detailed instructions on using the SonoSite X-Porte system, please refer to one of the following sources:

- ▶ SonoSite X-Porte on-board help, which is available on the system. To access help on the system, tap **MORE**, and then tap **Help**.
- ▶ *SonoSite X-Porte User Guide*, included with the system and on the FUJIFILM SonoSite website (www.sonosite.com)

To use the system

perform these basic operating steps:

- 1 Connect a transducer.
- 2 Turn on the system.
- 3 Select the transducer and exam type (or use the default selections).
- 4 (Optional) Enter patient information.
- 5 Scan.

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