SonoSite X-Porte

User Guide Errata

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SONOSITE, X-Porte, and the SONOSITE logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P21471-02

Publication date: December 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

0086

SonoSite X-Porte User Guide Errata

Introduction	1
Document conventions	1
Getting help	2
Getting started	2
Setting up the system	3
USB Settings	3
eFilm Lite image-Viewer	3
Measurement references	3
Cardiac references	3
Cleaning and disinfecting	4
Specifications	4
Dimensions	4

Introduction

The content in this document was either not included or incorrect in the system user guide; the revisions will be made in the next update.

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the system user guide.

Getting help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

The following content was either not included or incorrect in the system user guide; the revisions will be made in the next update.

Getting started

The following caution has been added.

Caution

When performing a search in the system Help, do not enter the word “and” following your search word. The system locks up and requires a restart.

Setting up the system

USB Settings

Information about the eFilm Lite image-viewer has been removed.

To specify export options

- 1 On the **USB** settings page, select an **Export Type**:
 - ▶ **DICOM Export** creates DICOMDIR files readable by a DICOM reader. DICOM is an optional feature.
 - ▶ **Multimedia Export** organizes files in a standard folder structure. Video clips export in MJPEG video saved as AVI. To view them, FUJIFILM SonoSite recommends Windows Media Player or QuickTime 7.0 or later.
- 2 Select an **Image Format** for your export type. For JPEG image format, also select a JPEG compression. A high compression has a smaller file size but less detail (see “Limitations of JPEG format”). The image format affects both still images and video clips.

For optimal DICOM image quality, select RGB image format and low compression.
- 3 (**Multimedia Export** only) Select a sort order from the **Sort By** list.
- 4 Select **Auto Export** to enable automatic export of ended exams. See also “To export patient exams automatically to a USB memory stick.”
- 5 (**DICOM Export** only) Select any of the following:
 - ▶ **Include CDA** to export the CDA report. See “CDA Report settings.”
 - ▶ **Include SR** to export the Structured Report.
- 6 Tap **Done**.

eFilm Lite image-Viewer

This section has been removed.

Measurement references

Cardiac references

The following reference has been added.

Left Atrial Area

Lang, R. et al. "Recommendations for Cardiac Chamber Quantification by Echocardiography in Adults: An Update from the American Society of Echocardiography and the European Association of Cardiovascular Imaging." *Journal of the American Society of Echocardiography*. (2015), 28: p. 25-26.

Cleaning and disinfecting

The following website was corrected.

www.sonosite.com/products/transducers

Specifications

Dimensions

Stand

	<u>First configuration</u>	<u>Second configuration</u>
Length	26.4 in. (67.1 cm)	26.4 in. (67.1 cm)
Width	21.2 in. (58.3 cm)	21.2 in. (58.3 cm)
Height (max with monitor up)	64 in. (162.6 cm)	65 in. (165.6 cm)
Height (min with monitor down)	42.2 in. (107.2 cm)	42.2 in. (107.2 cm)

Clinical monitor

	<u>First configuration</u>	<u>Second configuration</u>
Length (exterior dimension)	18.8 in. (47.8 cm)	20.2 in. (51.4 cm)
Height (exterior dimension)	14.2 in. (36 cm)	14.33 in. (36.4 cm)
Diagonal (viewable area)	19 in. (48.3 cm)	19.9 in. (50.6 cm)
Resolution	1680 x 1050	1728 x 1080

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