V-UNIVERSAL STAND

User Guide

CE

Manufacturer

FUJIFILM SonoSite, Inc.

21919 30th Drive SE
Bothell, WA 98021
USA
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite, Ltd.

Alexander House
40A Wilbury Way
Hitchin
Herts SG4 0AP
UK
T: +44-1462-444800
F: +44-1462-444801

Caution: | United States federal law restricts this device to sale by or on the order of a physician.

NanoMaxx, S Series, V-Universal, SonoSite, and the SonoSite logo are trademarks or registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

P09557-02 1/2013

Copyright © 2013 by FUJIFILM SonoSite, Inc.

All rights reserved.

V-Universal Stand User Guide

Contents

Introduction	1
Stand features	2
Stand basics	4
Attaching the ultrasound system	4
Removing the ultrasound system	6
Using the printer	6
Using the Triple Transducer Connect	7
Using the DVD recorder	8
Troubleshooting	9
Cleaning and disinfecting	10
Safety	10
Specifications	12

Introduction

The *V-Universal Stand User Guide* explains how to use the V-Universal Stand and peripherals. For information on a peripheral, see the manufacturer's instructions included with that product. For information on the transducers, NanoMaxx ultrasound system, or S Series ultrasound system, see the ultrasound system user guide.

If you need to disassemble the stand, refer to the V-Universal Stand assembly instructions, available with the stand and at www.sonosite.com.

- WARNING:** To avoid the risk of patient injury, misdiagnosis, or user injury, read all the warnings in the ultrasound system user guide and supplemental guides.
- WARNING:** To avoid injury or equipment damage that could result from the stand toppling over, do not apply undue force to the stand. Positioning the stand platform at a low level may help to alleviate this potential hazard.
- WARNING:** To avoid possible injury or equipment damage, place the system in the lowest possible height on the stand before system transport.

Conventions

The user guide follows these conventions:

- A **WARNING** describes precautions necessary to prevent injury or loss of life.
- A **Caution** describes precautions necessary to protect the products.
- Numbered steps in procedures must be performed in order.
- Items in bulleted lists do not require a sequence.
- Single-step procedures begin with ❖.

Symbols and terms used on the system are explained in the ultrasound system user guide.

Customer assistance and comments

Questions and comments are encouraged. SonoSite is interested in your feedback regarding the system and the user guide. Please call SonoSite at 888-482-9449 in the U.S. Outside the U.S., call the nearest SonoSite representative.

For technical support, please contact SonoSite as follows:

SonoSite Technical Support

Phone (U.S. or Canada):	877-657-8118
Phone (Outside U.S. and Canada):	425-951-1330 Or call your local representative.
Fax:	425-951-6700
E-mail:	service@sonosite.com
Web site:	www.sonosite.com

Europe Service Center

Phone:	+44-(0)1462-444-800
E-mail:	uk.service@sonosite.com

Stand features

The stand provides a mobile work platform and a storage area for transducers and other supplies. It also provides connections for accessories and peripherals.

WARNING: To avoid the risk of electrical shock or injury, the equipment shall be connected to a center-tapped single-phase supply circuit when users in the United States connect the equipment to a 240V supply system.

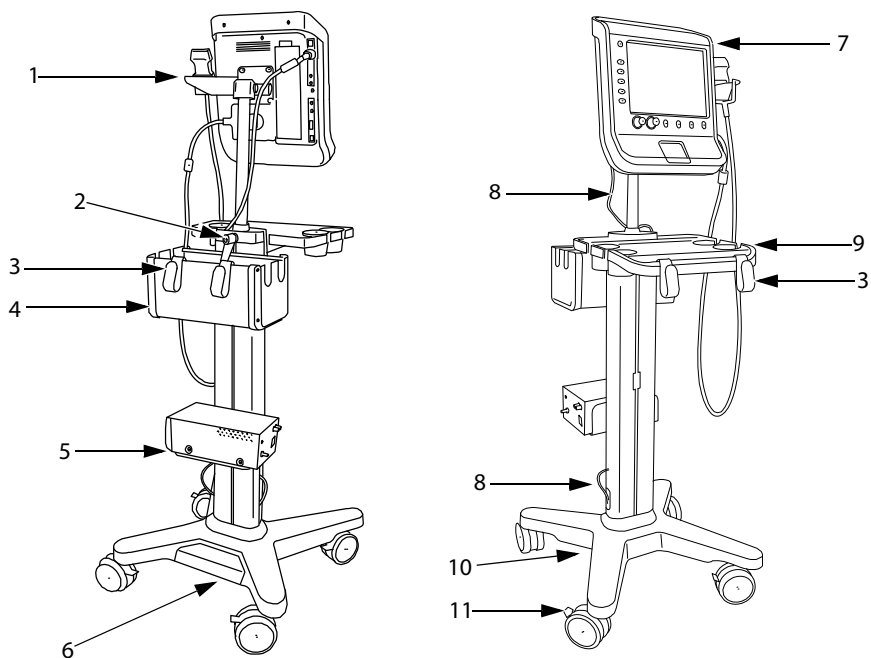
Figure 1 V-Universal Stand: Back and front views (shown with S Series ultrasound system and optional printer, basket, and tray)

1	Transducer holder
2	Height-adjustment lever
3	Hooks for cables and other items (included with tray and basket)
4	Basket (optional)
5	Shelf for black-and-white printer (optional)
6	AC adapter 1: Connector for system power cord
7	Ultrasound system
8	DC power cord
9	Tray (optional)
10	AC adapter 2: Connector for AC power cord (under stand base)
11	Locking lever on wheel

Stand basics

To raise or lower the ultrasound system

- 1 Make sure that the area above the column is clear.
- 2 Turn the height-adjustment lever counter-clockwise.
The pole may rise automatically from the column.
- 3 Raise or lower the pole to the desired height, and then tighten the lever.
To tighten the lever, turn it clockwise.
If lowering the pole, avoid grasping its base. Skin could get caught and pinched in the gap.
- 4 If you want to reposition the lever (for example, parallel to the pole), pull the lever out, turn it to the desired position, and release.

To lock or unlock a wheel

- ❖ Push the locking lever on the wheel:
 - Push down to lock.
 - Push up to unlock.

To adjust the system angle

- 1 Lock two or more wheels.
- 2 Using both hands, rotate the system up and down or side to side.

Attaching the ultrasound system

The transducer holder attaches to the system before the system attaches to the stand mounting plate.

To attach the transducer holder

- 1 Place the transducer holder on the back of the system, with the cup facing toward the front.
For NanoMaxx, the small end of the cup points upward.
- 2 Using a Phillips screwdriver, tighten the four screws. See [Figure 2](#).
For NanoMaxx, the holes for the screws are offset.

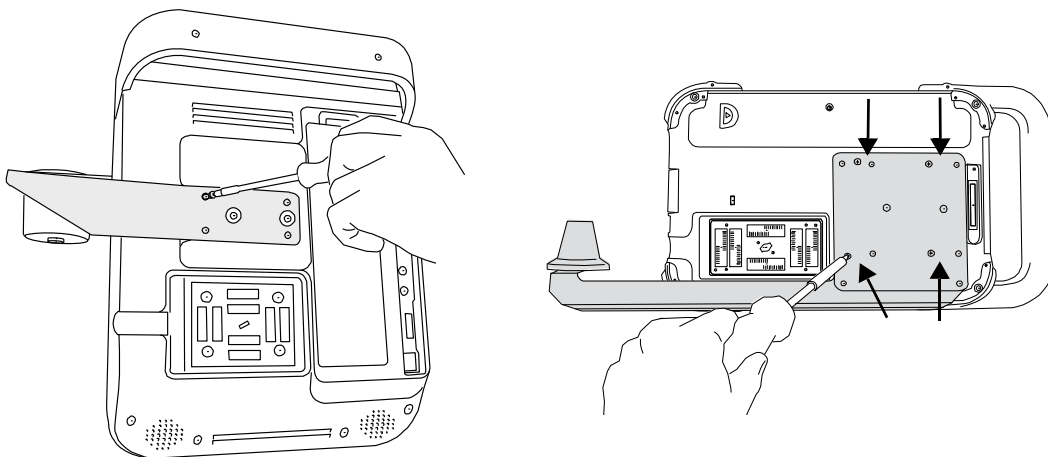
Figure 2 Turn screws on transducer holder: S Series (on left) and NanoMaxx (on right)

To attach the ultrasound system

- 1 (NanoMaxx only) Connect the NanoMaxx dock or power-only adapter to the system.
- 2 Attach the transducer holder to the system.
- 3 Place the back of the system onto the mounting plate.

The two pegs on the mounting plate pass through the transducer holder and, on the S Series system, into the two large holes on the system.

- 4 Turn the screw in each of the four corners into the system until the system is secure. Tighten with a Phillips screwdriver. See [Figure 3](#).

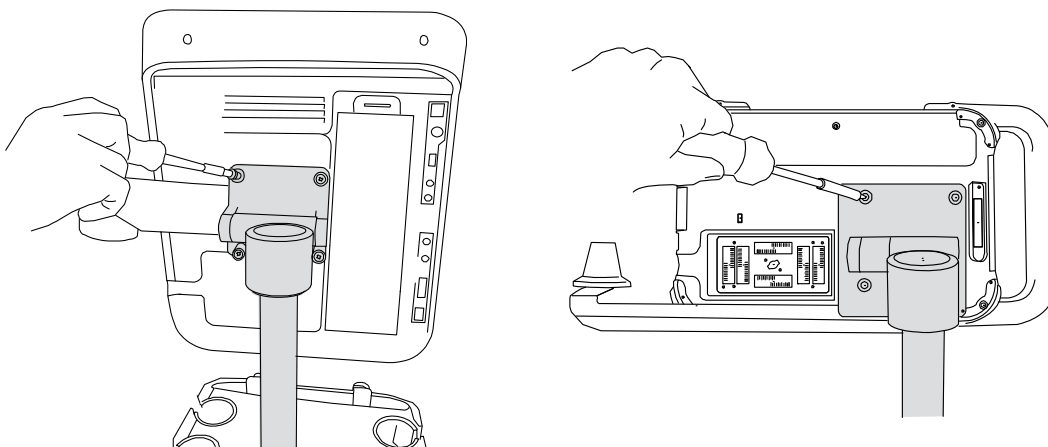
Figure 3 Turn screws on mounting plate: S Series (on left) and NanoMaxx (on right)

To connect AC power

- ❖ Connect the following cords. See [Figure 1](#) on page 3.
 - DC power cord
 - System AC power cord

Removing the ultrasound system

To remove the ultrasound system

- 1 Support the ultrasound system, and loosen the screw in each of the four corners of the mounting plate. If necessary, use a Phillips screwdriver. See [Figure 3](#).

Supporting the system ensures that it doesn't drop when loosened.

- 2 Lift the system off the stand.

The system lifts off the two mounting-plate pegs.

To remove the transducer holder

- ❖ Using a Phillips screwdriver, loosen the four screws on the back of the transducer holder. See [Figure 2](#).

Using the printer

WARNING: To avoid the risk of electrical shock or injury, do not connect more than one peripheral at a time.

Caution: To avoid damaging the system, use only peripherals recommended by SonoSite with the ultrasound system.

Printer

The NanoMaxx and S Series ultrasound systems support the printer.

For specific information on using the printer, including warnings and cautions, see the manufacturer's operating instructions.

To configure the system for the printer, see the ultrasound system user guide. You configure the system on the Connectivity setup page.

To connect the printer, see the V-Universal Stand assembly instructions.

To print the screen

Note: Test the printer before a patient exam to ensure that all connectivity is correct.

- ❖ Use the controls on the printer for printing images. Follow the manufacturer's instructions.

You can also save images and print them from the patient list. For instructions, see the ultrasound system user guide.

Using the Triple Transducer Connect

The S Series ultrasound system supports the Triple Transducer Connect (TTC). The TTC is an optional three-transducer module that lets you simultaneously connect up to three transducers while the ultrasound system is docked.

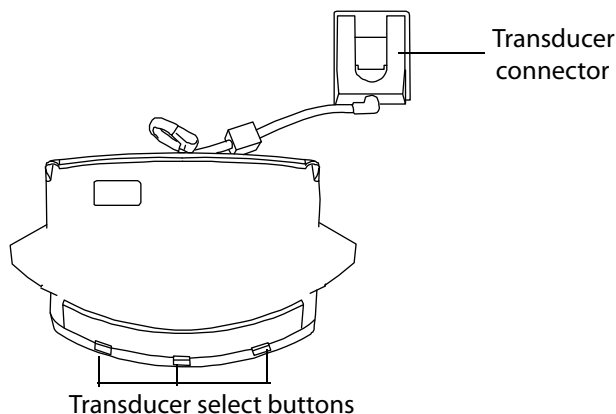
Figure 4 Triple Transducer Connect

To connect a transducer to the TTC

- 1 Pull the transducer latch up and rotate 90°.
- 2 Align and insert the transducer connector to the connector on the bottom of the TTC.
- 3 Turn the latch and press down, securing the transducer connector to the TTC.

To select a transducer

Note: When you use the TTC, there may be a minor reduction of transducer penetration.

- ❖ With the system turned on, push a transducer select button on the TTC to select the desired transducer.

When selecting another transducer, the light on the button flashes momentarily and then turns solid green.

The transducer appears in the system status section of the screen, and the system selects the default exam type for the selected transducer.

Only one transducer is accessible at a time.

To remove the TTC for service

- 1 Remove any attached transducers.
- 2 Disconnect the transducer connector from the system.
- 3 Pry open the snap covers on the cover of the TTC with a flathead screwdriver.

- 4 Hold the bottom of the TTC to support it and use the 5/32" (larger) Allen wrench to remove the four screws beneath the snap covers.

Using the DVD recorder

WARNING: To avoid the risk of electrical shock or injury, do not connect more than one peripheral at a time.

Caution: To avoid damaging the system, use only peripherals recommended by SonoSite with the ultrasound system.

Note: As of July 2012, the DVD recorder is not supported with the V-Universal stand.

The S Series ultrasound system supports the DVD recorder.

For specific information on using the DVD recorder, including warnings and cautions, see the manufacturer's operating instructions.

To configure the system for the DVD recorder, see the ultrasound system user guide. You configure the system on the Connectivity setup page.

To connect the DVD recorder, see the V-Universal Stand assembly instructions.

To record images

Note: Test the recorder before a patient exam to ensure that all connectivity is correct.

- 1 Turn on the DVD recorder.
- 2 Insert a recordable DVD.
- 3 Use controls on the DVD recorder to start and stop recording.

Each time you record, a new file saves to the DVD.

To play back images

- 1 On the ultrasound system, hold down the touchpad key, and press the lower-right control key.
- 2 Press PLAY on the DVD recorder.

For more information, see the manufacturer's instructions.

Troubleshooting the DVD recorder

Symptom	Solution
DVD recorder does not record	<ul style="list-style-type: none"> • Check the serial port setting on the Connectivity setup page. See the ultrasound system user guide. • Check the cable connections on the stand and DVD recorder. See the V-Universal Stand assembly instructions. • Ensure that the DVD record is turned on and set up properly. See the DVD recorder manufacturer's instructions, if necessary.
DVD recorder does not play back	<ul style="list-style-type: none"> • Check the cable connections on stand and DVD recorder. See the V-Universal Stand assembly instructions. • Ensure that the display setting is set to S-video. Follow the manufacturer's instructions.

Troubleshooting

Symptom	Solution
Printer does not print.	<ul style="list-style-type: none"> • Check the printer selection on the Connectivity setup page. See the ultrasound system user guide. • Check the cable connections. See the V-Universal Stand assembly instructions. • Ensure that the printer is turned on and set up properly. See the printer manufacturer's instructions, if necessary.
Stand tips over more easily than expected.	<ul style="list-style-type: none"> • Lower the pole to lower the ultrasound system and optional tray. • Avoid placing or hanging heavy objects on the stand. Too much weight can cause the stand to become less stable.

Cleaning and disinfecting

- WARNING:** To avoid electrical shock, before cleaning, always disconnect the system from the power supply and remove it from the stand.
- WARNING:** To avoid injury, always use protective eyewear and gloves when performing cleaning and disinfecting procedures.
- WARNING:** To avoid infection, ensure that the solution expiration date has not passed.
- WARNING:** To avoid infection, the level of disinfection required for a product is dictated by the type of tissue it contacts during use. Ensure that the solution strength and duration of contact are appropriate for the equipment. For information, see the disinfectant label instructions and the recommendations of the Association for Professionals in Infection Control and Epidemiology (APIC) and FDA.
- Caution:** Use only recommended cleaners or disinfectants on surfaces. Immersion-type disinfectants are not tested for use on surfaces.

To clean and disinfect the stand

You can clean and disinfect the exterior surface of the stand using a recommended cleaner or disinfectant. For the most recent list of approved products, see www.sonosite.com.

- 1 Remove the system, unplug the power supply, and detach any cables.
- 2 Clean the surfaces using a soft cloth lightly dampened in a mild soap or detergent cleaning solution.
Apply the solution to the cloth rather than the surface of the stand.
- 3 Wipe the surfaces with the disinfecting solution.
- 4 Air dry or towel dry with a clean cloth.

For information about cleaning the peripherals, see the manufacturer's instructions.

Safety

For maximum safety, observe the following warnings and cautions before using accessories and peripherals with the ultrasound system and transducers.

Electrical safety

- WARNING:** To avoid the risk of a shock hazard, do not plug the power cord of the V-Universal Stand into a multiple portable socket outlet (MPSO) or use an extension cord.
- WARNING:** To avoid the risk of electrical shock, use only properly grounded equipment. Shock hazards exist if the power supply is not properly grounded. Grounding reliability can only be achieved when equipment is connected to a receptacle marked "Hospital Only" or "Hospital Grade" or the equivalent. The grounding wire must not be removed or defeated.
- WARNING:** To avoid the risk of electrical shock and fire hazard, inspect the power supply, plugs, cables, and power cords on a regular basis for damage.
- WARNING:** To avoid the risk of electrical shock, use only accessories and peripherals recommended by SonoSite, including the power supply. Connection of accessories and peripherals not recommended by SonoSite could result in electrical shock. Contact SonoSite or your local representative for a list of accessories and peripherals available from or recommended by SonoSite.
- WARNING:** To avoid possible electrical shock or electromagnetic interference, verify proper operation and compliance with relevant safety standards for all equipment before clinical use. Connecting additional equipment to the ultrasound system constitutes configuring a medical system. SonoSite recommends verifying that the system, all combinations of equipment, and accessories connected to the ultrasound system comply with JACHO installation requirements and/or safety standards AAMI-ES1, NFPA 99, IEC 60601-1-1, and IEC 60601-1-2, and are certified according to IEC 60950 (Information Technology Equipment (ITE)).

Equipment safety

- Caution:** Excessive bending or twisting of cables can cause a failure or intermittent operation.
- Caution:** Improper cleaning or disinfecting of any part of the accessories and peripherals can cause permanent damage. For cleaning and disinfecting instructions, see "[Cleaning and disinfecting](#)" on page 10 and the manufacturer's operating instructions.
- Caution:** Accessible metal on the signal inputs and outputs on the back and side of the ultrasound system is not protectively earthed. Do not perform a high current ground impedance test involving this part.

Clinical safety

WARNING: To avoid misdiagnosis, do not use non-medical (commercial) grade peripheral monitors. These monitors have not been verified or validated by SonoSite as being suitable for diagnosis.

Electromagnetic compatibility (EMC)

See the safety section of the ultrasound system user guide and supplemental guides for information on electromagnetic compatibility with IEC 60601-1-2, including the Manufacturer's Declaration.

Specifications

For information on the peripherals, see the manufacturer's specifications.

Stand dimensions

Width: 20 in. (51 cm)

Length: 23 in. (59 cm)

Height: 54.5 in. max. (138.5 cm), 42.5 in. min. (108 cm)

Height range: 12 in. (30.5 cm)

Weight: 37 lbs. (16.8 kg) basic, 45 lbs. (20.4 kg) deluxe

Temperature and humidity limits

Operating: 10–40°C; 15–95% R.H.

Shipping/Storage: -35–65°C; 15–95% R.H.

Peripherals

Medical grade

Black-and-white printer

For printer paper, contact Sony at 1-800-686-7669 or www.sony.com/professional to order paper or to find the local distributor.

DVD recorder (S Series only. As of July 2012, the DVD recorder is not supported with the V-Universal stand.)

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Copyright © 2013 FUJIFILM SonoSite, Inc.
Copyright © 2013 par FUJIFILM SonoSite, Inc.