Sending and Receiving DICOM Data on SonoSite Systems

User Guide Errata

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: +1-888-482-9449 or
+1-425-951-1200
F: +1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution | United States law restricts this device to sale by or on the order of a physician.

FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation. Edge, M-Turbo, NanoMaxx, S Series, SonoSite, and the SonoSite logo are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

DICOM is a registered trademark of the National Electrical Manufacturers Association.

All other trademarks are the property of their respective owners.

Part Number: P23658-01

Publication Date: December 2018

Copyright © 2018 FUJIFILM SonoSite, Inc. All Rights reserved.

DICOM and SiteLink User Guide Errata

Introduction	1
Document conventions	1
Getting Help	2
Getting Started	2
Configuration setup fields (DICOM)	2
Configuring the ultrasound system (SiteLink)	3

Introduction

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the user guide.

Getting Help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

Getting Started

The following content was incorrect in the *Sending and Receiving DICOM Data on SonoSite Systems User Guide* and the *SiteLink User Guide*; revisions will be made to the guides in the next update.

Configuration setup fields (DICOM)

Location configuration (page 1)

Host Name Unique network name for the ultrasound system. Default is SonoSite.

Domain Name of the authentication server.

Note

You can only use this option if you have a wireless connection; it does not work with a LAN connection.

Configuring the ultrasound system (SiteLink)

To configure SiteLink locations

- 1 Ensure that the transfer mode is set to **SiteLink**. See “[To set the transfer mode.](#)”
- 2 On the Connectivity setup page, select **SiteLink Setup**.

Select **New** and fill in the following fields:

Host Name Unique network name for the ultrasound system.

Domain Name of the authentication server.

Note

You can only use this option if you have a wireless connection; it does not work with a LAN connection.

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