NanoMaxx Ultrasound System User Guide Errata P15533-04

Software licensing

The user guide incorrectly states that grace period time is not used while the NanoMaxx® ultrasound system is asleep. In fact, the system does use grace period time while asleep. To avoid using grace period time, turn off the system while not in use.

Cleaning and disinfecting

The following statement was updated in the user guide.

For a list of recommended cleaners and disinfectants, see the cleaners and disinfection tool available at www.sonosite.com/support/cleaners-disinfectants.

Clinical safety

The following warning has been updated in the user guide.

WARNING

FUJIFILM SonoSite does not recommend the use of high-frequency electromedical devices in proximity to its systems. FUJIFILM SonoSite equipment has not been validated for use with high-frequency electrosurgical devices or procedures. Use of high-frequency electrosurgical devices in proximity to its systems may lead to abnormal system behavior or shutdown of the system.

To avoid the risk of a burn hazard, do not use the transducer with high-frequency surgical equipment. Such a hazard may occur in the event of a defect in the high-frequency surgical neutral electrode connection.

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Copyright © 2019
All rights reserved.
SonoSite and the SonoSite logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.
FUJIFILM SonoSite, Inc.
FUJIFILM SonoSite B.V.**
1114 AB Amsterdam,