Workflow Solutions

1.1 SP1

Installation Instructions	3
Instructions d'installation	16

SonoSite, Inc.

21919 30th Drive SE
Bothell, WA USA 98021
T: 1-888-482-9449 or 1-425-951-1200
F: 1-425-951-1201

SonoSite Ltd

Alexander House
40A Wilbury Way
Hitchin
Herts SG4 0AP
UK
T: +44-1462-444800
F: +44-1462-444801

SonoSite, SWS, and the SonoSite logo are registered (in some jurisdictions) and unregistered trademarks owned by SonoSite, Inc.

Non-SonoSite product names may be trademarks or registered trademarks of their respective owners.

03/2011

Copyright 2011 by SonoSite, Inc.

All rights reserved.

SonoSite Workflow Solutions

Installation Instructions

Installing SonoSite® Workflow Solutions (also called SWS™ software) involves the following:

- Installing the SWS server software
- Installing the SWS client on each user's computer

System requirements

Computer with the following:

- One of the following operating systems:
 - Microsoft® Windows Vista® Business with SP2 (32-bit), Windows Server® 2008 Standard with SP2 (32-bit), or Windows XP with SP3 (32-bit)
 - Mac OS 10.5.6 or later
- DVD drive

Additional requirements for server

- 250 (Windows) or 320 (Mac OS) GB hard-disk space, minimum
- 3 GHz (Windows) or 2 GHz (Mac OS) or faster processor
- 4 GB RAM, minimum

Additional requirements for client

- A browser: FireFox 3.5 or later, Internet Explorer 7.0 or later, Safari 4.0 or later, or Google Chrome 2.0 or later
- Adobe Reader 8.1 or later (See www.adobe.com.)

Setup

If using a firewall, allow inbound access to the following ports:

- 8080, for communication with SWS clients
- 3306, for communication with mySQL
- 11112, for transferring data from the ultrasound system

Note: 11112 is the default port for SWS to receive images and clips. You can change this port during installation, but modify your firewall to allow inbound access to the port you choose.

If using Windows Server 2008, allow inbound echo requests (ICMPv4).

Make sure to install the SWS server software on a computer that has a static IP address.

Installing the server software

The server-software installation also includes installing a client on the host machine.

Windows

To install SWS server or update SWS server 1.0 on Windows

- 1 Log on to Windows as administrator.
- 2 Insert the SWS disc into the DVD drive.

The SWS installation wizard appears.

Note: If the wizard does not appear, double-click Install_SWS_for_Windows on the SWS disc.

- 3 If updating from SWS 1.0, do the following:
 - a In the SWS 1.0 Detected dialog box, click **Proceed**.
Clicking Proceed allows the installer to automatically create a backup file of existing data.
 - b Enter your password for SWS 1.0 server and then click **Proceed**. (For assistance with your password, contact SonoSite Technical Support.)
- 4 Follow the instructions in the wizard.

If the Windows Security Alert regarding Java(TM) Platform SE binary appears, click **Keep Blocking**.

The wizard prompts you to accept the license agreement; to specify an installation location, storage location, AE title, port number, and other items; and to enter a SWS database password.

Important: Make sure to remember the SWS database password. You will need it if reinstalling, updating, or reconfiguring the SWS product. Also, make a note of any non-default installation or storage location you specify.
Password: _____

- 5 In the Launch SWS Client Web Install Page dialog box, click **OK**.
The browser starts.
- 6 Drag the browser off the wizard, and click **Done** in the wizard. (Do not close the browser.)
- 7 Install the client. See “[Installing clients](#)” on page 10. For host server, enter `http://localhost:8080`.
- 8 License the product and specify an archive location. See “[To license the product and specify an archive location](#)” on page 12.

To update SWS server 1.1 on Windows

- 1 Log on to Windows as administrator.
- 2 Insert the SWS disc into the DVD drive.
The SWS installation wizard appears.
Note: If the wizard does not appear, double-click `Install_SWS_for_Windows` on the SWS disc.
- 3 In the SWS 1.1 Detected dialog box, click **Proceed**.
- 4 Follow the instructions in the wizard.
If the Windows Security Alert regarding Java(TM) Platform SE binary appears, click **Keep Blocking**.
- 5 In the Launch SWS Client Web Install Page dialog box, click **OK**.
The browser starts.
- 6 Drag the browser off the wizard, and click **Done** in the wizard. (Do not close the browser.)

- 7 Install or update the client. See “Installing clients” on page 10. For host server, enter `http://localhost:8080`.
- 8 License the product and specify an archive location. See “To license the product and specify an archive location” on page 12.

Mac OS

To install SWS server on Mac OS

- 1 Make sure that JDK 6 is installed: Confirm that the **1.6** folder is in the following location:

Macintosh HD/System/Library/Frameworks/
JavaVM.framework/Versions

If there is no 1.6 folder, install JDK 6:

- a Connect to the Internet.
- b Choose Apple menu > System Preferences.
- c Click **Software Update**.
- d Click **Check Now**.

A list of updates appears. Follow the on-screen prompts to install **Java for Mac OS X 10.5 Update**.

Note: If Java for Mac OS X 10.5 Update is not in the list, install the items that are in the list. Then start the procedure again from the beginning.

- 2 Insert the SWS disc into the DVD drive.
- 3 Open a terminal window: In the **Applications/Utilities** folder, double-click **Terminal**.

- 4 At the command prompt, type the following text, and then press ENTER:

```
sudo /Volumes/SWS111/Mac/MacOSX/install.sh
```

Note: Include the space after “sudo” but no spaces elsewhere.

- 5 At the **Password** prompt, type the administrator password for the system, and then press ENTER.

The SWS software end-user license agreement appears in a separate window.

- 6 In the terminal window, type Y to accept the license agreement and continue.
- 7 If updating from SWS 1.1, do the following:
 - a Type Y at the prompt to proceed.

Several lines of text appear as SWS installs. The process may take a few minutes. Afterward, the SWS server automatically starts as a service, and the following message appears:

SWS 1.1 SP1 has been successfully installed. . . .

- b Skip to step 13 below.
- 8 At the **Do you wish to install to the default folder path (/Library) (Y/N)?** prompt, do one of the following:
 - (Recommended) Press Y to accept the default installation folder path, and then press ENTER.
 - Press N to specify an installation folder path, and then press ENTER. Follow the prompt to enter the desired path.

- 9 If updating SWS from a previous version, enter your password at the **Please enter your SWS 1.0 database password** prompt, and then press ENTER. (For assistance with your password, contact SonoSite Technical Support.)
- 10 At the **Please enter the database password you would like to use for SWS 1.1 SP1** prompt, type the password, and then press ENTER. At the confirmation prompt, retype the password, and then press ENTER.

Important: Make sure to remember the SWS database password. You will need it if reinstalling, updating, or reconfiguring the SWS product.

Password: _____

- 11 For each of the next four prompts that appear, do one of the following:
- (Recommended) Press Y to accept the default setting, and then press ENTER.
 - Press N to specify a non-default setting, and then press ENTER. Follow the prompt to enter the setting. If specifying a folder path, use a folder path that contains no spaces.
- Important:** Make a note of the folder path you specify. You will need it if reinstalling, updating, or reconfiguring the SWS product.

The four prompts are as follows:

Would you like to use the default DICOM AE Title (SWS) (Y/N)?

Would you like to use the default DICOM Port (11112) (Y/N)?

Do you wish to set the database data folder to the default folder path (/SWS_MYSQL_DB) (Y/N)?

Do you wish to set the SWS image/clip storage folder to the default folder path (/SWS_STORE) (Y/N)?

After you address the four prompts, several lines of text appear as SWS installs. The process may take a few minutes. Afterward, the SWS server automatically starts as a service, and the following message appears:

SWS 1.1 SP1 has been successfully installed. . . .

- 12** If updating from SWS 1.0, uninstall the SWS 1.0 client: Delete the **Applications/SonoSite** folder.
- 13** Install the client. See [“Installing clients”](#) on page 10. For host server, enter `http://localhost:8080`.
- 14** License the product and specify an archive location. See [“To license the product and specify an archive location”](#) on page 12.

Installing clients

Install a client on each user’s computer.

If a client from an earlier version is already installed, you must update it. To determine the version of a currently installed client, see its Info pane.

To install a new SWS client or update a SWS 1.0 client

- 1** If using Windows, make sure that you are logged into Windows as administrator.

- 2 Uninstall the SWS 1.0 client (if present):
 - For Windows, use the **Programs and Features** (Vista) or **Add or Remove Programs** (XP) utility to remove the “SonoSite Workflow Solutions” entry.
 - For Mac OS, delete the **/Applications/SonoSite** folder.
- 3 In the browser, enter the host server where SWS is installed, in the following format:

`http://host server:8080`

For example, type `http://172.20.33.100:8080` (in the text box at the top of the browser), and press ENTER.

For the host server, see your SWS administrator.

- 4 Follow the instructions that appear in the browser.

Note: For a copy of these instructions, see “Appendix” on page 14.

Make sure to install Adobe AIR, even if it is already installed. The latest version is included with SWS.

To update a SWS 1.1 client

- 1 If using Windows, make sure that you are logged into Windows as administrator.
- 2 Start the SWS client.
- 3 Click **Update**.

The client updates automatically.

To log in the first time

For security purposes, make sure to change the password after you log in the first time.

- 1 In the SWS client, enter `admin` in both the **User Name** box and **Password** box.
- 2 Click Log in.

To license the product and specify an archive location

- 1 Log into the client. (See [“To log in the first time”](#) on page 11.)
- 2 Click **SETTINGS**, and then license the product:
 - a Click the **License** tab, and type the license key in the **License Key** boxes.
 - b Click **Submit**.
- 3 Specify a location for archiving:
 - a Click the **Connect** tab.
 - b In the **Path** box, type the path for a folder that the server can access.

Type the complete path, including the drive letter or name. For example:

`c:\SWS-archive`

`Users/Shared/archive`
 - c Click **Submit**.
- 4 If updating from SWS 1.0, add the full name and professional degree to each user account. (For editing instructions, see the *SonoSite Workflow Solutions Administrator Guide*.)

In the client, you can complete additional tasks such as setting up user accounts. See [“Getting help”](#) on page 13.

Getting help

If you need assistance, please refer to the *SonoSite Workflow Solutions Administrator Guide* for troubleshooting suggestions.

The *SonoSite Workflow Solutions Administrator Guide* and *SonoSite Workflow Solutions User Guide* are available in the following locations:

- SWS disc
- **HELP** link in the SWS client
- SonoSite Web site at www.sonosite.com

For technical support, contact SonoSite as follows:

SonoSite Technical Support

Phone (US or Canada): 877-657-8118

Phone (Outside US and Canada): 425-951-1330
Or call your local representative.

Fax: 425-951-6700

E-mail: service@sonosite.com

Web site: www.sonosite.com

Appendix

On-screen client installation instructions

These instructions appear in the browser during client installation.

- 1** Install Adobe® AIR™.
 - a** Click your operating system: Mac OS or Windows.
 - b** Download the Adobe AIR Installer by selecting **Save**.
 - c** Double-click the downloaded file (AdobeAIRInstaller.exe on Windows, AdobeAIR.dmg on Mac OS).

The file is located in your browser's downloads folder.
 - d** (Mac OS only) Double-click Adobe AIR Installer in the Finder.
 - e** Follow the instructions in the wizard.
- 2** Download and install SonoSite® Workflow Solutions.
 - a** Click SonoSite Workflow Solutions.
 - b** Click **Open**. (If a dialog box with Open does not appear, double-click the IMAGE.air file in your browser's downloads folder.)
 - c** Click **Install**.
 - d** Click **Continue**.
- 3** Enter server information in login screen.
 - a** In the login screen, click **View Server Information**, and then click **Edit**.

- b** In the **Host** box, type the host server where SonoSite Workflow Solutions is installed (the same entry used to display this page; for example, 172.20.33.100).
 - c** In the **Port** box, type 8080, and then click **Submit**.
 - d** Verify that a number for **Server Version** appears (if no number appears, click **Retry** or see Troubleshooting).
- 4** Install Adobe Reader 8.1 or later.
See <http://get.adobe.com/reader>.
Adobe Reader lets you create, view, save, and print reports in SonoSite Workflow Solutions.

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