
TEE Transducer Care

Step 1 of 2: Clean the transducer


- WARNINGS:**
- ▶ Be sure to use the appropriate Personal Protective Equipment (PPE) whenever necessary.
 - ▶ **DO NOT** skip any steps or abbreviate the cleaning process in any way.
- Cautions:**
- ▶ **DO NOT** allow bodily fluids to dry on the transducer. Always clean the transducer immediately after use.
 - ▶ **DO NOT** bend the endoscopic shaft smaller than a 20 cm (8 inch) curve.
 - ▶ **DO NOT** use unapproved cleaning agents such as alcohol or bleach (for example: Sani-Cloth).
 - ▶ **DO NOT** spray or immerse the controls, the cord, or the connector with any fluid. Be careful to prevent moisture from getting on the connector.
 - ▶ **DO NOT** soak the transducer longer than recommended by the chemical manufacturer.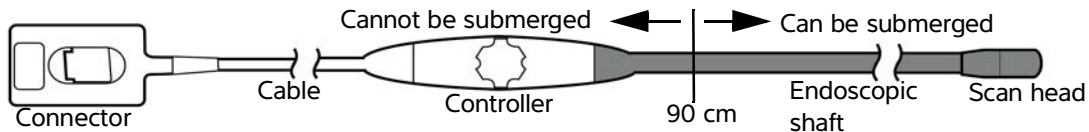


Enzymatic cleaners

Cleaner	Concentration/Temperature	Duration
Prolystica	1 - 5% at 33-43° C, 91-109° F	2 - 5 minutes, then rinse immediately
Hexanios G+R	0.5%, at less than 45° C, 113° F	15 minutes
Aniosyme DD1	0.5%, at 38° C, 100° F	15 minutes
Salvanios pH7	0.5%, at 38° C, 100° F	15 minutes
Cidezyme/Enzol	1 - 10%, at 38° C, 100° F	1 - 3 minutes


A Immediately after extraction from the patient, **Wipe** the transducer, including the cable, controller, endoscopic shaft, and scan head with a cloth or wipe moistened with water. Be sure to remove all visible biological material. Do not wipe the connector.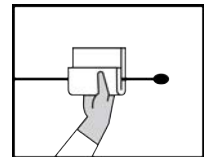


B At the cleaning station, check the expiration date of the cleaner. Check the temperature and concentration of the cleaning solution. Secure the control handle so it cannot fall into the cleaning solution. **Soak** the endoscopic shaft in a plastic container filled with enzymatic cleaning solution according to the concentration and soak time indicated in the *Enzymatic cleaners* table. **Scrub** the endoscopic shaft for at least three minutes using a soft brush or single-use endoscopic sponge moistened with enzymatic cleaner. Follow the chemical manufacturer's precautions and instructions.

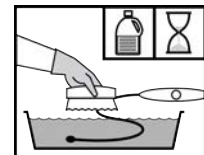C While the endoscope is soaking, **Clean** the control handle and cable by wiping with a clean, non-linting cloth or single-use endoscopic sponge moistened with one of the cleaners listed in the *Enzymatic cleaners* table. To remove any residual cleaner, **Wipe** both components again with a clean, non-linting cloth or single-use endoscopic sponge moistened with water. Do not wipe the connector.

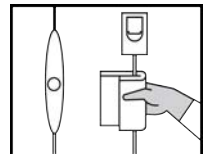
D **Rinse** the endoscopic shaft by soaking it in a large volume of clean, lukewarm water (for example, eight liters) for at least three minutes to remove residual cleaning solution. Residual cleaners left on the transducer can cause damage. Visually **Inspect** the scan head and endoscopic shaft for remaining biological material. If any is found, repeat the cleaning process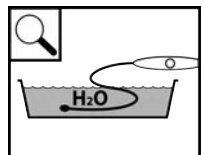

E **Test** the transducer for bite holes. For more information about leakage and bite hole detection, see the *TEE Transducer User Guide*.

Dry the transducer with a clean, non-linting towel.
Examine the transducer and cable for damage, such as cracks or splitting where fluid can enter. Discontinue use of the transducer if damage is evident.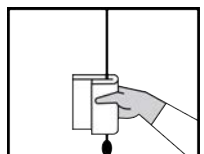

Disinfection procedure on other side

TEE Transducer Care

Step 2 of 2: Disinfect and store the transducer




- WARNINGS:**
- ▶ Be sure to use the appropriate Personal Protective Equipment (PPE) whenever necessary.
 - ▶ Chemical disinfectants can cause harm to the patient if not completely removed from the transducer.
 - ▶ **DO NOT** skip any steps or abbreviate the disinfection process in any way.
- Cautions:**
- ▶ **DO NOT** bend the endoscopic shaft smaller than a 20 cm (8 inch) curve.
 - ▶ **DO NOT** use unapproved disinfecting agents such as alcohol or bleach (for example: Sani-Cloth™)
 - ▶ **DO NOT** immerse the system cord, connector, or the control handle in any fluid.
 - ▶ **DO NOT** soak the transducer longer than recommended by the chemical manufacturer.

Approved disinfectants and soak times

Disinfectant	Temperature	Duration
Anioxyde 1000	23° C, 73° F	15 minutes
Cidex	20° C, 68° F	45 minutes
Cidex OPA	25° C, 77° F	12 minutes
Korsolex extra	less than 45° C, 113° F	15 minutes
Metricide	25° C, 77° F	45 minutes
Nu-Cidex	20° C, 68° F	15 minutes
PeraSafe	less than 45° C, 113° F	15 minutes
Sekusept Aktiv	less than 45° C, 113° F	15 minutes
TD-100 & TD-5	Auto	Auto
Tristel Generator Solution	Auto	Auto
Wavicide-01	24° C, 75° F	45 minutes

If using an automated disinfection process, follow manufacturer procedures. For manual disinfection, do the following:

- At the disinfection station, check the expiration of the disinfectant. Check the temperature and the concentration of the solution. Secure the control handle so it cannot fall into the disinfectant solution.
- A** **Disinfect** the transducer by soaking the shaft in the disinfection fluid. Do not exceed the soak time as indicated in the *Approved disinfectants and soak times* table or by the chemical manufacturer. **Disinfect** the cable and controller by wiping them with a sterile, non-linting cloth or single-use endoscopic sponge moistened with disinfectant solution.
- B** **Rinse** the transducer by soaking it for at least **one minute, three separate times**, in a large volume of sterile, deionized water (for example, eight liters). **Discard** the rinse water after each rinse. **Rinse** the cable and controller by wiping them with a sterile, non-linting cloth or single-use endoscopic sponge moistened with sterile water.
- Chemical disinfectants can cause harm to the patient if not completely removed.
- C** **Dry** the transducer with a sterile, non-linting towel or medical-grade air. **Examine** the transducer and cable for damage, such as cracks or splitting where fluid can enter. Discontinue use of the transducer if damage is evident. **Store** the transducer in accordance with your organization's policies, or refer to the *TEE Transducer User Guide* for specific instructions.

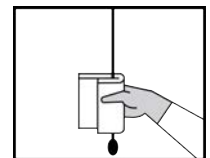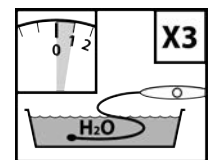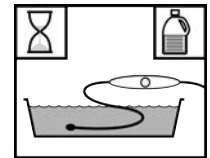For more information on cleaning, disinfecting, transporting, and storing the TEE transducer, refer to the **TEE Transducer User Guide**.

SONOSITE and the SONOSITE logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value From Innovation is a trademark of FUJIFILM Holdings America Corporation. Value From Innovation is a trademark of FUJIFILM Holdings America Corporation. All other trademarks are the property of their respective owners.

Copyright © 2019 FUJIFILM SonoSite, Inc. All rights reserved.

P28249-01

Manufacturer:
FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 9802, USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative
FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor
FUJIFILM SonoSite
Australasia Pty Ltd
114 Old Pittwater Road
Brookvale, NSW, 2100
Australia

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Copyright © 2019 FUJIFILM SonoSite, Inc. Tous droits réservés.
Copyright© 2019 FUJIFILM SonoSite, Inc.
Copyright © 2019 FUJIFILM SonoSite, Inc. Med enerett.
Copyright © 2019 FUJIFILM SonoSite, Inc. Med ensamrätt.