SonoSite X-Porte

Desktop

User Guide Supplement

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite, and X-Porte are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P16515-04

Publication date: June 2019

Copyright © 2019 FUJIFILM SonoSite, Inc. All rights reserved.

CE
2797

SonoSite X-Porte Desktop User Guide Supplement

Introduction	1
Document conventions	1
Getting Help	2
Assembling the SonoSite X-Porte Desktop	3
Ports	7
Using the mouse	7
Control panel and clinical displays	8
Control panel display	8
Clinical display	8
Saving images and video clips	9
System messages and alerts	9
Video clip controls	9
Using the system with the stand	10
Cleaning and disinfecting	13
Safety	14
Compatible accessories and peripherals	15
Specifications and standards	15
Electrical	15

Introduction

This user guide supplement provides information specific to using the SonoSite X-Porte ultrasound system on a desktop (SonoSite X-Porte Desktop), without the stand or the touch panel.

Refer to the Help included with the SonoSite X-Porte ultrasound system, or the *SonoSite X-Porte User Guide* for all other instructions for use.

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the ultrasound system user guide.

Getting Help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

Assembling the SonoSite X-Porte Desktop

WARNINGS

- ▶ Verify that the hospital supply voltage corresponds to the power supply voltage range.
- ▶ Plug the isolation transformer into a grounded hospital-grade outlet only.
- ▶ The isolation transformer has unique settings for use with 120 or 240 V power sources. Verify the isolation transformer selector switch has been set to the desired setting.
- ▶ Use only power cords provided by FUJIFILM SonoSite with the system.

Caution

Do not use the system if an error message appears on the monitor. Note the error code and turn off the system. Call FUJIFILM SonoSite or your local representative.

The SonoSite X-Porte Desktop consists of the ultrasound engine, a FUJIFILM SonoSite clinical monitor with pedestal mount, a mouse, an isolation transformer, power supplies, one or more transducers, and various optional accessories and peripherals (see “[Compatible accessories and peripherals](#)” on page 15).

System components

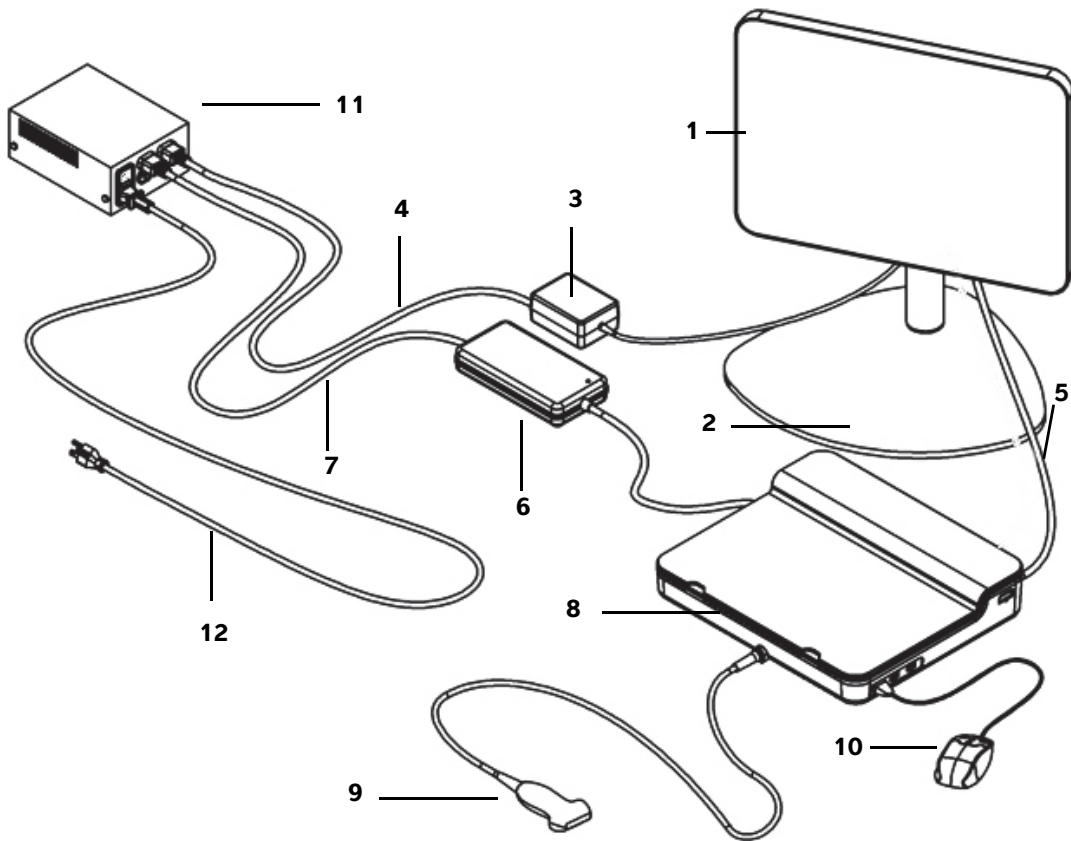
Figure 1 The SonoSite X-Porte Desktop

- | | | | |
|---|-------------------------------------|----|---|
| 1 | Clinical monitor | 7 | Ultrasound engine power supply extension cord |
| 2 | Monitor pedestal mount | 8 | Ultrasound engine |
| 3 | Monitor power supply | 9 | Transducer |
| 4 | Monitor power supply extension cord | 10 | USB mouse |
| 5 | Digital video cable | 11 | Isolation transformer |
| 6 | Ultrasound engine power supply | 12 | System power cord |

To assemble the SonoSite X-Porte Desktop

Caution

Connect only the clinical monitor power supply extension cord, the ultrasound engine power supply extension cord, and the (optional) printer power supply to the isolation transformer. Do not use the fourth output on the isolation transformer.

Note

To avoid unexpected results, follow the assembly instructions in the order they are presented.

Before you begin, make sure that the isolation transformer is disconnected from power and that the isolation transformer's power switch is in the OFF position.

1. Connect a transducer and the ultrasound engine power supply to the bottom of the ultrasound engine

- a Place the ultrasound engine upside down on a soft surface, and connect a transducer (see [Figure 2](#) on page 5 and "Preparing transducers" in the SonoSite X-Porte Help or in the *SonoSite X-Porte User Guide*).
- b Connect the ultrasound engine power supply to the ultrasound engine: Remove the plastic guard from the power connector (if present), and connect the power connector to the engine (see [Figure 2](#) on page 5).
- c Invert the ultrasound engine, so that it sits flat on the block attached to the power connector cord.

If necessary, adjust the position of the power connector cord to make sure that the ultrasound engine sits flat on the block attached to the power connector cord and is stable.

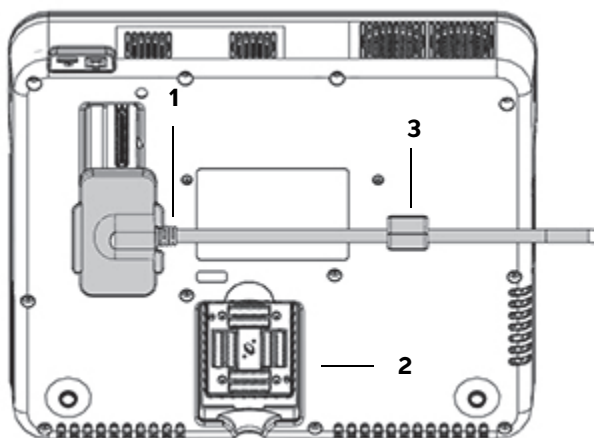
Figure 2 Underside of the ultrasound engine: Power connector and transducer connector

- 1 Power connector 2 Transducer connector 3 Block

2. Connect the digital video cable

- a Connect the digital video cable to the digital video port on the back of the monitor.
- b Connect the digital video cable from the monitor to the video-in port on the back of the ultrasound engine (see [Figure 5](#) on page 7).

3. Connect ultrasound engine and clinical monitor power supplies to the isolation transformer

- a Connect the ultrasound engine power supply extension cord from the ultrasound engine power supply to the isolation transformer.
- b Connect the clinical monitor power supply:
 - i Connect the clinical monitor power supply to the power connector on the back of the monitor.
 - ii Connect the clinical monitor power supply extension cord from the monitor power supply to the isolation transformer.

4. Connect the mouse and optional peripherals

- a Connect the mouse to a USB port on the side of the ultrasound engine (see [Figure 3](#) on page 7).
- b (Optional) If connecting to the network by Ethernet, connect an Ethernet cable to the Ethernet port on the back of the ultrasound engine (see [Figure 5](#) on page 7). For more information, see “Connecting to the network” in the *SonoSite X-Porte Help* or in the *SonoSite X-Porte User Guide*.
- c (Optional) Connect the printer:
 - i Connect the printer USB cable to a USB port on the side of the ultrasound engine (see [Figure 3](#) on page 7).
 - ii Connect the printer power supply cord from the printer power cable to the isolation transformer.

5. Mount the clinical monitor on the pedestal mount

For instructions, see the *GCX Mounting Assembly Installation Guide*, included with the system.

6. Connect to power and turn on the system and optional peripherals

- a Connect the system power cord from the isolation transformer to a grounded hospital-grade outlet.
- b Turn the isolation transformer power switch to the ON position.
- c Press the power button on the ultrasound engine.

The power button is green while the system is on. (When the system is in sleep mode, the power button blinks green.)
- d (Optional) Turn the printer power switch to the ON position.

Ports

The SonoSite X-Porte Desktop includes three USB ports on the right side of the ultrasound engine, an ECG port and Kensington security slot on the left side of the engine, and video-in and Ethernet ports on the back of the engine.

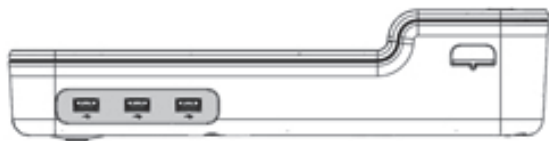
Figure 3 Right side of the ultrasound engine: USB ports (shaded)

Figure 4 Left side of the ultrasound engine: ECG port and Kensington security slot (shaded)

Figure 5 Video-in and Ethernet ports (shaded) on the back of the ultrasound engine

Using the mouse

In the SonoSite X-Porte Desktop, you can access all controls by clicking with the mouse. You can access the **Freeze**, **Unfreeze**, and **Select** controls by pressing either the left or right mouse buttons (see below).

Note

The mouse cursor does not appear in the image area on the clinical display.

To configure the mouse for right- or left-handed operation

- 1 Click **MORE**, and then click **System Settings**.
- 2 Click **Presets** in the list on the left.
- 3 Click **User Profile**.

4 On the User Profile setup page, select **Right Handed** or **Left Handed** from the **Mouse Operation** list.

To access controls using the mouse buttons in the right-handed configuration

- ❖ Do either of the following:
 - ▶ To select, press the left mouse button.
 - ▶ To freeze or unfreeze, press the right mouse button.

To access controls using the mouse buttons in the left-handed configuration

- ❖ Do either of the following:
 - ▶ To select, press the right mouse button.
 - ▶ To freeze or unfreeze, press the left mouse button.

Control panel and clinical displays

When you use the SonoSite X-Porte Desktop, you can view the control panel display or the clinical display on the clinical monitor, but not both at the same time. The clinical display appears by default.

Control panel display

The control panel display shows the ultrasound image and lets you access all controls and system settings, except for the **Save Image**, **Save Image & Calcs**, and the **Save Video Clip** controls.

Clinical display

The clinical display shows the ultrasound image, patient information, measurement and calculation information, system information, system status, and these controls:

- ▶ Four custom controls, two of which must be **Save Image** and **Save Video Clip** (see “[Saving images and video clips](#)” on page 9), if you want to save images or video clips.
- ▶ **Depth** controls

- ▶ Near Gain, far Gain, and overall Gain sliders 

- ▶ Control Panel 

- ▶ **Freeze and Unfreeze**

Note

When you freeze an image, the system displays the control panel display.

- ▶ Video clip controls in **Review**

To switch between the control panel display and the clinical display

- ❖ Do one of the following:
 - ▶ To switch from the control panel display to the clinical display, press the scroll up button on the mouse.
 - ▶ To switch from the clinical display to the control panel display, do any of the following:
 - ▶ Press the scroll down button on the mouse.

- ▶ Click **Control Panel** .

- ▶ Freeze the image.

Note

In a form that obscures the ultrasound image completely on the control panel display, such as the patient form, patient list, or worksheets, you cannot use the mouse scroll buttons to switch to the clinical display.

Saving images and video clips

To save images or video clips in the SonoSite X-Porte Desktop, you must configure the **Save Image** and **Save Video Clip** controls as custom controls. If you want to save calculations with images, you must also configure **Save Image & Calc** as a custom control. See “To configure custom controls” in the SonoSite X-Porte Help or in the *SonoSite X-Porte User Guide*.

The system stops image and video clip acquisition in the following circumstances:

- ▶ When you view a visual guide on the SonoSite X-Porte Desktop, or when you are in a form, such as the patient form or worksheets.
- ▶ When you switch from the clinical display to the control panel display.

System messages and alerts

System messages and alerts appear on the right side of both the control panel display and the clinical display.

Video clip controls

Video clip controls (see “Images and clips” in the SonoSite X-Porte Help or in the *SonoSite X-Porte User Guide*) appear on the control panel display or on the clinical display:

- ▶ Video clip controls in the cine buffer appear on the control panel display.

- ▶ Video clip controls in **Review** appear on both the clinical display and the control panel display.
- ▶ When you select **Preview Video Clip** in **Video Clip Settings**, video clip controls appear on the control panel display.

Using the system with the stand

You can insert the ultrasound engine into the SonoSite X-Porte stand, if desired.

To insert the engine into the stand

- 1 Turn off the ultrasound engine, and then turn off the isolation transformer.
- 2 Disconnect any devices attached to the ultrasound engine (such as USB memory sticks, ECG cables, mouse, ethernet cables, transducers, power cables, or digital video cables).
- 3 Place the front edge of the engine into the stand head just beneath the two metal retaining tabs at the front edge of the stand head.

Figure 6 Inserting the engine into the stand

- 4 Push down firmly on the top back edge of the engine until you hear the stand's latches engage.

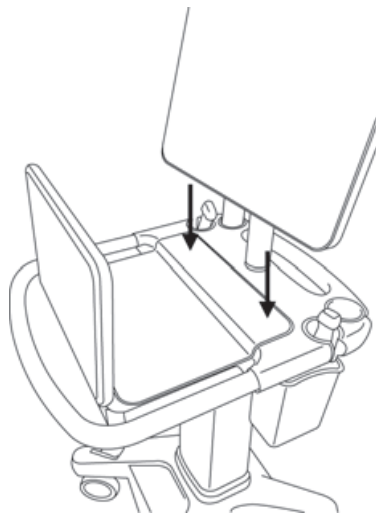
Figure 7 The engine inserted in the stand

5 Reconnect any devices you removed from the engine.

To remove the engine from the stand

Caution

Disconnect any devices attached to the ultrasound engine, such as the Triple Transducer Connector (TTC), USB memory sticks, transducers, power cables, or ECG cables, before removing it from the stand.

1 Disconnect the Triple Transducer Connector from the engine by pulling the latch up and rotating it clockwise.

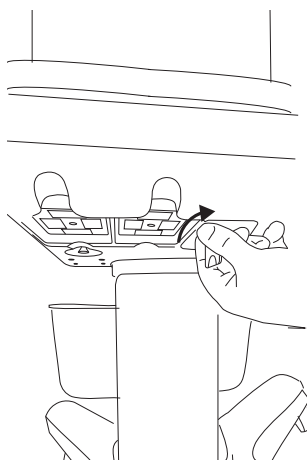
Figure 8 Loosening the TTC

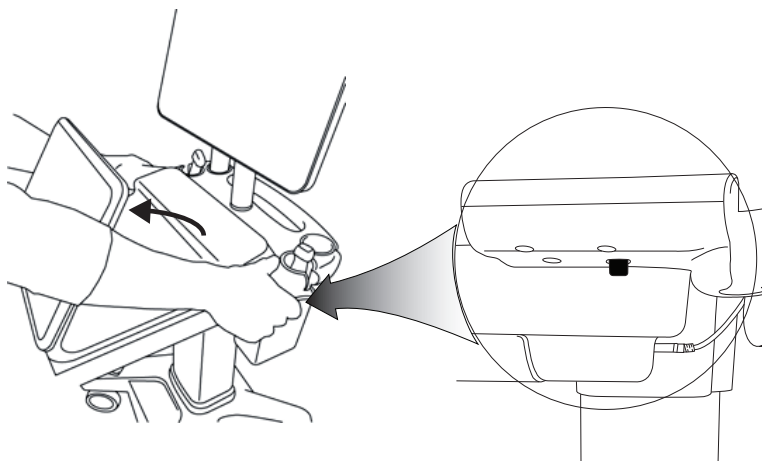
Figure 9 To loosen the engine from the stand, raise the control panel, and then release the two latches on either side of the stand head.

2 Remove the engine by lifting the back end out first.

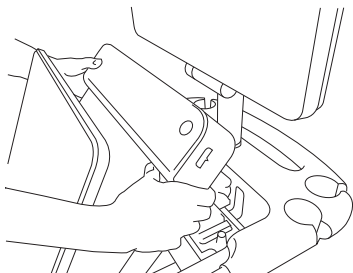
Figure 10 Lifting the engine out of the stand

Troubleshooting

Mouse cursor disappears. If you cannot see the mouse cursor on the clinical display, move it out of the image area, or switch to the control panel display.

Mouse does not respond. Disconnect and then reconnect the mouse to a USB port on the side of the ultrasound engine.

System does not turn on. Check all power connections, including the power connection on the underside of the ultrasound engine.

Cleaning and disinfecting

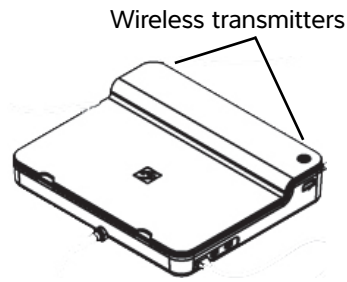
Instructions for cleaning and disinfecting the monitor pedestal mount are covered in the *GCX Mounting Assembly Installation Guide*, included with the system.

Instructions for cleaning and disinfecting the system, transducers, and accessories are covered in the “Cleaning and Disinfecting” chapter in the *SonoSite X-Porte User Guide*. Refer to the cleaning and disinfecting online tool available at www.sonosite.com for a complete list of the most current cleaners and disinfectants.

Safety

WARNINGS

- ▶ Ensure a minimum separation distance of 20 cm is maintained between the wireless transmitters on top of the ultrasound engine and all persons.

- ▶ To avoid the risk of burn, avoid prolonged contact with the power supply.
- ▶ To avoid the risk of tripping and equipment damage, keep the power cords away from trafficked areas.
- ▶ FUJIFILM SonoSite does not recommend using a monitor other than the clinical monitor provided by FUJIFILM SonoSite.
- ▶ Do not allow the mouse to come in contact with the patient.
- ▶ To avoid the risk of electrical shock and fire hazard, inspect the power supply, AC power cord, cables, and plugs on a regular basis. Ensure that they are not damaged.
- ▶ To avoid the risk of electrical shock, burn, or fire, use only AC power supplies specified by FUJIFILM SonoSite.

Cautions

- ▶ The glass on the clinical monitor is fragile and may shatter if the monitor tips or falls. To avoid the risk of tipping, do not place the monitor on top of the engine.
- ▶ To avoid overheating the system, do not block the fan outlets on the ultrasound engine.
- ▶ At higher elevations (greater than 1948 meters (6391 feet) above sea level), always operate the ultrasound engine in the stand.

Compatible accessories and peripherals

The SonoSite X-Porte Desktop supports the same accessories and peripherals as the SonoSite X-Porte stand-based ultrasound system, with the exception of the DVR, batteries, PowerPark docking station, Triple Transducer Connect (TTC), and touch panel.

See the *SonoSite X-Porte User Guide* for a full list of accessories and peripherals available with the system.

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