

Bar Code Scanner (USB)

Installation Instructions

Follow these instructions to connect and program any of these USB bar code scanners:

- ▶ **JADAK flexpoint HS-1M**
- ▶ **Symbol LS2208**

When connected, the bar code scanner is listed on the system's USB or USB Devices setup page.

See the *Bar Code Scanner User Guide* from FUJIFILM SonoSite for information on safety and cleaning.

Configuring the JADAK bar code scanner

To connect and program the JadaK bar code scanner

- 1 Attach the scanner cord to the bar code scanner and to a USB port on the ultrasound system.

See *JADAK flexpoint HS-1^M Quick Start Guide* included with the bar code scanner.

For location of USB ports, see the ultrasound system user guide.

- 2 Turn on the ultrasound system.
- 3 Scan the following two bar codes in the order listed:

USB Kybrd
STX Prefix
ETX Suffix



DOS CTRL
ASCII MODE



- 4 To ensure the scanner has been configured to work with your ultrasound system, FUJIFILM SonoSite recommends opening a patient information form on your system and scanning a bar code. Your system is successfully configured when you scan a bar code and the patient ID (and associated patient information, if any) populates the patient information form.

Configuring the Symbol bar code scanner

To connect and program the Symbol bar code scanner

- 1 Attach the scanner cord to the bar code scanner and to a USB port on the ultrasound system.

See *Symbol LS2208 Quick Start Guide* included with the bar code scanner.

For location of USB ports, see the ultrasound system user guide.

2 Turn on the ultrasound system.

3 Scan the following 9 bar codes in the order listed:

For instructions to scan, see *Symbol LS2208 Quick Start Guide*.

Note

If you have trouble scanning a bar code, try holding the scanner farther away from the bar code or at a different angle.

Set All Defaults



Code 39 - Any Length



Erase All Rules



Begin New Rule



Send Control B



Send All Data
That Remains



Send Control C



Beep Once



Save Rule



- 4 Scan additional programming bar codes as desired.
- 5 To ensure the scanner has been configured to work with your ultrasound system, FUJIFILM SonoSite recommends opening a patient information form on your system and scanning a bar code. Your system is successfully configured when you scan a bar code and the patient ID (and associated patient information, if any) populates the patient information form.

Note

You can program the scanner with various user preferences, such as beeper volume. Refer to *Symbol LS2208 Quick Start Guide* or, for more comprehensive information, to the *Symbol LS 2208 Product Reference Guide*, included with the scanner and available on the Motorola website at www.motorola.com.

SonoSite and the SonoSite logo are registered or unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions.

All other trademarks are the property of their respective owners.

Copyright© 2017 FUJIFILM SonoSite, Inc. All rights reserved.

<u>Manufacturer</u>	<u>EC Authorized Representative</u>	<u>Australia Sponsor</u>	11/2017
FUJIFILM SonoSite, Inc. 21919 30th Drive SE Bothell, WA 98021 USA T: 1-888-482-9449 or 1-425-951-1200 F: 1-425-951-1201	FUJIFILM SonoSite B.V. Joop Geesinkweg 140 1114 AB Amsterdam, The Netherlands	FUJIFILM SonoSite Australasia Pty Ltd 114 Old Pittwater Road BROOKVALE, NSW, 2100 Australia	



P12173-08

