Bar Code Scanner

User Guide Supplement

FUJIFILM
Value from Innovation

SonoSite

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

EC Authorized Representative

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite and the SonoSite logo are registered and unregistered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P22992-01

Publication date: December 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

Bar Code Scanner User Guide Supplement

Introduction	1
Document conventions	1
Getting help	2
Using the bar code scanner to enter patient information	2
Specifications	3

Introduction

This user guide supplement provides information on the custom programming feature now available with the JADAK flexpoint HS-1M bar code scanner for use with FUJIFILM SonoSite ultrasound systems.

For information from the manufacturer, see the *JADAK flexpoint HS-1M Quick Start Guide*.

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the system user guide.

Getting help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

Using the bar code scanner to enter patient information

Custom programming is available for the JADAK flexpoint HS-1M bar code scanner in English only. Contact your customer representative for further information.

Note

Before using the bar code scanner with patients, verify that your custom program is working as expected.

Specifications

The ultrasound system with the JADAK flexpoint HS-1M bar code scanner supports the following scanning formats:

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