Barcode Scanner Installation Instructions

The Sonosite PX ultrasound system is designed to work specifically with a preconfigured JADAK JDK-10175 USB barcode scanner. Do not use other JADAK Flexpoint HS-1M models. See the *JADAK Flexpoint HS-1M Quick Start Guide* or the system user guide from FUJIFILM Sonosite for information on safety and cleaning.

To install the JADAK barcode scanner on your ultrasound system

- 1 Connect the barcode scanner to a USB port on the ultrasound system.
See the *JADAK Flexpoint HS-1M Quick Start Guide* included with the barcode scanner.
For location of USB ports, see the Sonosite PX system user guide.
- 2 Turn on the ultrasound system, and select **Barcode auto query** on the General settings page..
The barcode scanner should appear on the system’s USB setup page.
- 3 To confirm that the barcode scanner is operating properly, open a patient information form on your system, scan a patient’s barcode, and verify that the information populates into the appropriate fields (see the system user guide for details).

Getting help

For technical support, please contact FUJIFILM Sonosite as follows:

Phone (U.S. or Canada)	+1 (877) 657-8118
Phone (Outside U.S. or Canada)	+1 (425) 951-1330, or call your local representative
Fax	+1 (425) 951-6700
E-mail	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+61 2 9938 8700 ffss-apacme-service@fujifilm.com

Manufacturer

FUJIFILM SonoSite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: +1-888-482-9449 or
+1-425-951-1200
F: +1-425-951-1201

**EC Authorized
Representative**

FUJIFILM SonoSite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM SonoSite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia

8/2020

SONOSITE is a registered trademark of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation in various jurisdictions. Value From Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Copyright © 2020 FUJIFILM SonoSite, Inc. All rights reserved.

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Copyright © 2020 FUJIFILM SonoSite, Inc. Alle Rechte vorbehalten.
Copyright © 2020 FUJIFILM SonoSite, Inc. Reservados todos los derechos.
Copyright © 2020 FUJIFILM SonoSite, Inc. Tous droits réservés.
Copyright © 2020 FUJIFILM SonoSite, Inc. Tutti i diritti riservati.
Copyright © 2020 FUJIFILM SonoSite, Inc. Alle rechten voorbehouden.
Copyright © 2020 FUJIFILM SonoSite, Inc. Alle rettigheder forbeholdes.