Barcode Scanner

Installation Instructions

Manufacturer	EC Authorized Representative	UK Responsible Person	Australia Sponsor
FUJIFILM SonoSite, Inc. 21919 30th Drive SE Bothell, WA 98021 USA T: 1-888-482-9449 or 1-425-951-1200 F: 1-425-951-1201	FUJIFILM SonoSite B.V. Joop Geesinkweg 140 1114 AB Amsterdam, The Netherlands	Fujifilm UK Limited Fujifilm House Whitbread Way Bedford MK42 0ZE United Kingdom	FUJIFILM SonoSite Australasia Pty Ltd 114 Old Pittwater Road BROOKVALE, NSW, 2100 Australia

SONOSITE is a registered trademark of FUJIFILM SonoSite, Inc., in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation. All other trademarks are the property of their respective owners.

Patent: www.sonosite.com/patents

P22236-05

Published Jul 24, 2023

Copyright © 2023 FUJIFILM SonoSite, Inc. All rights reserved.

Installing the barcode scanner on your ultrasound system

The Sonosite PX, Sonosite LX, Sonosite ZX, and Sonosite ST ultrasound systems are designed to work specifically with a preconfigured JADAK JDK-10175 USB barcode scanner. Do not use other JADAK Flexpoint HS-1M models. See the ultrasound system user guide for information on safety and cleaning.

1. Connect the barcode scanner to a USB port on the ultrasound system.
For location of USB ports, see the system user guide.
2. Turn on the ultrasound system.
The barcode scanner should appear on the system's USB setup page.
3. To confirm that the barcode scanner is operating properly, open the patient information form on your system, select a text field, scan a patient's barcode, and verify that the information populates into the appropriate fields (see the system user guide for details).

Getting help

- FUJIFILM Sonosite Technical Support:

United States and Canada	+1 877-657-8118
Europe and Middle East	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia and Pacific	+61 2 9938 8700
Other regions	+1 425-951-1330 or call your local representative
Fax	+1 425-951-6700
Email	General: ffss-service@fujifilm.com United Kingdom: uk-service@fujifilm.com Europe, Middle East, and Africa: eraf-service@fujifilm.com Asia and Pacific: ffss-apacme-service@fujifilm.com
Web	www.sonosite.com

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