## M-Turbo® Ultrasound System

### User Guide Errata P16206-02

#### Attaching and removing USB devices

If you attach a USB device to the system, wait at least 30 seconds before removing it or attaching another USB device. Similarly, if you remove a USB device from the system, wait at least 30 seconds before reattaching it or attaching another USB device.

Attaching or removing USB devices (for example, a USB printer or a USB storage device) too quickly may cause the system to stop responding and to display a blue screen with the Maintenance  and Attention  icons.

#### Patient report

If you transfer or export an exam from the M-Turbo® ultrasound system, the following measurements do not appear in the exported patient report. If you need to record them, save the report as an image (press the SAVE key) before transferring or exporting, or write them manually into the patient record.

- GYN exam: Follicle
- Cardiac exam: Automatically calculated CO values (CO, SV, CI, SI, VTI, Vmax, Vmean, PGmax, PGmean, LVOT D, LVOT area)

#### Language selection

If you need to change the Language selection on the Presets setup page, make sure to first archive, transfer, or export all saved patient data you want to keep. The system may not correctly maintain the saved patient information after the Language selection is changed.

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Copyright 2015 by
All rights reserved.
FUJIFILM SonoSite, Inc.
Australasia Pty Ltd