SonoSite Workflow Solutions

English

User Guide 1

Administrator Guide 15

Français

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Guide d'administration 45

SonoSite, Inc.

21919 30th Drive SE
Bothell, WA 98021
USA

T: 1-888-482-9449 or 1-425-951-1200

F: 1-425-951-1201

SonoSite Ltd

Alexander House
40A Wilbury Way
Hitchin
Herts SG4 OAP
UK

T: +44-1462-444800

F: +44-1462-444801

Caution: Federal (United States) law restricts this product to sale by or on the order of a physician.

M-Turbo, MicroMaxx, NanoMaxx, S Series, SonoSite, SWS, and the SonoSite logo are registered (in some jurisdictions) and unregistered trademarks of SonoSite, Inc.

DICOM is the registered trademark of the National Electrical Manufacturers Association for its standards publications relating to digital communications of medical information.

Non-SonoSite product names may be trademarks or registered trademarks of their respective owners.

The SonoSite ultrasound systems referenced in this document may be covered by one or more of the following U.S. patents: 5722412, 5817024, 5893363, 6135961, 6203498, 6364839, 6371918, 6383139, 6416475, 6447451, 6471651, 6569101, 6648826, 6575908, 6604630, 6817982, 6835177, 6962566, 7169108, 7449640, 7534211, 7549961, 7588541, 7591786, 7604596, 7643040, 7686766, 7694814, 7727153, 7740586, 7804970, 7809400, 7819807, 7841575, 7849250, 7867168, 7883276, D456509, D461895, D509900, D538432, D544962, D558351, D559390, D591423, D592750, D592760, D625014, D625015, and by the following counterpart foreign patents: AU727381, AU730822, CA2372152, CA2372158, CA2373065, CN ZL 97113678.5, CN ZL 98106133.8, CN ZL 98108973.9, CN ZL 200830007734.8, DE60021552.0, DE60029777.2, DE60034670.6, DE69730563.5, DE6980539.6, DE69831698.3, DE60 2004 23 816.3-08, FR0815793, FR0875203, FR0881492, FR1175713, FR1180970, FR1589878, GB0875203, GB0881492, GB1175713, GB1180970, GB1180971, GB1589878, IT0815793, IT0881492, IT1175713, IT1589878, KR528102, KR532359, NO326202, NO326814, NZ542968, RCD000897368-0001, SP0815793, SP0881492, SP1589878. Patents pending.

P12963-03 03/2011

Copyright 2011 by SonoSite, Inc.

All rights reserved.

SonoSite Workflow Solutions

User Guide

Contents

Getting started	1
Starting SWS and logging in	2
About image quality	3
Browse view	3
Transferring and importing exams	3
Organizing the list	4
Searching for exams	5
Exporting worksheet data	6
Review view	7
Images and clips	7
Patient information	9
Worksheets	9
Creating and sending reports	11
Troubleshooting	12
Technical support	13

Getting started

SonoSite[®] Workflow Solutions, also called SWS[™] software, lets you quickly organize and manage exams from DICOM-compatible ultrasound systems. Supported SonoSite systems include the M-Turbo[®], S Series[™], NanoMaxx[®], and MicroMaxx[®] ultrasound system.

You can update and correct patient information, fill out worksheets, create PDF reports, send HL7 reports, and export images for teaching and billing.

SWS runs on a *host* server, which you access through a *client*.

SWS has two views: Browse and Review.

Starting SWS and logging in

SWS requires users to log in. A maximum of five users can be logged in at any one time.

SWS automatically logs you out after 20 minutes of inactivity. Before logging you out, it saves any changes.

To start SWS

You start SWS the same as other standard applications.

- ❖ Do the following, depending on preferences set during installation:
 - (Windows) On the **Start** menu, point to **Programs**, point to **SonoSite**, and click **SonoSite Workflow Solutions**. Or double-click the SonoSite Workflow Solutions icon on the desktop.
 - (Mac OS) In the **Applications** folder, double-click **SonoSite Workflow Solutions**. Or click the SonoSite Workflow Solutions icon in the dock.

To log in

- ❖ In the login window, type your user name and password, and click **Login**.

For your user name and password, see the SWS administrator.

The login window appears when you start SWS.

If less than 10GB storage remains on the server, SWS displays an alert. See the SWS administrator about archiving and deleting backed-up exams to free up space.

To log out, click **LOG OUT**.

To change your password

- 1 In SWS, click **SETTINGS** in the upper right-hand corner.
- 2 Enter your old password.
- 3 Enter and reenter your new password.
- 4 Click **Submit**.
- 5 Click **BROWSE** to exit and return to Browse view.

To display server information

- ❖ In the login window, click **Server Information?**.

About image quality

On a computer, images and clips may not look exactly the same as on the ultrasound system. Factors such as the monitor; video card; and brightness, color, and resolution settings all affect the appearance of images and clips.

Before reviewing exams in SWS, view sample images and clips to make sure that the computer display settings are optimal. Using the computer's controls, adjust settings as necessary. For best appearance, use the following settings:

- Minimum color quality of 24-bit (32-bit recommended)
- Resolution of 1280 by 1024 pixels or, for a wide screen, 1440 by 900 pixels

Browse view

SonoSite Workflow Solutions (SWS™ software) starts in Browse view and displays a list of exams received within the last 24 hours.

You can search the list in several ways to locate specific exams for review. (See "Review view" on page 7.)

Note: Only a user with administrator access can delete an exam. See the SonoSite Workflow Solutions Administrator Guide.

Transferring and importing exams

You can transport exams from the ultrasound system to SWS by *transferring* them over the network or by *importing* them from a USB storage device. The methods available depend on your ultrasound system. See the ultrasound system's documentation or the SWS administrator.

To transfer exams over the network

❖ On the ultrasound system, end the exams.

Depending on the system, the exams transfer automatically. In SWS, click

Recent  to show the transferred exams.

Note: All images and clips may not arrive at once.

To import exams from a USB storage device

1 Insert the USB storage device (which contains the exams) into a USB port on the computer.

For instructions to export exams from the ultrasound system to a USB storage device, see the ultrasound system user guide.

- 2 In SWS, click **Import** .
- 3 Navigate to the USB storage device, and select the DICOMDIR file. Click **Open**.
- 4 Select the exams you want, and click **Import**.
- 5 When the import is complete, click **Exit**.
- 6 Click **Recent**  to show the imported exams.

Organizing the list

The list of exams is organized into the following columns. By default, the list is arranged alphabetically by patient name.

The patient information from the ultrasound system populates the list and is editable in Review view. See "[Patient information](#)" on page 9.

Name Patient Name

ID Patient ID

Date Date of the exam

Exam Type If multiple exam types were selected on the ultrasound system, they are separated by commas.

Images Number of images and clips in the exam:

x/y, where x is the images and y is the clips.

Worksheets Names of worksheets that are specified as **Completed** or as **Ready for Review**, or both. Both appear by default. To refine results, perform an Advanced search. (See "[Searching for exams](#)" on page 5.)

User Initials of who performed the exam (These initials are from the ultrasound system and may differ from the SWS user name.)

Clinician SWS user or users who specified a worksheet as **Completed** or **Ready for Review** or both. Both appear by default. To refine results, perform an Advanced search. (See "[Searching for exams](#)" on page 5.)

AE Title of the ultrasound system where the exam originated

 An icon indicates that the exam is currently locked because another user is reviewing it.

 An icon indicates that the exam was archived.

 An icon indicates that a PDF report has been created.

 If an overreader has entered comments, an icon appears:

 Agreement with the clinician's findings.

 Disagreement with the clinician's findings.

 Multiple comments. Open the exam for additional details.

To sort the list

❖ Click the column heading that you want to sort by. Click it again if sorting in reverse order.

To reposition columns

❖ Drag column headers.

You can also drag the column separators to adjust the width.

To display specific exams

❖ Do any of the following:

- To display all exams, click **SEARCH** with the textbox clear.
- To display only exams performed in the last 24 hours, click **Recent** .
- To display exams based on specific search criteria, see "[Searching for exams.](#)"

Searching for exams

To perform a basic search

1 In the textbox at the top, type characters to search on, and click **SEARCH**.

The list automatically populates with exams that contain those characters in at least one of the following columns: Name, ID, Exam Type, Worksheets, User, Clinician, and AE. The number of results appears in the lower left-hand corner.

2 If you want to add recently ended exams to a search, click **Update** . (The search is re-executed.)

To perform an advanced search

- 1 Click .

The Advanced Search fields appear.

- 2 Complete the fields to search on.

If searching for exams that have worksheets, select from the **Worksheet** list, and select one of the following:

Completed Worksheets specified as **Completed**.

Requested/Reviewed Worksheets specified as **Ready for Review** or overread.

All All worksheets.

- 3 Click **SEARCH**.

Each result contains matches to all the fields completed.

From and **To** specify the date range to search on. Click each calendar icon, and select a date. To clear these fields, click **Clear Date**.

- 4 If you want to add recently ended exams to the search, click **Update** . (The search is re-executed.)

For a basic search, click .

Exporting worksheet data

You can export worksheet data to Microsoft Excel (XLS) format from worksheets that are specified as Ready for Review, Completed, or both. The data include worksheet type, clinicians (initiating, completing, and overreader), overreader's evaluation, patient ID, and more. Using the exported data, you can, for example, perform statistical analysis and track progress of participants in an ultrasound accreditation program.

See also "[Worksheets](#)" on page 9.

Note: The initials in the User column in the XLS file are from the User entry on the SonoSite ultrasound system and may differ from the SWS user name.

To export worksheet data to XLS

- 1 Perform an advanced search to display the exams containing the worksheets of interest. (See "[To perform an advanced search](#)" on page 6.)

Make sure to include only worksheets that are specified as Ready for Review, as Completed, or both.

2 Click **Export to XLS** .

3 In the **Export credentialing data as** dialog box, specify a location and file name. Click **Save**.

Review view

In Review view, you can see exam details (including images and clips), edit patient information, complete worksheets, create reports, and export images and clips for training, presentations, or other purposes.

To open an exam

1 In Browse view, do one of the following:

- Double-click the exam.
- Select the exam and click **REVIEW** .

2 If the patient has multiple exams, select the exam in the **Select Procedure for Review** dialog box, and click **Review**.

To return to Browse view, click **BROWSE** .

Images and clips


By default, images and clips appear as thumbnails in the right-hand pane. You can resize the pane by dragging its left handlebar.

You can also change the image layout as needed.

To change the image layout

❖ In the upper right-hand corner, click the icon for the desired layout view. See [Figure 1.1](#) on page 7.

Double-click a thumbnail to display the full-size image or clip.

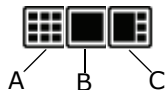*Figure 1.1 Image-layout views: **A**. All thumbnails, **B**. Selected image or clip above a horizontal row of thumbnails, and **C**. Thumbnails in a vertical row on the right.*

To adjust image size, brightness, or contrast

You can adjust the size, brightness, and contrast of an image or clip for optimal viewing. Settings revert to the original when you exit the exam.

- 1 Display the full-size image or clip.
- 2 In the upper right-hand corner of the window, click an Image Display control. See [Figure 1.2](#) on page 8.

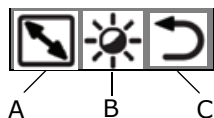
Figure 1.2 Image Display controls:

- A.** Enlarges and reduces the image
- B.** Adjusts the brightness and contrast. Dragging up and down on the image adjusts brightness. Dragging left and right adjusts contrast.
- C.** Resets the original brightness and contrast settings.

To print an image

- ❖ Select the image, and click **Print** .

To delete images

- 1 Select the images, and click **Delete** .
- 2 Click **Yes** to confirm deletion.

To export images and clips

You can export individual images and clips—for example, to use in training. If they are from a SonoSite system, patient information in the image headers is removed.

Clips export in FLV (Flash Video) format and can be played in any media player that supports FLV files (such as VLC media player). For information on a free VLC media player, see www.videolan.org.

- 1 Select the images and clips for export.
 - To select contiguous images and clips, Shift-click the first and last one in the range.
 - To select or deselect noncontiguous images and clips, Ctrl-click (Windows) or Command-click (Mac OS) them.
- 2 Click **Export** .
- 3 Select a folder for the images and clips, and click **OK** (Windows) or **Select** (Mac OS).

Patient information

Depending on the ultrasound system, the fields on the Patient pane are populated with entries from the exam.

The Patient pane is organized into three sections:

- Patient information
- Doctor and institutional information
- HL7 information, required if sending HL7 reports. (See also ["To create a PDF report or send an HL7 report"](#) on page 11.)
 - Placer Order: Order number assigned by the entity that ordered the ultrasound exam
 - Placer Application: HL7 ID of the software application that created the order number
 - Requested Service ID: Type of service that was ordered and performed

In addition, the ID field must match the patient's ID in the software application (EMR) to which you send the HL7 report.

To edit patient information

You can edit information that is not from a worklist.

1 Open the exam, and click the **Patient** tab.

2 Make changes, and click **Save** .

If a specific change is disallowed, SWS displays an alert.


Worksheets

You can fill out worksheets and do any of the following:

- Export the worksheet data for use in credentialing. EMED and Generic worksheets also have a section for overreader input. See ["Exporting worksheet data"](#) on page 6.
- Create reports. See ["Creating and sending reports"](#) on page 11.

EMED worksheet information entered on the M-Turbo 1.3 (or later) or S Series 1.3 (or later) ultrasound system is automatically populated in the respective worksheets in SWS.

Example Worksheet Workflow

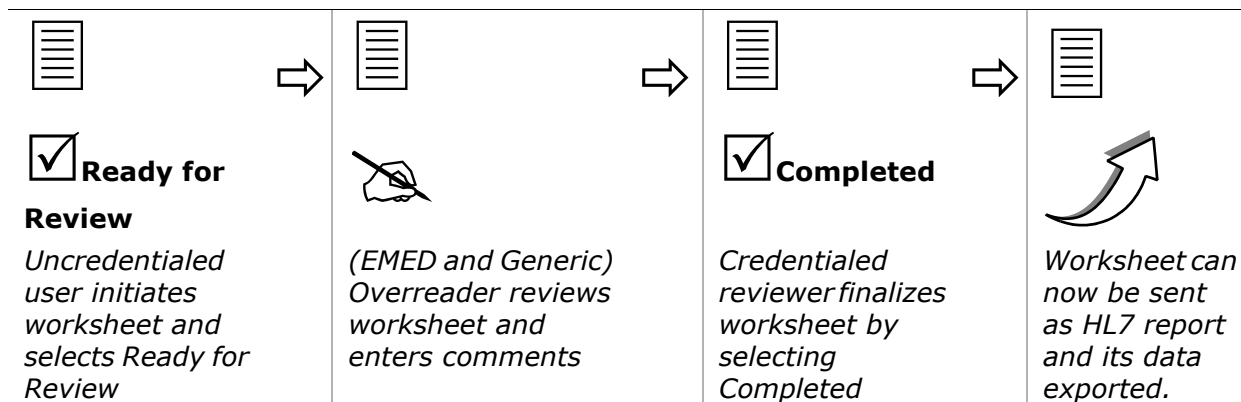To display a worksheet

- 1 Open the exam, and click the **Worksheets** tab.
- 2 Select the worksheet from the **WORKSHEET** list. For **Generic**, select also from the **Worksheet name** list that appears.

Note: Available worksheets are specified by the SWS administrator.

To initiate or complete a worksheet

- 1 Display the worksheet.
- 2 Fill out the fields.
- 3 Under **Clinician Declaration**, select any of the following:

Ready for Review The worksheet is ready for the overreader.

Completed The worksheet has been overread (if applicable) and is final. Its data can be sent as an HL7 report. This setting is available only to user accounts that have credentialing privileges.

- 4 Click **Save** .

SWS prompts you to save changes if you try to return to Browse view or create a report with unsaved changes.

To clear the worksheet, click **Clear**.

To overread a worksheet (EMED and Generic worksheets only)

Users with Overreader account privileges can overread worksheets. To locate worksheets ready for review, perform an advanced search. (See ["To perform an advanced search"](#) on page 6.)

1 Display the worksheet.

2 Under **Overreader Comments**, do the following:

- Select **Agree** or **Disagree**.
- Select **TP** (True Positive) or **TN** (True Negative), or **FP** (False Positive) or **FN** (False Negative)
- In the textbox, type any additional notes.

3 Click **Save** .

Creating and sending reports

You can create a PDF report for use in billing or other purposes. The report contains the patient and worksheet information (except overreader comments) and up to 10 still images in an organized format. (To customize the report stationery, see the SWS administrator.)

You can send an HL7 report for an exam if the following fields on the Patient pane have entries: Last, First, Accession, ID, Placer Order, Placer Application, and Requested Service ID. (See ["Patient information"](#) on page 9.)

Note: The destination for HL7 reports is specified by the SWS administrator.

To create a PDF report or send an HL7 report

1 Display the completed worksheet.

2 If including images, select up to 10 images, and select **Include selected images in report**.

To select contiguous images, Shift-click the first and last one in the range.

To select or deselect noncontiguous images, Ctrl-click (Windows) or Command-click (Mac OS) them.

3 Do any of the following:

- To create a PDF report, click **Create** .

The report appears in an Adobe Reader window. Using buttons on the toolbar, you can save, e-mail, or print the report.

- To send an HL7 report, click **Send** .

SWS displays the message, "Note: The completing clinician's name will appear in the report's electronic signature." Click **Proceed** to continue.

SWS displays a message that the report has been sent or that you must provide missing information.

Troubleshooting

If you encounter an issue not addressed or resolved by the following list, please contact SonoSite Technical Support.

Exams do not automatically transfer Make sure that the ultrasound system is configured properly. See the SWS administrator.

Only some images and clips transfer An exam's images and clips may not all transfer at once. If some images and clips appear missing after you click Recent, return to Browse view, and reopen the exam.

Login button is unavailable If SWS does not have the correct server connection information, the Login button is unavailable. In the login window, click **View Server Information**, click **Edit**, enter the host name and port, and then click **Submit**. To close the dialog box, click **Hide Server Information?**

No reports If you are unable to create reports, make sure that Adobe Reader 8.1 or later is installed.

Worksheets and other information do not transfer If some information from the ultrasound system does not transfer, make sure that you are using a version that supports this capability. Upgrade the ultrasound system's software, if necessary.

Only some worksheets export to XLS Make sure that each worksheet you want to export is specified as **Ready for Review** or **Completed** and that the exam with which it is associated appears in Browse view.

Technical support

For technical support, see the SWS administrator. Or contact SonoSite as follows:.

SonoSite Technical Support

Phone
(US or Canada): 877-657-8118

(Outside US and Canada): 425-951-1330
Or call your representative.

Fax: 425-951-6700

E-mail: service@sonosite.com

Web site: www.sonosite.com
On the Resources menu,
click Support & Service.

Europe Service Center

Phone: +44-(0)1462-444-800

E-mail: uk.service@sonosite.com

SonoSite Workflow Solutions

Administrator Guide

Contents

About administrator view	15
Configuring the ultrasound system	16
Configuring for transfer	16
Configuring for export	17
Adding, deleting, and modifying user accounts	17
Viewing and ending current sessions	18
Configuring the HL7 interface	19
Tracking activity	20
Customizing the Worksheet list and PDF report stationery	21
Archiving and deleting exams	21
Backing up and restoring data	22
Running the script for backing up	22
Running the script for restoring	23
Modifying the server connection information	26
Troubleshooting	25
Restarting the SWS server	25
Modifying the server connection information	26
Technical support	26

About administrator view

The administrator view in SonoSite® Workflow Solutions, also called SWS™ software, has tabbed panes for various tasks:

Account Lets users change their passwords. Available to all users.

Info View legal notices.

User Accounts Add, delete, and modify accounts.

Sessions View and manage current sessions.

Connect Enter the server connection information. Specify the archive location.

HL7 Configure the HL7 interface.

License License the software.

Diagnostics Used by SonoSite Technical Support to resolve connection and other issues, if necessary.

Audit Log Track activity.

Customize Specify items for the Worksheet list, and customize PDF report stationery.

To access administrator view

- 1 Log into SWS from an account with administrator privileges.
- 2 Click **SETTINGS** in the upper right-hand corner.

Configuring the ultrasound system

Before you can transfer or export patient exams from the ultrasound system to SonoSite Workflow Solutions (SWS software), you need to properly configure the ultrasound system.

Private tag information transfers and imports from SonoSite systems only. For more information, please refer to the *SWS DICOM Conformance Statement*.

Note: Most SonoSite systems require that the DICOM feature be licensed. For assistance, contact SonoSite Technical Support.

Configuring for transfer

To enable transferring, connect the ultrasound system to the network. In the connectivity settings, specify host name, IP address, and AE title; JPEG (recommended), monochrome, or RGB image format; and a DICOM transfer format. Also specify SWS as the destination for the transferred data. For assistance, see your system administrator or the manufacturer's documentation.

On SonoSite systems, these tasks involve the following:

- On the ultrasound system, configure a *location* and configure SWS as an *archiver*. Then, associate the archiver with the location. (See also "[Modifying the server connection information](#)" on page 26.)

When configuring the location, select to transfer images at the end of the exam. Also, select **Low** for JPEG compression.

When configuring the archiver, select **Include private tags** if present. Select **JPEG** for Images and for Clips, and set the retry Interval to 180 seconds.

For instructions to configure locations and archivers and to associate them, see the following documentation:

M-Turbo, S Series, or NanoMaxx: *Sending and Receiving DICOM Data on SonoSite Systems*

MicroMaxx: *MicroMaxx Ultrasound System User Guide*

- Connect the ultrasound system to the network, either wirelessly or by Ethernet. For instructions, see the following documentation:

	Wireless	Ethernet
M-Turbo S Series NanoMaxx	<i>Setting Up a Wireless Network</i>	<i>Sending and Receiving DICOM Data on SonoSite Systems</i>
MicroMaxx	<i>MicroMaxx Wireless Installation and User Instructions</i>	<i>MicroMaxx Ultrasound System User Guide</i>

Configuring for export

To enable exporting to a USB storage device in a format readable by SWS, specify a DICOM export format; the AE title; and JPEG (recommended), monochrome, or RGB image format.

For assistance, see your system administrator or the manufacturer's documentation. If configuring a SonoSite system, see *Sending and Receiving DICOM Data on SonoSite Systems*.

Adding, deleting, and modifying user accounts

You can add up to 100 user accounts. There are three levels of access. Up to five users can be logged in at one time. If five users are logged in, an administrator can also log in, for a total of six users.

Users can change their passwords on the Account pane, which is available to all users, regardless of access level.

To add a user

1 On the **User Accounts** pane, click **New User**.

2 Enter the following:

User Name User name for logging in

First Name, Middle Name, Last Name Name that appears on worksheets, in Browse view, and elsewhere.

Prof. Degree Professional degree or status

User Role Level of access. Select one:

Administrator Can access all features and administrator functions.

Standard Can access all but administrator functions

Restricted Can view exams and export images and clips

Overreader For Standard user, select if user has overreader privileges for worksheets. This setting is automatically selected for Administrator users and deselected for Restricted users.

Credentialed For Standard user, select if user has signing privileges for worksheets (That is, the user can mark the worksheet as Completed, enabling its contents to be exported or compiled in a report). This setting is automatically selected for Administrator users and deselected for Restricted users.

3 Enter and reenter the password.

4 Click **Submit**.

To delete users

1 On the **User Accounts** pane, select the check box for one or more users.

To select all users, click **Select All**.

2 Click **Delete Selected**.

3 Click **Yes** to confirm deletion.

To modify a user

1 Next to the user name, click **Edit**.

2 Modify entries, and click **Submit**.

Viewing and ending current sessions

On the Sessions pane, you can find out which users are logged in, send them an on-screen message, and log them out.

To display current sessions

❖ Click the **Sessions** tab.

The Sessions pane displays a list of users who are logged in.

To update the list, click **Refresh**.

To send an on-screen message

You can send a message to any users who are logged in (for example, “You will be logged off in 15 minutes”). The message appears on each recipient’s screen, which becomes unavailable until he or she closes the message.

1 On the **Sessions** pane, select one or more users.

To select all users, click **Select All**.

2 Type the message in the text box.

3 Click **Message Selected Users**.

To log out users

You can schedule a logout in advance.

1 On the **Sessions** pane, select one or more users.

To select all users, click **Select All**.

2 Select the logout time:

a Click the date field.

b Select the date in the calendar that appears.

c Select a time:

i Type or select the hour and the minutes.

ii Select **AM** or **PM**.

3 (Optional) Send an on-screen message about the logout.

4 Click **Logout Selected Users**.

Configuring the HL7 interface

On the HL7 pane, you configure the HL7 interface for sending HL7 reports. For assistance, see your clinical interface specialist and the *SonoSite Workflow Solutions HL7 Interface Specification*.

In order to send an exam as an HL7 report, the HL7 fields on the Patient pane must be filled out. See “[Patient information](#)” in the user guide.

To modify HL7 settings

1 On the HL7 pane, select **Enable** if you want to activate the interface.

2 Modify entries in the text boxes, and select an ORU format.

3 Click **Reset** to undo changes, or click **Save Changes**.

Tracking activity

On the Audit Log pane, you can run queries based on specified activity during a specified date range. You can base queries on the patient name or ID.

Results appear in a list with the following columns:

Event ID (Type)

Event Time/Receive Time

A Event Action

O Event Outcome

F Format

Audit Source

Active Participants

Participant Objects

To query the audit log

1 On the Audit Log pane, select the date range of activity you want to view:

a Click the **Start Date** field, and then click a date in the calendar that appears.

b Click the **End Date** field, and then click a date in the calendar that appears.

Clicking the arrow in the upper right-hand or left-hand corner of the calendar displays the next or previous month, respectively.

2 If you want to search on a specific patient, type the name or ID in the **Patient Name/ID** box.

3 From the **Operation Type** list, select the activity:

Any

Application Activity SWS server started or stopped

Audit Log Used

Instances Accessed Exams viewed; images or clips deleted; or exam or patient information edited

Instances Transferred Exams imported or transferred into SWS

Study Deleted Entire exam deleted

Instances Exported

User Authentication

4 Click **Submit**.

Customizing the Worksheet list and PDF report stationery

On the Customize pane you can specify which items appear in the Worksheet list. You can also customize the PDF report stationery with your institution's logo.

To specify items in the Worksheet list

- 1 Under **Worksheets** on the **Customize** pane, select check boxes for the exam types to include. For Generic worksheets, also type names for the worksheets in the text boxes.
- 2 Click **Save**.

To customize the report stationery

- 1 On the **Customize** pane, click **Select Logo**, and select a JPG file.
- 2 Click **Open** (Windows) or **Select** (Mac OS).

Archiving and deleting exams

Only a user with administrator access can archive or delete exams. You can delete exams either manually or when archived.

To archive exams

You can archive only exams that are closed. Exams archive in a format readable by a DICOM reader. To avoid conflicts, make sure that other users are logged out before you archive exams.

- 1 In Browse view, select the exams to archive.
- 2 Click **Archive** .
- 3 In the **Archive Name** text box, type a name. This name appears in the exam's information in the audit log for help in locating archive activity.
- 4 If you want to delete exams from SWS after they archive, select **Delete After Archiving**.
- 5 Click **Yes**.

The exams archive into a folder with a unique, time-stamped name. In Browse view, each of the exams has a blue checkmark icon  in the Archived Exams column. If you modify the exam, the icon disappears.

To manually delete exams

- 1 In Browse view, select the exams to delete.
- 2 Click **Delete** .
- 3 Click **Delete** to confirm deletion.

Backing up and restoring data

Make sure to back up your SWS data regularly. If you experience data loss because of equipment failure, equipment damage, power surge, or other such event, you can restore the data from the backup files. Because you can restore only data that is backed up, back up data frequently, according to standard best practices and your institutional policy.

SWS includes two command-line scripts that you run from the SWS server: one for backing up data and one for restoring data.

Running the script for backing up

When running the script for backing up, you need your SWS database password, the location of the data you are backing up, and the external location where you want to place the data.

To run the script for backing up (Windows)

- 1 Log on to Windows as administrator.
- 2 Open a Command Prompt window: Click the **Start** button, point to **Programs** or **All Programs**, point to **Accessories**, and do one of the following:
 - (Windows Vista or Server 2008) Right-click **Command Prompt** and click **Run as administrator**.
 - (Windows XP) Click **Command Prompt**.
- 3 At the command prompt, type the following, and then press ENTER:

```
BackupSWS.bat MySQLpassword "data destination"
```

where *MySQLpassword* is the MySQL password and *data destination* is the external location where you want to place the backup.

For example, type:

```
BackupSWS.bat MyP@ssword "E:\SWS_backups"
```

The data back up in a folder named BACKUP-yyyymmdd, where *yyyymmdd* is the current year, month, and day. The folder is located in the destination you specified (for example, E:\SWS_backups).

- 4 Close the Command Prompt window.

To run the script for backing up (Mac OS)

1 Open a terminal window: In the **Applications/Utilities** folder, double-click **Terminal**.

2 At the prompt, type the following and then press ENTER:

```
sudo -s
```

3 At the **password** prompt, type your system password and then press ENTER.

4 At the prompt, type the following and then press ENTER:

```
cd /install path/tools
```

where *install path* is the location where SWS is installed. For example, type:

```
cd /Library/SWS11Server/tools
```

5 At the prompt, type the following and then press ENTER:

```
bash BackupSWS.sh MySQLpassword "data destination"
```

where *MySQLpassword* is the MySQL password and *data destination* is the external location where you want to place the backup.

For example, type:

```
bash BackupSWS.sh MyP@ssword "/SWS_backups"
```

The data back up in a folder named BACKUP-yyyymmdd, where *yyyymmdd* is the current year, month, and day. The folder is located in the destination you specified (for example, /SWS_backups).

Running the script for restoring

When running the script for restoring, you need your SWS database password, and the location and name of the folder (BACKUP-yyyymmdd) you want to restore from.

Important: This script overwrites any current data (including images and clips) with the backed up data you specify. Any new data saved since backing up will be overwritten and lost.

To run the script for restoring (Windows)

1 Log on to Windows as administrator.

2 Open a Command Prompt window: Click the **Start** button, point to **Programs** or **All Programs**, point to **Accessories**, and do one of the following:

- (Windows Vista or Server 2008) Right-click **Command Prompt** and click **Run as administrator**.
- (Windows XP) Click **Command Prompt**.

3 At the command prompt, type the following, and then press ENTER.

```
RestoreSWS.bat MySQLpassword "backup location\BACKUP-yyyymmdd"
```

where *MySQLpassword* is your MySQL password, *backup location* is the location of the backed-up data, and *BACKUP-yyyymmdd* is the name of the folder you want to restore from (*yyyymmdd* is the date).

For example, type:

```
RestoreSWS.bat MyP@ssword "E:\SWS_backups\BACKUP-20091025"
```

4 Close the Command Prompt window.

To run the script for restoring (Mac OS)

1 Open a terminal window: In the **Applications/Utilities** folder, double-click **Terminal**.

2 At the prompt, type the following and then press ENTER:

```
sudo -s
```

3 At the **password** prompt, type your system password and then press ENTER.

4 At the prompt, type the following and then press ENTER:

```
cd /install path/tools
```

where *install path* is the location where SWS is installed. For example, type:

```
cd /Library/SWS11Server/tools
```

5 At the prompt, type the following and then press ENTER:

```
bash RestoreSWS.sh MySQLpassword "backup location/BACKUP-yyyymmdd"
```

where *MySQLpassword* is your MySQL password, *backup location* is the location of the backed-up data, and *BACKUP-yyyymmdd* is the name of the folder you want to restore from (*yyyymmdd* is the date).

For example, type:

```
bash RestoreSWS.sh MyP@ssword "/usr/local/backups/BACKUP-20101125"
```

Troubleshooting

If you encounter an SWS software issue not addressed or resolved by the following list, please contact SonoSite Technical Support.

Server does not start Make sure that JDK 6 is installed and that MySQL is installed, set up, and running. In Windows, make sure that the JAVA_HOME environment variable is configured. See *SonoSite Workflow Solutions Installation Instructions*.

Clients cannot connect to the server Make sure that your firewall is not blocking incoming connections over HTTP (8080) or to the MySQL database (3306). Also, allow inbound echo requests (ICMPv4).

If you have two network interfaces (for example, both Ethernet and wireless), make sure that you configured the ultrasound system and clients to recognize the same interface, and then disable the other one.

Cannot transfer exams Make sure that your firewall is not blocking incoming connections over the port specified in the connectivity settings (Default is 11112).

Make sure that the ultrasound system is properly configured. See ["Configuring for transfer"](#) on page 16.

If you have two network interfaces (for example, both Ethernet and wireless), make sure that you configured the ultrasound system and clients to recognize the same interface, and then disable the other one.

No space at storage location If the storage location runs out of space, you can either archive (and delete) data to free up space, or you can specify a different storage location that has more space. To specify a different storage location, back up your data and then reinstall the SWS server software, specifying the desired storage location when prompted. Afterward, restore your data to the new storage location.

See also ["Archiving and deleting exams"](#) on page 21, ["Backing up and restoring data"](#) on page 22, and the SWS installation guide.

Restarting the SWS server

To restart the SWS server in Windows

- 1 In Control Panel, click **Administrative Tools**, and then click **Services**.
- 2 Select **SonoSite Workflow Solutions**, and then click **Restart**.

To restart the SWS server in Mac OS

1 Open a terminal window: In the **Applications/Utilities** folder, double-click **Terminal**.

2 At the prompt, type the following and then press ENTER:

```
sudo -s
```

3 At the **password** prompt, type your system password and then press ENTER.

4 At the prompt, type the following and then press ENTER:

```
launchctl unload /Library/LaunchDaemons/com.sonosite.sws11sp1
```

5 At the prompt, type the following and then press ENTER:

```
launchctl load /Library/LaunchDaemons/com.sonosite.sws11sp1
```

Modifying the server connection information

You can modify the default server connection information. If you transfer exams over the network, make sure that the respective entries on the ultrasound system match the modifications. (See also "[Configuring for transfer](#)" on page 16.)

To modify the server connection information

1 On the **Connect** pane, type the following information for SWS:

- AE title (Default is SWS)
- Port Number (Default is 11112)

2 Click **Submit**.

3 Restart the server. See "[Restarting the SWS server](#)" on page 25.

Technical support

For technical support, contact SonoSite as follows.

SonoSite Technical Support

Phone

(US or Canada): 877-657-8118

(Outside US and 425-951-1330

Canada): Or call your representative.

Fax: 425-951-6700

E-mail: service@sonosite.com

SonoSite Technical Support (Continued)

Web site: www.sonosite.com
On the Resources menu,
click Support & Service.

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