

# Sonosite PX Stand and Accessory Assembly Instructions

---

Manufacturer	EU Authorized Representative	UK Responsible Person	Australia Sponsor
FUJIFILM SonoSite, Inc. 21919 30th Drive SE Bothell, WA 98021 USA T: 1-888-482-9449 or 1-425-951-1200 F: 1-425-951-1201	FUJIFILM SonoSite B.V. Joop Geesinkweg 140 1114 AB Amsterdam, The Netherlands	FUJIFILM Healthcare UK Limited Fujifilm House Whitbread Way Bedford MK42 0ZE United Kingdom	FUJIFILM SonoSite Australasia Pty Ltd 114 Old Pittwater Road BROOKVALE, NSW, 2100 Australia



**WARNING**

To avoid the risk of a shock hazard, do not plug the power cord into a multiple portable socket outlet (MPSO) or into an extension cord.



**CAUTION**

- To avoid damaging the system, use only accessories and peripherals recommended by FUJIFILM Sonosite.
- Federal (United States) law restricts this device to sale by or on the order of a physician.



**NOTE**

For more information on how to use the system, refer to the system user guide or [www.sonosite.com](http://www.sonosite.com).

SONOSITE is a registered trademark of FUJIFILM SonoSite, Inc., in various jurisdictions. FUJIFILM and FUJIFILM VALUE FROM INNOVATION are registered trademarks of FUJIFILM Corporation in various jurisdictions. All other trademarks are the property of their respective owners.

Patent: [www.sonosite.com/patents](http://www.sonosite.com/patents)



P21940-05

Published Aug 20, 2025

Copyright © 2025 FUJIFILM SonoSite, Inc. All rights reserved.


# Table of Contents

- Inside the boxes ..... 1**
- Unpack the boxes ..... 5**
- Attach the storage tray and printer ..... 6**
- Remove packaging ..... 8**
- Connect the stand battery ..... 10**
- Install covers over power receptacles ..... 12**
- Attach the stand accessories ..... 14**
- Attach the cable management system ..... 16**
- Raise the column ..... 17**
- Install the stand head ..... 18**
- Docking the system to the stand ..... 20**
- Connect the stand to power ..... 22**




# Inside the boxes

**Main box**

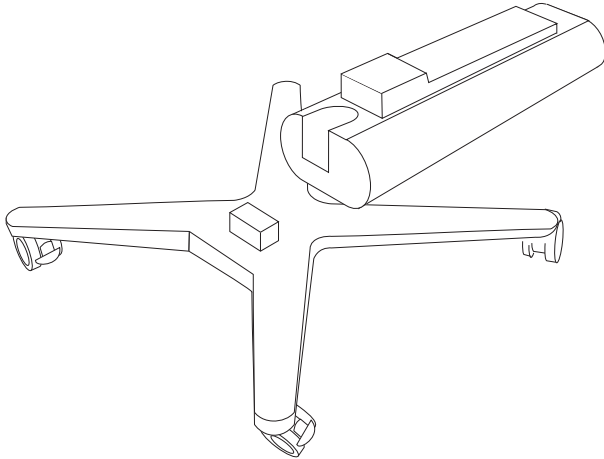


Column and base with battery pack

**Stand head box**

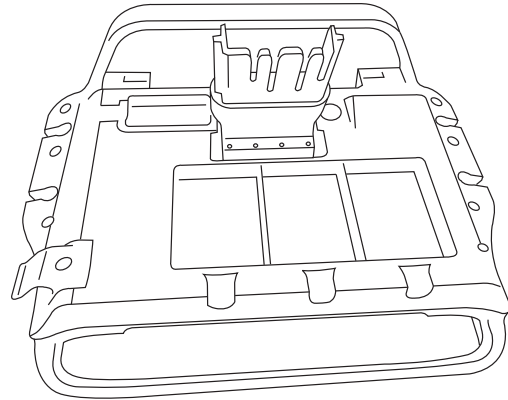


Stand head

**System box**

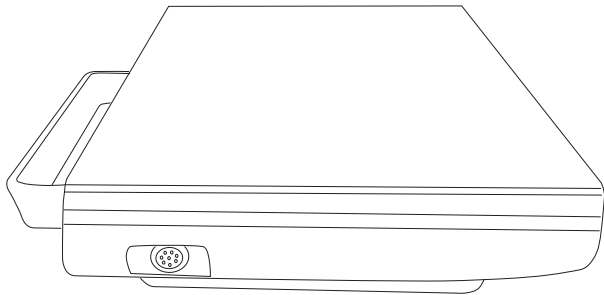


System

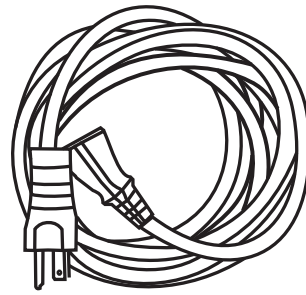

Power cord


## Kit 1

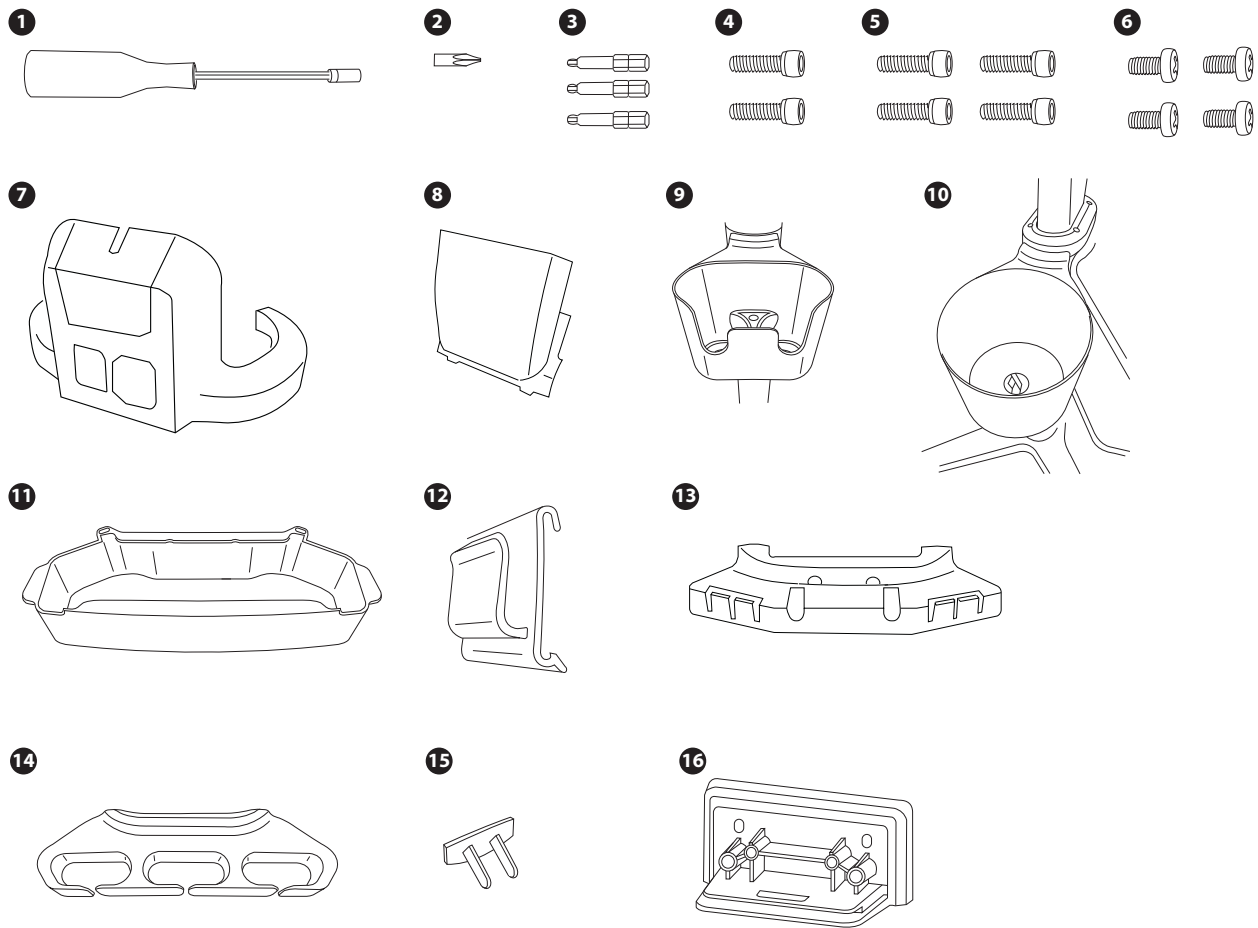


<b>1</b> Magnetic bit driver	<b>9</b> Gel bottle holder
<b>2</b> #1 Phillips bit	<b>10</b> Wipe holder
<b>3</b> 3 mm, 4 mm, and 5 mm hex bits (1 each)	<b>11</b> Lockable storage drawer
<b>4</b> M4 x 20 mm hex-head screws (2)	<b>12</b> Power cord wrap hook
<b>5</b> M6 x 18 mm hex-head screws (4)	<b>13</b> Cable management system mounting bracket
<b>6</b> M3 x 8 mm Phillips-head screws (4)	<b>14</b> Cable management system cover
<b>7</b> Rear base cover	<b>15</b> Outlet cover
<b>8</b> Bottom cover	<b>16</b> Cable stay mount

**Kit 2**

1

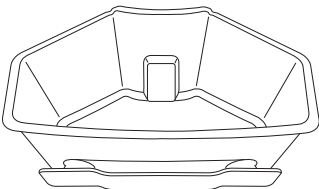

2

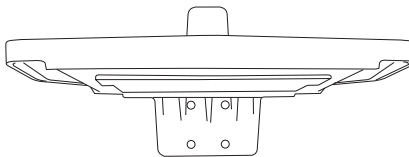

1


Storage container with power cord wrap

2


Storage tray

**Printer box (optional)**

1

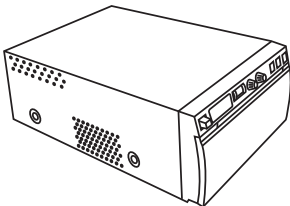


2

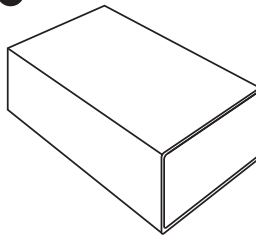

3



4

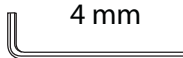

5



1

Printer and user documentation

4

Hex wrench

2

Printer mounting bracket and box


5

M6 x 12 mm hex-head screws (4)

3

Prying tool

Stand battery box

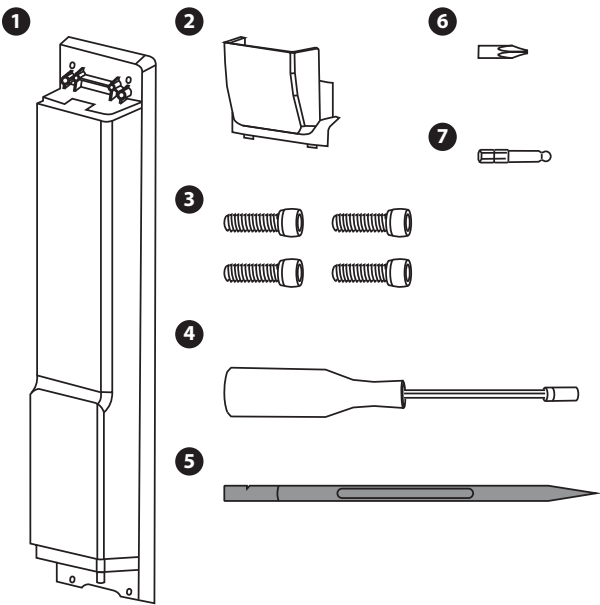


<b>1</b> Stand battery	<b>5</b> Pry tool
<b>2</b> Bottom cover	<b>6</b> #1 Phillips bit
<b>3</b> M4 x 20 mm hex-head screws	<b>7</b> 3 mm hex bit
<b>4</b> Bit driver	




# Unpack the boxes

1. Remove the upper part of the box.

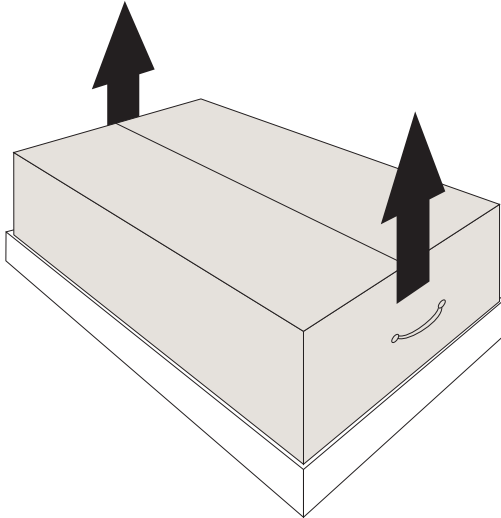


2. Unpack the inner boxes and remove the cardboard packing material.

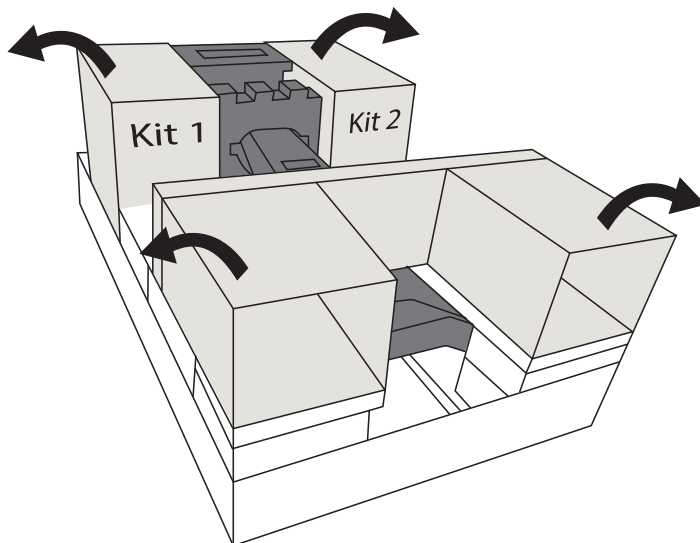


## Attach the storage tray and printer


1. Insert the 5 mm hex bit into the bit driver and use the four hex-head 18 mm long screws to attach the tray to the back of the stand column.

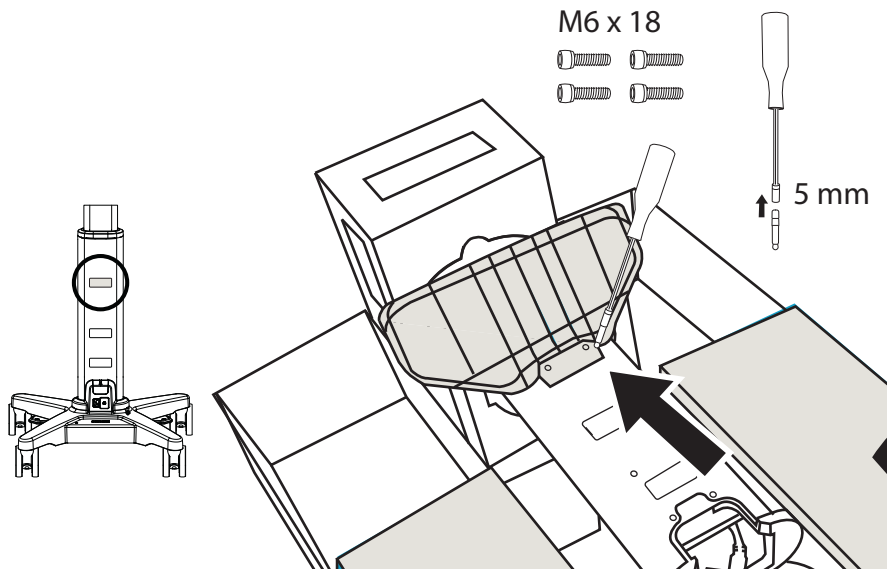


2. Slide the printer into the mounting box.

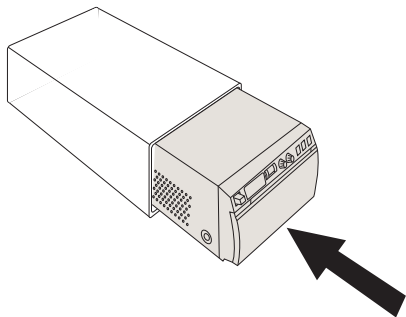


3. Use the bit driver with the #1 Phillips bit to tighten the four screws on the bottom of the mounting box.

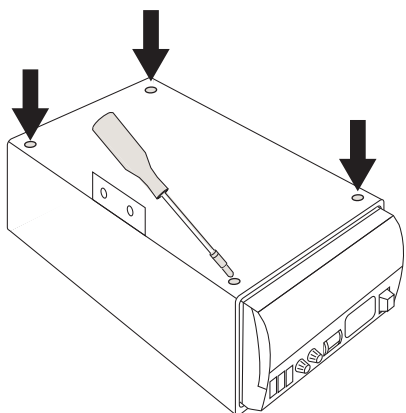


4. Remove the cover on the stand column.

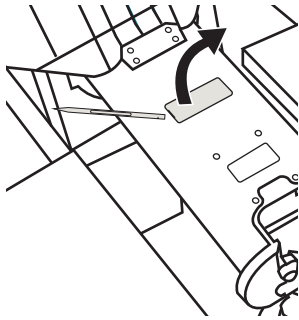


5. Use the bit driver with the 4 mm bit and the hex-head 18 mm screws to attach the printer mounting bracket to the stand.

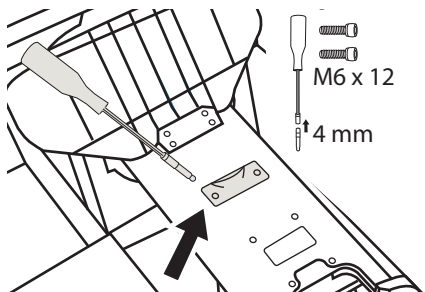

6. Slide the lip of the mounting box down onto the mounting bracket.

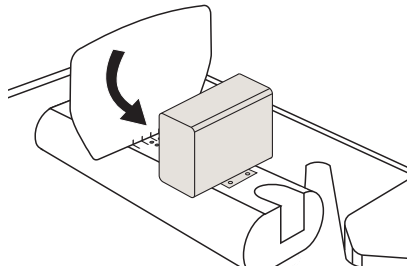


7. Tighten the screws located under the printer.

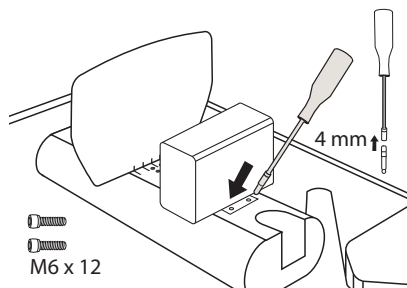


## Remove packaging


1. Raise the column upright.

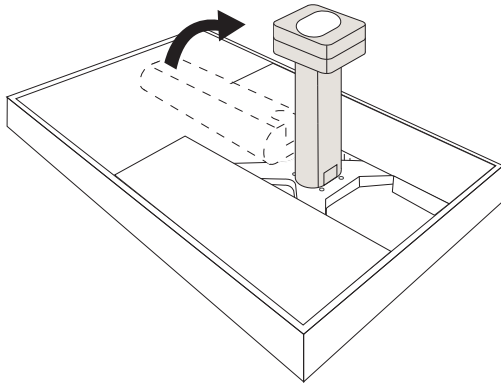


2. Use the bit driver with the 5 mm bit to tighten the four screws on the base of the column.

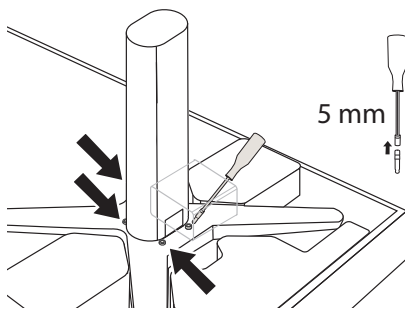


3. Remove the packing foam from on and around the stand.

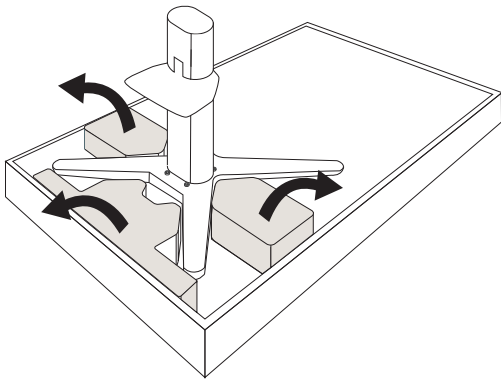


4. Cut the tape holding the front flap of the box up and lower the flap to the floor.

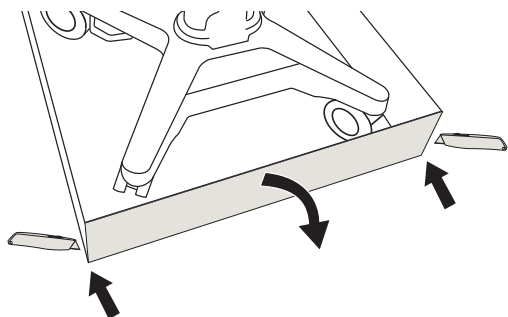

5. Maneuver the stand out of the packing box.

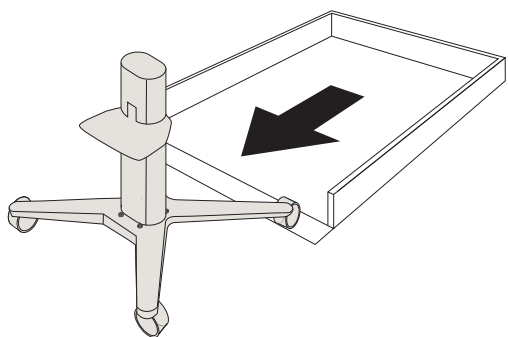

6. Lock two wheels.

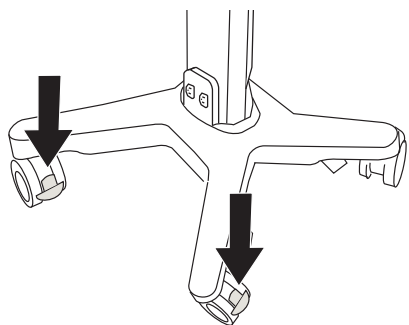


## Connect the stand battery




One option to make installation easier is to lay the stand down. To do this, first lock the rear wheels, and then tilt the stand down.


1. (Optional) Tilt the stand down so it is resting on the tray or the column.

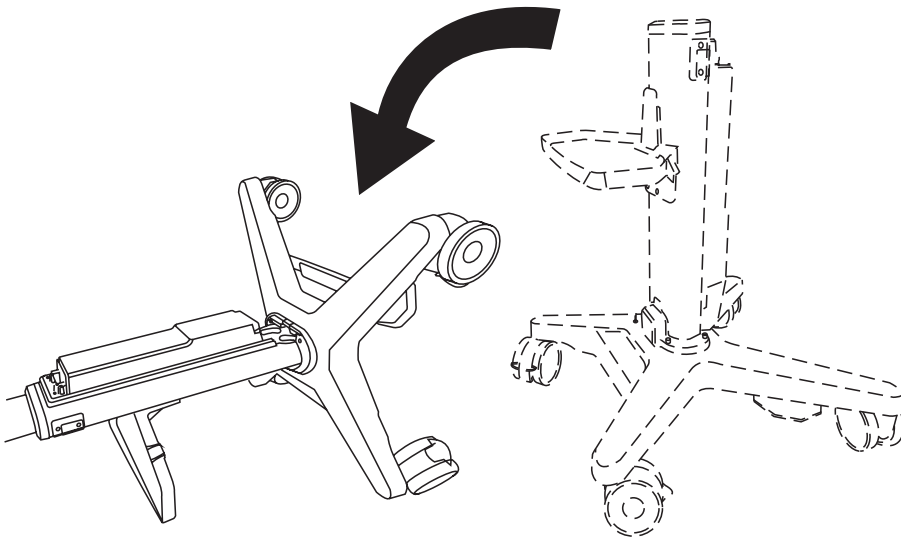


2. Align the cable connectors with the connectors on the bottom of the battery pack using the guide pins and notches. Make sure to orient the connectors as shown.

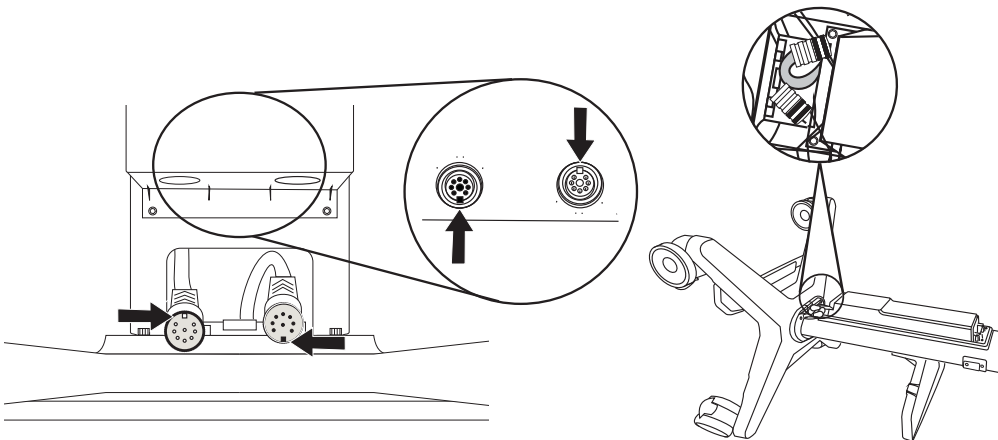


3. Firmly push the cable connectors into place. Once inserted, do not rotate the connector assemblies or you may damage the guide pins.

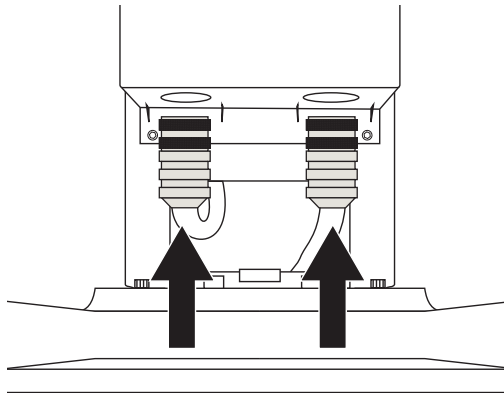

4. While maintaining proper alignment of the connectors, screw the connector collars to tighten.

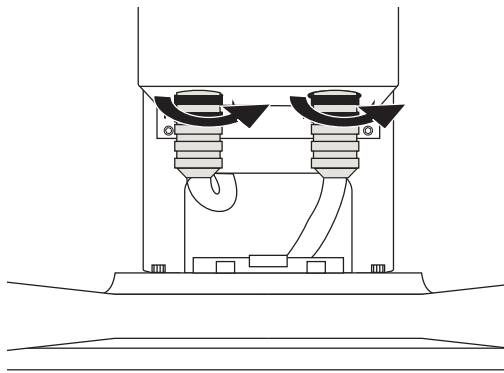


5. If the stand was lying down, tilt the stand back up.

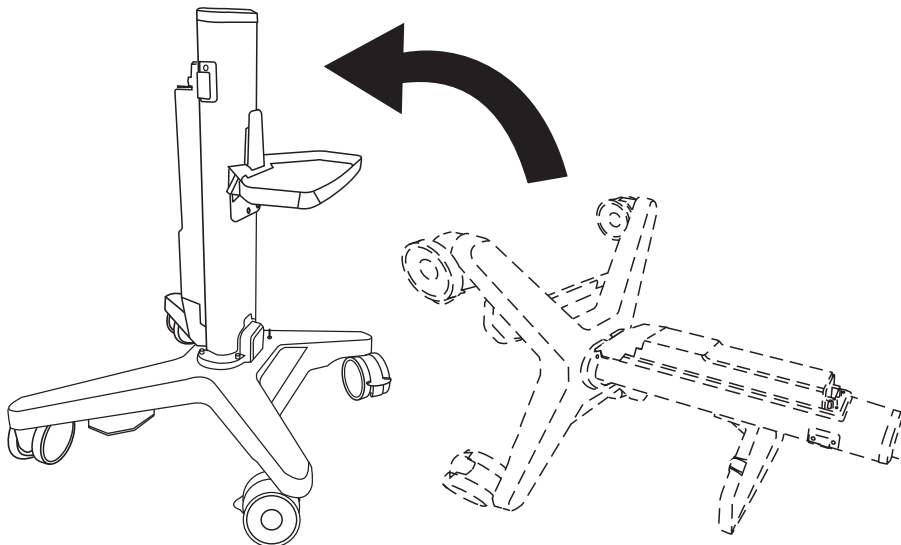


## Install covers over power receptacles

1. Fit the rear base cover around the stand column above the power receptacles, then press the cover straight down with equal pressure on both sides until it snaps into place.

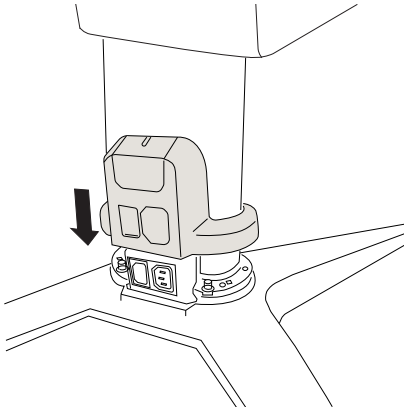


2. If you do not have a printer, cover the printer power outlet.

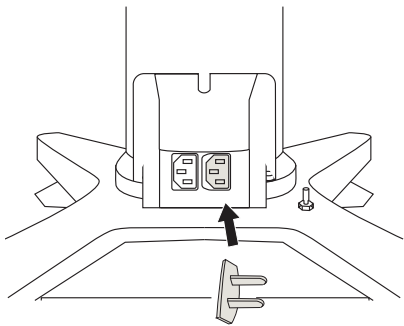




3. Install the cover at the bottom of the battery.

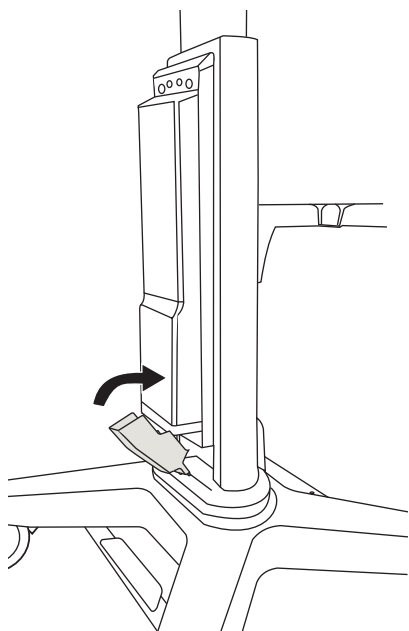


## Attach the stand accessories


1. Slide the storage container down over the tongue of the storage tray until it is resting on the tray.

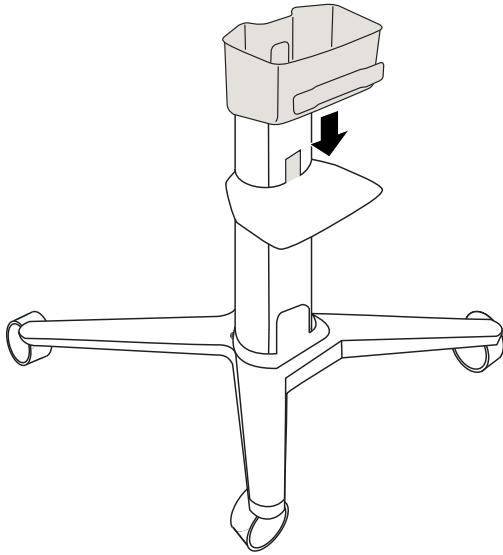


2. Slide the drawer into the slots on the underside of the tray.

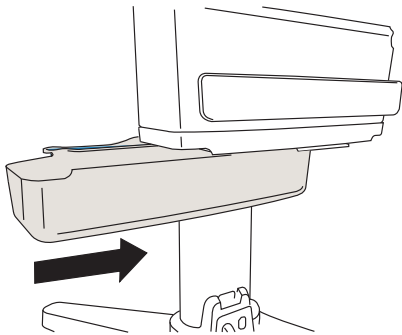


3. Slip the hook over the edge of the cord wrap on the front of the storage container.

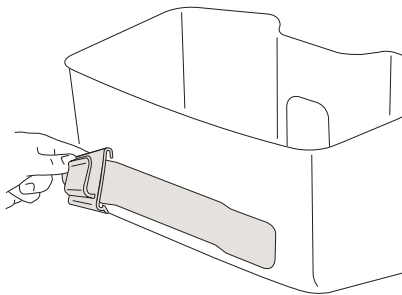

4. Slide the gel bottle holder down onto one of the mounting brackets on either side of the column.

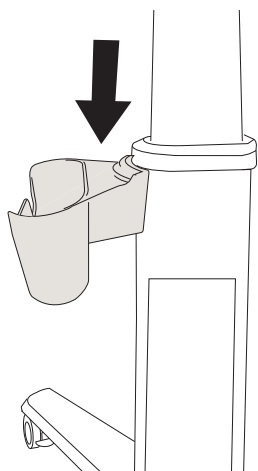


5. Slide the wipe holder down onto one of the mounting brackets on either side of the column.

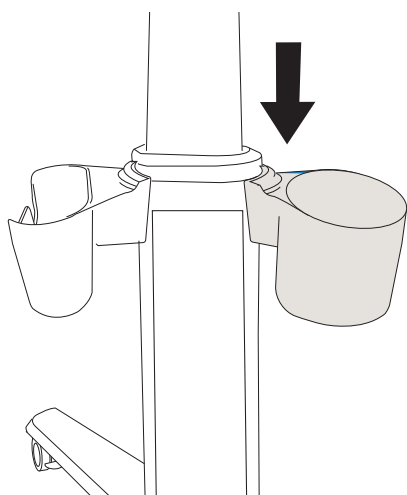


## Attach the cable management system


1. Snap the cable management bracket into position on top of the battery pack.

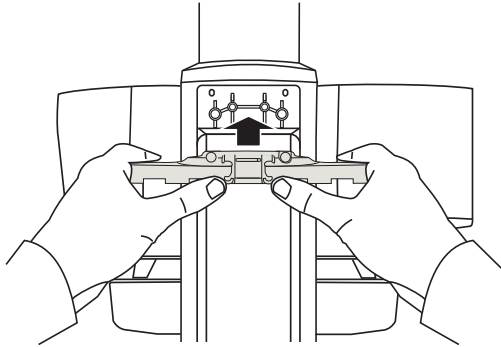

2. Attach the cable management bracket to the column using the bit driver with the Phillips bit and the two 8 mm screws.

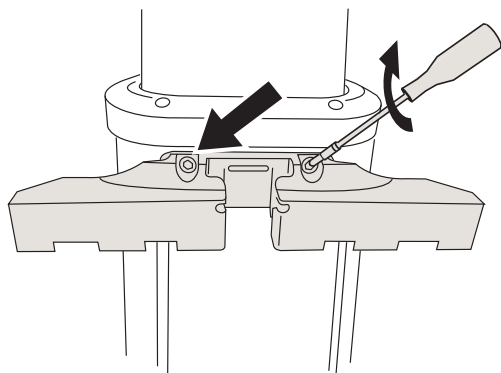


3. Slide the cable management cover down over the top of the cable management bracket until it snaps into place.

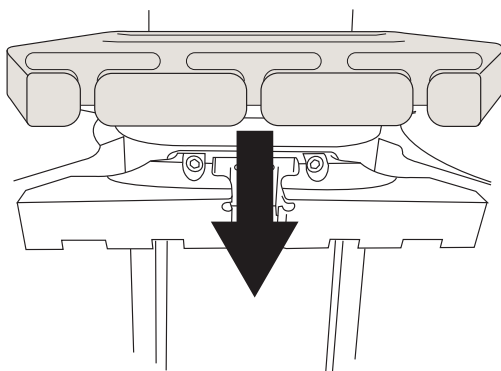


## Raise the column

1. Remove the hook and loop shipping strap that is immobilizing the foot pedal.

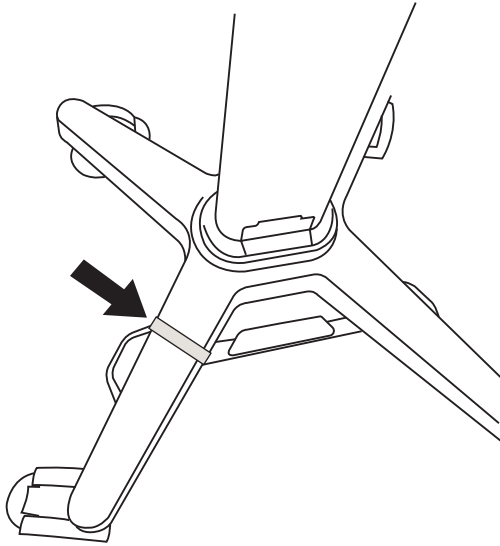


2. Place a hand on top of the platform to prevent it from rising too quickly. Press the foot pedal and raise the inner column partway.

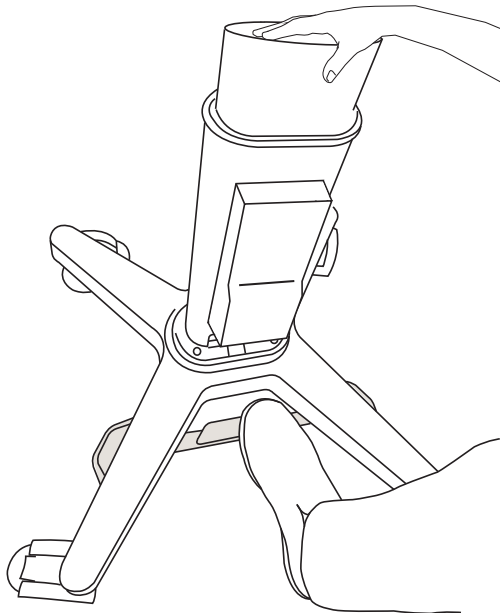


## Install the stand head


1. Insert the stand head pedestal onto the inner stand column.

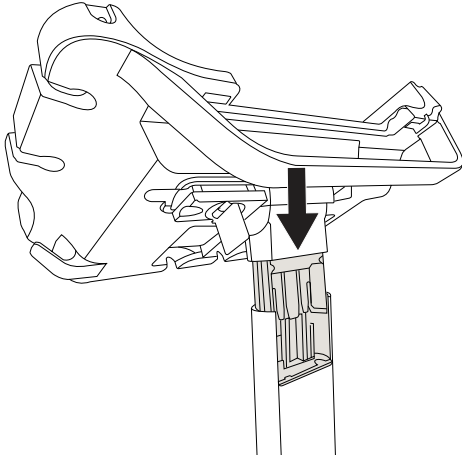


2. Use the bit driver with the 4 mm bit to tighten the four screws on the stand head.

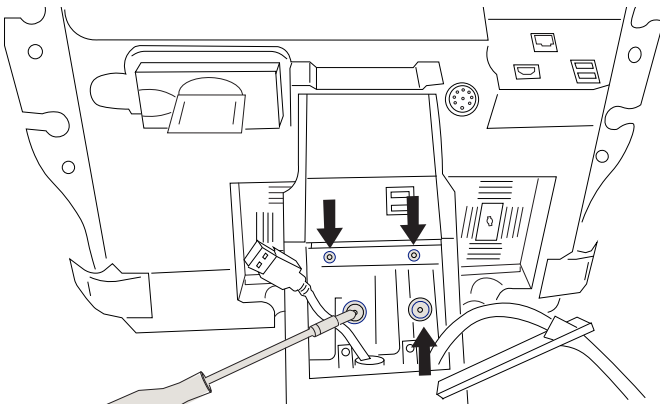


3. Connect the USB cable in the inner column to the USB port inside the stand head. Be certain that the USB cable is oriented correctly before pushing it in.

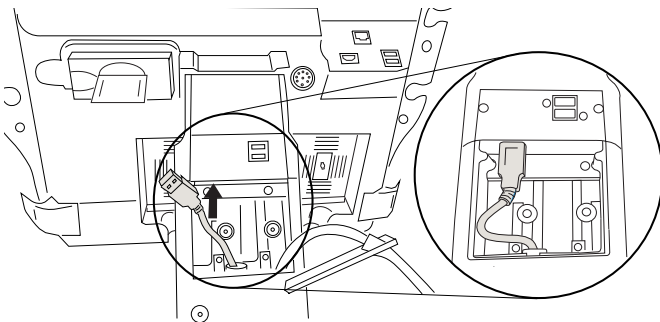


- Slide the power cable grommet into the column cover.

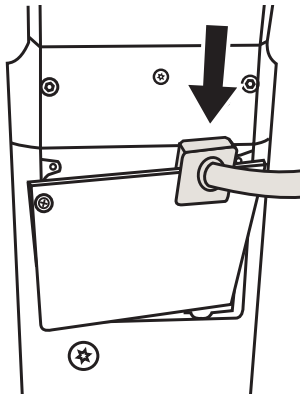

- Attach the column cover to the stand by tightening the two M3 screws with the bit driver and the Phillips bit.

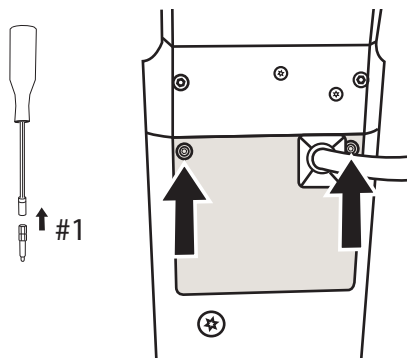


- Connect the column power cable to the stand head by pushing it into place and screwing it into the connector. Tug gently to make sure it is secure.

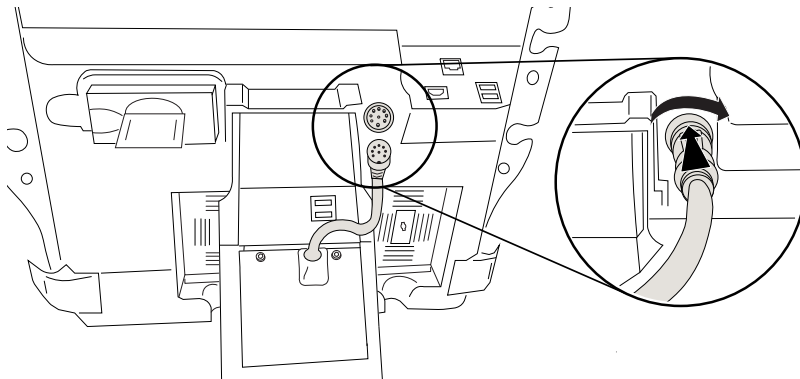


## Docking the system to the stand


1. Make sure that the connector located on the bottom of the Triple Transducer Connect (TTC) is loose by pulling the latch down then rotating it.

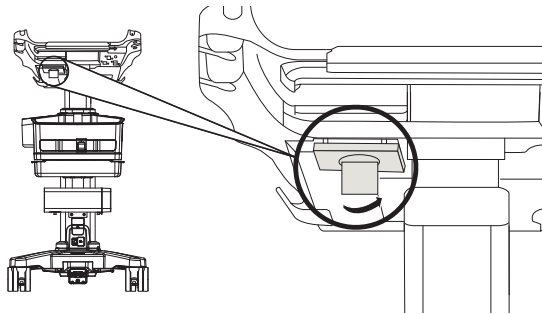

2. Dock the system to the stand head or platform.

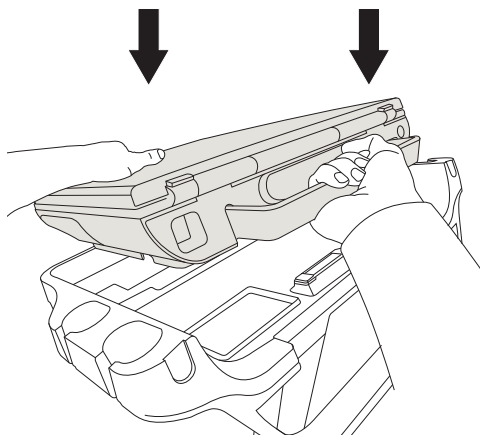


3. Secure the system by rotating the system handle down until it clicks.

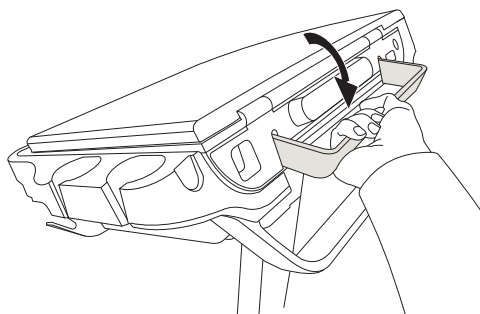




4. Tighten the connection between the system and the TTC by pushing the connector back into place, rotating the latch in the opposite direction, then pressing it up.

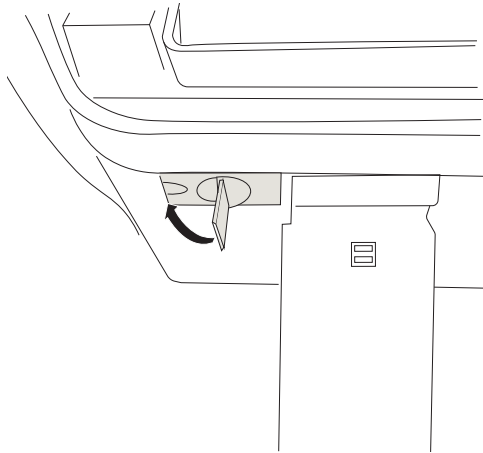


## Connect the stand to power


1. Connect AC power cord from the system box to the base of the stand.

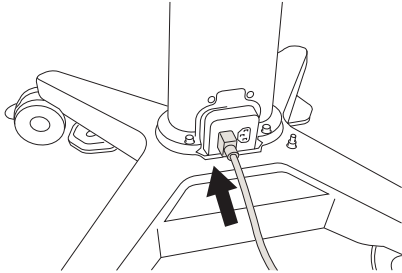


2. Run the AC power cord through the cord clip on the back of the storage tray.

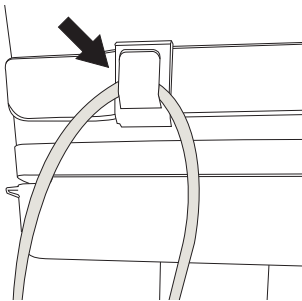


3. Connect the AC power cord to a hospital-grade AC mains outlet.

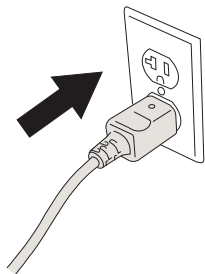


4. Make sure that the power status light on the rear of the stand head is on.

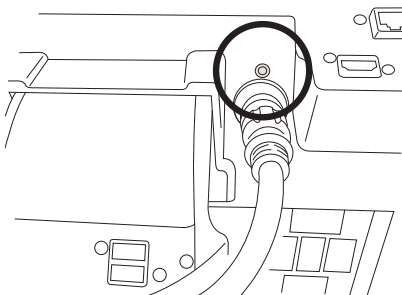


5. Make sure that the power status light on the base of the stand is on.

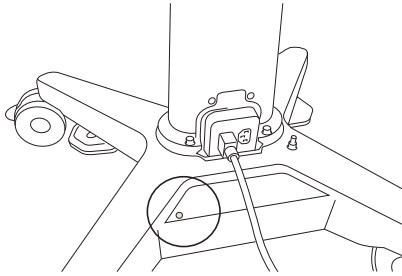


6. Connect the AC power cable to the printer, run the cable through the cable holder, and plug the cable into the stand.

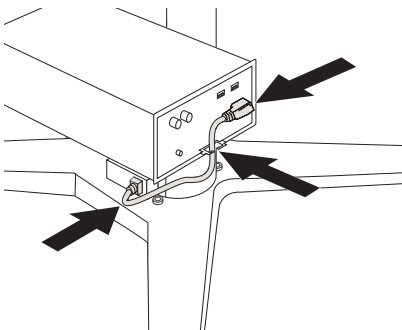


7. Connect the USB cable to the printer, and run the cable through the cable holder.

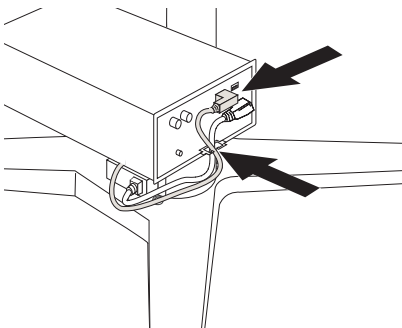


8. Plug the USB cable into the base of the stand.

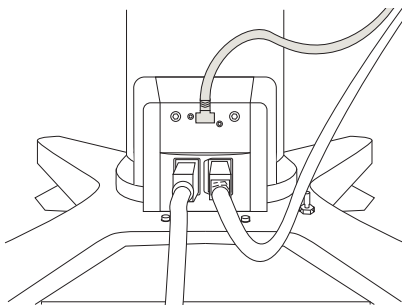

9. Install the cover over the USB connection.

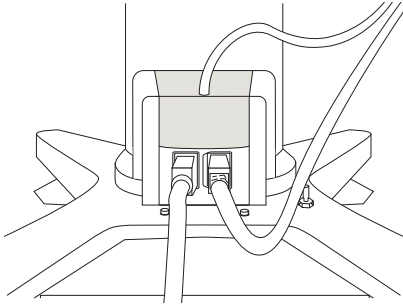

10. Attach the printer cover.

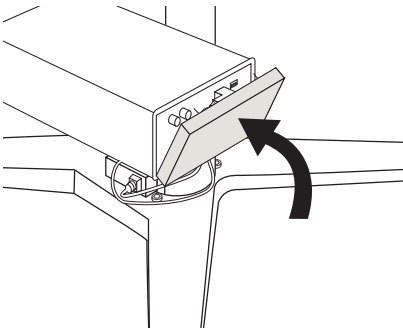

11. Allow the system to charge for six hours before using. This ensures the best battery life and user experience. The power status light will indicate a solid blue when fully charged.





# FUJIFILM

## SONOSITE

P21940-05

