# **Sonosite Ultrasound System**

## **User Guide Errata**

---

**Manufacturer**

FUJIFILM SonoSite, Inc.  
21919 30th Drive SE  
Bothell, WA 98021 USA  
T: 1-888-482-9449 or  
1-425-951-1200  
F: 1-425-951-1201

**EC Authorized Representative**

FUJIFILM SonoSite B.V.  
Joop Geesinkweg 140  
1114 AB Amsterdam,  
The Netherlands

**Australia Sponsor**

FUJIFILM SonoSite Australasia Pty Ltd  
114 Old Pittwater Road  
BROOKVALE, NSW, 2100  
Australia

**Caution**

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite, SONOSITE, SonoSite SII, M-Turbo, and SonoSite X-Porte are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P23353-01

Publication date: July 2021

Copyright © 2021 FUJIFILM SonoSite, Inc. All rights reserved.

# Sonosite Ultrasound System User Guide Errata

<b>Introduction</b> .....	<b>1</b>
Document conventions .....	1
Getting Help .....	2
<b>Cleaning and Disinfecting</b> .....	<b>2</b>
Cleaning Accessories .....	2
<b>Safety</b> .....	<b>3</b>

## Introduction

The content in this document corrects the following user guides. The revisions will be made in the next update.

- ▶ Sonosite Edge II ultrasound system
- ▶ Sonosite iViz ultrasound system
- ▶ Sonosite X-Porte ultrasound system
- ▶ Sonosite SII ultrasound system
- ▶ Sonosite M-Turbo ultrasound system

## Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see “Labeling Symbols” in the user guide.

## Getting Help

FUJIFILM Sonosite Technical Support

<b>United States or Canada</b>	+1 (877) 657-8118
<b>Europe and Middle East</b>	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
<b>Asia and Pacific</b>	+61 2 9938 8700
<b>Other regions</b>	+1 (425) 951-1330, or call your local representative
<b>Fax</b>	+1 (425) 951-6700
<b>E-mail</b>	General: ffss-service@fujifilm.com Europe, Middle East, and Africa: eraf-service@fujifilm.com Asia and Pacific: ffss-apacme-service@fujifilm.com
<b>Web</b>	www.sonosite.com

## Cleaning and Disinfecting

All references to blood and bodily fluids have been removed from this chapter.

The following note has been added to this chapter:

### Note

FUJIFILM Sonosite defines ultrasound guided percutaneous procedures as non-critical use applications. Following percutaneous procedures, users should clean and disinfect transducers for non-critical uses as outlined in this user guide.<sup>a</sup>

a. "Disinfection of Ultrasound Transducers Used for Percutaneous Procedures. Intersocietal Position Statement." *American Institute of Ultrasound in Medicine | J Ultrasound Med.* February 16, 2021; 9999: p. 1-3.

## Cleaning Accessories

To clean and disinfect the ECG leadwires, refer to the manufacturer's instructions.

## Safety

The following has been added to the safety chapter.

Please report any serious safety incident that occurs in relation to the ultrasound system to FUJIFILM Sonosite and to the competent authority of the country in which the user and patient are located.

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