Sonosite iLOOK Quick Guide

Charging the system

Charge the system before using it. Battery charging completes within four hours.

- 1. Make sure that the AC power cord is connected to the power brick and that the cable on the other end of the brick is connected to the connector on the back of the system.
- 2. Plug the AC power cord into a hospital-grade AC mains outlet.
- 3. Insert the transducer into the transducer holder with the label side facing the back of the holder where the wireless charger is located.

Turning the system on and off

- To turn on the display, press and hold the power button located on the back until the screen lights up.
- To turn on the transducer, press and hold the power button located on the bottom of the transducer. The power indicator flashes blue.
- To turn off the display, press and hold the power button. Tap **Power Off** when the selection screen appears.
- To turn off the transducer, press and hold the power button for three seconds.

Logging into the system

- 1. Enter your login credentials on the login page or select **Guest mode**.

After logging into the system, the home screen appears.

- 2. Select **Scan** to enter scan mode, **Help** to access the user guide, or **Settings** to configure the system.

Starting an exam

Scan mode allows you to connect to the transducer, enter patient information, and perform exams.

Connecting the transducer

- 1. Make sure that both the transducer and the display device are on.
- 2. To connect to a transducer for the first time, tap the system menu on the scan screen.
- 3. In the system menu, tap **Connect to transducer**.
- 4. Tap **Add transducer**, then tap the transducer's serial number in the list.
- 5. After connection, the transducer's indicator light turns green (if the battery is charged), and the scan screen appears.

Entering patient information

- 1. Tap the patient information text field at the top of the screen to enter the patient information screen.
- 2. On the patient information screen, tap each field and enter the name, ID, date of birth, and gender.

All data saved during the study are linked to the patient.

Selecting an exam type


- 1. At the top of the screen, tap on the exam type field to display a drop-down menu.
- 2. Select an exam type from the menu.

Scanning

- 2D is the default mode. Tap **2D** to return to the default mode.
- Tap **C** to scan in Color Doppler.
- To freeze the image, tap .

Color Doppler


- To select the position of the color box on the screen, tap , , or . You can also directly drag the box up or down to those positions.
- Tap to display the imaging control panel, then do one of the following:
 - To switch the color type, tap **CVD** (Color Velocity Doppler) or **CPD** (Color Power Doppler).
 - To steer the color box, tap , , or .
 - To adjust the color scale, tap **Low**, **Med**, or **High**.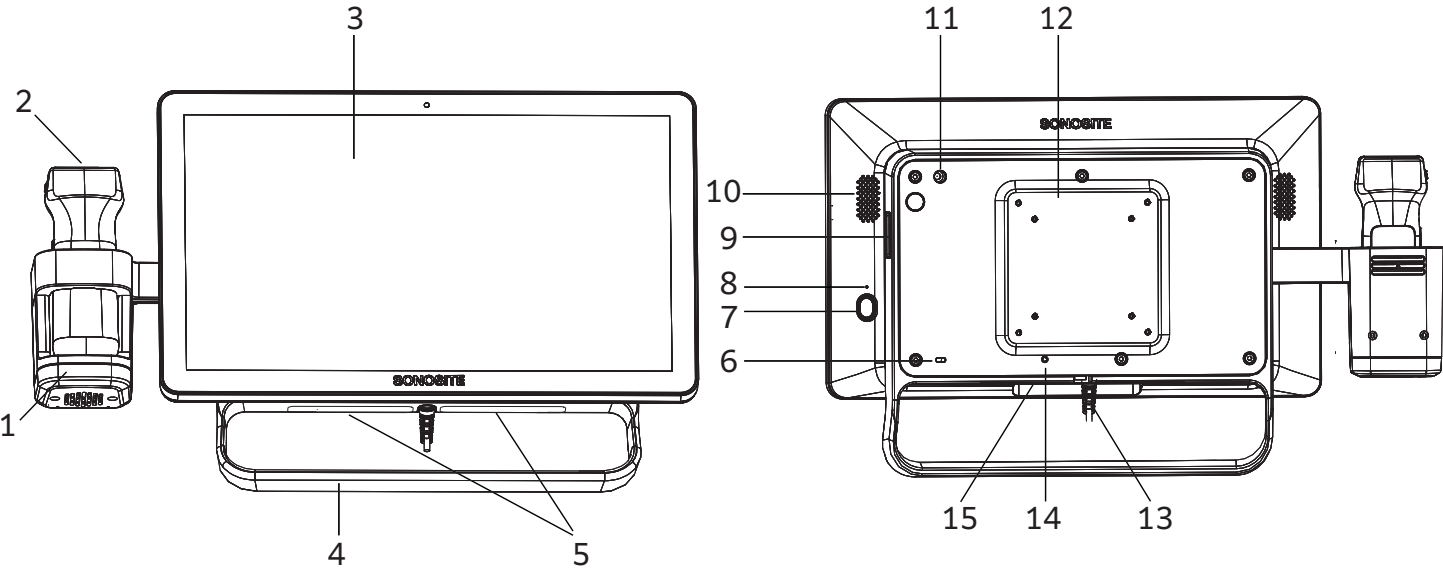

- | | | | | |
|--|-----------------------------|-----------------|--------------------|-----------------------|
| 1. Transducer holder with built-in charger and fan | 4. System handle | 7. Power button | 10. System speaker | 13. Cable connector |
| 2. Transducer | 5. System label | 8. Reset button | 11. Accessory port | 14. Charge indicator |
| 3. Clinical display with touchscreen | 6. Kensington security slot | 9. USB port | 12. VESA interface | 15. System microphone |

Sonosite iLOOK Quick Guide

Main scan screen

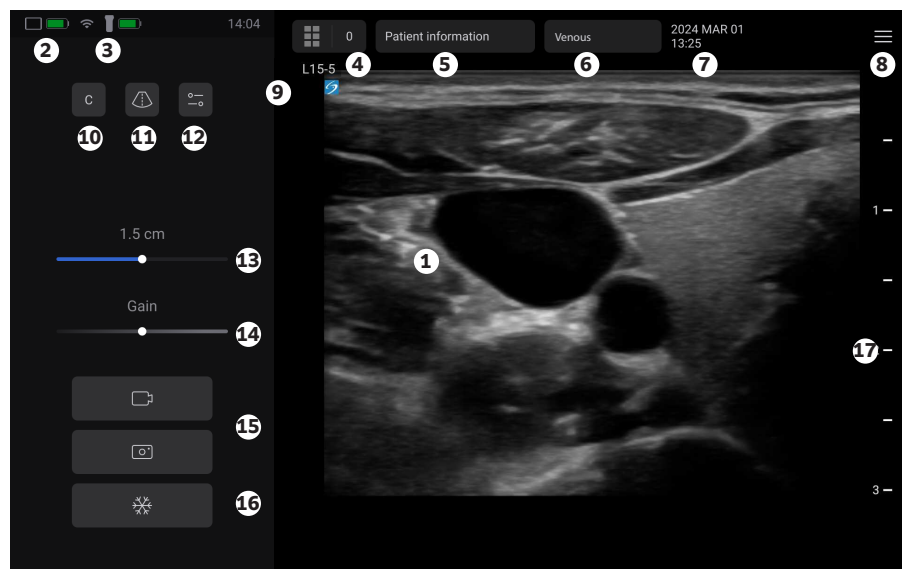1. Ultrasound image
2. Battery level of the display
3. Transducer connection status and battery level
4. Review button
5. Patient information
6. Exam type selection
7. Date and time
8. System menu
9. Orientation marker
10. Imaging modes
11. Centerline
12. Imaging control panel
13. Depth control
14. Gain or cine controls
15. Save buttons
16. Freeze
17. Depth marker

Adjusting depth, gain, and zoom

- To decrease or increase the depth, move the slider back and forth on the touchscreen.
- To decrease or increase the gain, move the slider back and forth on the touchscreen. When scanning in color, the gain slider adjusts the color gain.
- Adjust the zoom by pinching out or in on the touchscreen. You can double-tap the image to return to the original image size.

Measurements and calculations

Freeze the image  before completing a measurement or calculation.

- To take a distance measurement, tap  and adjust the calipers on the touchscreen.
- To calculate CVR (catheter to vessel ratio), tap **Vessel D (Fr)** or **Vessel D (G)**, use

the calipers to measure the diameter, then select the catheter size from the drop-down menu.

- To delete a measurement, tap  or  **All**.

Saving and reviewing

- Tap  to save an image with any current measurements displayed.
- Tap  to save a clip. Clips can only be saved on live images.
- To review images and clips for the current exam, tap  at the top of the screen then tap any thumbnail to display a full-size version.
- To view images and clips from a past exam, tap the menu  and then tap **Patient list**. Tap  to display the search screen, enter

the patient ID, name, or exam date and tap  to search the list. Tap the desired exam to display the review page.

Ending and exporting

- To end the exam while scanning, tap the **End study** button at the top right of the screen.
- To end the exam from another screen, tap the system menu  then tap **End study**.
- To export data to a USB, navigate to the **Patient list**, select the exam and tap **Send to** then **USB**. Select the check box to remove patient data before exporting.

www.sonosite.com/contactus

Copyright © 2026 FUJIFILM SonoSite, Inc. All rights reserved.

Part number: P33249-02

06/2026

SONOSITE and the SONOSITE logo are trademarks and registered trademarks of FUJIFILM Sonosite in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation in various jurisdictions. All other trademarks are the property of their respective owners.

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Copyright © 2026 FUJIFILM SonoSite, Inc. Tous droits réservés.
Copyright © 2026 FUJIFILM SonoSite, Inc. Alle rechten voorbehouden.
Copyright © 2026 FUJIFILM SonoSite, Inc. Med ensamrätt.
Copyright © 2026 FUJIFILM SonoSite, Inc. Alle rettigheder forbeholdes.