



Sonosite PX System Battery Assembly Instructions

Inside the box


System batteries (2)

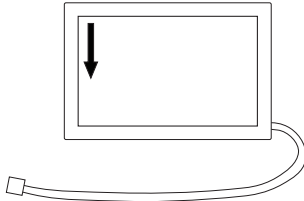

Battery compartment cover (1)

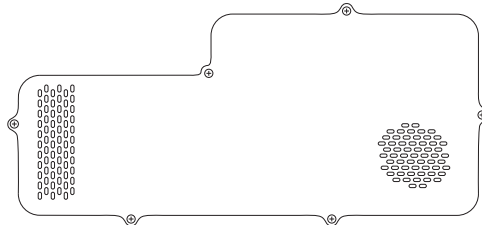


M3 x 8 mm Phillips-head screws (6)

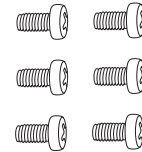

Installing the system battery


STEP 1

Access the system's battery compartment.

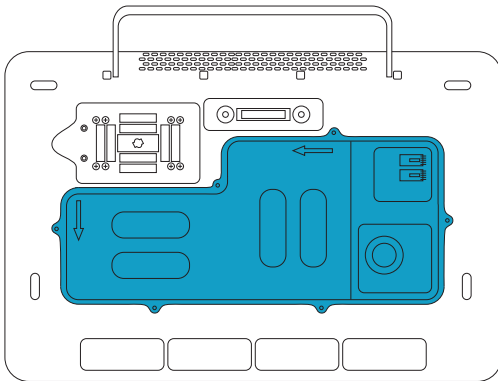


STEP 2

Place one of the batteries in the compartment as shown, then plug its cable into the connector. Make sure the cable is placed fully within the compartment.

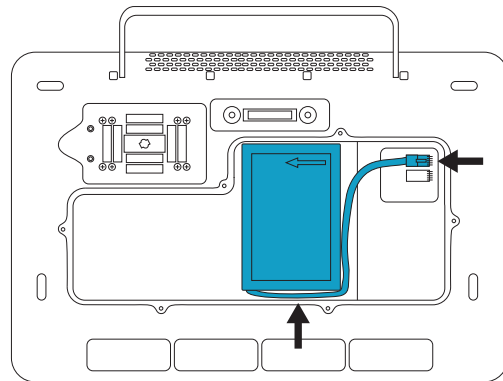


STEP 3

Place the other battery in the compartment as shown, then plug its cable into the connector. Make sure the cable is placed fully within the compartment.

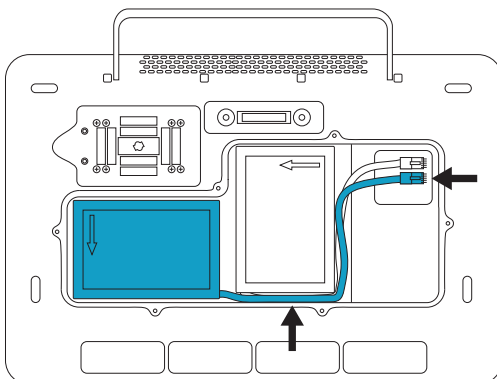

STEP 4

Attach the battery compartment cover using a Phillips screwdriver and the Phillips-head screws.*

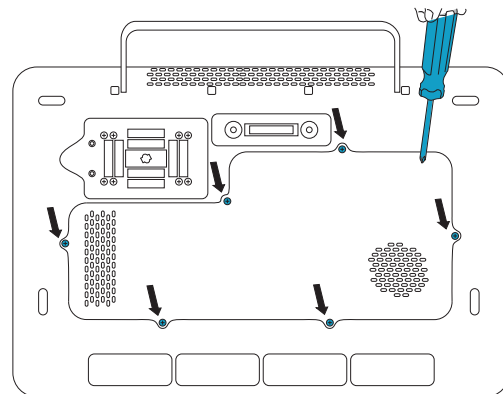

*A magnetic bit driver and Phillips #1 bit are available in the stand assembly kit.

SONOSITE is a registered trademark of FUJIFILM Sonosite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Published April 2020

Copyright © 2020 FUJIFILM Sonosite, Inc. All rights reserved.

Manufacturer

FUJIFILM Sonosite, Inc.
21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

**EC Authorized
Representative**

FUJIFILM Sonosite B.V.
Joop Geesinkweg 140
1114 AB Amsterdam,
The Netherlands

Australia Sponsor

FUJIFILM Sonosite Australasia Pty Ltd
114 Old Pittwater Road
BROOKVALE, NSW, 2100
Australia



P27891-01

