



# System Battery Assembly Instructions

## Inside the box


System batteries (2)

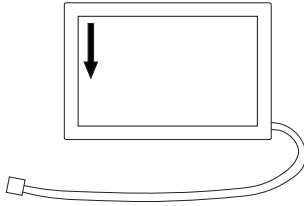

Battery compartment cover (1)

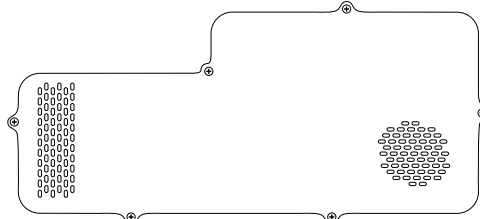


M3 x 8 mm Phillips-head screws (6)

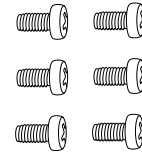

## Installing the system battery


### STEP 1

Access the system's battery compartment.

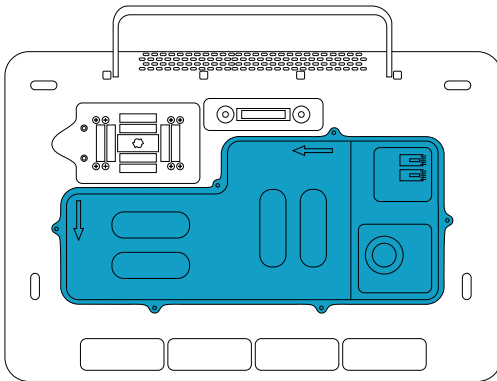


### STEP 2

Place one of the batteries in the compartment as shown, then plug its cable into the connector. Make sure the cable is placed fully within the compartment.

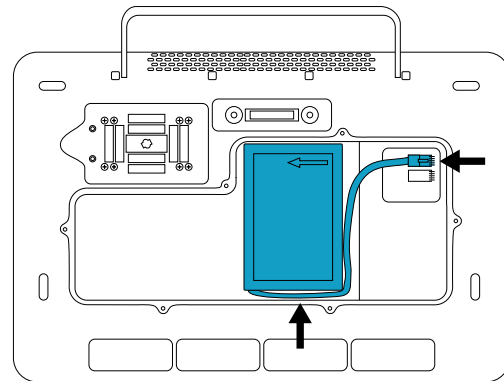


### STEP 3

Place the other battery in the compartment as shown, then plug its cable into the connector. Make sure the cable is placed fully within the compartment.

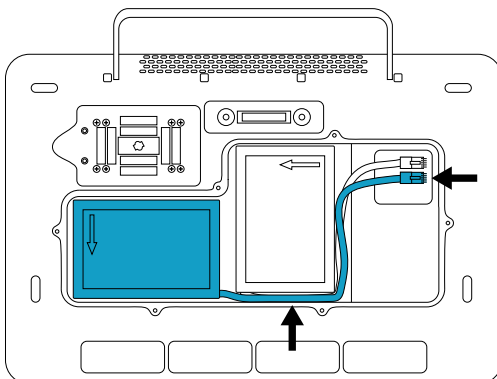

### STEP 4

Attach the battery compartment cover using a Phillips screwdriver and the Phillips-head screws.\*

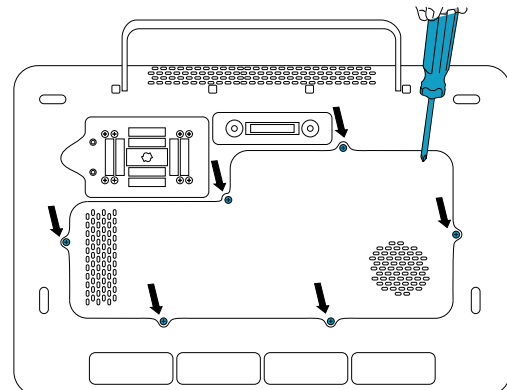

\*A magnetic bit driver and Phillips #1 bit are available in the stand assembly kit.

SONOSITE is a registered trademark of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation All other trademarks are the property of their respective owners.

Patent: <https://www.sonosite.com/patents>

Published February 2023

Copyright © 2023 FUJIFILM SonoSite, Inc. All rights reserved.

**Manufacturer**

FUJIFILM SonoSite, Inc.  
21919 30th Drive SE  
Bothell, WA 98021 USA  
T: 1-888-482-9449 or  
1-425-951-1200  
F: 1-425-951-1201

**EC Authorized  
Representative**

FUJIFILM SonoSite B.V.  
Joop Geesinkweg 140  
1114 AB Amsterdam,  
The Netherlands

**Australia Sponsor**

FUJIFILM SonoSite Australasia Pty  
Ltd  
114 Old Pittwater Road  
BROOKVALE, NSW, 2100  
Australia



P27891-02

