## T8-3 Transducer Care

### WARNINGS


- Be sure to use the appropriate Personal Protective Equipment (PPE) whenever necessary.
- **DO NOT** skip any steps or abbreviate the cleaning process in any way.

### Cautions

- **DO NOT** allow bodily fluids to dry on the transducer. Always clean the transducer immediately after use.
- **DO NOT** bend the endoscopic shaft smaller than a 20 cm (8 inch) curve.
- **DO NOT** drop the transducer. If it is dropped, perform an electrical safety test and temperature calibration prior to use.
- **DO NOT** use unapproved cleaning agents because they can damage the transducer and void the warranty. For more information about approved cleaners and disinfectants, see [www.sonosite.com/support/cleaners-disinfectants](http://www.sonosite.com/support/cleaners-disinfectants).
- **DO NOT** spray or immerse the controller, the cable, or the connector with any fluid. Be careful to prevent moisture from getting on the connector.
- **DO NOT** soak the transducer longer than recommended by the chemical manufacturer. Follow the manufacturer's recommendations for concentration, temperature, and duration.
- **Clean and disinfect** transducer prior to use if it is not housed in a protective covering.


Refer to the following diagram when caring for your transducer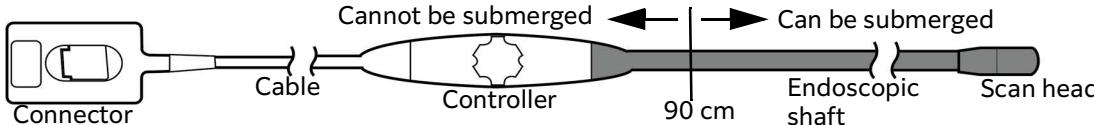

## Preventing damage

### STEP 1 ALWAYS


use a bite guard.  
Remove any dentures before placing the bite guard in the patient's mouth.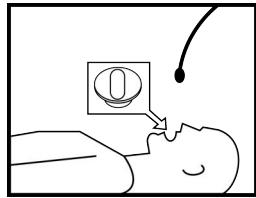

## Cleaning


### STEP 3 Rinse

the endoscopic shaft and scan head with clean, room temperature running water for a minimum of **1 minute**. **DO NOT** rinse the cable or controller.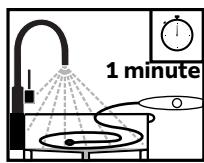

### STEP 4 Soak

up to 90 cm of the shaft and scan head in an **approved** cleaner. Make sure the controller cannot fall into the solution.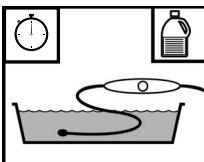



**CHECK** the expiration date of the cleaner, and follow **ALL** chemical

## Pre-cleaning

### STEP 2 DO NOT



allow biological material to dry on the transducer. Immediately after use:

- a Use a clean cloth moistened with water to wipe the cable and controller. **DO NOT** wipe the connector.
  - b Use a second clean cloth moistened with water to wipe the shaft and scan head.
- INSPECT** the transducer for any sign of damage. **DO NOT** use the transducer if any damage is found.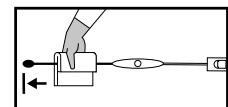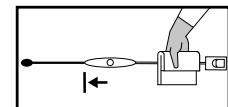


### STEP 5 During the soak:

- a **Wipe** cable and controller with a wipe approved for use on the cable and controller. **DO NOT** wipe the connector. **CHECK** the expiration date of the wipe.
- b **Wipe** shaft and scan head using a clean, soft cloth or sponge moistened with cleaner.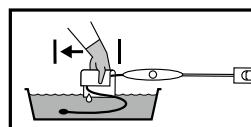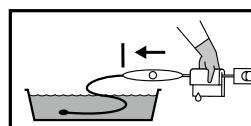

### STEP 6 Rinse


the shaft and scan head with room-temperature running water for a minimum of **3 minutes**. **DO NOT** rinse the cable or controller. **INSPECT** for any remaining biological material. Repeat cleaning if any is found.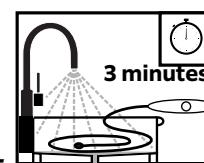

### STEP 7 Dry

the transducer with a clean, non-linting towel.

**INSPECT** for any sign of damage. **DO NOT** use the transducer if any damage is found.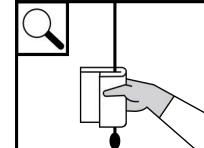


For a list of approved cleaners and disinfectants, refer to the cleaners and disinfection tool available at [www.sonosite.com/support/cleaners-disinfectants](http://www.sonosite.com/support/cleaners-disinfectants)

# Testing for electrical leakage

## STEP 8 Set up the equipment:


- a **Immerse** the endoscopic shaft up to the 90 cm mark. Secure the controller so it cannot fall into the test solution.
- b **Insert** the transducer connector into the transducer adapter.
- c **Connect** the transducer adapter to the leakage tester.
- d **Connect** the conductivity probe to the leakage tester.
- e **Immerse** the conductivity probe contacts into an approved leakage testing fluid. **DO NOT** allow the probe to touch the transducer.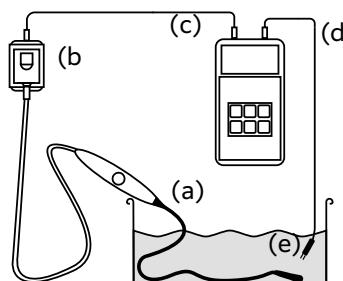

For complete instructions on using the leakage tester, refer to the **ULT2000 Series User Manual**.

## STEP 9 Run the leakage test:

- a **Power on** the ULT-2020 leakage tester.
- b **Press MODE** repeatedly until Device Configuration appears.
- c **Confirm** that the leakage tester is set to the **FUJIFILM Sonosite T8-3** transducer. If it isn't, change the setting as required.
- d **Confirm** that the endoscopic shaft, the scan head, and the conductivity probe contacts are all fully submerged in the leakage test fluid.
- e **Press FULL TEST** to start the test. Ensure the shaft, scan head, and probe contacts remain submerged during testing.
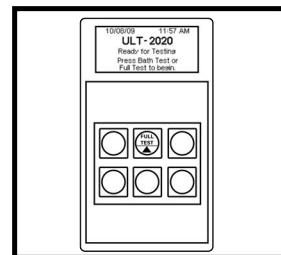


If the result is **Fail**, **DO NOT** use the transducer. Contact FUJIFILM Sonosite for repair.


## Disinfecting

### STEP 10 Disinfect the transducer:

- a **Wipe** the controller and cable with either a Sani-Cloth or a clean, non-linting cloth moistened with **disinfectant approved for use on cable and controller**. **DO NOT** wipe the connector. **CHECK** the expiration date of the wipe, and follow **ALL** manufacturer's instructions for contact time.
- b If it is in accordance with facility policies, **remove** disinfectant after specified soak time by wiping with a clean, non-linting cloth or sponge moistened with water.
- c **Soak** up to 90 cm of the endoscopic shaft and scan head in **disinfectant approved for immersing the shaft**. Make sure the controller cannot fall into the solution. **CHECK** the expiration date of the disinfectant, and follow **ALL** chemical manufacturer's instructions for soak time and temperature.

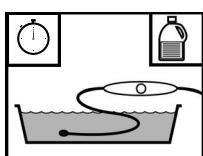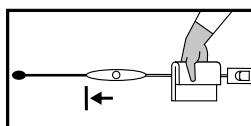
- ### STEP 11 Rinse
- the shaft and scan head with room-temperature running water for a minimum of **3 minutes**. **DO NOT** rinse the cable or controller.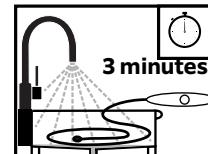

- ### STEP 12 Dry
- the transducer with a clean, non-linting cloth or medical-grade air.


**INSPECT** the transducer for any sign of damage. **DO NOT** use the transducer if any damage is found.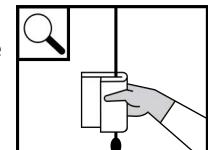

## Transporting and storing

### STEP 13 Transport and store

the transducer in accordance with your organization's policies.

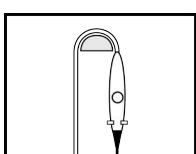For complete instructions on cleaning, disinfecting, transporting, and storing the T8-3 transducer, refer to the **T8-3 Transducer (P29465) User Guide**.

SONOSITE is a registered trademark of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Copyright © 2023 FUJIFILM SonoSite, Inc. All rights reserved.

P21459-02

**Manufacturer:**  
**FUJIFILM SonoSite, Inc.**  
21919 30th Drive SE  
Bothell, WA 9802, USA  
T: +1-888-482-9449 or  
+1-425-951-1200  
F: +1-425-951-1201

**EC Authorized Representative:**  
**FUJIFILM SonoSite B.V.**  
Joop Geesinkweg 140  
1114 AB Amsterdam,  
The Netherlands

**Australia Sponsor:**  
**FUJIFILM SonoSite Australasia Pty Ltd**  
114 Old Pittwater Road  
Brookvale, NSW, 2100  
Australia

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Copyright © 2023 FUJIFILM SonoSite, Inc. Reservados todos los derechos.
Copyright © 2023 FUJIFILM SonoSite, Inc. Alle Rechte vorbehalten.
Copyright © 2023 FUJIFILM SonoSite, Inc. Tous droits réservés.
Copyright © 2023 FUJIFILM SonoSite, Inc. Tutti i diritti riservati.
Copyright © 2023 FUJIFILM SonoSite, Inc. Todos os direitos reservados.
Copyright © 2023 FUJIFILM SonoSite, Inc. Alle rechten voorbehouden.