T8-3 Transducer Care

WARNINGS


- ▶ Be sure to use the appropriate Personal Protective Equipment (PPE) whenever necessary.
- ▶ **DO NOT** skip any steps or abbreviate the cleaning process in any way.

Cautions

- ▶ **DO NOT** allow bodily fluids to dry on the transducer. Always clean the transducer immediately after use.
- ▶ **DO NOT** bend the endoscopic shaft smaller than a 20 cm (8 inch) curve.
- ▶ **DO NOT** drop the transducer. If it is dropped, perform an electrical safety test and temperature calibration prior to use.
- ▶ **DO NOT** use unapproved cleaning agents because they can damage the transducer and void the warranty. For more information about approved cleaners and disinfectants, see www.sonosite.com/support/cleaners-disinfectants.
- ▶ **DO NOT** spray or immerse the controller, the cable, or the connector with any fluid. Be careful to prevent moisture from getting on the connector.
- ▶ **DO NOT** soak the transducer longer than recommended by the chemical manufacturer. Follow the manufacturer's recommendations for concentration, temperature, and duration.
- ▶ **Clean and disinfect** transducer prior to use if it is not housed in a protective covering.


Refer to the following diagram when caring for your transducer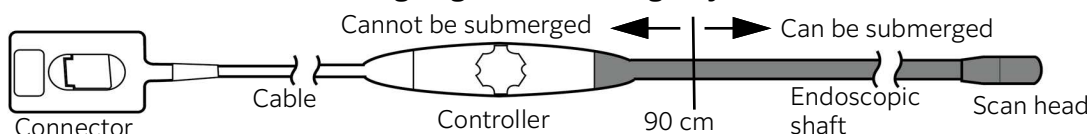

Preventing damage

STEP 1 ALWAYS

use a bite guard. Remove any dentures before placing the bite guard in the patient's mouth.
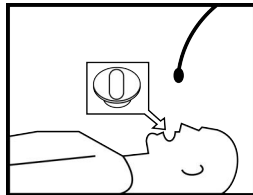


Pre-cleaning

STEP 2 DO NOT allow biological material to dry on the transducer. Immediately after use:

- Use a clean cloth moistened with water to wipe the cable and controller. **DO NOT** wipe the connector.
- Use a second clean cloth moistened with water to wipe the shaft and scan head.


INSPECT the transducer for any sign of damage. **DO NOT** use the transducer if any damage is found.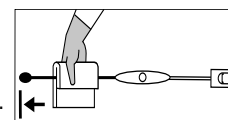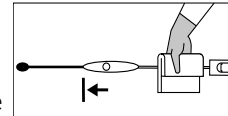

Cleaning


STEP 3 Rinse the

endoscopic shaft and scan head with clean, room temperature running water for a minimum of **1 minute**. **DO NOT** rinse the cable or controller.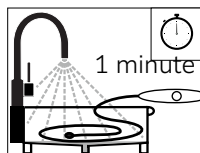



STEP 4 Soak up to

90 cm of the shaft and scan head in an approved cleaner. Make sure the controller cannot fall into the solution. **CHECK** the expiration date of the cleaner, and follow **ALL** chemical manufacturer's instructions for soak time and temperature.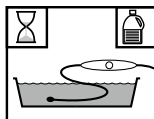


STEP 5 During the soak:

- Wipe** cable and controller with a wipe approved for use on the cable and controller. **DO NOT** wipe the connector. **CHECK** the expiration date of the wipe.
- Wipe** shaft and scan head using a clean, soft cloth or sponge moistened with cleaner.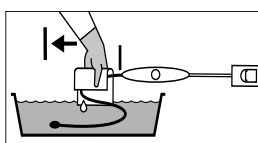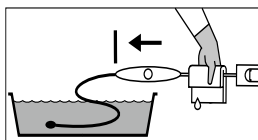


STEP 6 Rinse the

shaft and scan head with room-temperature running water for a minimum of **3 minutes**. **DO NOT** rinse the cable or controller. **INSPECT** for any remaining biological material. Repeat cleaning if any is found.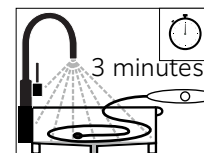

STEP 7 Dry the

transducer with a clean, non-linting towel. **INSPECT** for any sign of damage. **DO NOT** use the transducer if any damage is found.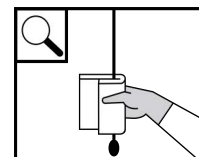


For a list of approved cleaners and disinfectants, refer to the cleaners and disinfection tool available at www.sonosite.com/support/cleaners-disinfectants.

Testing for electrical leakage

STEP 8 Set up the equipment:


- Immerse** the endoscopic shaft up to the 90 cm mark. Secure the controller so it cannot fall into the test solution.
- Insert** the transducer connector into the transducer adapter.
- Connect** the transducer adapter to the leakage tester.
- Connect** the conductivity probe to the leakage tester.
- Immerse** the conductivity probe contacts into an approved leakage testing fluid. **DO NOT** allow the probe to touch the transducer.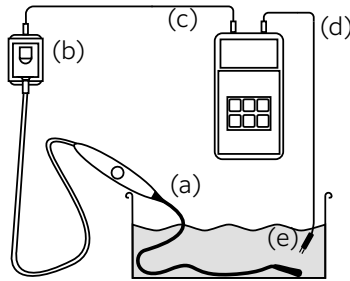

For complete instructions on using the leakage tester, refer to the **ULT2000 Series User Manual**.

STEP 9 Run the leakage test:

- Power on** the ULT-2020 leakage tester.
- Press **MODE** repeatedly until Device Configuration appears.
- Confirm** that the leakage tester is set to the **FUJIFILM Sonosite T8-3** transducer. If it isn't, change the setting as required.
- Confirm** that the endoscopic shaft, the scan head, and the conductivity probe contacts are all fully submerged in the leakage test fluid.
- Press **FULL TEST** to start the test. Ensure the shaft, scan head, and probe contacts remain submerged during testing.
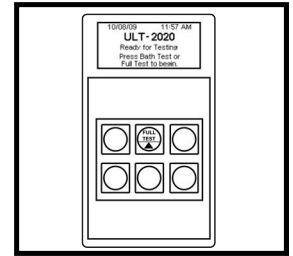


If the result is **Fail**, **DO NOT** use the transducer. Contact FUJIFILM Sonosite for repair.


Disinfecting

STEP 10 Disinfect the transducer:


- Wipe** the controller and cable with either a Sani-Cloth or a clean, non-linting cloth moistened with **disinfectant approved for use on cable and controller**. **DO NOT** wipe the connector. **CHECK** the expiration date of the wipe, and follow **ALL** manufacturer's instructions for contact time.
- If it is in accordance with facility policies, **remove** disinfectant after specified soak time by wiping with a clean, non-linting cloth or sponge moistened with water.
- Soak** up to 90 cm of the endoscopic shaft and scan head in **disinfectant approved for immersing the shaft**. Make sure the controller cannot fall into the solution. **CHECK** the expiration date of the disinfectant, and follow **ALL** chemical manufacturer's instructions for soak time and temperature.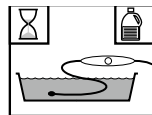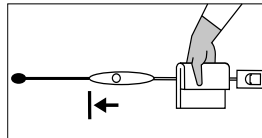

STEP 11 Rinse the shaft and scan head with room-temperature running water for a minimum of **3 minutes**. **DO NOT** rinse the cable or controller.

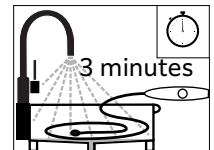
STEP 12 Dry the transducer with a clean, non-linting cloth or medical-grade air. **INSPECT** the transducer for any sign of damage. **DO NOT** use the transducer if any damage is found.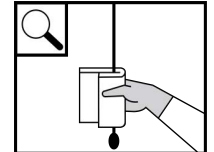

Transporting and storing

STEP 13 Transport and store the transducer in accordance with your organization's policies.

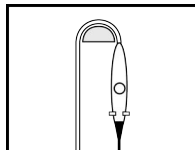For complete instructions on cleaning, disinfecting, transporting, and storing the T8-3 transducer, refer to the **T8-3 Transducer (P29465) User Guide**.

SONOSITE is a registered trademark of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Copyright © 2025 FUJIFILM SonoSite, Inc. All rights reserved.

P21459-03

Manufacturer:
FUJIFILM SonoSite, Inc.
 21919 30th Drive SE
 Bothell, WA 9802, USA
 T: +1-888-482-9449 or
 +1-425-951-1200
 F: +1-425-951-1201

EC Authorized Representative
FUJIFILM SonoSite B.V.
 Joop Geesinkweg 140
 1114 AB Amsterdam,
 The Netherlands

Australia Sponsor
FUJIFILM SonoSite Australasia Pty Ltd
 114 Old Pittwater Road
 Brookvale, NSW, 2100
 Australia

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Copyright © 2025 FUJIFILM SonoSite, Inc. Alle Rechte vorbehalten.
Copyright © 2025 FUJIFILM SonoSite, Inc. Reservados todos los derechos.
Copyright © 2025 FUJIFILM SonoSite, Inc. Tous droits réservés.
Copyright © 2025 FUJIFILM SonoSite, Inc. Tutti i diritti riservati.
Copyright © 2025 FUJIFILM SonoSite, Inc. Todos os direitos reservados.
Copyright © 2025 FUJIFILM SonoSite, Inc. Alle rechten voorbehouden.