---

# SonoSite X-Porte

## ***User Guide Errata***

**Manufacturer**

FUJIFILM SonoSite, Inc.

21919 30th Drive SE

Bothell, WA 98021 USA

T: 1-888-482-9449 or 1-425-951-1200

F: 1-425-951-1201

**EC Authorized Representative**

FUJIFILM SonoSite B.V.

Joop Geesinkweg 140

1114 AB Amsterdam,

The Netherlands

**Australia Sponsor**

FUJIFILM SonoSite Australasia Pty Ltd

114 Old Pittwater Road

BROOKVALE, NSW, 2100

Australia

**Caution**

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite, X-Porte, and the SonoSite logo are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. FUJIFILM is a registered trademark of FUJIFILM Corporation. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P21471-03

Publication date: April 2018

Copyright © 2018 FUJIFILM SonoSite, Inc. All rights reserved.

# SonoSite X-Porte User Guide Errata

<b>Introduction</b> .....	<b>1</b>
Document conventions .....	1
Getting help .....	1
<b>Getting started</b> .....	<b>2</b>
<b>Setting up the system</b> .....	<b>2</b>
USB Settings .....	2
eFilm Lite image-Viewer .....	3
<b>Measurement references</b> .....	<b>3</b>
Cardiac references .....	3
<b>Cleaning and disinfecting</b> .....	<b>3</b>
<b>Clinical safety</b> .....	<b>4</b>
<b>Specifications</b> .....	<b>4</b>
Dimensions .....	4

## Introduction

The content in this document was either not included or incorrect in the system user guide; the revisions will be made in the next update.

## Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.
- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ❖.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the system user guide.

## Getting help

For technical support, please contact FUJIFILM SonoSite as follows:

**Phone** 877-657-8118  
**(U.S. or Canada)**

<b>Phone (outside U.S. or Canada)</b>	425-951-1330, or call your local representative
<b>Fax</b>	425-951-6700
<b>Email</b>	ffss-service@fujifilm.com
<b>Web</b>	www.sonosite.com
<b>Europe Service Center</b>	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
<b>Asia Service Center</b>	+65 6380-5581

Printed in the U.S.

## Getting started

The following caution has been added.

### Caution

When performing a search in the system Help, do not enter the word "and" following your search word. The system locks up and requires a restart.

## Setting up the system

### USB Settings

Information about the eFilm Lite image-viewer has been removed.

#### To specify export options

- 1 On the **USB** settings page, select an **Export Type**:
  - ▶ **DICOM Export** creates DICOMDIR files readable by a DICOM reader. DICOM is an optional feature.
  - ▶ **Multimedia Export** organizes files in a standard folder structure. Video clips export in MJPEG video saved as AVI. To view them, FUJIFILM SonoSite recommends Windows Media Player or QuickTime 7.0 or later.

2 Select an **Image Format** for your export type. For JPEG image format, also select a JPEG compression. A high compression has a smaller file size but less detail (see “Limitations of JPEG format”). The image format affects both still images and video clips.

For optimal DICOM image quality, select RGB image format and low compression.

3 (**Multimedia Export** only) Select a sort order from the **Sort By** list.

4 Select **Auto Export** to enable automatic export of ended exams. See also “To export patient exams automatically to a USB memory stick.”

5 (**DICOM Export** only) Select any of the following:

- ▶ **Include CDA** to export the CDA report. See “CDA Report settings.”
- ▶ **Include SR** to export the Structured Report.

6 Tap **Done**.

## eFilm Lite image-Viewer

This section has been removed.

## Measurement references

### Cardiac references

The following reference has been added.

#### Left Atrial Area

Lang, R. et al. “Recommendations for Cardiac Chamber Quantification by Echocardiography in Adults: An Update from the American Society of Echocardiography and the European Association of Cardiovascular Imaging.” *Journal of the American Society of Echocardiography*. (2015), 28: p. 25-26.

## Cleaning and disinfecting

The following website was corrected.

[www.sonosite.com/products/transducers](http://www.sonosite.com/products/transducers)

The following footnote was updated in the cleaning and disinfecting tables.

Refer to the cleaners and disinfection tool available at [www.sonosite.com/support/cleaners-disinfectants](http://www.sonosite.com/support/cleaners-disinfectants) for a more complete list of approved cleaners and disinfectants.

## Clinical safety

The following warning has been updated.

**WARNING** FUJIFILM SonoSite does not recommend the use of high-frequency electromedical devices in proximity to its systems. FUJIFILM SonoSite equipment has not been validated for use with high-frequency electrosurgical devices or procedures. Use of high-frequency electrosurgical devices in proximity to its systems may lead to abnormal system behavior or shutdown of the system.

To avoid the risk of a burn hazard, do not use the transducer with high-frequency surgical equipment. Such a hazard may occur in the event of a defect in the high-frequency surgical neutral electrode connection.

## Specifications

### Dimensions

#### Stand

	<u>First configuration</u>	<u>Second configuration</u>
<b>Length</b>	26.4 in. (67.1 cm)	26.4 in. (67.1 cm)
<b>Width</b>	21.2 in. (58.3 cm)	21.2 in. (58.3 cm)
<b>Height</b> (max with monitor up)	64 in. (162.6 cm)	65 in. (165.6 cm)
<b>Height</b> (min with monitor down)	42.2 in. (107.2 cm)	42.2 in. (107.2 cm)

#### Clinical monitor

	<u>First configuration</u>	<u>Second configuration</u>
<b>Length</b> (exterior dimension)	18.8 in. (47.8 cm)	20.2 in. (51.4 cm)
<b>Height</b> (exterior dimension)	14.2 in. (36 cm)	14.33 in. (36.4 cm)
<b>Diagonal</b> (viewable area)	19 in. (48.3 cm)	19.9 in. (50.6 cm)
<b>Resolution</b>	1680 x 1050	1728 x 1080

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