SonoSite iViz

User Guide Supplement

Manufacturer	EC Authorized Representative	Australia Sponsor
FUJIFILM SonoSite, Inc.	FUJIFILM SonoSite B.V.	FUJIFILM SonoSite Australasia Pty Ltd
21919 30th Drive SE	Joop Geesinkweg 140	114 Old Pittwater Road
Bothell, WA 98021 USA	1114 AB Amsterdam,	BROOKVALE, NSW, 2100
T: 1-888-482-9449 or 1-425-951-1200	The Netherlands	Australia
F: 1-425-951-1201		

Caution

United States federal law restricts this device to sale by or on the order of a physician.

SonoSite, the SonoSite logo, and SonoSite iViz are trademarks and registered trademarks of FUJIFILM SonoSite, Inc. in various jurisdictions. Value from Innovation is a trademark of FUJIFILM Holdings America Corporation.

All other trademarks are the property of their respective owners.

Part number: P22312-04

Publication date: October 2017

Copyright © 2017 FUJIFILM SonoSite, Inc. All rights reserved.

SonoSite iViz User Guide Supplement

Introduction	1
Document conventions	1
Getting Help	2
Configuring SonoSite iViz	3
Configuring SonoSite iViz settings	3
Configuring preferences	3
Configuring OB measurements and calculations	5
Managing Patient Records	5
Managing studies	5
Sharing a study	5
Managing reports	6
Editing a report	6
Performing an Exam	7
Centerline	7
Accessing guided protocols	8
eFAST	8
Managing Images and Clips	8
Reviewing an image or clip	8
Measurements and Calculations	9
Exam-based calculations	9
Cardiac calculations	9
Troubleshooting and Maintenance	9
Troubleshooting	9
Creating a bug report	9

Introduction

The content in this document was either not included or incorrect in the system user guide; the revisions will be made in the next update.

Document conventions

The document follows these conventions:

- ▶ A **WARNING** describes precautions necessary to prevent injury or loss of life.
- ▶ A **Caution** describes precautions necessary to protect the products.
- ▶ A **Note** provides supplemental information.
- ▶ Numbered and lettered steps must be performed in a specific order.

- ▶ Bulleted lists present information in list format but do not imply a sequence.
- ▶ Single-step procedures begin with ♦.

For a description of labeling symbols that appear on the product, see "Labeling Symbols" in the ultrasound system user guide.

Getting Help

For technical support, please contact FUJIFILM SonoSite as follows:

Phone (U.S. or Canada)	877-657-8118
Phone (outside U.S. or Canada)	425-951-1330, or call your local representative
Fax	425-951-6700
Email	ffss-service@fujifilm.com
Web	www.sonosite.com
Europe Service Center	Main: +31 20 751 2020 English support: +44 14 6234 1151 French support: +33 1 8288 0702 German support: +49 69 8088 4030 Italian support: +39 02 9475 3655 Spanish support: +34 91 123 8451
Asia Service Center	+65 6380-5581

Printed in the U.S.

Getting Started

General interaction

Using the touchscreen

The Depth label has now been removed from the scan screen.

Configuring SonoSite iViz

Configuring SonoSite iViz settings

Configuring preferences


Two new preferences have been added:

- ▶ Anonymize on USB Share
- ▶ USB Structured Reports

To configure preferences

- 1 From the SonoSite iViz **Settings** screen, tap **Preferences**.
- 2 From the **Preferences** screen, the following settings are available:
 - ▶ **Units** - Select the Height and Weight units used for patient information.
 - ▶ **Generic Volume Calcs** - Can be measured two different ways, depending on the type of structures you are measuring. Select either of the following:
 - ▶ **Ellipsoid** - (default) Length x width x height x $\pi/6$.
 - ▶ **Cuboid** - Length x width x height.
 - ▶ **Thermal Index** - Select any of the following:
 - ▶ **TIB** - Thermal index in bone
 - ▶ **TIC** - Thermal index in the cranium
 - ▶ **TIS** - Thermal index in soft tissue
 - ▶ **Auto Save Patient Form** - When turned on, this feature automatically saves the patient form periodically during editing to help prevent the loss of patient information.
 - ▶ **Clip Recording** - Select either of the following:
 - ▶ **Prospective** - During a scan, this option records a standard length clip of the scan after tapping **Save**.
 - ▶ **Retrospective** - During a scan, this option records a standard length clip of the scan before tapping **Save**.
 - ▶ **Clip Length** - From the drop-down menu, choose a standard recording length for clips.

- ▶ **Anonymize on USB Share** - When turned on, this feature removes identifying information from images and AVI clips saved to USB, as well as the folder name. Structured reports and DICOM data will not be created on the USB. The default setting is OFF.
- ▶ **USB Structured Reports** - When turned on, this feature saves information to USB as a structured report. The default setting is OFF.
- ▶ **Scan Power Management** - Choose the settings that will provide the best balance between performance and battery life while scanning: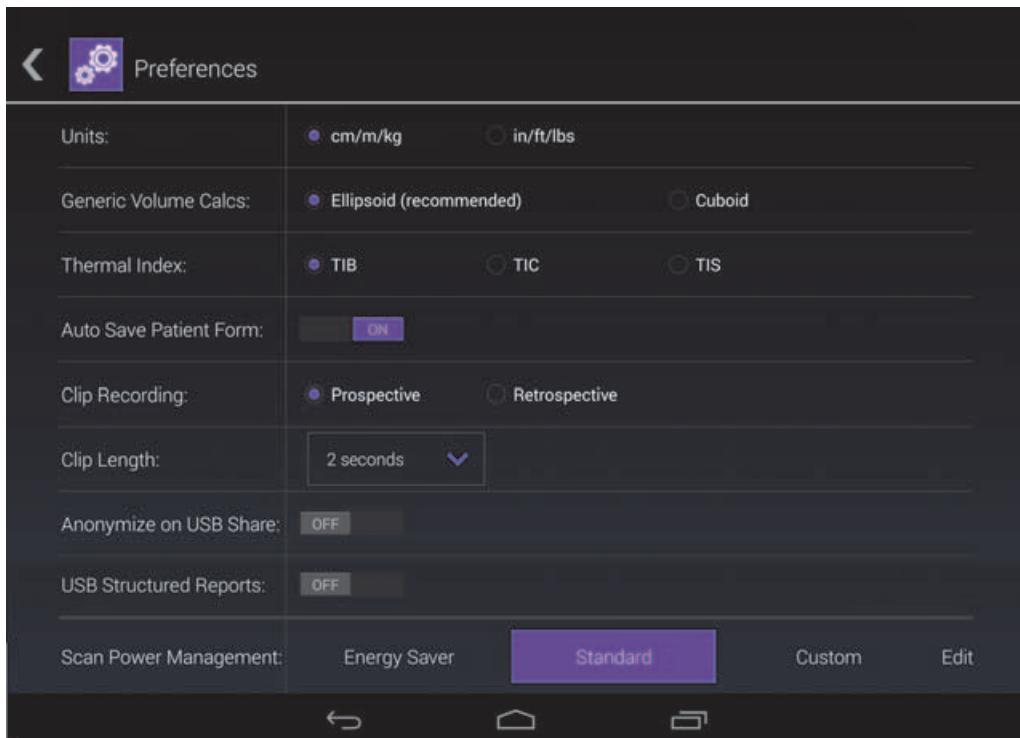

- ▶ **Energy Saver** - Engages low-power modes quickly during periods of inactivity to get the most scanning time from your battery charge.
- ▶ **Standard** - Allows longer periods of inactivity before engaging low-power modes to provide a reasonable balance between scanning availability and battery life.
- ▶ **Custom** - Choose your own power management settings:
 - ▶ **Slow frame rate** - Activates in scanning mode after a period of inactivity. Automatically returns to normal operation when scanning resumes or you touch the controls. When active, a flashing white  appears on the screen. To turn this feature off, select **Never**.

- ▶ **Freeze** – Activates after a continued period of inactivity after switching to Slow frame rate mode. You must unfreeze manually to resume scanning. When active, a solid blue  appears on the screen. To turn this feature off, select **Never**.
- ▶ **Sleep** – Activates after a continued period of inactivity after switching to Freeze mode. To wake the system, quickly press and release the power button. To turn this feature off, select **Never**.
- ▶ **Power off** – Turns off the iViz system after a period of time in sleep mode, regardless of whether scanning or not. The system must be powered on manually.

Configuring OB measurements and calculations

You can now restore the default gestational age settings when configuring OB measurements and calculations.

To restore default OB settings

- 1 From the SonoSite iViz **Settings** screen, tap **OB Calcs**.
- 2 In Gestational Age, tap **Restore**.

Managing Patient Records

Managing studies

Sharing a study

If you have **Anonymize on USB Share** selected you are now given the option to continue or cancel when exporting or sharing data.

Note

You cannot anonymize gallery images.

The **Email** option for sharing a study, image, or clip is no longer available.

To use Tricefy to send or share a study

To use the Tricefy medical image-sharing tool to share a study, image, or clip, you must have an active Tricefy account, and be connected to a wireless network.

- 1 From the Patient module, tap the **iViz Studies** tab.
- 2 Tap **Select**. Check the boxes next to the studies that you want to share.
- 3 Tap .

- 4** Tap **Tricefy**.
- 5** Tap **Yes** to activate Tricefy Uplink.
- 6** Follow the prompts on the Tricefy screen to login or register for Tricefy using a valid email address.
Once your account is active, you will receive an activation email.
- 7** Access the email invitation to set up an online account.
- 8** Using your browser, select **Accept Invitation** to access Tricefy4.com.
- 9** Follow the prompts, and enter a password.
- 10** Once your registration is complete, share or upload images, studies, and clips to the cloud.

Note

If you select the **Share** option, an email with a link to a collaborating page will be sent to the specified email or phone number. If you select the **Archive to Cloud** option, the study, images, or clips will be uploaded to the cloud.

To deactivate your device from Tricefy

- 1** From the Patient module, tap the **iViz Studies** tab.
- 2** Tap .
- 3** Tap **Tricefy**.
- 4** On the Tricefy screen, tap **Change Activation**.
- 5** Tap **Yes** to deactivate your device from Tricefy.

Managing reports

You can now **Save** and/or **Close** your report directly from the Save Report Dialog box.

Editing a report

You can now edit the report after you have ended the study, including adding and editing images, notes, and anatomy checklists. Measurements, calculations, and biophysical data cannot be changed.

To add an image to a report

- 1** Open the study that contains the report you want to edit.
- 2** In the open study, tap . The report opens.
- 3** To add images, scroll down and tap **Add images to Report**.
- 4** In the next screen, select the images you want to add.

5 Tap **Add**.

The images are added to the Patient Report Dialog.

6 Tap **Edit** to select and delete any added images.

7 Tap **Done** to return to the Patient Report.

8 Select **Save** to save the report.

Performing an Exam

Centerline

The Centerline feature is now available with the L25v and L38v transducers. To access the feature, tap **Centerline**.

The centerline graphic aligns with the center mark of the transducer and serves as a reference mark for the center of the displayed image.

When using the Centerline feature as a reference during a freehand procedure, be aware that the centerline represents only the center of the ultrasound image and is not an accurate predictor of the path the needle will take.

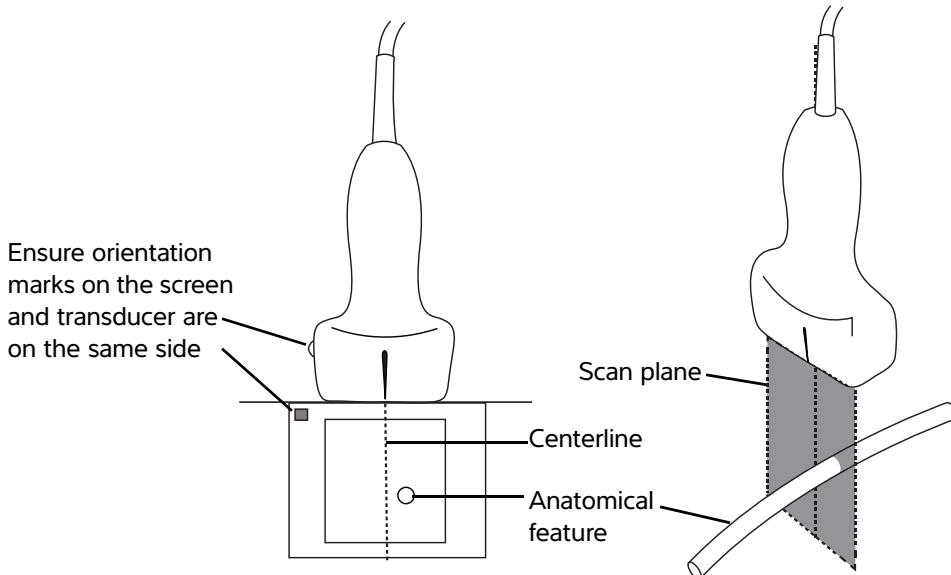
Figure 1 Relationship of the centerline graphic to the transducer and the ultrasound image.

Small tilts or rotations of the transducer can affect the relationship between any external reference points and the anatomy that appears on the ultrasound image.

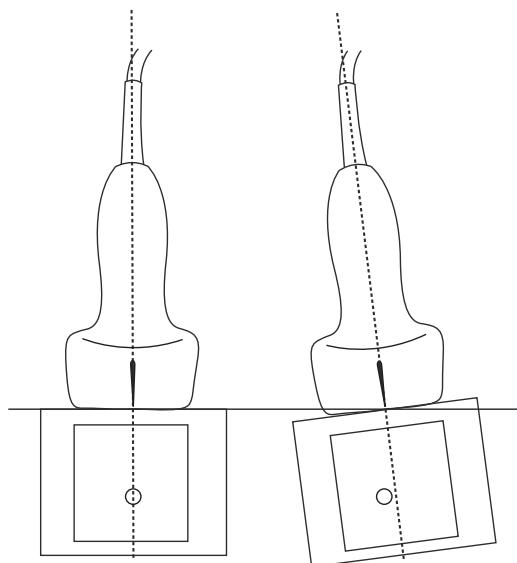
Figure 2 Relationship of the ultrasound image to the transducer angle or tilt.

Accessing guided protocols

eFAST

The Subxiphoid view and label has now been added to the exam.

Managing Images and Clips

Reviewing an image or clip

You can now more easily play back .avi MJPEG files exported from your system on your computer.

Measurements and Calculations

Exam-based calculations

Cardiac calculations

The ACS calculation has been removed from the 2D calculation list.

Troubleshooting and Maintenance

Troubleshooting

Creating a bug report

You can now create and send a bug report to technical support. The report does not include patient data.

To create and send a bug report

1 From the SonoSite iViz Home screen, tap **Settings** .

2 Tap **About iViz**.

3 Tap **Send bug report**.

4 Select **OK** to turn on the Wi Fi.

The report will generate.

5 Select **OK** to send the report to FUJIFILM SonoSite.

6 If the server connection fails, select **Retry** to try again or **Cancel** to cancel the report.

7 You will receive a notification after the report uploads to the server.

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